## R-010160 / R-010170

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The easy locking articulators (R-010160 for full dentures and R-010170 for partial dentures) are two new innovative articulators that save working time.

## PROCESSING INSTRUCTION

1. Trim a rough plaster cast as shown in figures.

- In case of the easy locking articulator for partial dentures (R-010170), trim equally the width of the upper and lower plaster cast as shown in Fig. 1.
- Also trim equally the end part of upper and lower plaster cast as shown in Fig. 2 for the easy locking articulator for full dentures (R-010160).
- Use n. 18, 19, 20 and 21 rubbers to make plaster cast easily.

Fig. 1



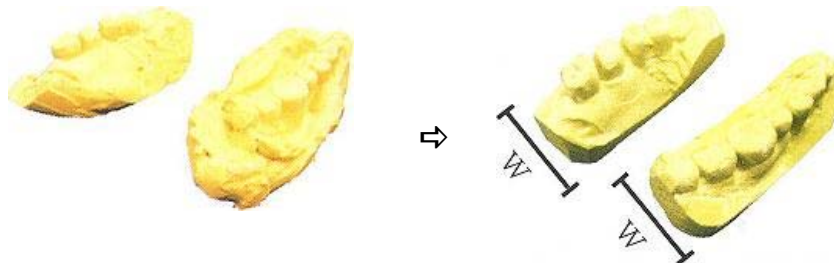
Fig. 2


2. Fix upper & lower plaster cast with using clamping screws (7, 8) and use n. 9 rubbers in case of need. The upper body of articulator, R-010170, can be rotated in case of need (following the arrow).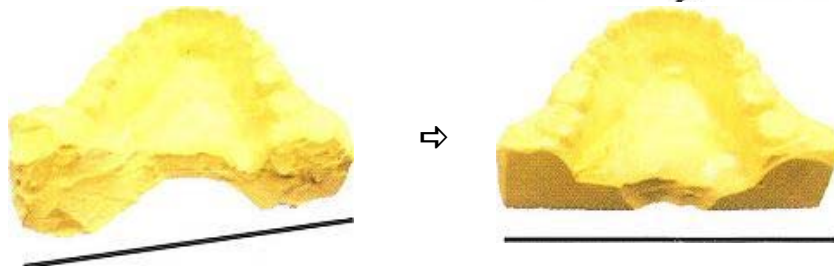

3. Connect and fix the upper and lower plaster cast on right position with using dental wax.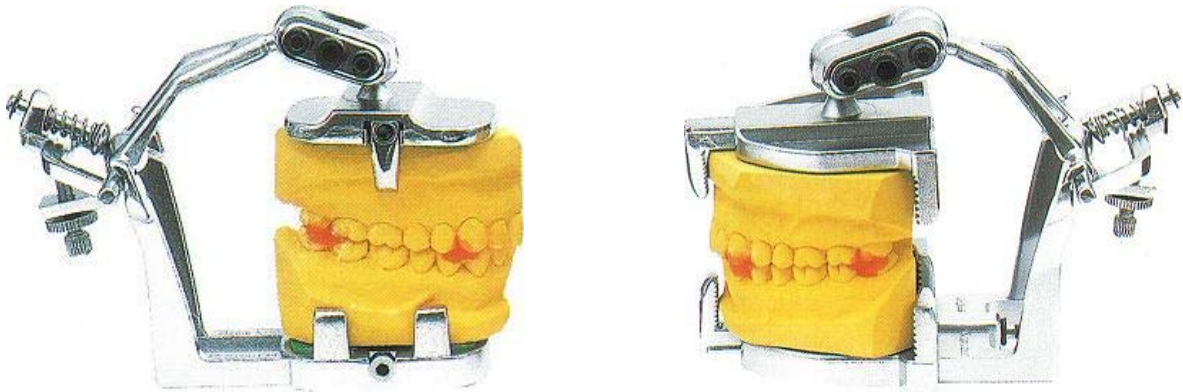

4. Please hold the ball holder carefully and screw on the ball lightly by n. 14 bolts with using hand or screw driver. And then tighten the ball across by n. 15 bolts with using screw driver. Do not contact the end of ball holder to the neck of the ball.

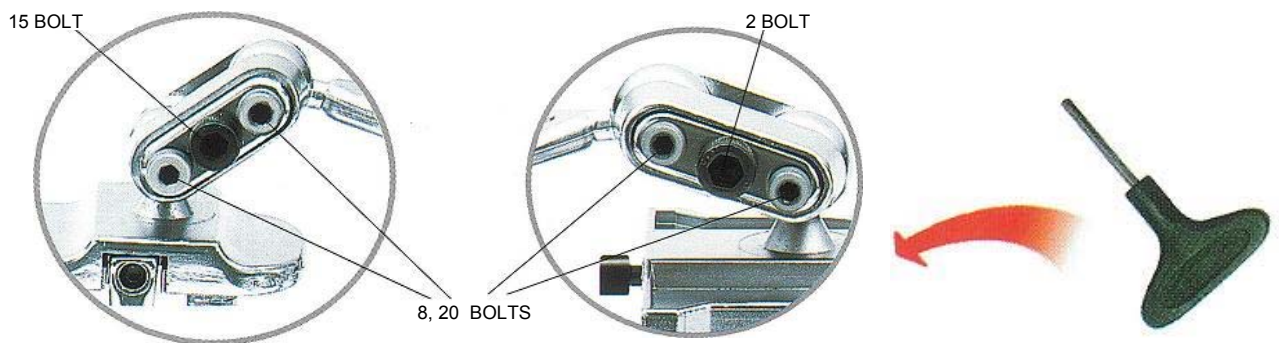5. In case of coming loose between the ball and ball holder, please polish the surface of the ball and the inside of the ball holder by sand blasting.

## CAUTION

- ◆ Do not drop and not force excessively the articulator in order to prevent losing bolts.
- ◆ Do not force excessively when fix plaster cast by screw.
- ◆ Do not use too big, small and weak plaster cast.
- ◆ Keep the articulator free from moisture to protect screws of articulator from corrosion.

**MESTRA<sup>®</sup>**

**TALLERES MESTRAITUA S.L.**

Txori-Erri Etorbidea, 60

Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25

E-mail: [mestra@mestra.es](mailto:mestra@mestra.es) - [www.mestra.es](http://www.mestra.es)

**48150 SONDIKA - BILBAO - ESPAÑA**