Lava  
POLIMERIZADORAS

LAVA-POLIMERIZADORA E2 A. INOX.  
**E2 WASHING AND POLYMERIZING MACHINE INOX.**  
 LAVEUSE POLYMERISEUSE INOX

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 **This machine is easy and safe to use, and saves time as all its processes are automatic. It polymerises 12 flasks. It efficiently softens and eliminates wax from models. It avoids unnecessary manipulation during scalding. Electricity and water consumption is low (closed circuits), and maintenance is simple.**

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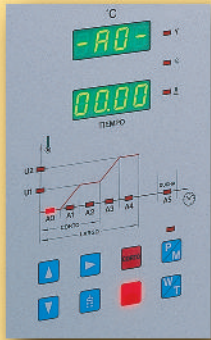
**Frame made entirely of stainless steel.**

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**ERGONOMICS**

**Twin display screen: one for temperature, the other for time. Graphic showing shower temperature increase and holding, with indication of what process is ongoing.**

**EASY TO HANDLE**

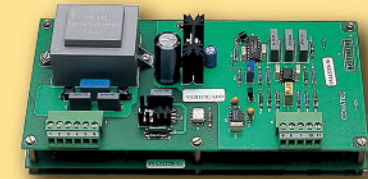
- ▶ **SEARCH KEY:** press to enter programming mode. The setting to be changed on the graph flashes.
- ▲ **UP KEY:** press to increase the time or temperature setting flashing on the graphic.
- ▼ **DOWN KEY:** press to decrease the time or temperature setting flashing on the graphic.
- 🔍 **ESCAPE KEY:** press to exit programming mode and memorize settings.
- 🔍 **STOP/START KEY:** hold down for 5 seconds to turn the washing & polymerising machine on and off.
- 🚿 **SHOWER KEY:** press to start the shower for the programmed time.
- S** **SHORT PROGRAM KEY**
- L** **LONG PROGRAM KEY**

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**LATEST GENERATION MICROPROCESSORS**

- Time & temperature control.
- Shower time & temperature control.
- Buzzer to indicate end of program.
- Last program memory.
- Automatic disconnection at end of long program.
- Heating control by triac.
- Thermocouple breakage indicator.
- Protection against overheating during shower process (cavitation effect).
- Pump protection on cold start-up.
- Protection against incorrect resets after power cuts (during shower process).
- Protection against voltage variations.
- Protection against electromagnetic emissions.
- Protection against electrical power cut (last figure is memorised).

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**Fast spray tree assembly & disassembly system.**

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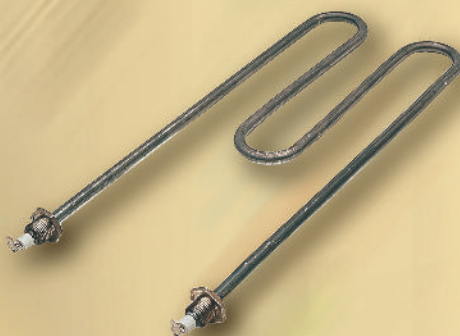
**Large compartment with room for 6 clamps with 2 flasks each.**

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**High-quality, oversized heating element.**

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**The washing system with 6 high-pressure water injectors ensures proper scalding of flasks and perfect wax removal.**

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**High-power motor pump of proven reliability, protected by heat relay.**

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#### FEATURES

Using the microprocessor the user can set:

- short or long program.
- delayed start time in hours and minutes (00.00 to 99.59)
- how long the machine takes to reach a temperature, or temperature hold time up to 99 hours 59 minutes
- temperature to which water is to be heated (max: boiling)
- shower time.
- when the program ends the microprocessor buzzes for one minute and then disconnects: all display screens and pilot lamps flash.

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**Completely made of stainless steel.**  
**Together with the Washing and Polymerising Machine R-080060,**

**they make a practical, aesthetic and unchanging combination in dental labs. It has three inner shelves in order to keep adequately flasks, clamps and any other material used in polymerising processes.**

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**The biggest problem in laboratories using polymerisation techniques is the condensation and water vapour produced by boiling water. High levels of humidity are harmful for health and can damage objects. MESTRA has reduced this problem by providing this polymeriser with a precise control which enables temperatures to be set in tenths of**

**a degree. Water can thus be heated to 98.5° without boiling it, and therefore without affecting the quality of polymerisation.**

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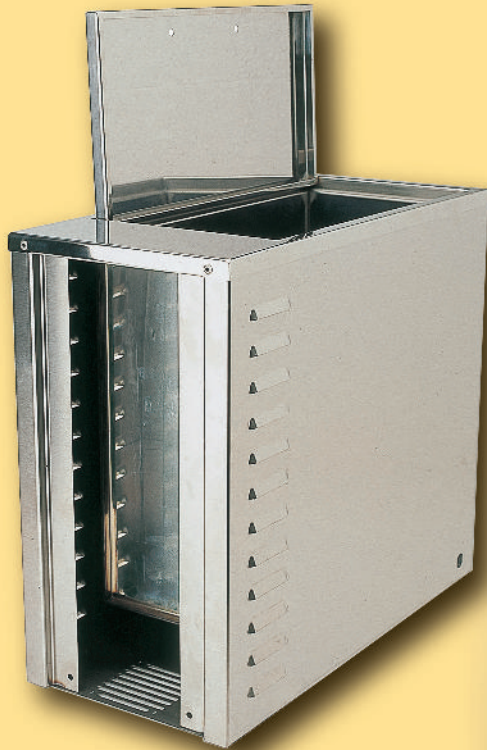
**It is exactly the same as the R-080400 except for its size that can hold up to 18 flasks.**

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Frame made entirely of stainless steel.

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#### EASY TO HANDLE

- ▶ Search key.
- ▲ Up key.
- ▼ Down key.
- ☒ Escape key.
- ☒ Stop/ start key.

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#### FEATURES:

- a) Delayed start time from 99h. 59 min.
- b) Time taken to reach temperature or holding time from 00.00 to 99 h. 59 m.
- c) Water temperature (max: boiling).
- d) Buzzer and automatic disconnection.

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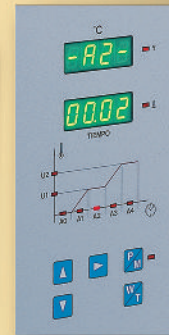
#### Room for 3 clamps with three flasks each.

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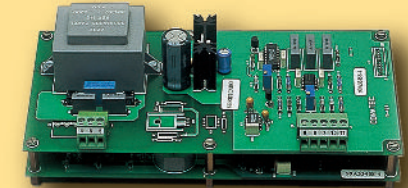
#### MICROPROCESSOR

- Time & temperature control.
- Buzzer to indicate end of program.
- Last program memory.
- Automatic disconnection.
- Static relay for heating control
- Thermocouple breakage indicator.
- Protection against voltage variations
- Protection against electrical power cut.

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