
R-080516

ES	MEZCLADORA DE VACÍO IRIS 2.....	5
EN	IRIS 2 VACUUM MIXER.....	7
FR	MALAXEUR SOUS VIDE IRIS 2	9
DE	IRIS 2-VAKUUMMISCHGERÄT	11
IT	MESCOLATORE SOTTO VUOTO IRIS 2.....	13
PO.....	BATEDORA DE VÁCUO IRIS 2.....	15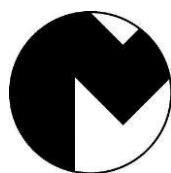

MESTRA®

TALLERES MESTRAITUA S.L.


Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 21/04/16

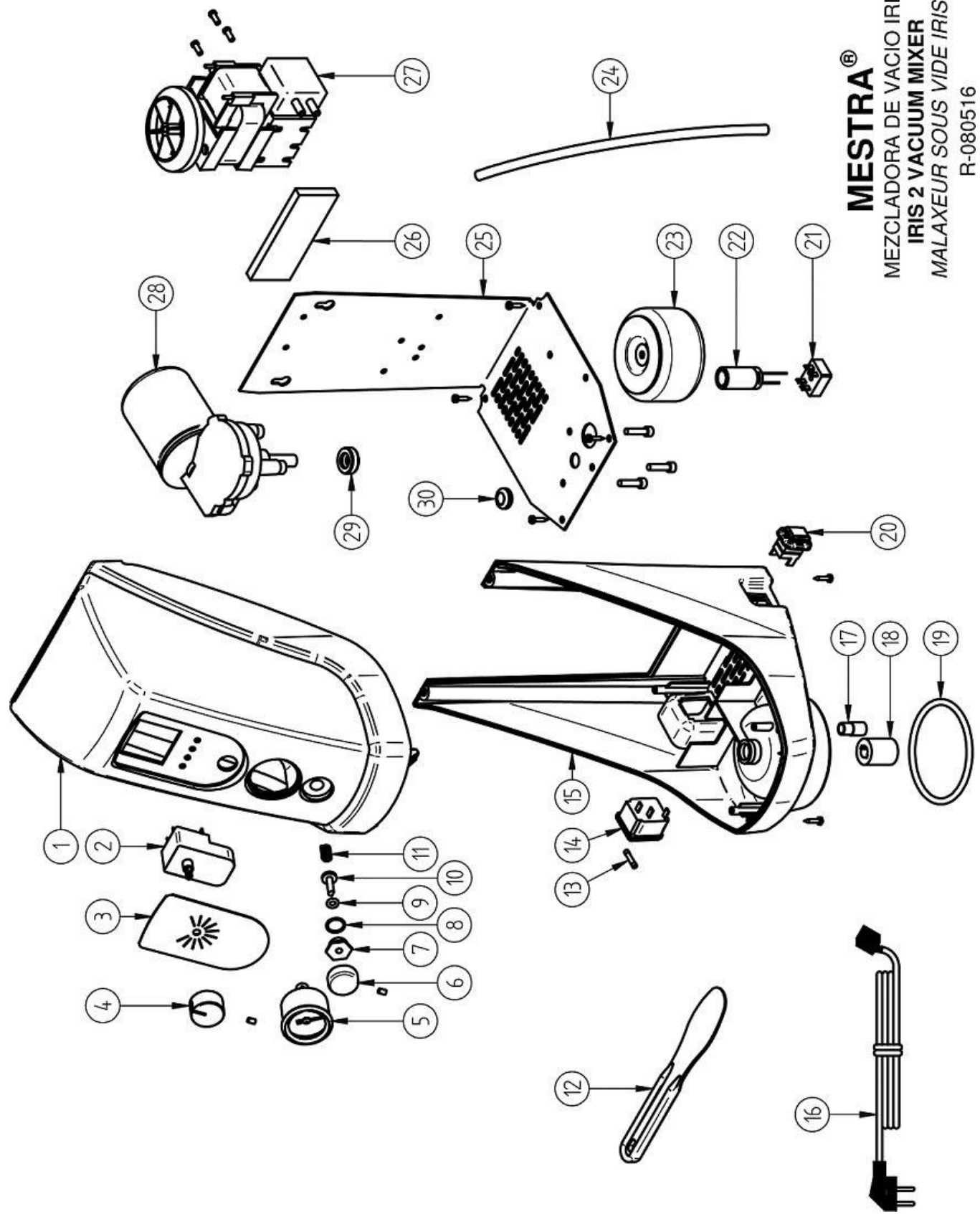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The IRIS 2 vacuum mixer (R-080516) is used to make investmets, plaster and similar mixtures. The machine has been designed as a simple and reliable tool for Dental Labs. For optimum performance and a long service life, please read the instructions for use carefully.

INSTALLING THE UNIT

1. Carefully unpack your IRIS vacuum mixer, and check for these fittings in the packaging:
 - Power cable.
 - Spare filter.
 - 550 cc. mixing bowl (D).
 - Kit to secure the mixer to the wall.
 - Plastic spatula for plaster-alginate (R-060002).
2. Choose a wall well away from any sources of heat, vibrations or moisture to set up the mixer unit. It must also be positioned on a firm, rigid, level surface.
3. Make two marks with a marker pen exactly 80 mm apart at the chosen location. Ensure that the imaginary line linking the two marks is perfectly horizontal. A level may be used if required.
4. Drill out the holes marked by hand using a Ø 6 mm widia bur.
5. Insert the nylon studs included in the vacuum mixer kit in the drill holes. Then screw in the cheese-head shed bolts, allowing the heads to protrude around 11 mm.
6. Use the bolts to secure the IRIS vacuum mixer firmly to the wall.
7. Use the pen to mark the position of the fixing eye on the lower section of the unit. Remove the mixer and drill out the hole manually. Insert a nylon stud in the orifice and put the unit back on.
8. Put the bolt through the lower hole. Ensure the unit has been firmly secured.
9. Hook up the mixer's power cable to an proper power supply with earthing.

OPERATING THE MIXER

The mixer is now ready for use:

1. Use the main switch (E) to turn on the machine.
2. Put the mixing ingredients into the bowl (D). **Pre-mix the products in the bowl manually with a spatula.**
3. Use gentle pressure to position the bowl and the mixer connection (F) with a good sealing, and then select the mixing time in seconds by turning the timer control (A) clockwise. The unit will then start up.
4. Use the vacuum meter (B) to check the vacuum in the bowl. If there is no vacuum in the bowl, move it gently to improve its seating. Keep hold of the bowl until the meter needle reaches yellow. You may now release the bowl.
5. During mixing operations, the needle must remain in the "VACUUM" section. When the pre-set time has elapsed, the unit will stop.
6. Use the vacuum drain valve (C) to remove the bowl from the unit. The mix is now ready to be used.

A cycle may be cancelled during operation by moving the timer control anticlockwise to "0".

IMPORTANT

This machine has a "diaphragm" vacuum pump (maintenance-free). If the unit is started at very low room temperature, the membrane rubber may not be sufficiently elastic for best operation. Thus we recommend that the mixer be run for approximately 1 minute before starting operations in cold environments.

MAINTENANCE AND CLEANING

For best reliability and a long service life, we recommend the following:

- Clean the mixing bowl carefully each time it is used. This will prevent dirt

building up and seizure of the blade shaft. Make sure the bowl seat and joint are cleaned for a complete sealing.

- Keep the unit and the surroundings clean. Do not use solvents or any other inflammable products.
- The filter inside the bowl connection (F) must be replaced periodically. Unscrew the filter-silencer and replace it with a new one. You can use a 11 mm box spanner.

- We recommend using always original MESTRA spare parts for mixer maintenance.

PRECAUTIONARY MEASURES

- ◆ Before connecting up the unit, check the power supply is adequate and with earthing.
- ◆ Do not fit the bowl to the unit when the mixer motor is on. Make sure the blades-bowl set is properly assembled.
- ◆ Do not let children or unqualified staff handle the unit.
- ◆ Choose a solid well-leveled wall to place the mixer, and make sure that fixing bolts are firmly screwed into the wall.
- ◆ Wait for the motor to stop completely after finishing a cycle before removing the bowl.
- ◆ Disconnect the unit from the power supply during cleaning operations.

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TALLERES MESTRAITUA S.L. is a company manufacturing tools and appliances for the dental prosthesis sector, from the business address shown at the bottom of the page.

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I HEREBY STATE: That the **“IRIS 2” VACUUM MIXER R-080516**, with the following characteristics:

Alto / Height:	350 mm
Ancho / Width:	155 mm
Fondo / Depth:	205 mm
Peso / Weight:	4,8 kg
Tensión / Supply voltage:	230 V, 50/60 Hz 115 V, 50/60 Hz
Potencia / Power:	100 W
RPM / RPM:	240 rpm
Rango de tiempo / Timer:	0 ~ 120 ”

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Meets the low voltage requirements set out in Directive 2014/35/UE, in Directive 2014/30/UE concerning electromagnetic compatibility and in Directive 2006/42/EC concerning machinery safety.

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In witness whereof, the company issues this declaration of conformity in Sondika, on 09 July 2019.

Firma / Signed:

Ignacio Mestraitua,
Director Gerente / *Managing Director*

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA