
R-100007

E.....	ESPÁTULA ELECTRONICA PARA MODELAR CERA.....	4
GB.....	ELECTRONIC SPATULA FOR WAX MODELLING.....	6
D.....	ELEKTRONISCHES WACHSMESSER.....	8
P.....	ESPÁTULA ELECTRÓNICA PARA MODELAR CERA.....	10
I.....	SPATOLA ELETTRONICA PER MODELLARE LA CERA.....	12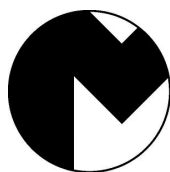

MESTRA®

TALLERES MESTRAITUA S.L.


Txori-Erri Etorbidea, 60

48150 SONDIKA - BILBAO - ESPAÑA

Tfno. + 34 944530388 - Fax + 34 944711725

E-mail: mestra@mestra.es - www.mestra.es

Rev. 14/03/12

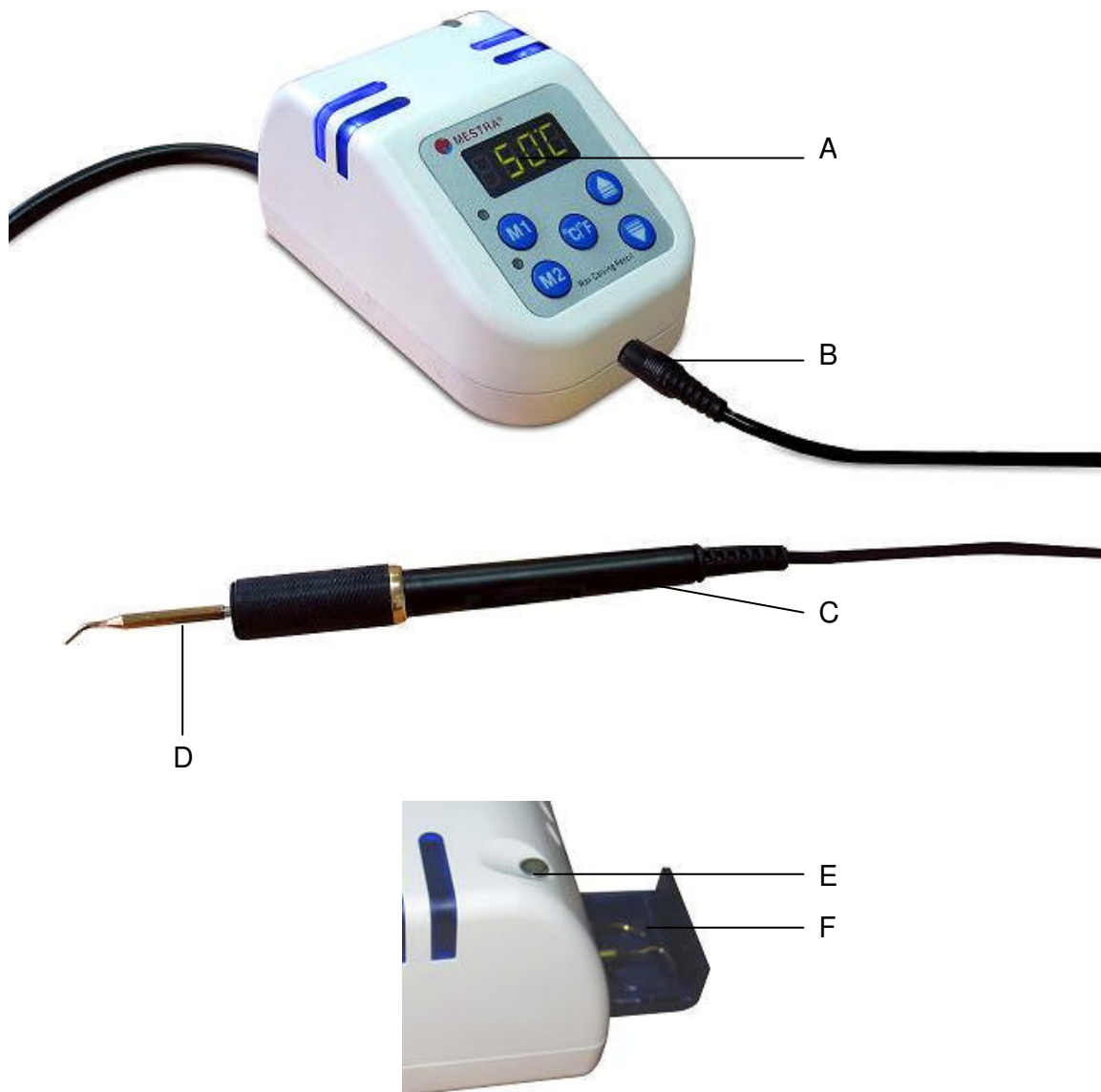A	Display / Display / <i>Display</i> / Display / Display
B	Orificio de conexión / Connection hole / <i>Anschlussöffnung</i> / Tomada de ligação / Entrata collegamento
C	Mango / Handle / <i>Handstück</i> / Punho / Manico
D	Punta / Point / <i>Spitze</i> / Ponta / Punta
E	Soporte magnético / Magnetic holder / <i>Magnethalter</i> / Suporte magnético / Supporto magnetico
F	Cajón para puntas / Case for points / <i>Schublade für Spitzen</i> / Gaveta para pontas / Cassetto per punte

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- Carefully read this manual. It contains important information on safety during the installation, operation and maintenance of the unit.
- Keep this manual in a suitable place to consult it in the future.
- Check the overall state of the unit just upon unpacking it.
- Before connecting the unit, make sure the voltage shown in the identification plate matches that of the power source to be used. The installation must fulfil all the applicable safety standards and the instructions of the manufacturer.
- The manufacturer will not be responsible of the damage caused by an incorrect ground connection. The damage caused by a faulty installation or usage of the unit will not be covered by the warranty.
- In case of doubt, do not start the unit. Contact a qualified and authorised technician.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Do not touch the spatula heads while the unit is in operation, since they reach high temperatures.
- After turning the main switch off, wait a time before touching the spatula heads.

- The machine features a precision circuit. Prevent fortuitous impacts and stresses that could cause mechanical damage.

- Remove any dirt or water remains from the cord connection.
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not insert plastic objects into the bowls or any other material that melts at low temperatures.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents,...).

- Do not allow children or non-qualified personnel handling the unit.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.

- Contact the supplier for questions or detailed information about problems with the unit.

INTRODUCTION

The R-10007 is an electronic spatula for wax modelling specially designed to be used by professionals in the field of dental prosthesis. Its ergonomics, ease of use, and small size make it an optimal tool for this kind of work. The accuracy of the electronic controls is also remarkable, since it maintains the temperature in the spatula head within a narrow error margin. The working temperature is shown on the digital display in the controls box. The device is factory supplied with two different heads.

INSTALLATION

- Place the unit on a flat, horizontal and stiff surface, away from vibration or heat sources. Make sure there is not any flammable material nearby.
- Insert the jacks of the handles into the OUT1 and OUT2 outlets.
- Connect the unit to a grounded 230 V, 50/60 Hz power source.

OPERATION

- Insert the handle jack (C) in the connection hole (B).
- Select one of the interchangeable heads and fix it to the handle.
- Turn on the main switch in the right side of the unit.
- Press the (°C/°F) to shift to Celsius or Fahrenheit degrees.
- Press the (▼▲) keys to select the temperature you want the wax to reach.
- Press M1 or M2 to select the temperatures stored in the memory
- To change the M1 and M2 temperatures, select the temperature you wish and press M1 or M2, and keep it pressed for a while. The new temperature will be stored in the memory.
- While you are not using the spatula, lean it on the magnetic holder (E).
- After finishing the operation, disconnect the unit by turning the main switch off, and handle it cautiously for several minutes until it is cooled down completely.
- You can store the heads in the head box (F).

MAINTENANCE AND CLEANING

To ensure a long life of the unit and an optimal output, we strongly recommend:

- To clean the wax remains from the unit periodically. To do this you can use a small spatula. Make sure the unit has been previously disconnected from the power source.
- Do not spill melted wax over the outer casing of the unit. Handle the spatula with caution.

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