Ref. 080077

ES.....	2	Manual original
EN.....	4	
FR.....	6	

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Ignacio Mestraitua
Director Gerente
Sondika, 15.10.2019

NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

Meaning of the symbols on the unit

Danger. Hot surface.

Read completely these instructions before operating the unit.

Precautions

Danger. The device surface may be very hot. Be careful when touching, also to select the temperature or pressing the ON/OFF button.

- Keep this manual in a suitable place to consult it in the future.
- Check the overall state of the unit just upon unpacking it.
- The damage caused by a faulty installation or usage of the unit will not be covered by the warranty.
- The risk of electric shock, this equipment must only be connected to a properly grounded power source. Before connecting the appliance, check that the identification label indicates the correct voltage. The installation must comply with all applicable safety standards.
- Remove any dirt or water remains from the cord connection.
- Do not pull, twist or fold the power cord excessively.
- Do not touch the unit or the power cord with wet hands.
- Place the unit in a dry place, away from dust, water splashes and flammable materials (alcohol, solvents, etc.).
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not allow children or non-qualified personnel handling the unit.
- Do not use the unit if any doubts. Contact the supplier for questions or detailed information about problems with the unit.
- Do not dismount, repair or modify the unit.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Unplug the unit before cleaning, do not use chemical products.
- Contact the supplier for questions or detailed information about problems with the unit.

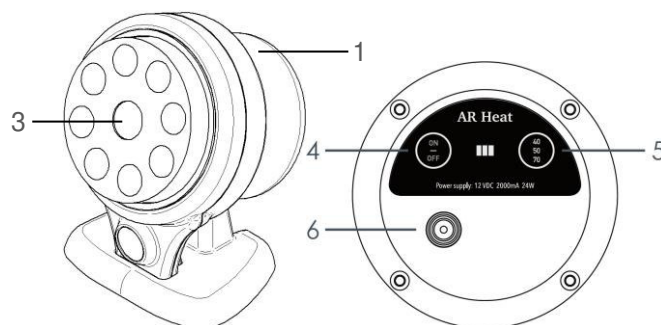
- Before connecting the device, make sure it is placed on a firm and solid surface.
- The cord and transformer may only be replaced by one with the same characteristics as supplied by the manufacturer.
- Use the unit only for its intended use.

Intended use

This heater has been designed for heating composite resin material, improve the flow of its adhesive properties; At the same time, it can also be used to heat adhesive when used manual equipment.

According to your choice of material and personal preference temperature can be set up to: 40, 50 and 70 °C.

Unit parts

- 1 – Heating main body.
- 2 – Power adaptor.
- 3 – Temperature indicator light.
- 4 – Turn on/off button.
- 5 – Temperature selection button.
- 6 – Power DC socket.

Installation

Environmental conditions:

- Use indoor only.
- Environmental temperature: 5-40 °C [41-104 °F]
- Max. relative humidity: 0-95 %

1. Carefully unpack the unit. Make sure the unit has not suffered any damage during the transport.
2. Place the unit on a flat and firm surface, away from any heat or vibration source.
3. Connect the unit to a proper and grounded power source. Then, the three temperature indicators will light at the same time

Operation

1. Press ON/OFF button. Only one indicator will light.
2. Press the temperature selection button:

Temperature	Indicator light color
40 °C	White
50 °C	Green
70 °C	Red

The indicator light will blink until reaching the temperature.

 **3 indicator lights blinking together means temperature sensor failure.**

 **Danger. The device surface may be very hot. Be careful when touching, also to select the temperature or pressing the ON/OFF button.**

3. Insert the composite material of syringe into a slot in the heating body.

Maintenance and cleaning

To ensure a correct, reliable and durable performance of the unit we strongly recommend:

- Do not use alcohol or any other solvent to clean the unit.
- Never immerse the unit in a liquid.
- If you notice unusual noises or vibrations, send the unit to a technical assistance service.
- Use only original Mestra spare parts.
- Prevent material leakage into the heating body.
- Keep the unit clean.
- All the parts can be cleaned with disinfectant dish-cloth.

- Don't put any parts immersed in any liquid cleaning and disinfection,
- Don't disinfect under high temperature and high pressure.
- 3 indicator lights blinking together means temperature sensor failure.

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Meets the requirements set out in the following directives:

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