## R-080125

E .....	EXTRACTOR DE HUMOS.....	2
GB .....	SMOKE EXTRACTOR.....	3
F .....	EXTRACTEUR DE FUMÉES.....	4
P .....	EXTRACTOR DE FUMOS .....	5
I .....	ESTRATTORE FUMI.....	6

**MESTRA®**

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25

E-mail: [mestra@mestra.es](mailto:mestra@mestra.es) - [www.mestra.es](http://www.mestra.es)

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 05/10/09

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The Smoke Extractor is an interesting attachment for the HP Preheating Furnaces, although it may also be fitted to other manufacturers' furnaces. The device removes furnace gases produced in the muffle continuously and smoothly.

### FURNACE INSTALLATION

1. Choose a well ventilated location in the lab for the Furnace-Extractor. Remember that the Extractor sweeps away smoke generated inside the muffle, but does not purify it. The ideal location would be under an extractor hood with an exterior outlet.
2. When coupling the Extractor to one of the MESTRA HP furnaces, insert the Extractor inlet into the flue tube at the rear of the Furnace, and then tighten the fastening screw on the Extractor inlet.
3. If the Extractor is connected to a non-MESTRA Furnace, first check that the diameter of the Furnace flue tube does not exceed 38 mm, and then proceed as in the previous point. If there is a great difference between the Extractor inlet and the Furnace flue tube, a better fit can be obtained by rolling up a piece of textile thermal insulation on the Furnace flue tube (1200 °C minimum temperature).
4. The smoke Extractor has a hose for water drainage, which could appear inside the Extractor due to condensation. Place a collection receptacle near the Extractor, and position the end of the hose inside.
5. Mestra HP-Furnaces have a socket on the left-hand side of each Furnace. If the extractor is connected to this socket, the extractor runs from 75 °C until the furnace reaches 850 °C, or the second heating ramp begins. For continuous operation, the extractor must be hooked up to a separate connection.

### MAINTENANCE AND CLEANING

1. Empty the receptacle for condensed liquids on a regular basis.
2. Clean the whole Extractor periodically using a wet cloth and soapy water. Use a disposable cellulose sheet to clean wax or charcoal inside the Extractor.
3. If the drainage hose is blocked, clean it out using a cylindrical object of suitable diameter with a blunt edge.

### PRECAUTIONS

- ◆ Working temperature could be high, and so you must not touch the unit if you are not sure that it is cold.
- ◆ If the Extractor is connected to a socket other than the Furnace socket, make sure this is earthed.
- ◆ During cleaning and maintenance operations, disconnect the Extractor from the mains.
- ◆ The Extractor does not have a fuse or a mains switch, and this should be borne in mind when connecting it to a switch other than on the Mestra Furnace.
- ◆ Make sure the Extractor outlet is placed at a well-ventilated location, since gases produced inside the muffle can be toxic. Never inhale these gases.
- ◆ Do not place any object near the Extractor air inlet located on the lower section. Leave at least 15 cm between this inlet and the nearest object.

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