## R-080518

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The Iris 2 Advanced vacuum mixer (R-080518) is used to make investments, plaster and similar mixtures. The machine has been designed as a simple and reliable tool for Dental Labs.

For optimum performance and a long service life, please read the instructions for use carefully.

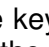
- |  |  |
|--|--|
|  | <ul style="list-style-type: none"> <li>- Read these instructions carefully before operating the unit. They contain very important information about safety during the installation, operation and maintenance of the unit.</li> <li>- Keep these instructions for further consultation.</li> <li>- Check out the state of the unit upon unpacking it.</li> <li>- Before connecting the device, make sure the voltage in the identification plate matches the power source to be used. During the installation all the safety requirements must be observed and the instructions of the manufacturer followed.</li> <li>- The manufacturer will not be liable for the damage caused by an inappropriate grounding system.</li> <li>- Damage caused by a faulty installation or a wrong use of the unit is not covered by the warranty.</li> <li>- When in doubt, do not start the unit. Contact the supplier for questions or detailed information about problems with the unit.</li> </ul> |
|  | <ul style="list-style-type: none"> <li>- Choose a solid well-leveled wall to place the mixer, and make sure that fixing bolts are firmly screwed into the wall.</li> <li>- Do not fit the bowl to the unit when the mixer motor is on. Make sure the blades-bowl set is properly assembled.</li> <li>- Wait for the motor to stop completely after finishing a cycle before removing the bowl.</li> </ul>  |
|  | <ul style="list-style-type: none"> <li>- Do not touch the unit or the power cord with wet hands.</li> <li>- Unplug the unit in case of storm and when you are not going to use it for a long time.</li> <li>- Place the unit in a dry place, away from dust and water splashes.</li> <li>- Do not pull, twist or fold the power cord excessively.</li> <li>- Do not wet the unit; do not use chemical products to clean it.</li> </ul>   |
|  | <ul style="list-style-type: none"> <li>- <b>WARNING:</b> to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).</li> </ul>   |
|  | <ul style="list-style-type: none"> <li>- Do not allow children or non-qualified personnel handling the unit.</li> <li>- Do not dismount, repair or modify the unit.</li> <li>- Do not ever place or remove the tanks while the unit is in operation.</li> </ul>  |
|  | <ul style="list-style-type: none"> <li>- Contact the supplier for questions or detailed information about problems with the unit.</li> </ul>   |

### INSTALLING THE UNIT

1. Carefully unpack your Iris 2 Advanced vacuum mixer, and check for these fittings in the packaging:
    - Power cable.
    - Spare filter.
    - 550 cc. mixing bowl (D).
    - Kit to secure the mixer to the wall.
    - Plastic spatula for plaster-alginate (R-060002).
  2. Choose a wall well away from any sources of heat, vibrations or moisture to set up the mixer unit. It must also be positioned on a firm, rigid, level surface.
  3. Make two marks with a marker pen exactly 80 mm. apart at the chosen location. Ensure that the imaginary line linking the two marks is perfectly horizontal. A level may be used if required.
  4. Drill out the holes marked by hand using a Ø 6 mm widia bur.
  5. Insert the nylon studs included in the vacuum mixer kit in the drill holes. Then screw in the cheese-head shed bolts, allowing the heads to protrude around 12 mm.
  6. Use the bolts to secure the IRIS vacuum mixer firmly to the wall.
  7. Use the pen to mark the position of the fixing eye on the lower section of the unit. Remove the mixer and drill out the hole manually. Insert a nylon stud in the orifice and put the unit back on.
  8. Use a wrench to put the bolt through the lower hole. Ensure the unit has been firmly secured.
1. Hook up the mixer's power cable to 230 V, 50/60 Hz with earthing.

### FUNCTIONS

- The Iris 2 Advanced vacuum mixer (R-080518) has several possible functions that you can select or not.
- In order to go to the function selection mode, press the key  for some seconds.
- You will see that the icon of the first function lights.

- Shortly pressing the key  you will advance on the different functions.
- With the keys **▲▼**, at upper display, select the value "YES" for activate or "NO" for deactivate it.
- If you select "YES", shortly press the key  to go to the lower display and with the keys **▲▼** you will be able to modify the parameters for each function. A dot at the rightside shows the active display.

#### PRE-VACUUM

The vacuum pump works but blades will not turn.

At the lower display you can chose the time in seconds for the *pre-vacuum*.

#### PRE-MIXING

A first show mixing is obtained. You can chose 3 different types of *pre-mixing*:

1. 5 seconds at 100 rpm.
2. 2 seconds at 100 rpm clockwise and 5 seconds at 250 rpm anticlockwise.
3. 2 seconds at 100 rpm clockwise and 2 anticlockwise (twice).

#### MIXING

Blades will mix at speed selected by the user.

Once out of the function selection mode, you can select mixing time and speed.

If you select "YES", you will activate *spin inversion*.

#### SPIN INVERSION

While the mixing, the motor alternatively turns clockwise and anticlockwise for 10 seconds (not editable value).

#### TURBO

Last seconds of the mixing will be at 400 rpm.

At the lower display you can select the *turbo* time (5 - 30 seconds).

#### POST-VACUUM

The pump continues working for some seconds after the mixing.

At the lower display you can select the *post-vacuum* time in seconds.

#### ALARM

If the *post-vacuum* function is activated, when finish, the alarm will automatically sound. Press any key to stop.

## OPERATING THE MIXER

The mixer is now ready for use:

1. Use the main switch (E) to turn on the machine.
2. The upper display will show the spin speed in rpm and the lower display will show the working time in seconds.
3. You can modify both of them by pressing the **▲▼** keys. The speed can be modified even when the unit is working.
4. To shift from a display to the other press the **↔** key.
5. At the right of the display, you will see lighting the icons of the functions that are going to work.
6. Put the mixing ingredients into the bowl (D). **Pre-mix the products in the bowl** manually with a spatula or by activating the function **✂ PRE-MIXING**.
7. Use gentle pressure to position the bowl and the mixer connection (F) with a good sealing.
8. Press the **START/STOP** **⊙** key for the unit to start a cycle.
9. Upper display will show the blades speed.
10. Lower display will show the remaining time to finish the work. Notice that the rest of the functions will be added to the *mixing* time.
11. Use the vacuum meter (B) to check the vacuum in the bowl. If there is no vacuum in the bowl, move it gently to improve its seating. Keep hold of the bowl until the meter needle moves and the cup keeps holded. You may now release the bowl.
12. During mixing operations, the needle must remain in the "VACUUM" section. When the pre-set time has elapsed, the unit will stop.
13. At the right of the display, the icon of the function running will blink. Already done functions will keep lighting.
14. Use the vacuum drain valve (C) to remove the bowl from the unit. The mix is now ready to be used.
15. To stop the operation of the unit at any moment keep the **START/STOP** **⊙** key pressed.

### IMPORTANT

*This machine has a "diaphragm" vacuum pump (free of maintenance). If the unit is started at very low room temperature, the membrane rubber may not be sufficiently elastic for best operation. Thus we recommend that the mixer be run for approximately 1 minute before starting operations in cold environments.*

## SAVED CONFIGURATIONS

The Iris 2 Advanced has the following configurations saved in memory:

P1	- Mixing with spin inversion.
P2	- Pre-mixing type 1. - Mixing with spin inversion. - Turbo for 5 seconds.
P3	- Pre-vacuum for 5 seconds. - Pre-mixing type 1. - Mixing with spin inversion. - Turbo for 5 seconds.
P4	- Pre-vacuum for 5 seconds. - Pre-mixing type 1. - Mixing with spin inversion. - Turbo for 5 seconds. - Post-vacuum for 5 seconds.

## USING THE DOSAGES STORED IN THE MEMORY

- To use a configuration stored in the memory, press the **P** key as many times as required to select the program you want (P1, P2, P3 or P4).
- Press the key to display the values stored.
- Press the **START/STOP** **⊙** key and start working.

## NEW CONFIGURATION STORING

The electronic controls of the Iris 2 Advanced allow you to store up to 4 different configurations. To store a configuration, proceed as follows:

- First of all, select the configuration you want to store, as explained in previous sections.
- Once you have selected the values you want to store, select the program (P1, P2, P3 or P4) in which you want to store the configuration by pressing the **P** key as many times as required.
- Finally, keep the **P** key pressed for a few seconds. The display will flicker once to indicate that the values have been stored in the memory.

## MAINTENANCE AND CLEANING

For best reliability and a long service life, we recommend the following:

- Clean the mixing bowl carefully each time it is used. This will prevent dirt building up and seizure of the blade shaft. Make sure the bowl seat and joint are cleaned for a complete sealing.
- Keep the unit and the surroundings clean. Do not use solvents or any other inflammable products.
- The filter inside the bowl connection (F) must be replaced periodically. Unscrew the filter-silencer and replace it with a new one. You can use a 10 mm box spanner.

- We recommend using always original MESTRA spare parts for mixer maintenance.

## PRECAUTIONARY MEASURES

- ◆ Before connecting up the unit, check the power supply is 230 V, 50/60 Hz with earthing.
- ◆ Do not fit the bowl to the unit when the mixer motor is on. Make sure the blades-bowl set is properly assembled.
- ◆ Do not let children or unqualified staff handle the unit.
- ◆ Choose a solid well-leveled wall to place the mixer, and make sure that fixing bolts are firmly screwed into the wall.
- ◆ Wait for the motor to stop completely after finishing a cycle before removing the bowl.
- ◆ Disconnect the unit from the power supply during cleaning operations.

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