

INSTRUCCIONES

R-100131, R-100133, R-100134

INSTALACIÓN

- Colocación en pared:
 - Afloje los dos tornillos del lateral, coloque los ganchos y sujételos al aparato apretando los tornillos de nuevo.
 - Coloque dos tornillos o clavos en la pared. La distancia entre los dos tornillos es 32,5 (horizontal) o 12 cm (vertical)
 - Cuelgue el aparato.
- Colocación sobremesa:
 - Retire el soporte de la parte trasera.
 - Gire el soporte y encájelo en los enganches.

FUNCIONAMIENTO

Conecte el adaptador AC y accione el interruptor ON/OFF para encender el aparato.

LEER CON ATENCIÓN

- **NO** intente desmontar ni reparar el aparato ni tocar los circuitos eléctricos de su interior.
- **NO** deje el aparato en un lugar con alta humedad ni alta temperatura, ni lo exponga directamente a la luz del sol.
- **NO** derrame agua ni cualquier otro líquido sobre el aparato.
- Por favor, consulte las regulaciones locales para el tratamiento de residuos peligrosos cuando tire las lámparas.

CARACTERÍSTICAS

	R-100131	R-100133	R-100134
Medidas exteriores:	240 x 360 x 200 mm	520 x 400 x 23 mm	700 x 520 x 33 mm
Área de visión:	210 x 300 mm	420 x 300 mm	580 x 400 mm
Duración de las lámparas:	Aprox. 10000 horas		
Consumo:	19 W	18 W	30 W
Alimentación (adaptador externo):	AC 90V ~ 240V (12VDC, 1.0 A)	AC 90V ~ 240V (12VDC, 1.5 A)	AC 90 ~ 240V (12VDC, 2.0 A)
Peso neto:	1.7 kg	2.6 kg	4.5 kg

INSTRUCTIONS

R-100131, R-100133, R-100134

INSTALATION

- On wall:
 - Fasten the prop stand from the back.
 - Fasten them with nails or screws at the proper place on the wall. Distance between 2 nails is 32.5 (horizontal) or 12 cm (vertical)
 - Hang the light box accordingly.
- On desk:
 - Remove the prop stand from the back.
 - Reverse the stand and lock it into the hook until it secures in position.

HOW TO OPERATE

Please connect the AC adapter and then flip the ON/OFF slide switch to operate the unit.

PLEASE READ CAREFULLY

- **DO NOT** try to disassemble and repair the unit or touch any part of the electrical circuits inside.
- **DO NOT** leave unit viewer in high humidity and under hot temperature or expose it to direct sunshine.
- **DO NOT** spill water or any other liquid over the unit.
- Please consult local regulations concerning treatment of hazardous wastes when disposing of the lamps.

CHARACTERISTICS

	R-100131	R-100133	R-100134
External area:	240 x 360 x 200 mm	520 x 400 x 23 mm	700 x 520 x 33 mm
Viewing area:	210 x 300 mm	420 x 300 mm	580 x 400 mm
Life of lamps:	Approx. 10000 hours		
Power consumption:	19 W	18 W	30 W
Power supply (external switching adaptor):	AC 90V ~ 240V (12VDC, 1.0 A)	AC 90V ~ 240V (12VDC, 1.5 A)	AC 90 ~ 240V (12VDC, 2.0 A)
Net weight:	1.7 kg	2.6 kg	4.5 kg