


LIGHT BALANCED ARTICULATOR R-010090

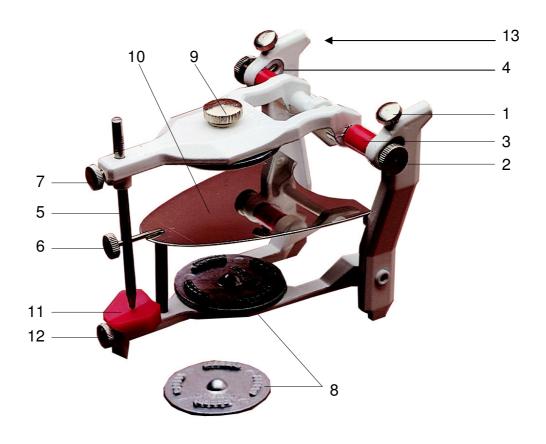

1.	010090-01	Condilar bolt
2.	010090-02	Condile prolongation bolt
3.	010090-03	Condiles
4.	010090-04	Spring
	080270-43	Spring shaft
5.	010090-05	Incisal guide
6.	010090-06	Mouth center reference point
7.	010090-01	Vertical dimensión bolt
8.	010090-16	Removable aluminum base
9.	010090-12	Aluminum base bolt
10.	010090-10	Occlusal plane
11.	010090-11	Plastic base
12.	010090-01	Base bolt
13.	010090-13	Stud

LIGHT BALANCED ARTICULATOR R-010090

- The MESTRA articulator reproduces mandible movements based on average records obtained from clinical diagnoses.
- The springs above the condyles allow the maxilla to be moved and then returned to its original position.
- The MESTRA articulator is equipped with removable aluminum bases, enabling it to work with several models at the same time facilitating pattern waxing.
- No movement is possible when the condylar screws are tight.
- The incisal guide has red painted notches as a guide to parallelism. The central notch indicates that the maxilla is parallel to the occlusal plane.
- The light weight of the alloys used means that the MESTRA articulator is easy to handle.