Ref. 080089

ES	DECANTADORA DE ALTA CAPACIDAD	5
EN	HIGH CAPACITY DECANTER.....	6
FR	BAC DE DECANTATION GRANDE CAPACITE	7
PO	DECANTADORA DE ALTA CAPACIDADE	8
IT	DECANTATORE AD ALTA CAPACITÀ.....	9

MESTRA®


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

+34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

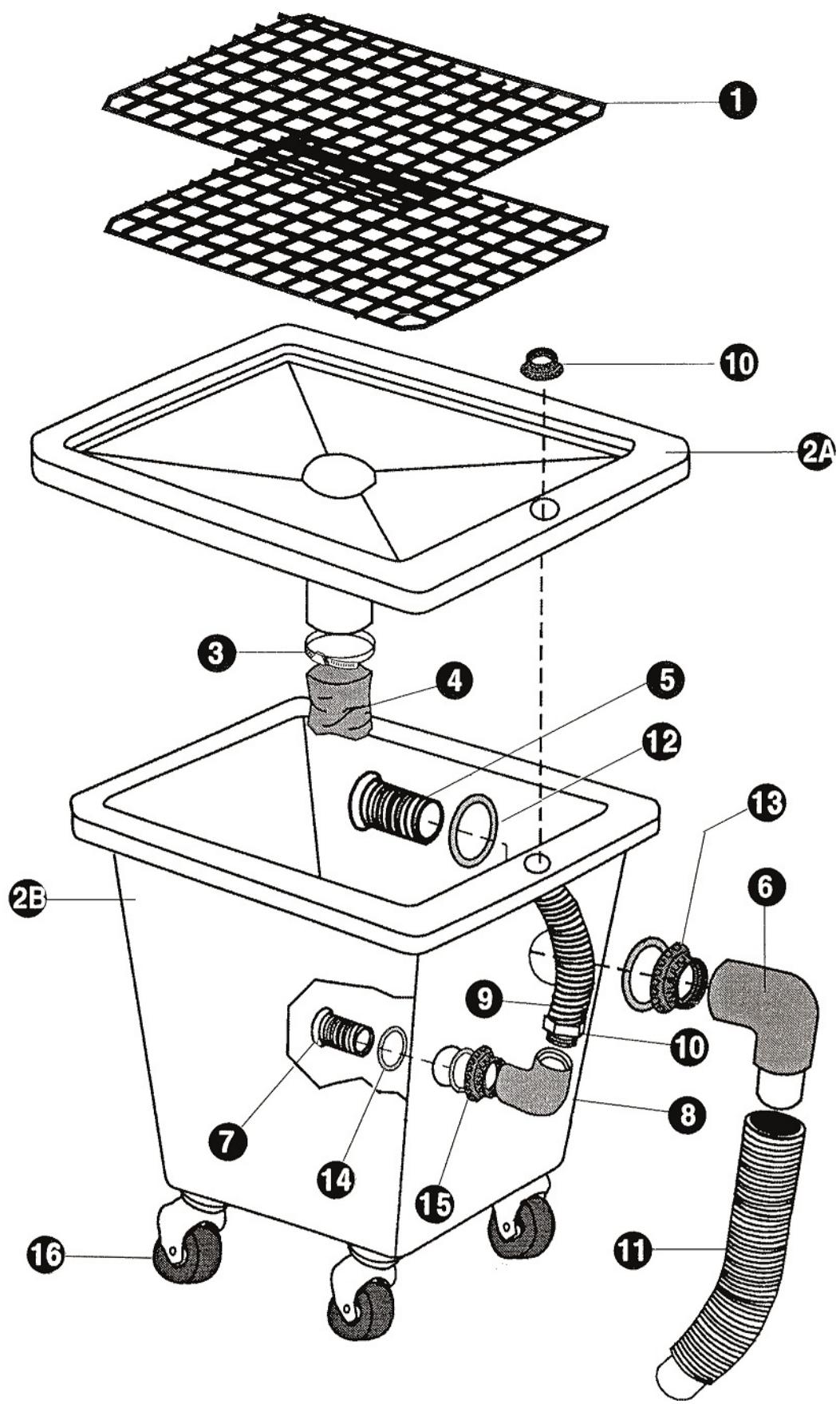Nº	Ref.	Descripción	Description
1	080089-01	Rejillas de decantadora	Decanter grilles
2A	080089-11	Tapa	Lid
2B	080089-12	Cuerpo	Body
3	080089-03	Abrazadera 80-100	Hose clamp 80-100
4	080089-04	Manguito	Sleeves
5	080089-05	Pasamuros grande	Large bushing
6	080089-06	Codo grande	Large elbow
7	080089-07	Pasamuros pequeño	Small bushing
8	080089-08	Codo pequeño	Small elbow
9	080089-09	Tubo pequeño	Small tube
10	080086-03	Manguito con tuerca	Bush with nut
11	080089-10	Tubo grande	Large pipe
12	080089-14	Junta de pasamuros grande	Large bushing O-ring
13	080089-15	Tuerca de pasamuros grande	Large bushing nut
14	080089-17	Junta racor pasamuros pequeño	Small bushing connector O-ring
15	080089-18	Tuerca racor pasamuros pequeño	Small bushing connector nut
16	080086-06	Tuerca de manguito decantadoras	Decanter bush nut
17	080086-07	Manguito decantadoras	Decanter bush

- | | |
|---|--|
| A | Rejilla / Screen / Grille / Grelha / Griglia |
| B | Fijación tubo / Tube fixation / Fixation du tube / Fixação tubo / Fissaggio tubo |
| C | Tubo vaciado / Draining pipe / Tube de vidange / Tubo de despejo / Tubo scarico |
| D | Tubo desagüe / Drainage tube / Tube de purge / Tubo de drenagem / Tubo di scolo |

Fig. 1

INSTALACIÓN

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INSTALLATION

The R-080089 Decanter has been designed to be used under a sink. For its installation we recommend:

- Carefully unpack the decanter.
- For the Decanter correct operation it is necessary that the sink drainpipe has just a unique straight outlet tube whose end has to be at least 62 mm from the floor. If others, you should eliminate u-bends or any other sink elements (Picture 1).
- Get ready a connexion between drainpipe (D) and the lab general drainpipe. Choose the most suitable adapters in accordance with characteristics of general installation of the lab drainpipe. The machine is supplied with a 50 mm drainage outlet.
- Join by a drainage tube (D) the decanter outlet and the general drainage connexion. The machine is ready to be used.

CLEANING

Plaster wastes inside the tank should be cleaned periodically. Cleaning frequency depends on the use of the Decanter, so it is difficult to specify it.

The Decanter has a maximum level mark that marks the maximum height that plaster wastes can reach inside the tank with machine normal operation. Check that this level is not exceeded. Anyway, cleaning frequency should never be longer than 2 or 3 months, even if plaster wastes do not exceed the maximum level mark, thus avoiding unpleasant smells produced by plaster rotting.

For Decanter cleaning, follow the following instructions:

- Take out the drainage tube (D) from its fixation to the lab general drainpipe. Put the end of the tube into a bucket or similar container to avoid dripping.
- Once the Decanter is free, move it outwards the sink and place it on a lab comfortable place. Thanks to its wheels (E) movement is easy.
- Unscrew tube fixation (B) to free the draining pipe (C). Turn the draining pipe and unload water into a bucket or other container.
- Once there is no water inside the tank, remove the screen and lid (A) and take out with a dustpan the plaster waste on the bottom of the tank.

Place the Decanter again on its place under the sink following the reverse described process.

PRECAUTIONS

- ◆ For placing the Decanter choose a rigid, flat and levelled area.
- ◆ Avoid presence of wires or circuits near the Decanter. Minimum recommended safe distance should be at least 1 m.
- ◆ Transport and movement of the Decanter should be done carefully, avoiding risk of machine overturn.
- ◆ If there are difficulties in finding the optimum way to adapt the Decanter to the lab general drainpipe, consult a plumbing professional.

TECHNICAL CHARACTERISTICS

Height:	615 mm
Width:	520 mm
Depth:	600 mm
Weight:	10 kg
Capacity:	70 L

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GUARANTEE

All **MESTRA®** units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA®** will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

MESTRA® will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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