

Ref. 080118

ES.....	3	Manual original
EN.....	5	
FR.....	7	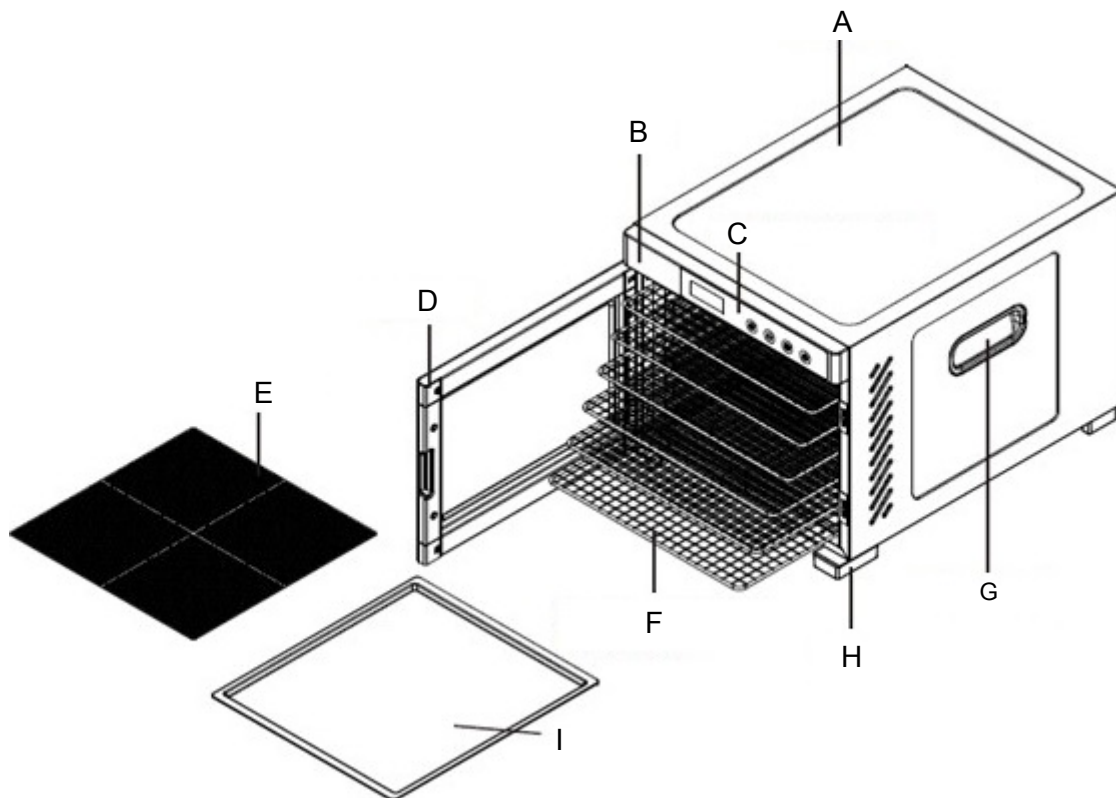

A	Carcasa / <b>Housing</b> / Carcasse
B	Panel / <b>Pannel</b> / Panneau
C	Controles / <b>Controls</b> / Contrôles
D	Puerta / <b>Door</b> / Porte
E	Rejilla plástico / <b>Plastic grid</b> / Grille plastique
F	Baldas / <b>Shelves</b> / Plateaux
G	Asa / <b>Handle</b> / Poignée
H	Pie / <b>Foot</b> / Pied
I	Bandeja de plástico / <b>Plastic tray</b> / Plateau plastique

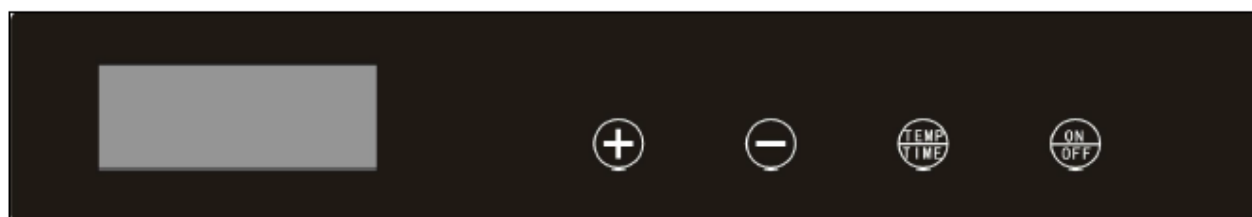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## Warnings

### Preventive measures

- Attention: before using the machine, please completely read this operating manual.
- Keep this operating manual properly for future.
- Do not allow children or non-qualified personnel handling the unit.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- The machine features a precision circuit. Prevent fortuitous impacts and stresses that could cause mechanical damage.

- Do not mix potentially explosive substances.
- Remove any dirt or water from the cable connection.
- Unplug the appliance immediately if you detect noise, odor or smoke.
- Install the product away from heat sources.
- Do not use the appliance in particularly humid places, or near flammable materials (alcohol, solvents, etc.).

- Do not use the machine with products different than defined in the intended use.
- To prevent accidents, be cautious when using the unit.
- Always keep the device in idle mode, except when mixing.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.
- Do not dismount, repair or modify the machine.
- If you find a defective component or a problem do not use the machine.

- Contact us for questions or detailed troubleshooting information.

## Intended use

The drying oven (ref. 080118) is designed to extract moisture and keep the thermoforming gaskets dry avoiding the formation of pores, for drying instruments, plaster models, etc. in dental laboratories and clinics.

A fan circulates the hot air producing a uniform drying.

For optimum output and performance, we advise you to read the following instructions with great care.

## Installation

- Unpack the appliance carefully. You will find the following items:
  - Main unit.
  - 1 plastic tray.
  - 1 plastic grid.
  - 6 stainless steel shelves.
- Make sure the drying oven sits firmly on a solid, level surface.
- Place a plastic tray on the bottom of the oven interior.
- Place the plastic grid inside the tray.
- Place the 6 metal shelves on their supports.
- Connect the power cord to a grounded 230 V, 50/60 Hz electrical source.

## Operation

1. Place the sheets or objects on the trays. Do not overload the trays:  
Max. load = 600 g per shelf.
2. Close the door.
3. Press the **TEMP/TIME** button to toggle between temperature and time.
4. Using the + / - buttons, select the desired temperature and operating time.
5. The default setting is 10 hours and 70 °C.
6. Press the **ON/OFF** button to start the cycle or to stop it.
7. The control will deduct the time until "0:00". Heating will stop and the fan will continue to run for 10 seconds until an audible warning sounds.
8. Unplug the appliance when not in use.

## Maintenance and cleaning

To ensure a lengthy service life, we recommend the following:

- **ATTENTION!** Do not operate the device for more than 24 hours in a single use. After 24 hours allow it to cool down for at least 2 hours before working again.
- After using, disconnect the drying machine from the power source and let it cool down before cleaning. Dry all parts with a dry cloth.
- Unplug the power cord before cleaning.
- Do not clean the appliance with aggressive or abrasive chemicals so as not to damage the surface.
- Do not spray cleaners inside the appliance.
- If you are not going to use the appliance for a long time, unplug it and store it in a clean, dry place, at room temperature and in a stable position.

## Technical characteristics

Power supply: .....220-240 V, 50/60 Hz  
Consumption: .....550-650 W  
Height: .....310 mm  
Width: .....340 mm  
Depth: .....450 mm  
Weight: .....8.3 kg  
Temperature range: .....35-75 °C  
Timer: ..... 0-24 h  
Max. load: .....600 g per shelf  
Number of shelves: ..... 6 (stainless steel)  
Housing: ..... Stainless steel

## Declaration of conformity UE

The firm **Talleres Mestraitua S.L.**  
with social address: Txori-erri Etorbidea, 60  
48150 Sondika - Bilbao - Spain

Hereby declares that the product

Drying oven, ref. 080118

Meets the requirements set out in the following directives:

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