## R-080150 / R-080160

ES.....	ELECTROPULIDORA.....	7
EN .....	ELECTRO-POLISHER.....	9
FR.....	TOUR A POLIR .....	11
DE .....	ELEKTRISCHE POLIERMASCHINE .....	13
PO .....	ELECTRO-POLIDORA.....	15
IT .....	PULITRICE ELETTRICA.....	17

**MESTRA®**

TALLERES MESTRAITUA S.L.

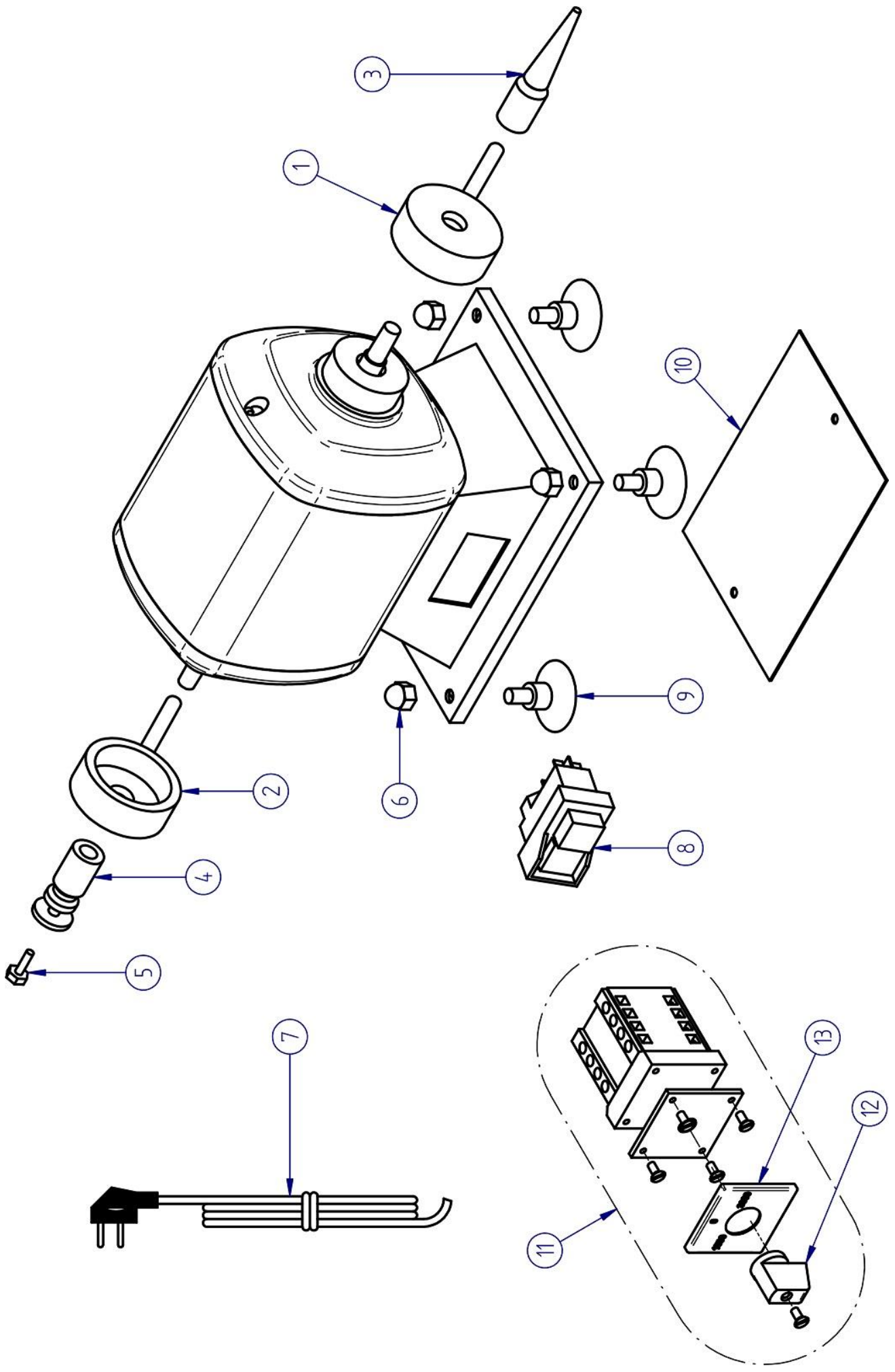
Txori-Erri Etorbidea, 60

Tel. + 34 944 530 388 - Fax + 34 944 711 725

mestra@mestra.es - www.mestra.es


48150 SONDIKA - BILBAO - ESPAÑA

Rev. 24/02/10

Nº	Ref.	DESCRIPCIÓN	DESCRIPTION
1	080150-02	Extractor derecha	Right extractor
2	080150-18	Extractor izquierda	Left extractor
3	080160-03	Porta cepillos derecha	Right brush holder
	080175	Porta cepillos izquierda	Left brush holder
4	080160-04	Porta piedras izquierda pulidora	Left stone holder
	080170	Porta piedras derecha pulidora	Right stone holder
5	080160-11	Tornillo porta piedras izquierda	Left brush holder bolt
	080170-01	Tornillo porta piedras derecha	Right stone holder bolt
6	080150-05	Tuerca ciega	Cap nut
7	000200	Cable	Cable
8	080080-40	Interruptor seguridad	Safe switch
9	080150-03	Ventosa	Suction pad
10	080150-07	Tapa inferior	Lower lid
11*	080160-02	Interruptor velocidad	Speed switch
12*	080500-58	Pomo interruptor velocidad	Speed switch knob
13*	080160-09	Marco interruptor velocidad	Speed switch frame

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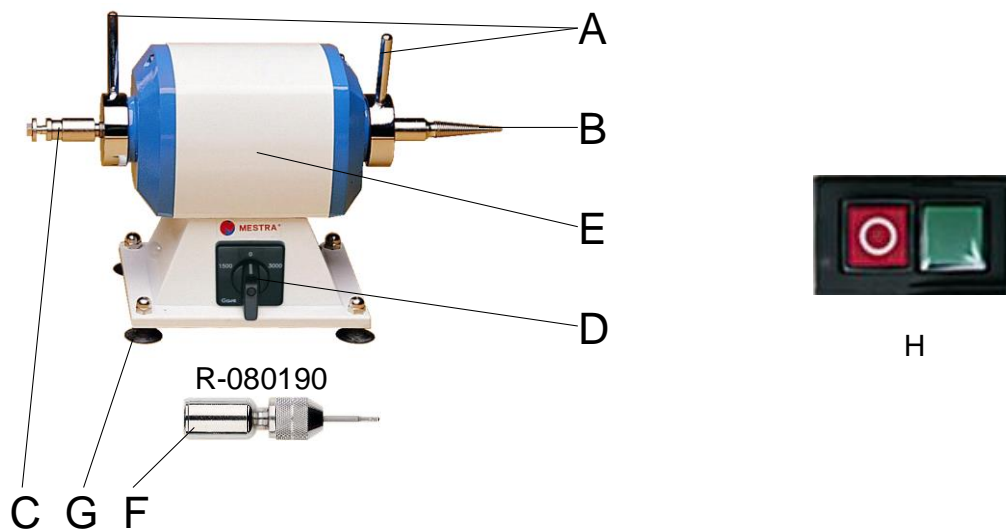## R-080150 / R-080160

A	080150-02 080150-18	Extractor derecha / <b>Extractor right</b> / <i>Extracteur droit</i> / Abzieher richtig / <b>Extractor direito</b> / <i>Estrattori destra</i> Extractor izquierda / <b>Extractor left</b> / <i>Extracteur gauche</i> / Abzieher verließ / <b>Extractor esquerdo</b> / <i>Estrattori sinistra</i>
B	080160-03 080175	Porta cepillos derecha / <b>Brush holder right</b> / <i>Mandrin porte-brosses droit</i> / Bürstenhalter richtig / <b>Porta escovas direito</b> / <i>Porta spazzole destra</i> Porta cepillos izquierda / <b>Brush holder left</b> / <i>Mandrin porte-brosses gauche</i> / Bürstenhalter verließ / <b>Porta escovas esquerdo</b> / <i>Porta spazzole sinistra</i>
C	080170 080160-04	Porta muelas derecha / <b>Wheel holders right</b> / <i>Mandrin porte-meules droit</i> / Schleifscheibenhalter richtig / <b>Porta mós direito</b> / <i>Porta mole destra</i> Porta muelas izquierda / <b>Wheel holders left</b> / <i>Mandrin porte-meules gauche</i> / Schleifscheibenhalter verließ / <b>Porta mós esquerdo</b> / <i>Porta mole sinistra</i>
D	080162-06	Conmutador de velocidad / <b>Speed switch</b> / <i>Commutateur de vitesse</i> / Geschwindigkeitsumschalter / <b>Comutador de velocidade</b> / <i>Commutatore di velocità</i>
E	-	Motor / <b>Motor</b> / <i>Moteur</i> / Motor / Motor / <i>Motore</i>
F	080190	Porta fresas 2,35 mm / <b>2.35 mm cutter holder</b> / <i>Porte fraises 2,35 mm</i> / Fräsenhalter 2,35 mm / <b>Porta fresas 2,35 mm</b> / <i>Porta frese 2,35 mm</i>
G	080150-03	Ventosas de base / <b>Base suction caps</b> / <i>Ventouses</i> / Ventosas de base / <b>Ventose base</b> / <i>Saugfüße</i>
H	080080-40	Interruptor de seguridad / <b>Safe switch</b> / <i>Interrupteur de sécurité</i>

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The R-080150 / R-080160 is a simple electro-polisher especially designed to polish and grind small items in the fields of jewellery and dental mechanics. It is supplied with the following accessories:

- 1 right-hand thread brush holder (B).
- 1 left-hand thread grinding wheel holder (C).

As an option, a cutter holder (F) (**MESTRA** catalogue ref. R-080190) can also be fitted, for cutters with 2.35 mm shafts. Left-hand and right-hand brush holders are also available.

### **FITTING THE BRUSH TO THE BRUSH HOLDER**

For the brush to be properly gripped in the brush-holder (B), proceed as follows:

1. Choose an appropriate brush-holder. If you are facing the ON switch and wish to fit the brush-holder on the right-hand end of the motor shaft, select a right-hand (clockwise) thread brush-holder, so that the friction torque produced when the unit is used will help tighten the brush-holder. If you wish to fit it to the left-hand end of the shaft, select a left-hand (anti-clockwise) thread brush-holder.
2. Attach the threadless part of the brush-holder to a bench vise.
3. Insert the brush into the brush-holder, ensuring that the brush-holder shaft is perpendicular to the plane of the brush.
4. Screw the brush onto the brush-holder as tightly as possible by hand.
5. Take the brush/brush-holder assembly out of the bench vise.
6. Visually check that the brush-holder shaft is perpendicular to the plane of the brush by spinning the assembly in your hand.

### **FITTING GRINDING-WHEELS TO THE WHEEL-HOLDER**

1. Select the grinding-wheel-holder (C) and take the wheel attachment screw off the end.
2. Attach the other end of the wheel-holder to a bench vise.
3. Fit the wheel to the wheel-holder.
4. Secure the wheel to the wheel-holder by tightening the wheel fixing screw with a metric size 10 fixed wrench.
5. Take the wheel/wheel-holder out of the bench vise.
6. Visually check that the wheel-holder shaft is perpendicular to the plane of the wheel by spinning the assembly in your hands.

### **FITTING THE BRUSH-HOLDER AND WHEEL-HOLDER TO THE SHAFT ON THE UNIT**

1. Fit the end of the wheel-holder or brush-holder with the hole on the end of the motor shaft and push it in firmly by hand.
2. Tap the opposite end of the wheel-holder or brush-holder with a nylon mallet until it is properly fitted.
3. Turn the motor shaft by hand and visually check that the wheel or brush is perpendicular to the axis of rotation. Also check that the motor shaft is properly lined up with the wheel-holder or brush-holder shaft.
4. Start up the unit: the spin of the motor will reveal whether the wheel or brush is off line. If there is more than 2 mm of slip, dismount and re-fit the wheel-holder or brush-holder.
5. Remember that the friction torque produced during work after a brush is fitted to the brush-holder will keep it tight. This effect can be favoured by a suitable workpiece attack angle.

### **DISMOUNTING THE BRUSH-HOLDER OR WHEEL-HOLDER FROM THE UNIT SHAFT**

The unit comes with two extractors (A) which enable brush-holders or wheel-holders to be removed easily. Grip the handle of the extractor firmly and turn it anti-clockwise on the right-hand end of the shaft or clockwise on the left-hand end. The pressure exerted by the extractor as it advances will smoothly release the brush-holder or wheel-holder.

### **CONDITIONS OF USE**

For best results use the unit as follows:

- Mount an appropriate brush-holder/wheel-holder as indicated above.
- Connect the unit to a 230 V, 50/60 Hz power source.
- Select the desired spin speed (1500/3000 rpm) via switch D (R-080160).
- Before polishing or grinding a workpiece, check the motor spin direction, as the initial workpiece attack angle depends on this. Incorrect attack angles can be extremely dangerous.
- The following protective equipment is recommended for polishing or grinding:
  - safety goggles;
  - safety gauntlets;
  - face-mask;
  - proper exhausters;

## English

- a protective screen suited to the characteristics and shape of the unit. TALLERES MESTRAITUA, S.L. recommends R-080200 or R-080210 screens, which are especially designed for use with the polisher.
  - Proper clothes. No baggy clothes, ties, necklaces, rings, ropa holgada, corbatas, collares, anillos, bracelets or elements that could be hooked by the machine.
  - Be careful with long hair.
  - No objects similar to ropes or chains that could be hooked by the machine.
- After using the unit, clean the inside of the protective screen.

## MAINTENANCE AND CLEANING

To ensure a long, trouble-free useful life for your unit, proceed as follows:

- After using the unit, clean dust off the inside of the protective screen with a brush. Clean the whole screen regularly with a cloth moistened in soapy water.
- To clean the motor and base used a cloth moistened in soapy water. Never use alcohol, solvents or other inflammable liquids.
- Never immerse the unit in any liquid.
- If you notice unusual noises or vibrations, send the unit to your technical assistance service.
- This unit is designed to work correctly on the four suction caps on base G.

## PRECAUTIONS

- Before connecting the unit to the mains, check that the power supply is 230 V, 50/60 Hz.
- Discard any cracked or damaged brush or grinding wheel: if these elements break during operation they could cause serious accidents.
- Before starting work, ensure that the brush-holder/wheel holder is firmly attached to the motor shaft.
- When handling workpieces, use safety goggles, gauntlets, a face mask and a suitable safety screen. We recommend MESTRA R-080200 or R-080210 screens.
- Do not allow children or unskilled personnel to handle the unit.
- If the motor shaft locks during normal operation, disconnect the unit from the mains before attempting to release it by turning it manually.
- Never polish, brush or grind materials which may give off particles which are toxic if inhaled.

- Before starting to polish or grind a workpiece, check the motor spin direction: the attack angle depends on this direction.
- Never use the unit close to solvents, alcohol or other inflammable materials. Take all possible precautions when grinding materials which may give off sparks.
- Never use the unit without proper exhausters.

## TECHNICAL CHARACTERISTICS

Tension:	230 V, 50/60 Hz
Power:	125 W
Speed R-080150:	2750 rpm
Speed R-080160:	1500/3000 rpm
Height:	200 mm
Height of shaft centre line:	140 mm
Overall shaft length:	270 mm
Weight:	7 kg

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