Ref. 080233

ES.....	ARENADORA COSMOS.....	5
EN.....	SANDBLASTER COSMOS.....	9
FR.....	SABLEUSE AUTOMATIQUE COSMOS.....	12

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

48150 SONDIKA - BILBAO - ESPAÑA

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

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The Cosmos R-080233 is a sand blaster specially designed for sandblasting small pieces within the field of dental prosthesis.

To working modes are possible:

- A fix nozzle that recycles the abrasive of the big tank under the cabin (particles up to 300 microns).
- Two precision mobile nozzles with independent tanks without abrasive recycling (particles up to 150 and 250 microns).

In order to achieve an optimal operation and performance of the unit, we strongly recommend you to read carefully the following instructions:

INSTALLATION

- Carefully unpack the unit. You will find the following accessories inside the cabin:
 - A pedal for the mobile nozzles.
 - A Ø 4 / 6 mm pipe to connect the unit to the air supply.
 - A 90° PVC elbow to be fitted into the outlet aspiration hose.
 - Two plastic covers to protect the glass (R-080235-50).
 - A plastic cover to Project the controls.
- Choose a firm, flat surface to place the unit, far from heat or vibration sources. The room atmosphere should never be damp. An excess of humidity could compact the aluminium oxide particles inside the cabin and, therefore, obstruct the blasting nozzles. Finally, place the unit at a height that allows an easy manipulation of the objects inside the cabin.
- Fit the connecting pipe supplied with the unit into the inlet adaptor placed at the right side panel of the machine. The other end should be fitted into an intake of pressured air fed by a compressor with a minimum flow of 100 litre/minute.
- Connect the power cord to a grounded 230 V, 50/60 Hz power source.

Fix nozzle

- Fill the hopper in the cabin with aluminium oxide of a maximum diameter of 300 micron. To obtain a maximum autonomy, load the hopper up to a maximum height of 2 cm below the tray.

NOTE: the unit is optimized for 200-micron particles and getting an stable jet. With smaller particle diameters (100 microns) the jet may be more irregular.

The unit is factory supplied with a 3 mm boron carbide nozzle (R-080235-25), material much more abrasion-resistant than the traditional tungsten.

Mobile nozzles

- Connect the pedal drive tube to the coupling nozzle on the right-hand side of the unit. Then position the pedal for comfortable foot operation. Depending on operational requirements and/or handling comfort, it is possible to position the pedal for knee operation.
- Unscrew the sandblasting module covers (D), and fill the tanks with emery. During this operation, **it is most important not to fill the tanks to more than ¾ of their capacity.** Put the covers back on, and ensure they have been screwed on tightly.
- Mobile nozzles are located on the right side of the camera. If you wish, you can also pull the tubes for comfortable use with the left hand.
- In relation to the calibre of the nozzles, the machine is supplied ex-works to the following criterion of nozzle diameters and colours:

Blue	Ø 1 mm	(up to 150 microns)
Red	Ø 1.25 mm	(up to 250 microns)

PRESSURE REGULATION

Fix nozzle

The blowing pressure of the blasting gun can be regulated at will between 2 and 5 bar (0.2 and 0.5 MPa), according to the nature and special features of the work to be done. Take into account that the highest pressure will be limited by the maximum pressure of the air supply we are using. In any case, the pressure should never be higher than 6 bar (0.6 MPa).

The unit features a pressure regulation valve (G) to select the working pressure easily. To adjust the pressure, pull the valve outwards and turn it until the intended pressure has been reached. The value of the pressure is shown in the built-in pressure gauge (G). After adjusting the pressure, block the valve knob by pressing it inwards.

The pressure gauge shows the gun's blowing pressure at any time, even when the gun is not being used.

Mobile guns

The blowing pressure of the blasting guns can be regulated depending on the nature and

characteristics of the operation to be carried out. Use the pressure-adjustment knob (C) on the top right of the unit – pull the knob upwards, and rotate it to the required pressure. The unit has a pressure gauge (B) for easy selection of operating pressure. When the pressure has been adjusted, lock out the regulator knob by pushing it down to the initial position.

CONNECTING TO THE ASPIRATION UNIT

The blasting cabin should always be used connected to a pressured aspiration unit. For this purpose, there is an outlet hose on the upper left side panel of the unit. The aspiration unit to be used has to create a light depression inside the cabin for the dust particles to circulate smoothly and continuously towards the outlet hose.

NOTE: make sure that the aspiration unit used can suck abrasive particles. We strongly recommend the MESTRA Eolo Plus R-080531 aspiration unit.

The factory-supplied PVC elbow can be adapted according to the needs and shape of the pressured aspiration unit to be used.

OPERATION

1. Open the lid of the chamber, and put in the part you wish to blast. Parts may also be placed inside the chamber via sleeve (J).
2. Hold with your hand the pieces to sandblast inside the chamber.
3. Close the lid of the chamber, and ensure it is tightly shut (the glass should fit snug against the housing).
4. Suitable protection gear must be used: goggles, gauntlets and a mask.
5. Start up the forced suction facility
6. The unit has LED lighting for a better light in the chamber may be activated for a better view of the pieces.

Fix nozzle

- Position the gun (H) facing the pieces.
- Avoid the stream hitting the sides of the chamber directly, since this could damage the chamber. If you wish to test the blasting pressure or strength, direct the stream at the rubber dolly inside the chamber.
- Press button (F). The nozzle will blow abrasive until you press the button again.

Mobile nozzles

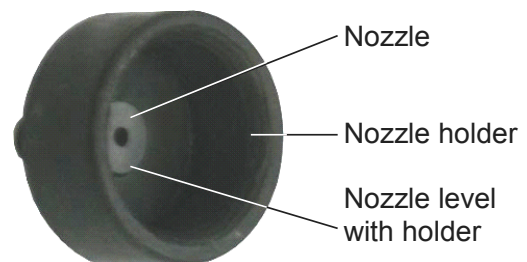
- Check there is enough emery in the tanks.
- Use the main switch (E) to select the tank.

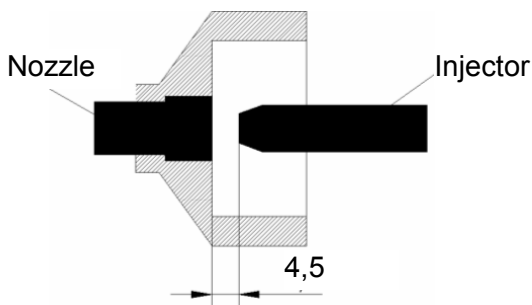
- Pick up the mobile gun selected inside the chamber. The colour of the gun's nozzle-holder matches the colour on the tank covers. When the gun has been selected, press the drive pedal to begin sandblasting.
- Direct the blast carefully to prevent the stream hitting the sides of the chamber directly, since this could damage the chamber. If you wish to test the blasting pressure or strength, direct the stream at the rubber dolly (E) inside the chamber.
- After the blasting operation, release the pedal, stopping the sand flow and activating immediate air discharge to relieve pressure in the tanks. Then switch off the suction unit, open the lid of the chamber, and take out the part.

MAINTENANCE AND CLEANING

To ensure a lengthy service life, we recommend the following:

- Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide.
- Preferably use dry-piston air compressors. The presence of oil particles in the air could make the sand compact and obstruct the gun's nozzle.
- To prevent the inner walls of the cabin wearing away due to abrasion, do not aim the aluminium oxide blast straight at the cabin walls.
- Periodically remove the aluminium oxide from the cabin. To do this, use the drain plug in the bottom of the hopper.
- For an overall cleaning of the unit, use a dry cloth or moistened with a proper product.
- In case of obstruction of the gun, dismount the nozzle holder. Clear the nozzle by blowing in the opposite direction to the air flow, or, if needed, by using a pin or a thin wire. Then set the gun again, making sure it is correctly centred.


If the nozzle came loose, glue it to its holder with cianocrilate. The nozzle should be level with the inner surface of the nozzle holder, to make sure that the distance from the injector to the nozzle is 4.5 mm and the Venturi effect takes place.

- Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide. The gun (\varnothing 3 mm nozzle) can blast aluminium oxide up to particles of \varnothing 300 micron.

MAINTENANCE OF THE PISTOL FLEX

Do not bend or twist excessively the flex that holds the pistol and allows positioning it. It could break and then the abrasive would not go up the pipe. When needed, replace it following these steps:

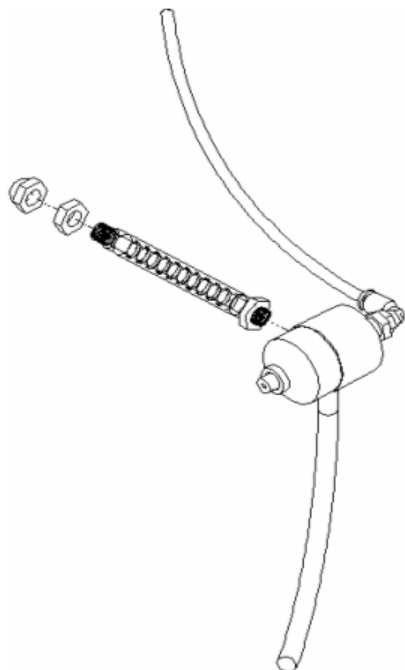1. Loosen the two lower bolts in the back cover and remove the cover.
2. Undo the cap nut and the normal nut that hold the flex from the back cover. They may be tight due to the sealant.

3. Disengage the white pipe from the pistol by pulling it.
4. Disengage the blue pipe from the pistol by pulling it while you press the adaptor.
5. Unscrew the flex from the body of the pistol with a pair of pliers. It may be very tight due to the

sealant. Hold the flex as near the pistol as possible.

6. Screw in the spare flex. Apply a sealant to prevent air leaks. Do not twist the flex. Hold the flex as near the pistol as possible.
7. Engage the two pipes again.
8. Fix the flex to the back cover with the two nuts. Apply a sealant to prevent air leaks.

UNIT CONTROLS PLASTIC PROTECTOR

In order to protect the unit controls from abrasive powder, you can stick the supplied plastic protector on the housing (see the picture).

PRECAUTIONS

- ◆ Never exceed a 6 bar pressure (0.6 MPa). The optimal working pressure lies between 2 and 5 bar (0.2 and 0.5 MPa).
- ◆ Always work with the cabin lid closed and correctly protected. Use protection mask and goggles.
- ◆ Connect the cabin's outlet hose to an aspiration unit.
- ◆ If the protection sleeves (J) tear or wear, replace them immediately.
- ◆ Do not allow children or unskilled personnel to operate the machine.
- ◆ Always use a high-quality aluminium oxide (it is strongly recommended to use **MESTRA aluminium oxide**). A very low-quality aluminium oxide can produce obstructions in the nozzle.
- ◆ Make sure that the aspiration unit can suck abrasive particles. We strongly recommend the **MESTRA Eolo Plus R-080531** aspiration unit.
- ◆ Connect the power supply cord to 230 V, 50/60 Hz, earthed.
- ◆ In case of leaks in the pressured air circuit, consult the installation engineer immediately.

TECHNICAL SPECIFICATIONS

Height:	550 mm
Width:	430 mm
Depth:	500 mm
Weight:	20 kg
Working pressure:	2 - 5 bares 0.2 - 0.5 MPa
Max. air consumption:	90 litros/min. a 5 bares
Voltage:	230 V, 50/60 Hz
Power consumption:	10 W

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