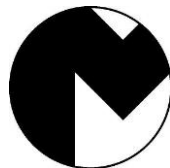
Ref. 080500

ES	PULIDORA CON ASPIRACIÓN	7
EN	POLISHER WITH EXHAUSTER	11
FR	STATION DE POLISSAGE MESTRA.....	14
DE	POLIERBOX MIT ABSAUGUNG	17
PO	POLIDORA COM ASPIRAÇÃO	21
IT	PULITRICE CON ASPIRAZIONE	24

MESTRA®

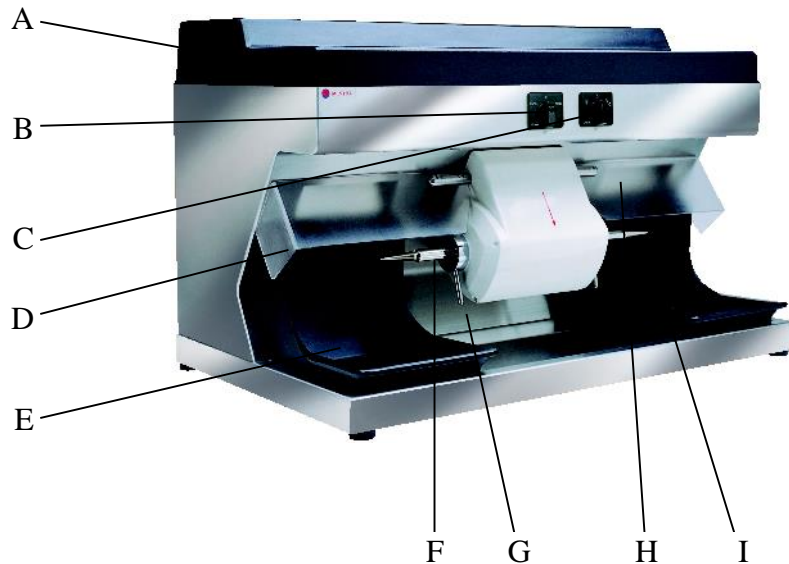
TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Ref. 080500

A	Tapa / Cover / <i>Couvercle</i> / Deckel / Tampa / Coperchio
B	Selector de velocidad / Speed selector / <i>Selecteur de vitesse</i> / Geschwindigkeitswähler / Selector de velocidade / <i>Selettore di velocità</i>
C	Interruptor de marcha / On switch / <i>Interrupteur marche-arret</i> / Betriebsschalter / Interruptor de funcionamento / <i>Interruttore di avvio</i>
D	Visera protectora / Protective visor / <i>Visiere de protection</i> / Schutzblende / Viseira protectora / <i>Visiera di protezione</i>
E	Pantalla / Screen / <i>Augette</i> / Gummischirm / Ecrã / Schermo
F	Portacepillos a Izda / Left-hand brush-holder / <i>Porte-brosses gauche</i> / Bürstenhalter links / Porta-escovas à esquerda / <i>Portaspazzole a sinistra</i>
G	Mango de extractor / Extractor bushing / <i>Manche d'extracteur</i> / Entnahmehebel / Maneta do extractor / <i>Manico di estrattore</i>
H	Entrada de aire / Air inlet / <i>Entree d'air</i> / Lufteintritt / Entrada de ar / <i>Entrata d'aria</i>
I	Portacepillos a Dcha. / Right-hand brush-holder / <i>Porte-brosses droit</i> / Bürstenhalter rechts / Porta-escovas à direita / <i>Portaspazzole a Destra</i>

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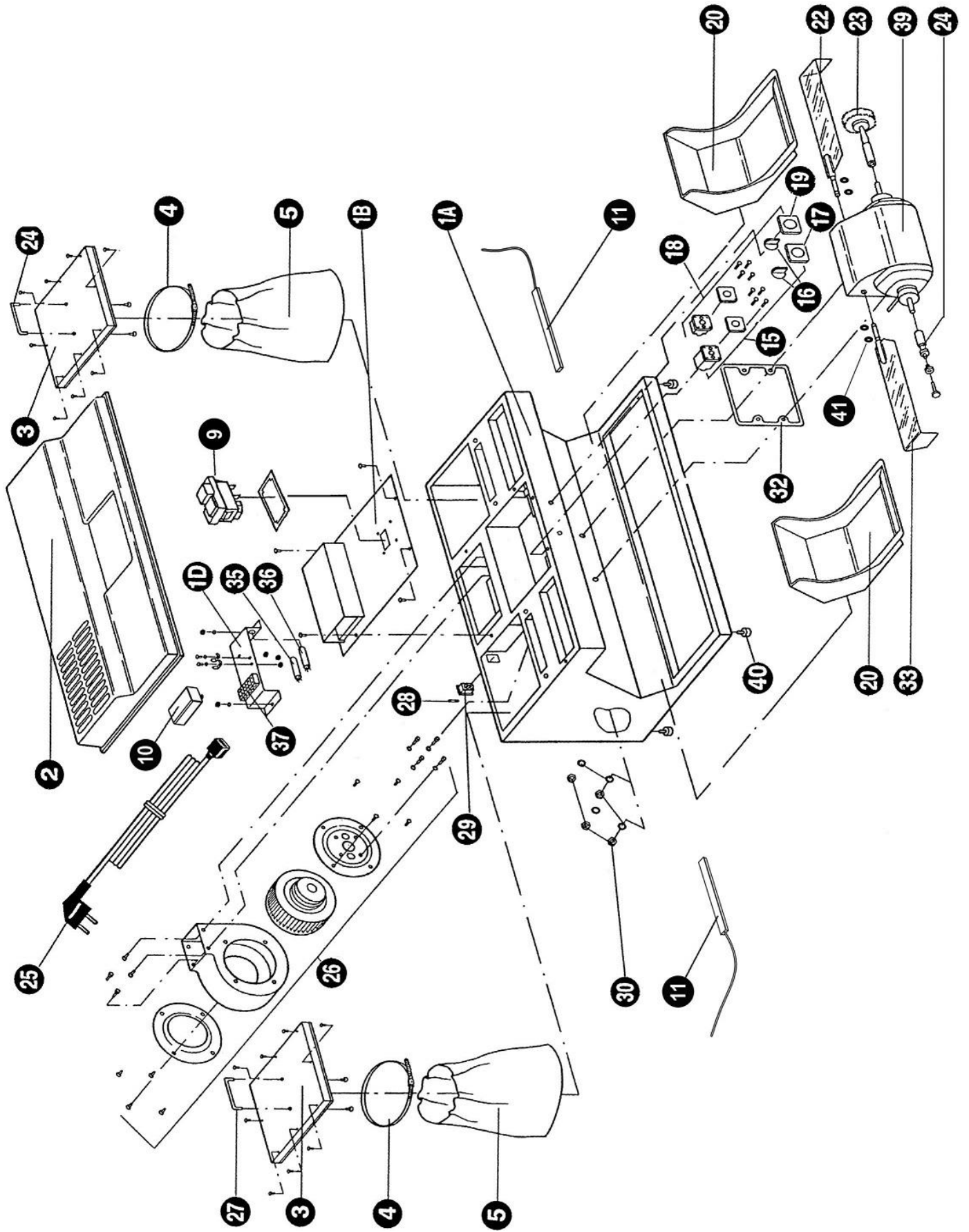
The polisher shown in the photograph has two brush-holders.

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GUARANTEE

All **MESTRA®** units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA®** will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

MESTRA® will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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The Ref. 080500 is a complete polisher equipped with an exhauster and other safety devices to European EC standards. It is especially designed to polish and grind small workpieces for the dental prosthesis sector. It is supplied from the factory with the following accessories:

- 1 left-hand thread brush-holder (F)
- 1 right-hand thread brush-holder (H)
- 1 right-hand thread stone holder (not shown in figure)
- 2 rubber screens

As an option the unit can be fitted with a mill holder (Mestra catalogue ref. R-080190) for 2.35 mm shank milling cutters. Left-hand brush-holders and right-hand wheel holders are also available.

INSTALLATION

Carefully unpack the Ref. 080500 polisher.

Set up the unit on a flat surface away from any sources of heat or vibration. A well ventilated area should be chose.

Connect the unit to a 230 V, 50/60 Hz earthed power outlet.

Mount the brush and/or grinding wheel on the motor shafts.

MOUNTING THE BRUSH IN THE BRUSH-HOLDER

To attach the brush correctly to the brush-holder (F) proceed as follows:

1. Choose a suitable brush-holder. For mounting on the right-hand end of the motor shaft as seen from the front of the unit, choose a right-hand (clockwise) thread brush-holder so that the friction torque tightens it during operation. Likewise, for mounting on the left-hand end of the unit choose a left-hand (anti-clockwise) thread brush-holder.
2. Attach the unthread part of the brush-holder to a bench vice.
3. Insert the brush into the brush holder, ensuring that the axis of the brush-holder is perpendicular to the plane of the brush.
4. Thread the brush onto the brush-holder as tightly as possible by hand.
5. Take the holder and brush assembly out of the bench vice.
6. Visually check the perpendicularity of the brush-holder axis to the plane of the brush by spinning the assembly in your hands.

MOUNTING GRINDING WHEELS ON THE WHEEL-HOLDER

1. Select the wheel-holder and remove the wheel attachment screw from the end.
2. Secure the other end of the wheel-holder to a bench vice.
3. Mount the wheel in the wheel-holder.
4. Secure the wheel in the wheel-holder by tightening the wheel attachment screw with a cross-head screwdriver.
5. Remove the wheel-holder/wheel assembly from the bench vice.
6. Visually check the perpendicularity of the wheel-holder axis to the plane of the wheel by spinning the assembly in your hands.

MOUNTING THE BRUSH-HOLDER AND WHEEL-HOLDER ON THE SHAFT OF THE UNIT

1. Place the end of the wheel-holder or brush-holder with the hole on the end of the motor shaft and push it on firmly by hand.
2. Tap the other end of the wheel-holder or brush-holder with a nylon mallet until it is properly fitted on.
3. Turn the motor shaft manually and check visually that the grinding wheel or brush is perpendicular to the axis of rotation. Also check the alignment between the motor shaft and the axis of the wheel-holder or brush-holder.
4. Start up the unit so that the motor spin shows up any tilt in the wheel or brush. If tilt or more than 1 or 2 mm is observed, remove and re-mount the wheel-holder or brush-holder.
5. Immediately after mounting a brush in the brush-holder, remember that the friction torque helps tighten it. Set the attack angle on the workpiece to be brushed so as to favour this effect.
6. The safety visors (D) are made of polycarbonate and are virtually unbreakable. They can be folded down for easier use. Raise them when mounting the brush-holder and wheel-holder for easier access to the motor shaft.

DISMOUNTING THE BRUSH-HOLDER AND WHEEL-HOLDER FROM THE SHAFT OF THE UNIT

The unit comes with two extractors (G) to help remove the brush-holder or wheel-holder. Grip the handle of the extractor firmly and turn it anti-

clockwise (right hand end of shaft) or clockwise (left-hand end). The pressure exerted by the extractor on the brush-holder will smoothly release them.

USING THE UNIT

To ensure correct use of the polisher, follow these instructions:

Mount a suitable brush-holder or wheel-holder as described above.

Lower the safety visor (D) and ensure that the rubber screens (E) are in place. If they are incorrectly positioned hazards can result and the machine's efficiency is impaired.

Before starting to polish or grind a workpiece, ensure that the motor spin direction is correct (red arrow on motor stator), as this affects the initial angle of attack on the workpiece. Incorrect angles of attack can be highly dangerous.

Start up the polisher via switch (B). Turn the switch right or left to select the desired spin speed (1500-3000 at 50 Hz or 1800-3600 at 60 Hz).

Turn switch (C) right to select simultaneous operation of the exhauster and the light. Turn it left for light without exhauster. This is intended only for highly unusual circumstances in which the exhauster might be harmful to the work being done. As a general rule switch (C) should always be turned to the right for simultaneous operation.

The following protective equipment is recommended for polishing or grind-ing:

- safety goggles;
- safety gauntlets;
- face-mask;
- As a general rule do not use the polisher without the exhauster.
- Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
- Be careful with long hair.
- No objects similar to ropes or chains that could be hooked by the machine.
- Place the protective screen properly.
- Work with the visor (D) in place.

After using the unit always clean the inside of the protective screen and the support tray.

CHANGING THE EXHAUSTER FILTERS

The Ref. 080500 has a closed exhauster circuit which picks up the particles of dust produced during polishing in two filters inside the machine. When these filters are dirty the machine works less

efficiently and the exhauster motor noise level increases. If this occurs, change the filter as soon as possible:

1. Disconnect the machine from the power supply and remove the dust guard (A).
2. Inside the machine you will find two square cast aluminium covers with handles. These are not attached to the polisher casing, and can be lifted directly.
3. Hanging from the bottom of each cover is a filter, attached to it by a clamp. Release this with a screwdriver.
4. Remove the dirty filter and fit a clean one. Talleres Mestraitua, S.L. recommends Mestra spares, as only they can ensure optimum performance and reliability.
5. Check that the filter mouth fits correctly onto the circular mouth of the cover, without creases, gaps or points through which air could leak. Then tighten the attachment clamp.
6. Re-mount the covers with the new filters in their original places. There are two centring lugs on the cover to indicate correct position and fit. Also check that the foam seals around the edge of the covers are seated perfectly on the supporting surface so that there are no air leaks.
7. Fit the dust guard (A) and the machine is ready for use.

One side of the machine may be used more than the other, and there are separate filters for the left and right sides. This means that sometimes only one filter may need replacing. Check filters regularly for clogging and change them as necessary.

Do not try to clean dirty filters: always fit new ones. Emptying the dust bag is not advisable, as particles of dirt will remain trapped in the filtering fabric. Filters which have been "cleaned" in this way considerably reduce exhauster performance.

MAINTENANCE AND CLEANING

To ensure correct, reliable, long-lasting operation of the unit we recommend the following:

Clean the dust off the inside of the protective screen with a brush after each use. Also clean the surface of the front tray.

Regularly clean the screens thoroughly. Also clean the stainless steel outer casing of the polisher with a rag moistened in soapy water.

Use a rag moistened in soapy water also to clean the motor and support. Never use alcohol, solvents or other inflammable liquids.

Replace the exhauster filter when it becomes dirty. Dirty filters are indicated by a drop in

exhauster performance and the appearance at times of an anomalous noise on starting up the exhauster. Replace filters as indicated above.

The unit has a protective system to prevent sudden start-ups. If the electrical power supply to the polisher motor is cut off it stops, but when power is restored the machine might start up unexpectedly and anyone manipulating the motor shafts at the time might suffer injury. To prevent this Talleres Mestraitua, S.L. has fitted the Ref. 080500 with an automatic switch which turns it off in case of a power failure. If this happens during operation the device will have to be reset before work can continue. Do this as follows:

1. Switch off the unit via switch (B).
2. Raise the top dust guard (A).
3. Locate the automatic off switch (between switches (B) and (C)).
4. Reset the automatic off switch by pressing it.
5. Fit the dust guard (A).

PRECAUTIONS

Before connecting the unit ensure that the power supply is 230 V, 50/60 Hz earthed power outlet.

Discard any brush or grinding wheel with cracks, knocks or flaws. Breakage of these components during operations could cause serious accidents.

Before starting work ensure that the brush-holder/wheel holder is firmly coupled to the motor shaft.

When handling workpieces use safety goggles, gautles and mask.

Do not allow children or unskilled personnel to handle the unit.

Never polish, brush or grind materials which might give off particles which are toxic if inhaled.

Before starting to polish or grind a workpiece, ensure that the motor shaft spin direction is correct (red arrow on motor stator). Attack angle depends on spin direction.

Do not use the unit near solvents, alcohol or other inflammable materials. Take extreme care when grinding materials which may give off sparks.

Before using the unit ensure that it is set up on a firm solid base.

Do not use the unit without the exhauster on unless the nature and characteristics of a specific job require that it be disconnected. In this case keep working time as short as possible.

TECHNICAL CHARACTERISTICS

Height:	500 mm
Width:	800 mm
Depth:	500 mm
Weight:	60 kg
Power supply:	230 V, 50/60 Hz
Speed (rpm):	1500/3000, 50 Hz 1800/3600, 60 Hz
Noise level:	67 dB
Exhauster flow rate:	500 m ³ /hora

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