

Ref. 100127

ES	MAQUINA DE SOLDAR POR PUNTOS	4
EN	SPOT WELDER	8
FR	SODEUSE A POINTS	12
DE	PUNKTSCHWEISSGERÄT	16
PO	MÁQUINA DE SOLDAR POR PONTOS	20
IT	SALDATRICE A PUNTI	24

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TALLERES MESTRAITUA S.L.
 Txori-Erri Etorbidea, 60
 Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25
 E-mail: mestra@mestra.es - www.mestra.es
 48150 SONDIKA - BILBAO - ESPAÑA

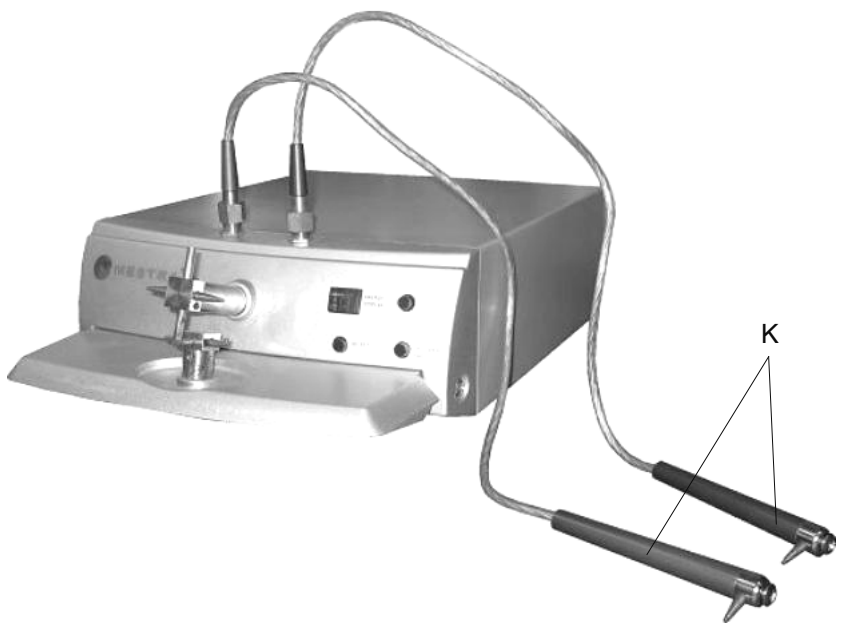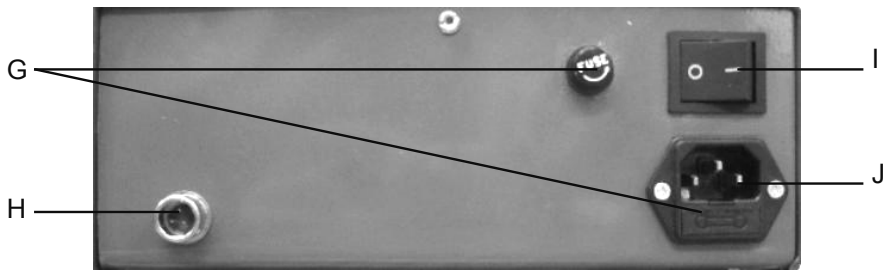- A Soporte para arcos / **Wire arc holder** / *Support pour arc* / Halterung für Bügel / Suporte para arcos / *Supporto per archi*
- B Display / **Display** / *Affichage* / Display / Display / *Display*
- C Tecla SOLDERING / **SOLDERING key** / *Touche SOUDURE* / SOLDERING-Taste / **Tecla SOLDERING** / *Tasto SOLDERING*
- D Tecla START / **START key** / *Touche START* / START-Taste / **Tecla START** / *Tasto START*
- E Tecla SELECT / **SELECT key** / *Touche SELECT* / SELECT-Taste / **Tecla SELECT** / *Tasto SELECT*
- F Electrodo / **Electrodes** / *Electrodes* / Elektroden / **Eléctrodos** / *Elettrodi*
- G Fusibles / **Fuses** / *Fusibles* / Sicherungen / **Fusíveis** / *Fusibili*
- H Conexión pedal / **Pedal connection** / *Connexion pédale* / Pedalanschluss / **Ligação pedal** / *Collegamento pedale*
- I Interruptor general / **Main switch** / *Interrupteur général* / Hauptschalter / **Interruptor geral** / *Interruttore generale*
- J Conexión cable / **Power cord connection** / *Connexion câble* / Kabelanschluss / **Ligação fio** / *Collegamento cavo*
- K Electrodo para soldadura manual / **Electrodes for manual welding** / *Electrodes pour soudures manuelles* / Elektroden zum manuellen Schweißen / **Eléctrodos para soldadura manual** / *Elettrodi per saldatura manuale*

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- Read these instructions carefully before operating the unit. They contain very important information about safety during the installation, operation and maintenance of the unit.
- Keep these instructions for further consultation.
- Check out the state of the unit upon unpacking it.
- Before connecting the device, make sure the voltage in the identification plate matches the power source to be used. During the installation all the safety requirements must be observed and the instructions of the manufacturer followed.
- The manufacturer will not be liable for the damage caused by an inappropriate grounding system.
- Damage caused by a faulty installation or a wrong use of the unit is not covered by the warranty.
- When in doubt, do not start the unit. Contact the supplier for questions or detailed information about problems with the unit.

- **Always wear goggles and gloves while operating the unit.**

- The unit operation generates electromagnetic fields. **Do not use the unit if you have a pacemaker.**

- **Do not touch the unit or the power cord with wet hands.**
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- Do not touch the electrodes while the unit is working.
- **Use the electrode crossheads (F) only for spot welding.** To operate in the “continuous” and “continuous for silver welding” modes, it is obligatory to use the wire arc holder (A) or the electrodes for manual welding (K); otherwise the machine would get too hot. **Do not use the continuous mode for more than 5 seconds.**

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents...).

- Do not allow children or non-qualified personnel handling the unit.
- Do not dismount, repair or modify the unit.


- Contact the supplier for questions or detailed information about problems with the unit.

The R-100127 is a practical welding machine specially designed for the professionals in the dental field. It allows welding two metal pieces by means of the passage of an electrical current of high intensity during a very short time. The machine operation is CPU controlled and therefore almost no spark is produced between the electrodes. You can work with a wide range of stainless steel and iron wires, by simply adjusting the working power.

INSTALLATION

- Carefully unpack the unit. Make sure it has suffered no damage during the transport.
- The unit is factory supplied with the following accessories:
 - Wire arc holder.
 - Electrodes for manual welding.
 - Operation pedal.
 - Power cord
- Choose a firm surface to place the unit, far from heat, vibration and damp sources.
- If you want, you can connect the operation pedal at the back of the unit (H).
- If you want to use the wire arc holder (A), place it in the upper terminals and secure it with the two bolts.
- If you want to use the electrodes for manual welding (K) place them in the upper terminals and secure them with the two bolts.
- Before working with the wire arc holder or the electrodes for manual welding, turn one of the electrode heads (F) 45° to prevent short-circuits and overheating the machine.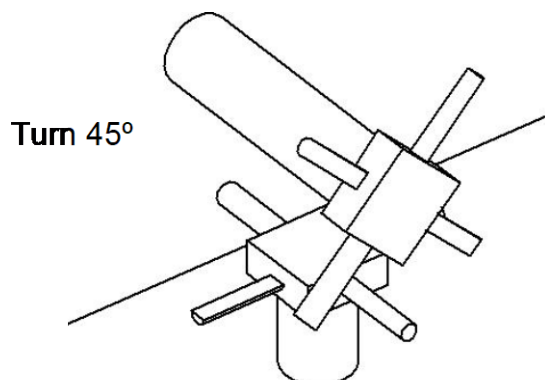

OPERATION

1. When operating the machine, protect yourself properly:
 - Wear safety goggles
 - Wear safety gloves
2. Check out the electrode before welding. If it is rusty, clean it with sandpaper.
3. Connect the unit to a proper power source and turn the main switch (I) on; the unit is ready to work.
4. There are three different operating modes: spot welding mode, continuous mode and continuous mode for silver welding.
5. To prevent damages to the unit, always select the mode and power level appropriate for each work.
6. Once the work has been completed, turn the main switch (I) off.

• **Spot welding**

In the spot welding mode, the machine produces an electrical current of short duration and controlled intensity that weld the two pieces together.

- a. For spot welding, press the SOLDERING key (C) until the display shows one of the following values: **00 – 01 – 02 - ... – 28 – 29**. Each of these values corresponds to a different power level (30 levels). Then select the power level by pressing the SELECT key (E). The higher the value, more power will be used in the welding. The following values may be helpful as a reference:

Level	00-02	03-17	18-29
Union thickness (Ø)	< 0.2 mm	0.2 - 1.6 mm	> 1.8 mm

- b. Turn the two sets of electrodes (F) until the two most suitable electrodes for the task are in position. Then place the work piece between the two electrodes.

If you prefer to use the manual welding electrodes (K), place them in the upper terminals and secure them with the two nuts. Then turn one of the electrode heads (F) 45° in order to prevent short-circuits and overheating the machine.


NOTE: for the spot welding of silver pieces use the thickest electrodes.

- c. To start welding, press the START key (D) or press the pedal.

• **Continuous mode (do not use for more than 5 seconds)**

In the continuous mode the electric current flows while the START key (D) or the pedal are pressed.

- a. To operate the unit in the continuous mode, press the SOLDERING key (C) until the display shows one of the following values: **t0 - t1 – t2 - ... – t8 – t9**. Each of these values corresponds to a different power level (10 levels). Then select the power level by pressing the SELECT key (E). The higher the value, more power will be used.
- b. Place the wire arc holder (A) or the manual welding electrodes (K) in the upper terminals and secure them with the two nuts. Then turn one of the electrode heads (F) 45° in order to prevent short-circuits and overheating the machine.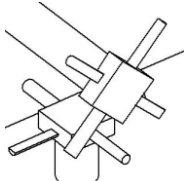


Whenever you operate the unit in continuous mode or in continuous mode for silver welding, turn the electrode heads (F) 45° to avoid contact; otherwise the unit could overheat.

- c. After selecting the parameters and securing the work piece, press the START key (D) or the pedal to start the process.

- **Continuous mode for silver welding (do not use for more than 5 seconds)**

When welding silver pieces, you can obtain better results by operating in the continuous mode. The current flows while the START key (D) or the pedal are pressed.

- To operate the unit in the continuous mode, press the SOLDERING key (C) until the display shows one of the following values: **q0 - q1 - q2 - ... - q8 - q9**. Each of these values corresponds to a different power level (10 levels). You can select the power level by pressing the SELECT key (E). The higher the value, more power will be used.
- Place the wire arc holder (A) or the manual welding electrodes (K) in the upper terminals and secure them with the two nuts. Then turn one of the electrode heads (F) 45° in order to prevent short-circuits and overheating the machine.

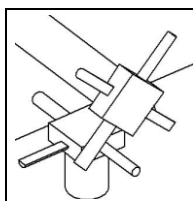Whenever you operate the unit in continuous mode or in continuous mode for silver welding, turn the electrode heads (F) 45° to avoid contact; otherwise the unit could overheat.

- After selecting the parameters and securing the work piece, press the START key (D) or the pedal to start the process.

NOTE: if you use the unit non-stop for a long time, the magnetic field may affect the electronic controls and they may fail to work properly. In that case, turn the main switch (I) off and give the unit a rest for several minutes.

NOTE: if you need electrodes of a different shape, you can modify any of the electrodes supplied with the unit at will with a hand tool.

MAINTENANCE

- Use the electrode sets (F) only for spot welding. In the continuous mode and continuous mode for silver welding the machine may overheat.
- Avoid strong vibrations when using the machine.
- Press the pedal smoothly.
- Turn the electrodes 45° when you are not working with them.
- Place the unit in a dry and well ventilated place to prevent damp.
- Check out the electrode before welding. If it is rusty, clean it with sandpaper. When it wears out, replace it with a spare one.

TECHNICAL SPECIFICATIONS

Voltage:	220 V, 50/60 Hz
Power:	350 W
Height:	160 mm
Length:	335 mm
Width:	220 mm
Weight:	5,5 kg

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