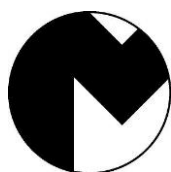
Ref. 100276

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GUARANTEE

All **MESTRA®** units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA®** will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

MESTRA® will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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The Ref. 100276 is a simple polishing lathe especially designed to polish and grind small items in the field of dental mechanics.

INSTALLATION

1. Carefully unpack the unit. Check that it has not sustained damage during shipment.

It is supplied with the following accessories:

- 1 right-hand thread brush holder.
 - 1 left-hand thread brush holder.
 - 1 cloth wheel Ø100.
 - 1 Allen wrench.
2. Place the unit on a flat and firm surface remote from any source of heat or vibration.
 3. Connect the unit to a 230 V, 50/60 Hz outlet with earth.

FITTING THE BRUSH TO THE BRUSH HOLDER

1. Insert the brush into the brush-holder, ensuring that the brush-holder shaft is perpendicular to the plane of the brush.
2. Screw the brush onto the brush-holder as tightly as possible by hand.
3. Visually check that the brush-holder shaft is perpendicular to the plane of the brush by spinning the assembly in your hand.
4. Start up the unit: the spin of the motor will reveal whether the wheel or brush is off line. If there is more than 2 mm of slip, dismount and re-fit the wheel-holder or brush-holder.
5. Remember that the friction torque produced during work after a brush is fitted to the brush-holder will keep it tight. This effect can be favoured by a suitable workpiece attack angle.

Note. Max. disc diameter: 125 mm.

CONDITIONS OF USE

For best results use the unit as follows:

- Connect the unit to a 230 V, 50/60 Hz power source.
- Turn on the main switch (G).
- Select the desired spin speed (3000-7000 rpm) turning selector (B).
- Before polishing or grinding a workpiece, check the motor spin direction, as the initial workpiece attack angle depends on this. Incorrect attack angles can be extremely dangerous.
- The following protective equipment is recommended for polishing or grinding:
 - safety goggles;
 - safety gauntlets;
 - face-mask;
 - proper exhausters;
 - a protective screen suited to the characteristics and shape of the unit. TALLERES MESTRAITUA, S.L. recommends Ref. 080200 / 080210 screens, which are especially designed for use with the polisher.
 - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
 - Be careful with long hair.
 - No objects similar to ropes or chains that could be hooked by the machine.
- After using the unit, clean the inside of the protective screen.

MAINTENANCE AND CLEANING

To ensure a long, trouble-free useful life for your unit, proceed as follows:

- To clean the motor and base use a cloth. Never use alcohol, solvents or other inflammable liquids.
- Never immerse the unit in any liquid.
- If you notice unusual noises or vibrations, send the unit to your technical assistance service.
- This unit is designed to work correctly on the four rubber legs (C).
- The motor has two brushes that will wear out with the use. To replace them, remove the plugs (D) using a screwdriver.

PRECAUTIONS

- Before connecting the unit to the mains, check that the power supply is 230 V, 50/60 Hz.
- Discard any cracked or damaged brush or grinding wheel: if these elements break during operation, they could cause serious accidents.
- Before starting work, ensure that the brush-holder/wheel holder is firmly attached to the motor shaft.
- When handling workpieces, use safety goggles, gauntlets, a face mask and a suitable safety screen. We recommend MESTRA Ref. 080200 / 080210 screens.
- Do not allow children or unskilled personnel to handle the unit.
- If the motor shaft locks during normal operation, disconnect the unit from the mains before attempting to release it by turning it manually.
- Never polish, brush or grind materials which may give off particles which are toxic if inhaled.
- Before starting to polish or grind a workpiece, check the motor spin direction: the attack angle depends on this direction.
- Never use the unit close to solvents, alcohol or other inflammable materials. Take all possible precautions when grinding materials which may give off sparks.
- Never use the unit without proper exhausters.
- Keep the working area clean and tidy in order to prevent accidents.
- Wear proper working cloth. Loose cloth, hanging or long hair could be caught by the machine.
- Make sure that you switch the power Off before plugging the unit, in order to avoid sudden start of the motor.
- Disconnect the unit from power supply before changing the wheel.

TECHNICAL CHARACTERISTICS

Power supply:	230 V, 50/60 Hz
Power:	1300 W
Speed:	3000-7000 rpm
Height:	16,5 cm
Height of shaft centre line:	12 cm
Width with brush holders:	36 cm
Weight:	6 kg