
MANUAL DE INSTRUCCIONES INSTRUCTION MANUAL

SAESHIN

LISTA DE PIEZAS / PART NAME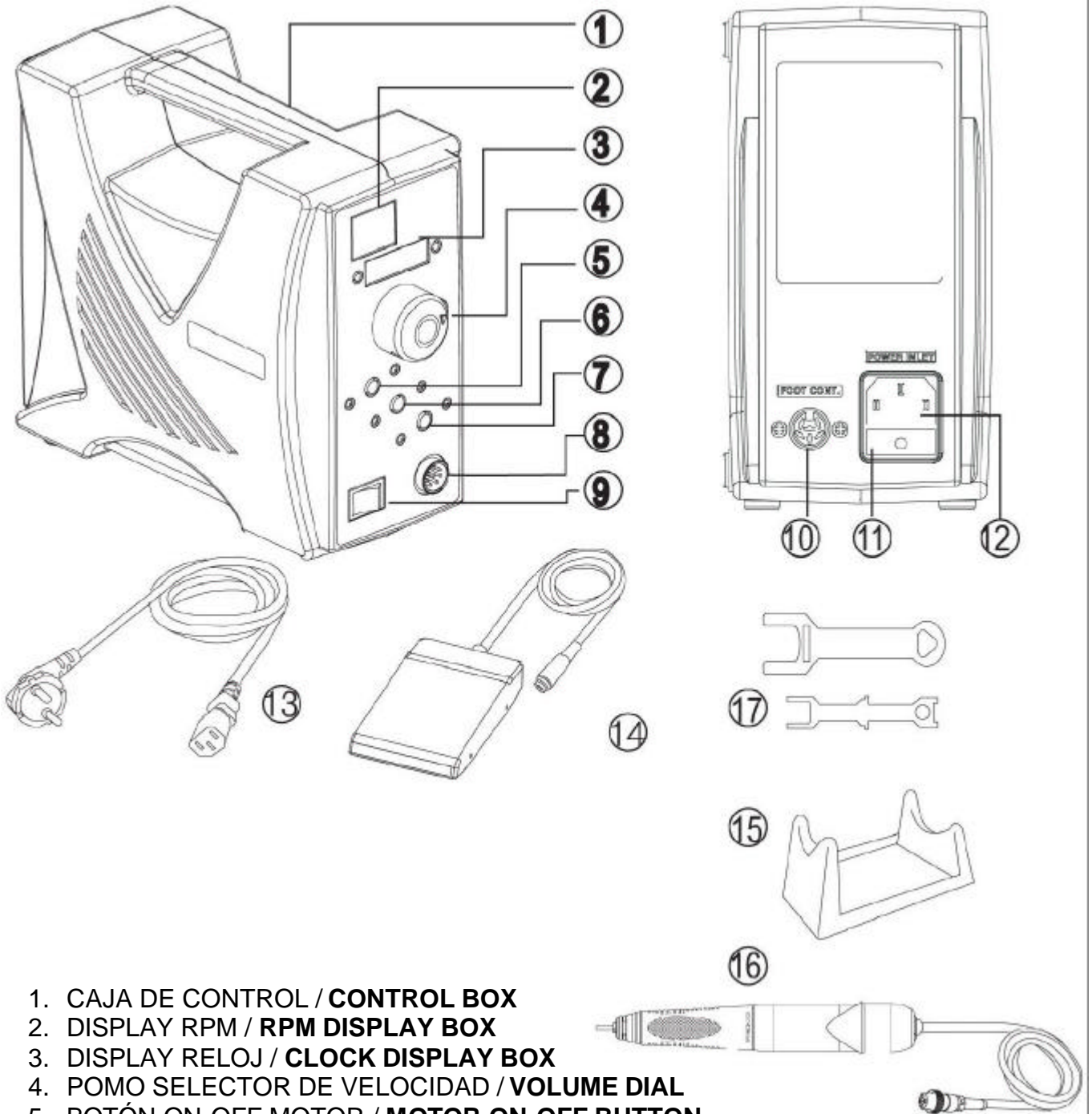

1. CAJA DE CONTROL / **CONTROL BOX**
2. DISPLAY RPM / **RPM DISPLAY BOX**
3. DISPLAY RELOJ / **CLOCK DISPLAY BOX**
4. POMO SELECTOR DE VELOCIDAD / **VOLUME DIAL**
5. BOTÓN ON-OFF MOTOR / **MOTOR ON-OFF BUTTON**
6. BOTÓN MANUAL-PEDAL / **HAND-FOOT BUTTON**
7. BOTÓN DERECHA/IZQUIERDA / **FORWARD-REVERSE BUTTON**
8. CONEXIÓN MOTOR / **MOTOR CONNECTION**
9. INTERRUPTOR DE ALIMENTACIÓN / **POWER SWITCH**
10. CONEXIÓN PEDAL / **FOOT PEDAL CONNECTION**
11. CAJA FUSIBLE / **FUSE BOX**
12. TOMA DE CORRIENTE / **POWER CONNECTION**
13. CABLE / **POWER CORD**
14. PEDAL DE CONTROL DE VELOCIDAD / **FOOT SPEED CONTROL PEDAL**
15. SOPORTE PIEZA DE MANO / **HANDPIECE STAND**
16. PIEZA DE MANO / **HANDPIECE**
17. LLAVES / **WRENCH**

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* Read all instructions before using. This product has been designed for being used at dental labs.

Thank you for purchasing our products.

Proper maintenance and repair brings long use of the products.

Keep the instruction for use for your reference.

The manufacturer is not responsible for accidents caused by an use not specified in the instructions.

Product appearance or specification can be subject to be changed without notice for the improvement.

The users are responsible for the operation and maintenance of the products.

By providing the model name and breakage condition, you will obtain a quick answer to your question.

WARRANTY

Handpiece - 1 year
Control box & pedal - 1 year

Warranty does not include misuse or normal wear of bearing, carbon brush, collet chuck, spindle and armature.

Other cases

- Faulty by natural causes (fire, sea wind or water, flood etc.).
- Replacing by the normal wear of the consumable parts (ball bearings, collet chuck, spindle, armature, etc.).


SERVICE CHARGE

In case of the fault or malfunction from the misuse of the user:


- Using with wrong input voltage.
- In case of user's misuse or improper use.
- Dropping down during setting or moving.
- Using consumable parts or accessories not recommended by manufacturer.
- Malfunction after repairing at the places not recommended by manufacturer.

* Design and specification of the product can be subject to be changed for improving the quality.

PRODUCT SPECIFICATION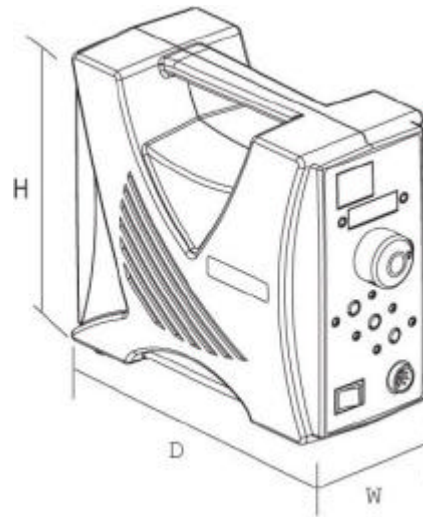


INPUT POWER SOURCE	AC 220 ~ 230 V 50/60 Hz
FUSE	2 A
WEIGHE	2.5 kg
DIMENSIONS (mm)	(W) 96 (D) 207 (H) 182
MAXIMUM LOAD CURRENT	2 A

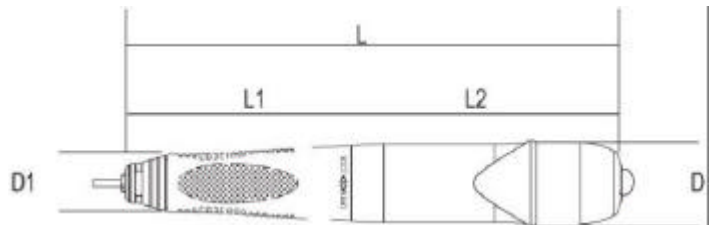NON LOAD RPM	50000 RPM				
OUTPUT	230 W				
MAX. TORQUE	7.8 N.cm				
WEIGHT	245 g				
DIMENSIONS	L	L1	L2	D	D1
	174	78	93	27	18

SETTING & OPERATION

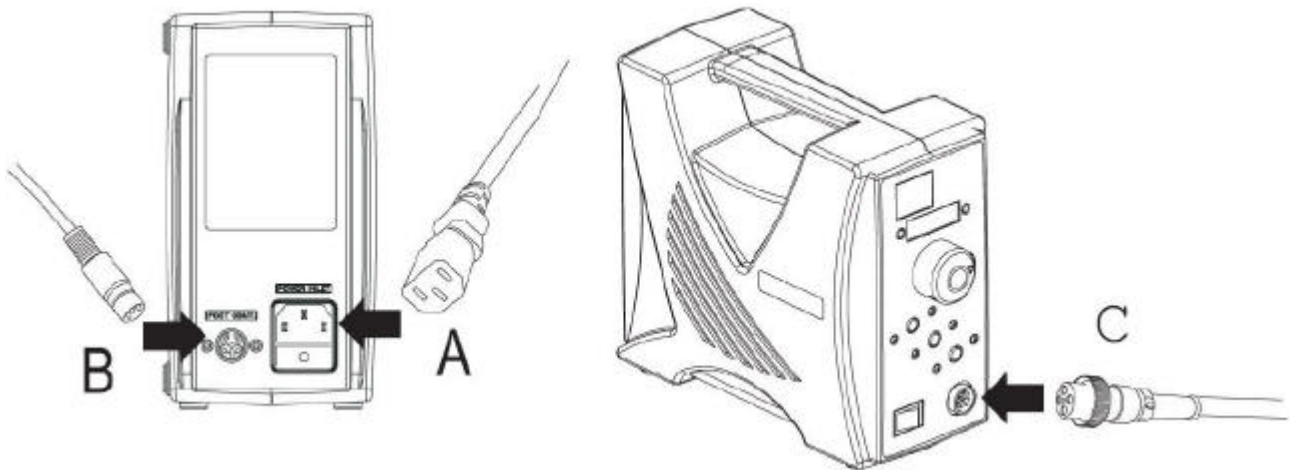Warning

Please make sure that power is off! (Otherwise, it might cause malfunction).

- 1 Check the power of the control box is OFF.
A- Connect the power cord.
B- Connect the foot control pedal to the foot pedal connector.
C- Connect the handpiece to the handpiece connector.
- 2 Put the speed control dial of the control box at "0".
- 3 Check the well-locked collet chuck turning the handpiece handle or pulling out the bur.
- 4 Hold the handpiece with one hand and turn on the control box power with the other hand.

POWER ON (I)

Turn on the power switch.

With bipper sound, it lights green on the power switch and shows the numbers on the rpm display box.

POWER OFF (O)

The green light on the power switch and the lights of the buttons are gone off. The number of the rpm display box is disappeared.

- 5 Please set the time on the digital display. Please set the hour with the left button and minute with the right button. There is no need to reset the time after turning on again. Hour display is shown from 01 to 24.

- 6 Select whether controlling by hand or foot by pressing the HAND/FOOT button. Each pushing, it changes Hand or Foot.

- 7 Choose the rotation direction by pressing the RIGHT/LEFT button. Each pushing, it changes Right and Left.

- 8 Use the motor ON/OFF button for running or stopping by hand or foot.

- 9 How to change the speed from 40000 rpm to 60000 rpm.
Put the volume dial at "0" and then press the HAND/FOOT button and MOTOR button at a time.
The RPM display will be changed 50000 rpm or 60000 rpm. 40 ⇌ 60
After the repetition, the RPM display will be returned 40000 rpm. 40 ⇌ 50
At max. 40000 rpm, the locking system works at 30000 rpm.
In case of max. 50000 or 60000 rpm, the RPM which the locking system functions is variable.


Operating: controlling by hand

- Push the HAND/FOOT button to select the Hand mode.
- Check if the handpiece works properly while adjusting the speed dial of the control box after pushing the motor button.

Operating: controlling by foot

- Push the MANUAL/PEDAL button to select the Foot mode.
- Check if the handpiece works properly while adjusting the pedal after pushing the MOTOR button.
The speed can be changed within the range of speed preset by the speed dial of the control box.

10 In case of overload

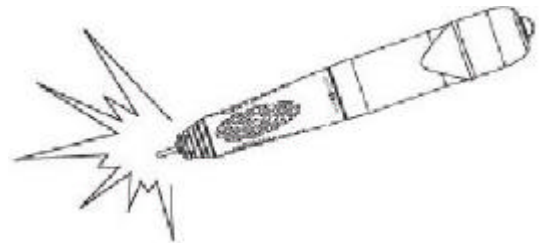In case of overload, rpm display box shows 00 and after 5 seconds, control box stops and shows Error 1 with beeper sound. For releasing, push motor button working by hand or take off the foot from the pedal in case of working by pedal.

SUGGESTION FOR MAINTENANCE

1. Clean the handpiece surroundings.
2. Remove the dust on handpiece with smooth clothes and avoid wet inside.
 - Do not apply oil or clean with water.
The grease inside of the bearings could melt away due to the oil.
3. Air cleaning is acceptable, but air pressure may badly affect to the hand piece.
4. Bur insertion and pulling out:
Open the chuck turning handle to the open direction. Chuck is released and bur is removed.
Turning the ring to lock direction, chuck is locked and bur is fixed.
Turn the handle until click sound.

WARNING

Do not disassemble except skilled technician. It may cause damage.

CHECK POINT IN CASE OF PROBLEM

If the product does not work after checking, please contact to the seller or manufacturer.


Problem	Check	Cause	Solution
The motor does not run	The power lamp is out when the power switch is on	The control box may have a problem	Check the fuse. Check the power switch.
	The handpiece does not run when the hand/foot switch is on and power lamp is on	Check the connection of power cord	Connect the power cord properly.
	The handpiece does not run, when the power lamp is on	Check if the chuck is open (E1 error)	Lock the chuck.
		Check the connection between the control box and handpiece cord	Connect the handpiece properly.
		Check if the cord is broken down or disconnected	Ask to the seller.
The RPM does not increase	The control box may have a problem	Ask to the seller.	

CAUTIONS

1. Memory:
Memory saves the conditions of direction and hand foot selection on turning off the power.
Therefore, be careful when the power is on.
2. Do not work at a dangerous place.
Do not use the control box in a humid place.
3. Transport and storage conditions:
Temperature: 10 – 40 °C
Humidity: 30 – 75 % RH
Atmospheric pressure: 700 hPa – 1060 hPa
4. Our handpiece and pedal need to be used with only our control box.
5. Check for any troubles, including vibration of bur, at low speed, and then start high speed.
6. Operate the speed control dial at control box softly.
7. Turn off the power or set the RPM at zero and then change the bur.
8. Do not turn on the power of control box while the collet chuck of handpiece is open (E1 error).
9. Maximum temperature of applied parts:
The head of handpiece is applied parts.
The maximum temperature of applied parts are below:
39 °C at ambient 25 °C or 54 °C at ambient 40 °C.
Therefore use this parts carefully

Please keep the RPM according to below diagram against the accident.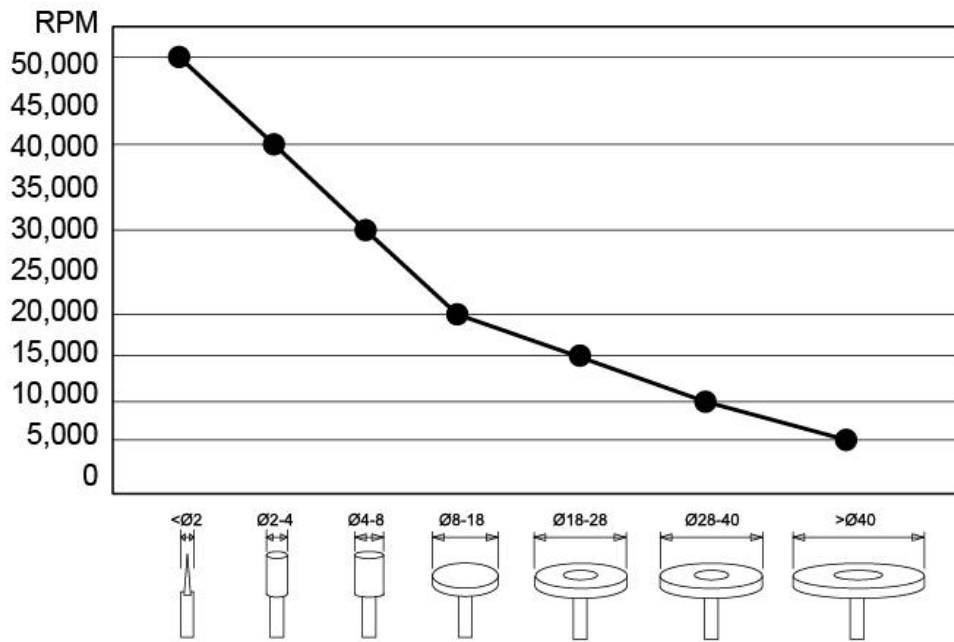

(The allowable RPM could be different depending on the bur material and type).

10. Bur

- Bur dimension: 2.35 mm (ISO7785-2 TYPE2).
 - Minimum fitting length of bur: 10 mm.
 - When locked chuck, the maximum projection length of bur should be at least 35 mm.
 - Use the bur ISO standard and avoid using a rusty and bent bur causing excessive wear of bearings.
- Keep safe speed and precautions recommended by bur manufacturer.

11. Be careful not to drop the handpiece as it may be damaged or broken. In case that it is dropped, check if the bur is bent or excessive heat is generated while operating.

12. If any problem occurs in handpiece, use after repairing.

13. Make sure that the switch is off before putting the power plug in the socket, in order to avoid an unexpected injury caused due to sudden start.

14. A circuit breaker is provided to protect the motor. It is actuated when there is an

overload switch the motor cannot stand. Work with a range of load which it is not actuated.

15. Do not open the chuck while operating.

16. Keep the handpiece with setting a bur to the collet chuck after operation.

17. Do not put the handpiece down before it stops completely.

18. Be sure to pull out the power cord before detaching the control box.

19. Use the specified voltage.

20. Use accessories suitable for this product.

21. For recommended speeds in various applications, follow the manufacturer instructions.

22. Wear proper protective clothes and equipment, including a dust mask and safety goggles. Use a proper suction unit and a ventilator.

23. Keep the workplace clean. Debris may cause an accident.

CHANGING FUSE







Fuse is placed in the small box beside of the power connector at the back side.
Take out the fuse box and change the fuse.

ERROR CODE

In case of overload, breakage and motor coil burning, motor stops and control box shows the error code on rpm display box. From this function, the reason of the breakage or faulty and conditions of each part can be noticed.

Error code	Cause	Releasing	Action
E1	Setting error Caused by abnormal motor initial operation		Check the power cord and motor connector connected well. Check if the chuck is open. Turn the power on again. If runs normally, error will be disappeared. Make inquiries to manufacturer in case of occurring same error
E2	Over current error Occurs in case of using the handpiece for a long time on overloaded, the wrong assembling after disassemble and f troubling of bearings and motors.	Push the motor button, or foot controler	Restart running the control box after keeping it stops for about 10 minutes. If the same error occurs continuously, ask to manufacturer.
E3	Control box error		Ask to manufacturer.
E0	Handpiece connect error		Please, disconnect the handpiece and reconnect again. In the same thing happens, please contact our headquaerters company.

CHANGE COLLET CHUCK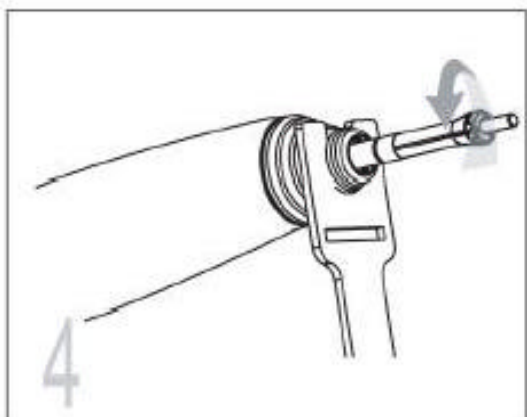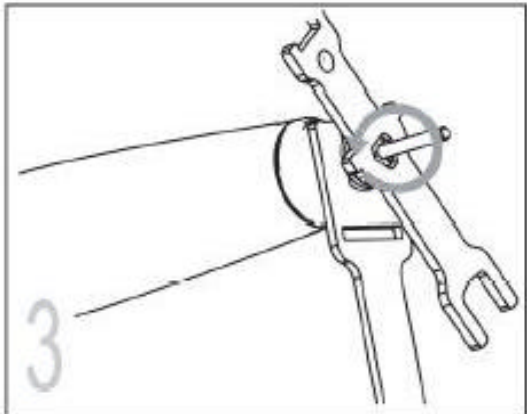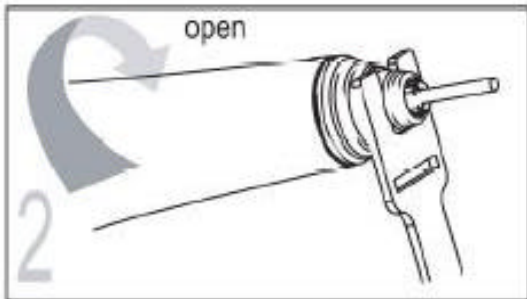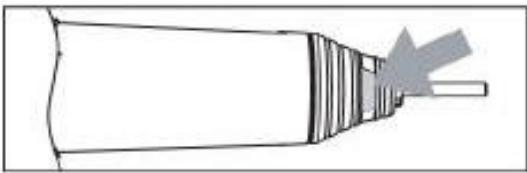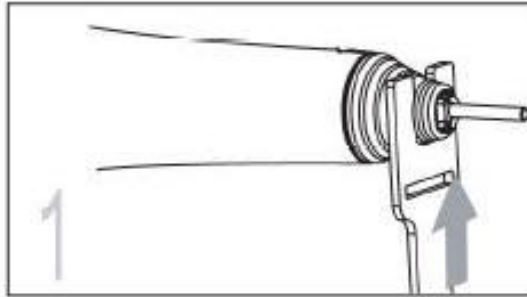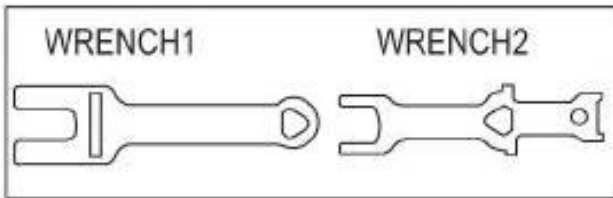

1. Set and fix the wrench turning the bur.
 2. Turn to open direction. Chuck open.
 3. Turn the chuck to the counter-clock wise. In case of not turning by hand, use wrench (4).
 4. Change the chuck.
- Fix the bur and chuck turning to clock wise. And then turn again to counter clock wise a bit and set the tension to remove the bur.

Attention


Clean and assemble the chuck very carefully.

If not skilled technician, do not disassemble the handle and motor part. It may cause the breakage.

Delin joint of handle part and the shaft of the motor part have to be fit each other.

If it doesn't, vibration, noise, heat and bearing damage may be caused.

- If the chuck is not open, please torn it with the wrench.

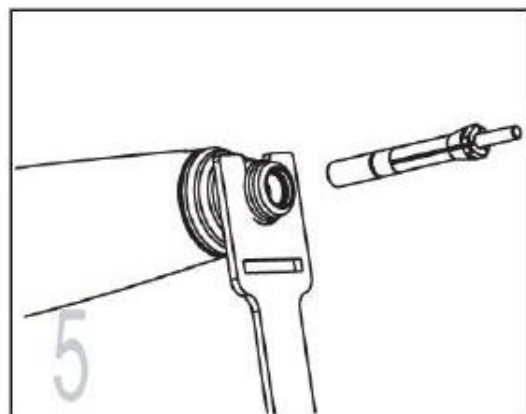For your safety, keep to the following

Warning

Do not use with damaged or faulty power cord, plug and outlet. Contact to the manufacturer. It may cause the danger of fire, electric shock.

When unplug the power cord, hold the plug and do not touch with wet hand. Do not use with loosing power plug causing the danger of electric shock and fire.

Do not bend the power cord excessively and do not press with heavy things. It may cause electric shock and fire.

Do not place near heat equipment and candles or cigarette causing fire.

Close supervision needs if this product is used by or near by the very young, the elderly or invalid.

Do not leave children unattended in any room where this product has been installed.

- ? Type of protection against electric shock: Class I equipment.
- ? Degree of protection against electric shock: Type B applied parts.
- ? Classification according to the degree of protection against ingress of water as detailed in the current edition of IEC 60529:
Equipment: IPX0, ordinary equipment
Foot switch: IPX1
- ? This equipment is not suitable for use in the presence of flammable anesthetics or oxygen.
- ? Mode of operation: continuous operation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does

cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving device.
- Increase the separation between the equipments.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) are connected.
- Consult the manufacturer or field service technician for help.

Use the grounding conductor.

Breakage or electric leak may cause the danger of electric shock.

For grounding, inquiry to electrician or to main office.

In case of using extension cord, do not use many electricity equipments in one time. It causes fire and electric shock.

Do not place or set up the product in the humid and rainy or snowy location.

It brings the danger of electric shock and fire.

Do not drop or damage the handpiece.

In case of malfunction, dropping, drowning, do not use and contact to the manufacturer (it may cause vibration of the bur, overheating, or damage of the bearings).

* Use the products in proper use as per the instruction and do not use not recommended by the manufacturer.

* It may cause injury or property loss if not following the instructions.

* Saeshin is not responsible for the accident causing by not following the instructions.

www.saeshin.com / sales@saeshin.com

MESTRA – TALLERES MESTRAITUA, S.L.
TXORI-ERRI ETORBIDEA 60,
48150 SONDIKA, BILBAO, SPAIN
T.34-944530388 / F.34-944711725

 **SAESHIN PRECISION CO., LTD.**

HEAD OFFICE & FACTORY 
#93-15 Paho-dong, Dalseo-gu, Daegu, Korea
Tel. 82-53-587-2340~1 / Fax. 82-53-587-2347

EU representative **EC REP**
Dongbang Acuprime
Gater House, Gater Lane, Palace Gate,
Exeter, Ex 1 1JL. United Kingdom
Tel. 44-1392-273908 Fax, 44-1393-273909
2010. 12 SOM-L006S-KE **Rev.4**