MESTRA®

**MANUAL DE INSTRUCCIONES
INSTRUCTION MANUAL**

FORTE

FORTE 200α

SAESHIN

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INTRODUCTION

Make sure to read this user manual before using. This user manual is to assure proper installation and use. Pay attention to read this manual in order to making the best use of this product and to assure the prolonged lifespan before using.

Specially, pay attention the contents with marks such as ,  and .

<Classification of equipment>

- Type of protection against electric shock
 - Control Box: Class II appliance
 - Handpiece: Class III construction
 - Foot Controller: Class III construction.
- Not suitable product in the atmosphere that exist flammable anesthetic mixture with air or with oxygen.
- Classification according to mode of operation: Intermittent operation.

Design and specification of the product can be subject to be changed without notice for improving the quality.

USER GUIDELINES

WARNING

- The setting conditions (Hand/Foot Select Button, Rotation Direction Select Button) are saved when power is off. Pay attention to use at the power on.
- Do not use as condition the collet chuck is opened. It causes product's malfunction.
- Do not open the collet chuck while the handpiece is operated.
- Check the power switch before connecting the power cord in order to decrease the danger by sudden operation of product.
- Connect the power cord after setting speed control dial at the off place (the end of counterclockwise).
- Circuit breaker is provided to protect the motor. It is actuated when an overload occurs which the motor cannot stand. Always use this product within allowable load range.
- Store the product as condition a bur is fixed at the collet chuck.
- Change the rotation direction after stopped the handpiece.
- Do not use the damaged bur. It causes product's malfunction and accident.
- Do not work in a dangerous place. Do not use the control box in a humid place.
- Do not disassemble or modify the product as one pleases. It causes accident.
- Use the compatible handpiece and foot controller with control box.
- Check the problems such as excessive vibration of bur, defective speed control and etc.
- Handle the speed control dial smoothly.
- Replace the bur after stopped the handpiece.
- Be careful not to drop the product as it may be damaged or broken. In case that it is dropped, check if the bur is bent or that any overheat is generated while carefully operating it.
- If any problem occurs in product, use after repairing.
- Do not put the handpiece down before it stops completely.
- Use the appropriate voltage for the product.
- Use the appropriate accessories for the product.
- Wear proper protective clothes and equipment, including a dust mask and safety goggles.
- Keep the workplace clean. Debris may cause an accident.
- Do not touch the power cord and handpiece cord with wet hand.
- Do not bend or press the power cord excessively. It causes fire or electric shock.
- Do not place the product nearby superheater, candle, lighter and etc. It causes fire.
- If use the extension cord, do not use many electric device at one time. It causes fire or electric shock.
- "This appliance can be used by children aged from 8 years and above and person with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in safety way and understand the hazards involved." "Children shall not play with the appliance." "Cleaning and user maintenance shall not be made by children without supervision"
- "If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard."
- Follow the user manual by manufacturer of bur to use bur.
- Keep the recommended speed limit by manufacturer of bur to use bur. If exceed the speed limit, it might cause an accident.

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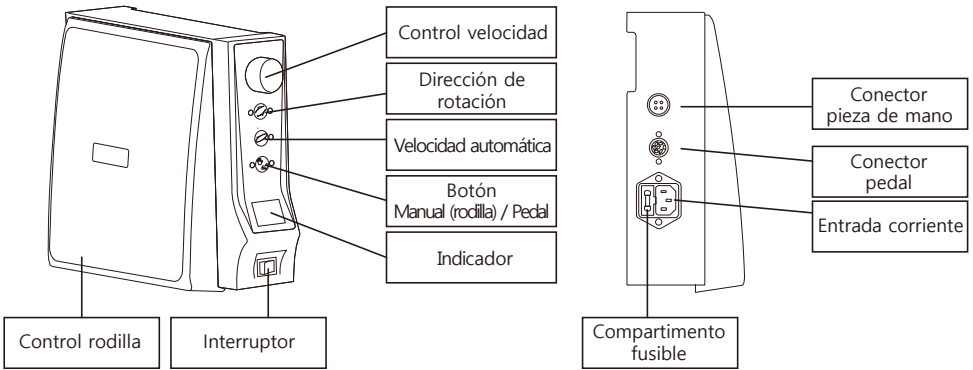
· Please check all components are included.

No.	Componente	Component
1	Caja de control (FORTE 200α)	Control Box (FORTE 200α)
2	Pieza de mano (FORTE 100αIII)	Handpiece (FORTE 100αIII)
3	Pedal (opcional)	Foot Controller (Option)
4	Cable de alimentación	Power Cord
5	Soporte para pieza de mano	Handpiece Stand
6	2 llaves	2 Type Spanner
7	Soporte de pared	Wall Mounting Bracket
8	Tornillos, pernos, tuercas	Bracket Fixing Screw, Bolt, Nut

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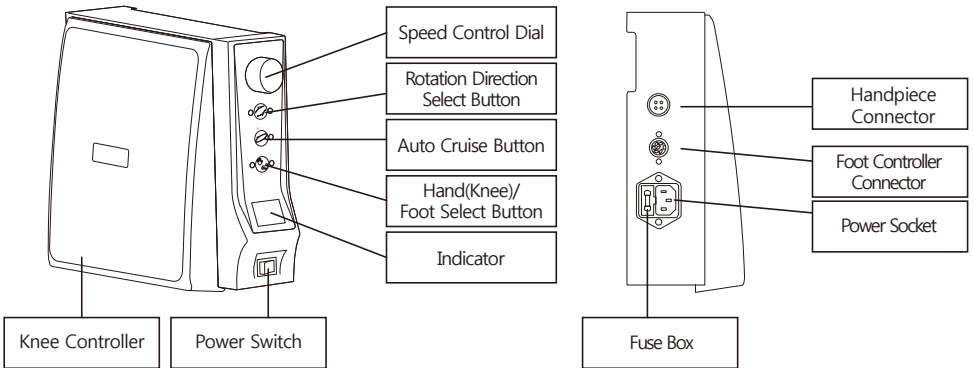
BUTTONS ON THE OPERATION PANEL

■ Speed Control Dial:

This dial is used to control RPM. Turn the clockwise, RPM is increased. Turn the reverse, RPM is decreased.

■ Rotation Direction Select Button:

This button is used to select rotation direction to the clockwise or reverse.

■ Auto Cruise Button:

This button is used to fix the speed of handpiece when operation by handpiece.

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■ Hand(Knee)/Foot Select Button:

This Button is used to select the handpiece operation method.

- Hand(Knee): Handpiece is operated by 'motor button'.
- Foot: Handpiece is operated by foot controller.

■ Indicator:

- RPM: RPM is displayed. Displayed number X 1,000 is actual RPM.
- Error Cord: When errors occur, the corresponding errors are displayed.

■ Power Switch:

This switch is used to turn on the power or off.

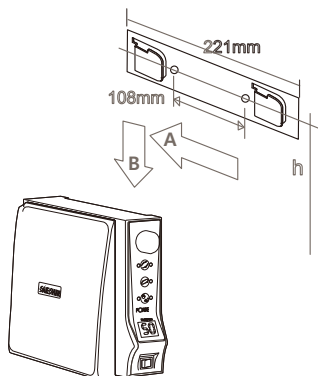
- I: Power On
- O: Power Off

■ Knee Controller:

This button is used to operate the handpiece when select operation method by hand(knee).

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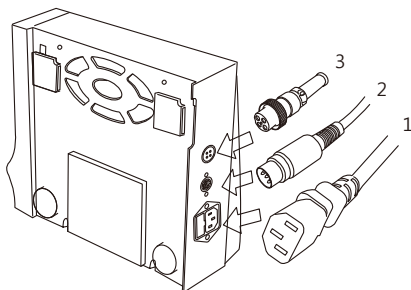
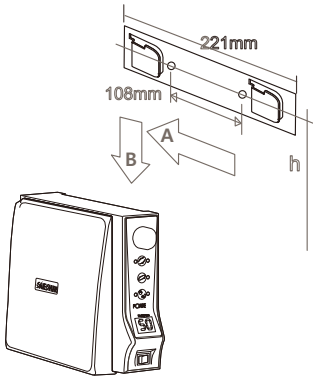
CONNECTION OF EACH PART

Wall Mounting Bracket Installation


1. Using the screws and bolts, Mount the wall mounting bracket at the lower side of workbench as the picture.

- Pay attention to not pop out the head of screw or bolt than surface of bracket.

2. After look the hole position, mount the control box to mounting bracket as direction like picture(A->B) without shaking.

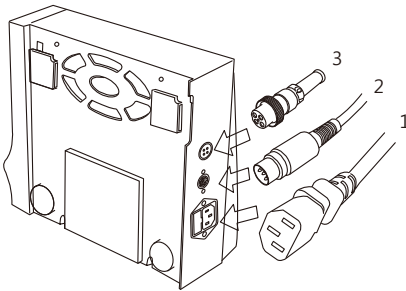Control Box Connection

1. Connect the power cord to the power socket.

- Before connect the power cord, make sure to check the power off.
- Make sure to check that the speed control dial is set to '0'.

2. Connect the foot controller cord.
3. Connect the handpiece cord.

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■ Bur Insertion

1. Turn the handle to open the handpiece chuck.

2. Insert the bur in the front of handpiece.

3. Lock the handpiece chuck by turning the handle.

※ Bur connection method is same regardless of the handpiece type.

- Be sure to lock the chuck before use. It may cause damage to the handpiece.
- Make sure that the bur is not damaged.
- Check the Bur dimension. (Bur Dimension: $\Phi 2.35\text{mm}$)

● OPERATION PROCESS

1. Put up the handpiece and turn on the control box.

(When the power is turned on, the beep sound is ring and the RPM is displayed on the indicator.)

- The first characters shown on the indicator when the power is turned on are software versions.

2. Select the rotation direction by pressing the Rotation Direction Select Button. When the left light is on, the forward direction is selected, when the right light is on, the reverse direction is selected.
3. Select the operation method by pressing the Hand(Knee)/Foot Select Button. When the left light is on, hand(knee) is selected, when the right light is on, foot is selected

■ How to use when select the hand(knee)

4. Control the desired speed by turning the speed control dial.
5. Operate the handpiece by controlling the knee controller. (Handpiece is worked within setting range.)
6. Knee controller is released, handpiece is stopped

■ How to use when select the foot

4. Control the desired speed by turning the speed control dial.
5. Operate the handpiece by controlling the foot controller. (Handpiece is worked within setting range.)
6. Foot controller is released, handpiece is stopped.

■ Auto Cruise Function

It is function to fix the speed of handpiece when operation by handpiece.

Operate the handpiece at the desired speed with the auto cruise function activated. Hold the speed for 3~5 seconds, the speed is kept with beep sound.

To release the function, press the auto cruise button or press the knee controller or foot controller again.

■ Maximum Speed Change Function (FORTE 200 α , FORTE 200 α d Only)

It is function to change the maximum speed from 40,000 rpm to 50,000 rpm.

Press the Hand(Knee)/Foot Select Button and Auto Cruise Button. Maximum speed is enlarged to 50,000 rpm. Re-press the two buttons, Maximum speed is returned.

■ Overload Protective Function

If handpiece is overloaded, after about 5 seconds, beep sound is ring and the error code 'E1' is displayed at the indicator and handpiece is stopped. To release the function, press the knee controller or foot controller again.

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■ Error Code

Error Code	Cause	Remedy
E0	<ul style="list-style-type: none"> · Defective connection of handpiece · Abnormal operation of motor 	<ul style="list-style-type: none"> · Check the connect condition of power cord and handpiece cord. · Re-turn on the power. · If the problem is not resolved, contact the manufacturer.
E1	<ul style="list-style-type: none"> · Handpiece Overload 	<ul style="list-style-type: none"> · Press the knee controller or foot controller. · Operate after cooling for about 10 minutes at the cool place. · If the problem is not resolved, contact the manufacturer
	<ul style="list-style-type: none"> · Operation as condition open the chuck 	<ul style="list-style-type: none"> · Use after closing the chuck of handpiece.
	<ul style="list-style-type: none"> · Defective handpiece assembly · Defective bearing · Defective motor 	<ul style="list-style-type: none"> · Contact the manufacturer.

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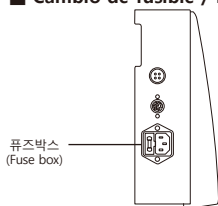
 Marcado Ce

WARRANTY

MESTRA guarantees out product for 1 year from invoice date, and product warranty means that MESTRA has responsibility of defective material or operation failure. The product warranty does not cover user's misuse, wrong installation, inappropriate maintenance and repair, and normal wear of consumables such as bearings, spindle, gear, and motor.

To check the warranty requires the operating condition, environment information, serial number of the product and stamped invoice. The warranty will be performed as repairs or exchanges by manufacturer's judgement and analysis of product.

Exception of warranty

- User's misuse or improper use and treatment.
- Use the product with incorrect input voltage (AC voltage).
- Drop the device during setting, moving, using.
- Use the not recommended consumables or accessories.
- Malfunction after repairing the product at the not recommended repair shop.
- Normal wear of the consumables such as ball bearing, gear, chuck, spindle case ass'y, motor.
- Force majeure.

Operating condition	Temperature (°C)	10-30
	Relative humidity (%)	10-80
	Atmospheric pressure (hPa)	700-1,060
Transport and Storage condition	Temperature (°C)	- 20-60
	Relative humidity (%)	10-90
	Atmospheric pressure (hPa)	500-1,060

SYMBOL

 On (Power)

 Off (Power)

 Manufacturer

 Consult Instruction For Use

 EC Representative

 General Warning Sign

 Serial Number

 Class II appliance

 Fuse

 Alternating Current

 This product should not be mixed with other commercial waste for disposal.

 CE Marking

FORTE

TALLERES MESTRAITUA S.L.
info@mestra.es - www.mestra.es
Tel. +34 944530388
Txori-Erri Etorbidea, 60
48150 SONDIKA - BILBAO - ESPAÑA

Rev. 09/06/2023