**INDIVIDUALIZADOR DE MODELOS
MODEL INDIVIDUALIZER
INDIVIDUALISATEUR DE MODELES**

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INSTRUCTIONS FOR USE

PREPARATION OF MODELS

Once the impression is taken, in either alginate or silicone, the palatine cavity is no longer of any interest, and indeed gets in the way emptying it onto the pan. It also necessitates too great a thickness of plaster. Therefore, cut away with a scalpel as much as possible of what protrudes from the plane of the dish.

EMPTYING THE IMPRESSION

Do this by adjusting to the individualizing pan, resting the impression on it so that the plaster is not too thick. The white expulsion plate should already be positioned so as to cover only the expulsion slot without overshooting it, and should be filled with the same plaster as used to empty the impression.

CLEANING THE MODEL

Once the plaster is hard, remove the expulsion plate and clean the plaster off the model. To achieve a good presentation,

R-050340

eliminate as much of the excess as possible from the model, through both the vestibulum and the palate.

ARTICULATION

Place the anchoring buttons in the pan. Position the model against its antagonist, attaching it with a few drops of wax, and fit the articulator as usual with white plaster.

INDIVIDUALIZATION

Dismount the model from the articulator and wash it to get rid of any traces of wax and plaster. Dismount the attachments holding the model on the plate and fit it onto the expulsion plate so that it can be extracted. Now the necessary pins can be sown. The plate's guides keep it correctly positioned.

All parts and the rest of the model should be kept free of dust and traces of plaster, etc., as this would make for a poor fit.

Note: Before emptying the impression is advisable to mark the line on the outside of the dish so that it will coincide with the pan and the model will be properly centred.

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