


**MESTRA<sup>®</sup>**  
NEGATOSCOPIO  
X-RAY VIEWER  
NEGATOSCOPE  
R-070460

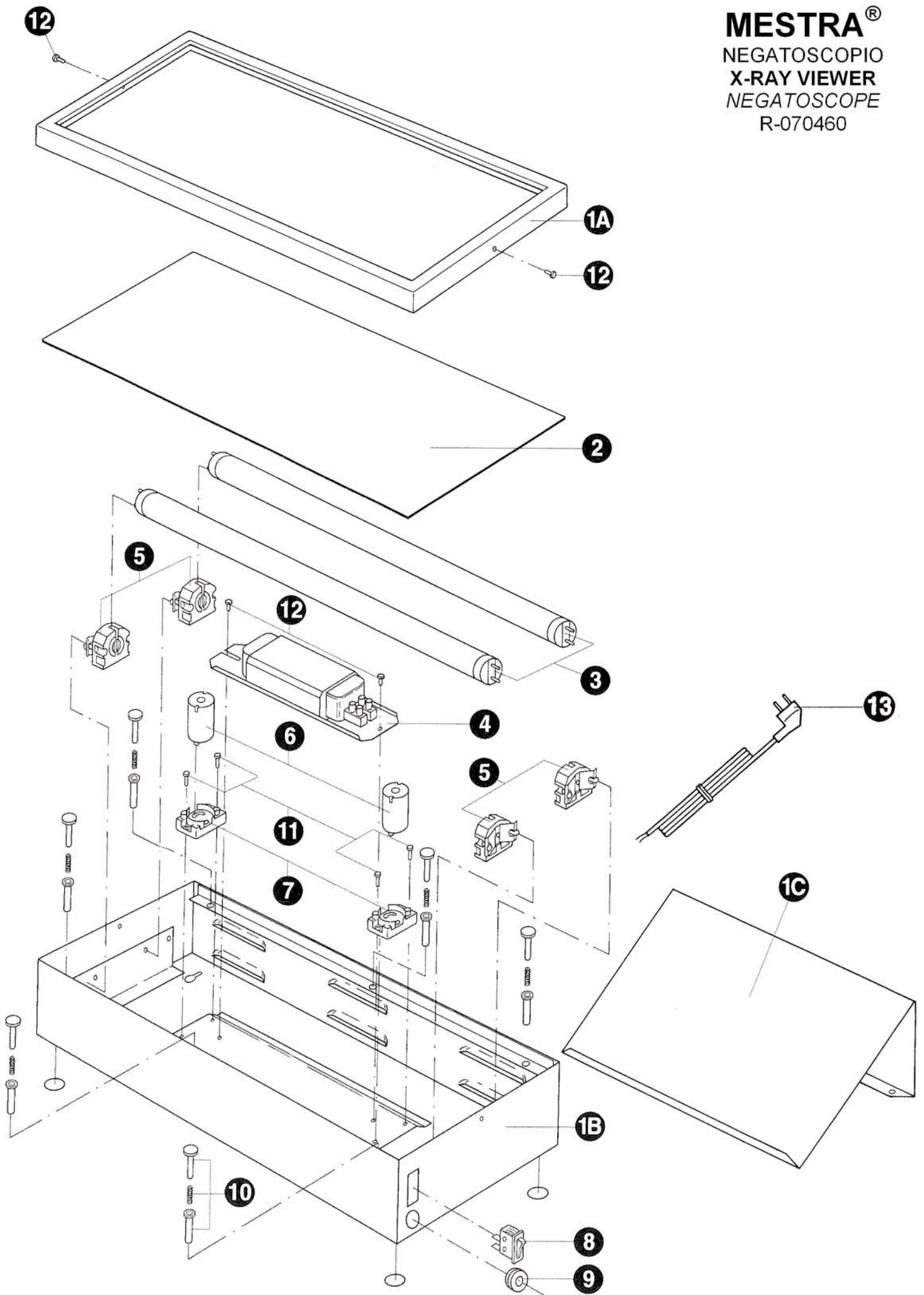

# MESTRA®

NEGATOSCOPIO

X-RAY VIEWER

NEGATOSCOPE

R-070460

Nº	Ref.	Descripción	Description
1A	070460 - 13	Carcasa tapa	Lid housing
1B	070460 - 14	Carcasa base	Base housing
1C	070460 - 15	Carcasa soporte	Support housing
2	070460 - 02	Metacrilato	Meth-acrylate
3	070460 - 03	Tubo fluorescente 230V	Fluorescent lighth 230V
	070460 - 03B	Tubo fluorescente 115V	Fluorescent lighth 115V
4	070460 - 04	Reactancia 230V	Reactance 230V
	070460 - 04B	Reactancia 115V	Reactance 115V
5	070460 - 05	Porta lámparas	Lamp holder
6	070460 - 06	Cebador	Starter
7	070460 - 07	Simón porta cebador	Starter holder
8	070460 - 08	Interruptor	Switch
9	080010 - 05	Pasa cables de goma	Cable bushing
10	070460 - 10	Conjunto amortiguador	Damper assembly
11	070460 - 11	Tornillos porta cebador	Starter holder screw
12	070460 - 12	Tornillos reactancia	Reactance screw
13	000200	Cable 230V	Cable 230V
	000202	Cable 115V	Cable 115V