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The Ref. 080001 is a micro vacuum pump specially designed for use with MESTRA Ref. 060220 and Ref. 060230 blenders. To ensure optimum performance, read the following instructions:

- Set up the micro-pump on a flat surface well away from sources of heat and moisture. The unit has 4 rubber bases (A) to ensure firm attachment.
- Connect the vacuum tube (B) to the connection point on the blender lid.
- Plug the unit into a 230 V, 50/60 Hz earthed power socket.
- Fill the blender body with the materials to be blended, position the lid and ensure that it is properly sealed (i.e. that the rubber seal on the blender lid fits tightly).
- Activate the main switch (C). The micro-pump will start up. After a few seconds the vacuum inside will ensure that the blender lid is tightly fixed to the body. Use reasonable force to ensure that the lid cannot be removed.
- Start blending. After blending, turn off the micro-pump at the main switch (D), disconnect the outlet tube (B) from the blender to release the vacuum, remove the blender lid and use the mix.
- Clean the blender.

MAINTENANCE & CLEANING

To ensure long, reliable service, we recommend the following:

- After each operation, clean away any plaster and other materials which may have dropped accidentally onto the micro-pump housing before they dry.

- Clean the outside of the unit regularly with a cloth moistened in soapy water. Always unplug the unit before cleaning it.
- To prevent the pump body against damage, a filter has been fitted to the outlet tube. Check this filter regularly through the viewing port. When it is dirty, replace it with an original **MESTRA** filter.

PRECAUTIONS

- ◆ Ensure that the mains power voltage is correct (230 V, 50/60 Hz earthed).
- ◆ Do not use the unit in very wet areas or close to inflammable materials (alcohol, solvents, etc.).
- ◆ Do not allow children or unskilled personnel to handle the unit.
- ◆ Do not pull, twist or bend the power cable excessively.
- ◆ Disconnect the unit from the mains before cleaning it.
- ◆ Do not submerge the unit in any liquid when cleaning it.
- ◆ Do not use the suction in the outlet tube for suctioning pipes or other objects.
- ◆ Do not place the outlet tube against your body while the pump is running.

TECHNICAL CHARACTERISTICS

Power supply:	230 V, 50/60 Hz
Consumption:	0.72 A
Vacuum:	550-600 mm/Hg
Dimensions:	150 x 120 x 100 mm
Weight:	1.9 kg

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