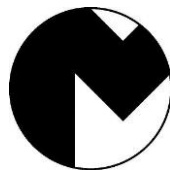
Ref. 080076

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The ref. 080076 High Capacity Wax Heater is a practical unit which keeps the wax temperature precisely as set by the user. The temperature range is between 30 and 100 °C. It has a number of applications in the dental sector. Please read the following instructions carefully for proper use of the machine, in order to obtain maximum output.

INSTALATION

1. Carefully unpack the unit. Check that it has not been damaged in transit.
2. Choose a flat and firm surface far from heat sources or vibrations.
3. Connect the machine to 230 V, 50/60 Hz with earthing.

HOW TO USE THE WAX HEATER

- Fill tank (C) with wax up to the minimum level (B), matching the horizontal line on the tank (approx. 30 mm from the upper rim).
- Activate the main switch at the back of the machine (on the supply cable base). Tank temperature will appear on the display. The machine **will start working immediatly**.
- If you wish to modify the temperature set (between 30 °C and 100 °C), press the [P] button for 3 seconds. Select the set temperature by pressing ▲/▼ keys. Press [P] to save.
- When the heating element is working, the pilot “OUT” lights.
- When set temperature is reached, the pilot “SET” lights.
- When the operation finishes, use the main switch on the back of the unit (on the supply cable base) to disconnect it.

MAINTENANCE AND CLEANING

- The machine has a protection device in case the level of wax in the tank falls below the

minimum level. If this happens, the heating element is disconnected automatically. Check the wax level regularly to prevent this.

- There is a fuse holder on the supply cable base. Change the fuse (4 A) whenever this is required.
- The machine should be cleaned periodically with a cloth. Before cleaning the machine, disconnect from the power supply.

PRECAUTIONS

- ◆ Before connecting the machine, make sure it is running at 230 V, 50/60 Hz and is earthed.
- ◆ When cleaning the machine, do not immerse it in any kind of liquid.
- ◆ Before cleaning the machine, disconnect from the power supply.
- ◆ Do not use the unit when the surroundings are too damp, or when it is too close to any inflammable products (alcohol or solvents).
- ◆ Do not pour solvents or any other inflammable product into the tank.
- ◆ Do not allow children or unqualified individuals to handle the machine.
- ◆ Do not start the machine if wax level is below the minimum. Check this level periodically, as wax may evaporate.

TECHNICAL CHARACTERISTICS

| | |
|--------------------|-----------------|
| Height: | 160 mm |
| Width: | 235 mm |
| Depth: | 200 mm |
| Weight: | 1 kg |
| Capacity: | 1 L |
| Power supply: | 230 V, 50/60 Hz |
| Power level: | 310 W |
| Fuse: | 4 A |
| Temperature range: | 30-100 °C |