**Ref. 080080 / 080081 / 080082 / 080083 / 080084**

ES.....	RECORTADORA DE MODELOS .....	7
EN .....	MODEL TRIMMERS .....	10
FR.....	TAILLE-PLATRE .....	12
DE .....	MODELLSCHLEIFER .....	15
PO .....	CORTADORA DE MODELOS .....	18
IT .....	TAGLIATRICE DI IMPRONTE .....	21

**MESTRA®**

**TALLERES MESTRAITUA S.L.**


Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

**48150 SONDIKA - BILBAO - ESPAÑA**

Rev.

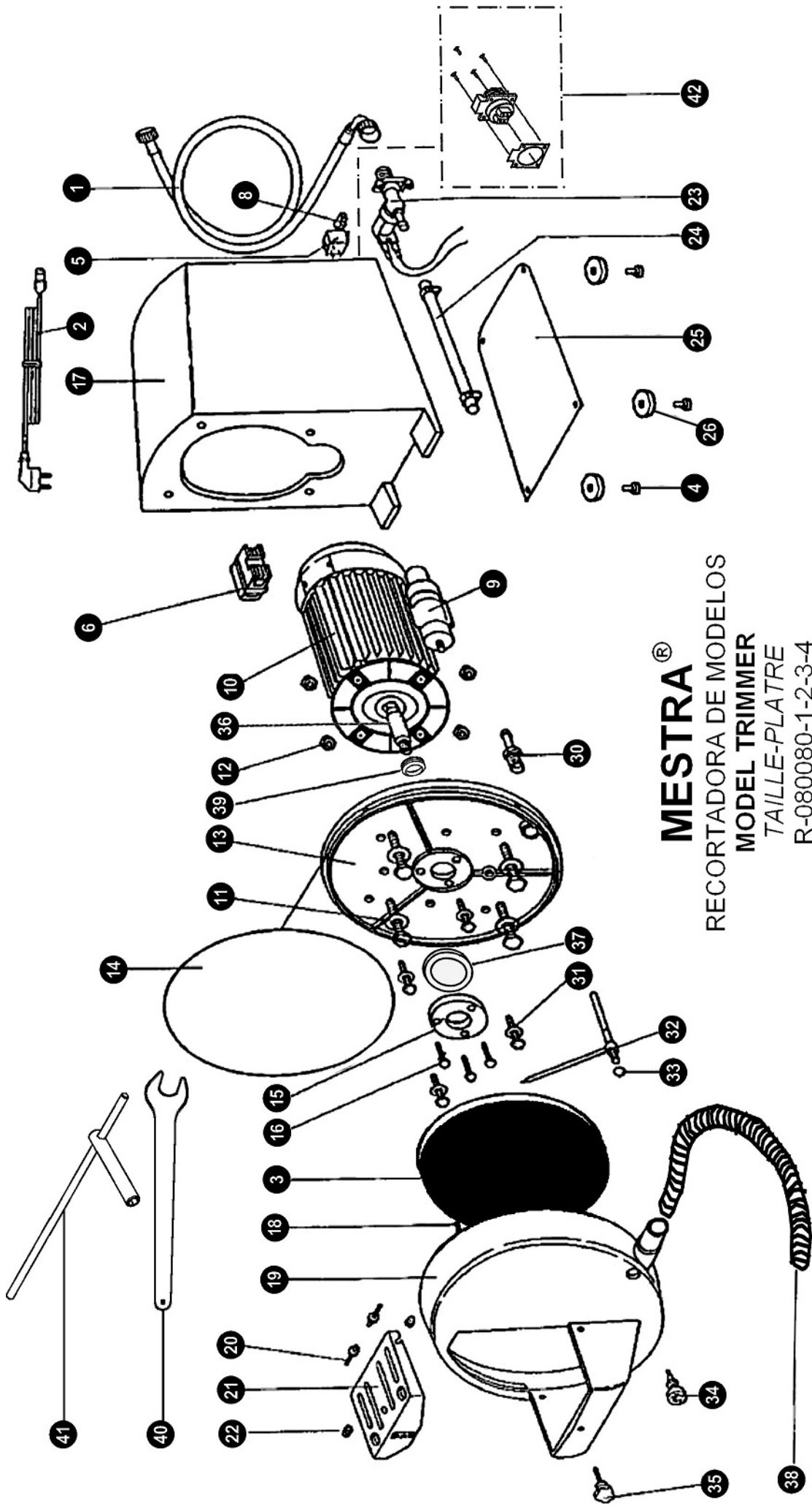**MESTRA®**  
 RECORTADORA DE MODELOS  
 MODEL TRIMMER  
 TAILLE-PLATRE  
 R-080080-1-2-3-4

Nº	Ref.	DESCRIPCION	DESCRIPTION
1*	080080-01	Manguera conexión agua	Water connection hose
2	000250	Cable 230V	Cable 230V
	000252	Cable 115V	Cable 115V
3	080080-03	Disco de carburo de silicio	Silicon carbide disc
	080081-01	Disco de diamante	Diamond disc
4	080251-09	Tornillo patas	Legs screws
5	080490-06	Base de conexión	Connection base
6	080080-40	Interruptor de seguridad	Safe switch
8	000236	Fusible 7A	Fuse 7A
	000237	Fusible 15A	Fuse 15A
	000238	Fusible 20A	Fuse 20A
9	080080-09	Condensador 230V	Capacitor 230V
	080080-09	Condensador 115V	Capacitor 115V
10	080080-41	Motor con condensador 1500 rpm 230V	Motor with capacitor 1500 rpm 230V
	080080-41B	Motor con condensador 1500 rpm 115V	Motor with capacitor 1500 rpm 115V
	080082-01	Motor con condensador 3000 rpm 230V	Motor with capacitor 3000 rpm 230V
	080082-01B	Motor con condensador 3000 rpm 115V	Motor with capacitor 3000 rpm 115V
11	080080-42	Tornillos sujeción motor	Motor camping screws
12	030230-04	Tuercas del motor	Motor nuts
13	080080-44	Tapa trasera	Rear cover
14	080080-14	Junta tórica tapa trasera	Rear cover O-ring
15	080080-15	Porta retén	Seal holder
16	040250-03	Tornillos de porta retén	Seal holder screws
17	080080-45	Tapa cubre motor	Motor hood
18	080080-18	Espárragos tapa delantera	Front cover allen screws
19	080080-19	Tapa delantera	Front cover
20	080080-20	Tornillos laterales tapa delantera	Front cover lateral screws
21	080080-21	Pieza basculante	Tilting device
22	080080-22	Tuercas ciegas	Tilting device blind nuts
23*	080080-23	Electroválvula 230V	Electrovalve 230V
	080080-23B	Electroválvula 115V	Electrovalve 115V
24*	080080-24	Manguera grifo	Tap hose
25	080080-46	Tapa de registro	Inspection cover
26	080080-26	Patatas de goma	Rubber legs
30*	080080-30	Racor de conexión grifo	Tap conector
31	080080-47	Tornillos unión de tapa trasera	Rear cover joining screws
32*	080080-32	Cuerpo de grifo (1500 rpm)	Tap body (1500 rpm)
	080082-02	Cuerpo de grifo (3000 rpm)	Tap body (3000 rpm)
33*	080080-33	Arandela de cobre	Copper washer
34*	080080-34	Mando de grifo	Tap control knob
35	080080-35	Tornillo baquelita	Bakelite screw
36	080080-56	Acoplamiento para motor	Motor coupling
37	080080-04	Reten para porta reten	Seal for seal holder
38	080080-54	Manguera desagüe	Sink hose
39	080080-49	Retén acoplamiento motor	Motor coupling seal
40	080080-53	Llave fija	Spanner
41	080080-55	Llave de tubo	Socket
42**	080084-01	Base empotrable	Base

\* No existe en 080084 / Does not exist in 080084

\*\* Sólo en 080084 / Only in 080084

- |   |   |
|---|---|
| A | Disco abrasivo / <b>Abrasive Disc</b> / <i>Disque Abrasif</i> / Schleifscheibe / <b>Disco abrasivo</b> / <i>Disco abrasivo</i>  |
| B | Superficie de apoyo / <b>Support surface</b> / <i>Plateau support</i> / Anlegefläche / <b>Superficie de apoio</b> / <i>Piano d'appoggio</i>   |
| C | Tornillo de fijación / <b>Attachment screw</b> / <i>Vis de fixation du support</i> / Befestigungsschraube / <b>Parafuso de fixação</b> / <i>Vite di fissaggio</i>                                   |
| D | Carcasa de protección de disco / <b>Front fairing</b> / <i>Carter de protection du disque</i> / Scheibenschutzgehäuse / <b>Carcaça de protecção de disco</b> / <i>Involucro di protezione disco</i> |
| E | Llave de paso de agua / <b>Water control</b> / <i>Bouton régulateur du débit d'eau</i> / Wasserdurchlaufhahn / <b>Torneira de água</b> / <i>Rubinetto d'intercettazione acqua</i>                   |
| F | Boca de desagüe / <b>Nozzle</b> / <i>Bouche d'évacuation d'eau</i> / Wasserablauföffnung / <b>Boca de despejo</b> / <i>Bocca di scarico</i>   |

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The Ref. 080080, 080081, 080082, 080083 and 080084 are easily-handled model cutters, especially designed for use in the dental prosthesis sector. They are structurally similar, with these small differences:

- The Ref. 080080 is fitted with a disc of special abrasive resins. It has a system for cleaning the disc with water.
- The Ref. 080081 is fitted with a diamond disc that enhances its durability, and reduces the work required in facing the models. This unit also has a disc-cleaning system that uses water.
- The Ref. 080082 is fitted with a silicon carbide disc. It has a system for cleaning the disc with water. The Ref. 080082 and the Ref. 080083 are the most powerful MESTRA trimmers.
- The Ref. 080083 is fitted with a diamond disc and also has a disc-cleaning system that uses water. Same power level as the 080082.
- The Ref. 080084 is fitted with a diamond disc, but the disc is not cleaned with water. This means greater stability and minimum effort in cutting, since the plaster does not get dampened.

#### INSTALLATION OF THE UNIT

1. Carefully unpack the model cutter. Check that it has not sustained damage during shipment.
2. Position the unit on a flat and firm surface remote from any source of heat or vibration. There should be a water tap nearby with a threaded outlet, as well as a drainage connection (except Ref. 080084).
3. Connect the right-angle end of the flexible tube onto the nozzle at the back of the unit. Connect the other end to the water source. To do this, screw the free nut at the end of the flexible tube onto the head of the tap (except Ref. 080084).
4. Fit the drainage tube onto the nozzle (F) at the lower front right of the model cutter. Connect the other end of the tube to the drainage (except Ref. 080084). Ensure that the drainage tube has a certain fall so as to prevent stagnation at the outlet and thus improve drainage.
5. In the case of the Ref. 080084, connect the nozzle (F) to an extractor to remove particles produced during abrasion.
6. The unit has four attachment suckers. In the design, special attention was given to ensuring sufficient weight and robustness to prevent movements during handling, so that it is not necessary to use additional fastening components.

7. Connect the unit to a 230 V, 50/60 Hz outlet with earth.
8. Check that the disc-cleaning system functions correctly (except Ref. 080084). To do this, turn on at the main switch, which is on the left side of the unit. The cutting machine will start up, and the servovalve controlling the inflow of water will open.
9. Open the tap at the main water intake. Turn the cutter control (E) anticlockwise until flow is right. After a few minutes of operation, check that there is no leak in the tube connections.

#### HANDLING OF THE UNIT

1. Adjust the support surface (B) by loosening the attachment screw. One may choose, depending on the nature of the work, one of three positions. Subsequently, tighten the attachment screw (C) until the support surface is firm.
2. Protect yourself suitably.
  - Use safety goggles.
  - Use safety gloves.
  - Use a face-mask
  - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
  - Be careful with long hair.
  - Always use the cutting machine with suction (only Ref. 080084).
3. Switch on the unit at the main switch, which is on the left side. The motor will start up.
4. Open the tap at the main water supply. Turn the control cock (B) until water flow is as desired. Flow increases when the cock is turned anticlockwise (except Ref. 080084).
5. Set the model cutting on the support surface (B). For higher precision in the cut, the support surface is provided with reference marks to facilitate estimation of the model attack angles.
6. When work is over, turn off at the main switch.
7. It is recommended that plaster be removed, after operation, from the support surface and the front housing (D) with a damp cloth.

#### CHANGE OF ABRASIVE DISC

Replace the abrasive element when it is worn or deteriorated. Always use **MESTRA** spares, since only these ensure perfect machine equilibrium and highest-quality abrasive resin. Proceed as follows:

1. Disconnect the unit from the power supply and close the main water tap.

2. Remove the front protective fairing of the disc (D) by undoing the four lock nuts that secure it to the after body. Use an M8 wrench.
3. The disc plate is attached to the motor shaft by a threaded coupling. The connection has probably hardened with the passage of time, and the application of loosening oil is advisable. Allow it a few minutes to take effect.
4. To remove the disc, connect the drive shaft by applying the M13 socket wrench supplied with the unit to the rear end of the shaft. The wrench has a useful rod-arm that can be so placed that it comes to a stop at the surface on which the unit rests, preventing unwanted shaft rotation while the disc is being changed.
5. Apply the M24 spanner (also supplied with the unit) to the two grooves in the plate neck. Turn it anticlockwise to free the thread. At the same time ensure that the socket wrench properly locks the motor shaft. If there is strong resistance, use a nylon hammer very lightly on the end of the spanner. Perform this last operation with extreme care, since otherwise the motor shaft could be damaged or put out of balance.
6. Fit the new disc, being especially careful not to damage the coupler shaft stop. Rotate it manually to check that it is free, and that there is no significant warping.
7. Fit the housing and retighten its four lock nuts. We recommend that loosening oil be applied to all threaded connections a few minutes before an abrasive disc is changed.

### MAINTENANCE AND CLEANING

To ensure correct, reliable, and enduring operation, we recommend:

- After use, clean dust and plaster from the support surface (B) and from the front fairing (D).
- For general cleaning, use a cloth dampened in soapy water. Do not use alcohol, solvents, or other liquids.
- Never immerse the unit in liquid.
- If you notice unusual vibration or noise, send the unit to a technical assistance centre.
- Change the abrasive disc when you notice a significant fall in performance.

- After changing the disc, check that there are no unusual vibrations.
- The unit was designed to endure sound operation when it rests on the four suction cups at the base. These are sufficient by themselves to withstand the force applied in facing.
- To ensure optimum performance, Talleres Mestraitua, S.L. recommends the use of the **MESTRA** Ref. 080086 catch basin, which is designed specifically for Ref. 080080, 080081, 080082 and 080083 cutters.

### PRECAUTIONS

- ◆ Before connecting the unit to the power supply, make sure the voltage is correct (230 V, 50/60 Hz with earth).
- ◆ Whenever you are carrying out a cut, protect yourself suitably. Use gloves, safety goggles, and mask.
- ◆ Always use suction (only in the case of Ref. 080084). In the case of cutters Ref. 080080 and 08001, always use the system with disc-cleaning by water.
- ◆ Do not allow children or unspecialised personnel to operate the unit.
- ◆ If the motor shaft jams during normal operation, disconnect the unit from the power supply before endeavouring to unlock the shaft by forcing its rotation manually.
- ◆ Do not polish, brush, or grind materials that could release particles that are toxic when inhaled.
- ◆ Before using the unit, make sure it is seated on a firm and solid base.
- ◆ Do not use the unit close to solvents or other inflammable materials.
- ◆ After changing the disc, make sure that the gasket belonging to the motor disc-shaft plate coupling is leak-proof. Similarly check the entire disc-flushing system.
- ◆ When installing the unit, make sure that the disc-cleaning circuit is perfectly leak-proof.
- ◆ Do not face materials other than those for which the unit has been designed, particularly metal objects.
- ◆ Do not interfere with the model supports when the unit is operating.

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