
Ref. 080096

MULHACEN

| | | |
|---------|---|----|
| ES..... | RECORTADORA DE MODELOS CON FRENO - SECO | 5 |
| EN..... | MODEL TRIMMER WITH BRAKE - DRY..... | 8 |
| FR..... | TAILLE-PLATRE AVEC SYSTEME DE FREIN - SEC | 11 |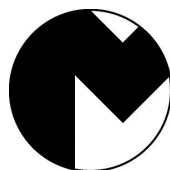

MESTRA®


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

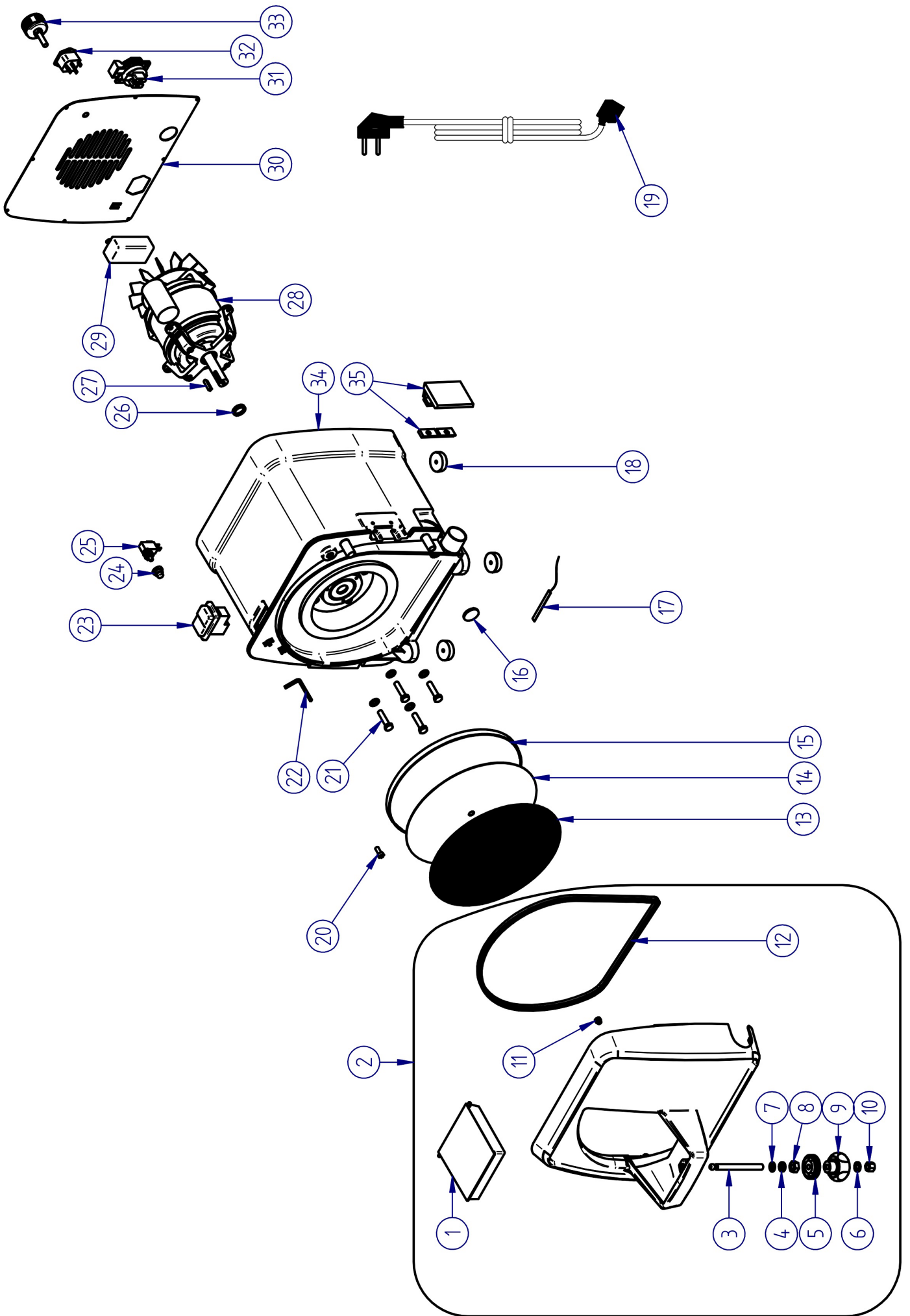| Nº | Ref. | Descripción | Description |
|----|------------|-----------------------------------|-------------------------------|
| 1 | 080093-35 | Mesa basculante | Tilting table |
| 2 | 080093-34 | Tapa delantera | Front cover |
| 3 | 080093-36 | Espárrago mesa elevación | Elevation table stud |
| 4 | 080093-23 | Casquillo plástico espárrago | Stud plastic bushing |
| 5 | 080093-02 | Tuerca regulación altura | Height regulation nut |
| 6 | 080093-58 | Arandela bloqueo | Lock washer |
| 7 | 080093-24 | Junta goma mesa | Table rubber gasket |
| 8 | 080093-41 | Tuerca de plástico | Plastic nut |
| 9 | 080093-06 | Pomo mesa elevación | Elevation table knob |
| 10 | 080093-17 | Tuerca pomo elevación | Elevation knob nut |
| 11 | 000110 | Goma tope microinterruptor | Microswitch top rubber |
| 12 | 080093-10 | Perfil goma cierre puerta | Cover rubber strip |
| 13 | 080093-04 | Disco de lija grano 80 grueso | Sandpaper disc grain 80 gross |
| 14 | 080093-03 | Disco velcro adhesivo | Adhesive velcro disc |
| 15 | 080093-39 | Plato aluminio para Velcro | Aluminium plate for Velcro |
| 16 | 080262-69 | Tapón | Plug |
| 17 | 080093-11 | Tira de LED | LED strip |
| 18 | 080080-26 | Pata | Leg |
| 19 | 000250 | Cable 230 V | Cable 230 V |
| | 000252 | Cable 115 V | Cable 115 V |
| 20 | 030120-03 | Tornillo sujeción plato | Plate holding bolt |
| 21 | 080093-48 | Tornillo sujeción motor | Motor holding bolt |
| 22 | 080093-52 | Llave Allen | Allen wrench |
| 23 | 080080-40 | Interruptor de seguridad 230 V | Safe switch 230 V |
| | 080080-40B | Interruptor de seguridad 115 V | Safe switch 115 V |
| 25 | 080093-22 | Capuchón microinterruptor | Microswitch hat |
| 25 | 080093-16 | Microinterruptor final de carrera | End stroke microswitch |
| 26 | 080093-46 | V-ring | V-ring |
| 27 | 080093-25 | Chaveta | Feather key |
| 28 | 080093-01 | Motor 230 V | Motor 230 V |
| | 080093-01B | Motor 115 V | Motor 115 V |
| 29 | 080093-53 | Fuente de alimentación 12 V | Power supply 12 V |
| 30 | 080093-71 | Tapa trasera | Back lid |
| 31 | 080084-01 | Base Schuko | Schuko plug base |
| 32 | 080502-12 | Base de conexión | Connection base |
| 33 | 080093-43 | Extractor del plato | Plate extractor |
| 34 | 080093-33 | Carcasa trasera | Rear housing |
| 35 | 080093-21 | Pestillo | Latch |

- | | |
|---|---|
| A | Disco abrasivo / Abrasive disc / <i>Disque abrasif</i> |
| B | Mesa de apoyo basculante / Adjustable table / <i>Table de travail basculante</i> |
| C | Tuerca de fijación altura / Locknut to fix the table / <i>Ecrou de fixation de la hauteur</i> |
| D | Pomo elevación de la mesa / Knob to adjust the table height / <i>Pommeau d'élévation de la table</i> |
| E | Indicador de funcionamiento / Operation light / <i>Témoin lumineux de fonctionnement</i> |
| F | Cierre de la tapa / Cover lock / <i>Système de fermeture de la façade</i> |
| G | Conexión aspiración / Aspiration connection / <i>Evacuation</i> |

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The Ref. 080096 is an easy to handle model trimmer specially designed for the field of dental prosthesis.

Its 3000 r.p.m. powerful motor has a brake that stops the spin after turning the main switch off.

Its dimensions give the unit an excellent stability during operation. It features a 122 mm window (one of the widest in the market).

Its design allows to place a big storage tray (R-100062) on the unit in order to keep the models well arranged and the workbench clear.

The Ref. 080096 has a factory installed 80 grit abrasive disc and a second unit to replace.

Replacements:

80 grit abrasive disc (coarse) Ref. 080093-04 (5 u.)

The following items are also factory supplied:

- A water inlet hose.
- An extensible drainpipe.

INSTALLATION OF THE UNIT

1. Carefully unpack the model trimmer. Make sure the unit has not suffered any damage during the transport.
2. Place the unit on a flat and firm surface, away from any heat or vibration source.
3. Connect an aspiration unit to the outlet (H) in the right bottom front part of the unit.
4. The unit has four rubber pads. When designing the unit, special care has been taken to give the unit enough weight and robustness to prevent any movement during its operation. No additional fixing element is needed.

5. Connect the unit to a 230 V, 50/60 Hz grounded power source.

OPERATION OF THE UNIT

1. Adjust the table (B) with the knob (E), according to the type of work. Once the working position has been selected, screw the locknut (C) up as far as it goes to fix the table.
2. If you want to trim at a vertical 90° angle, align the lower edge of the table with the edge of the casing:

3. Take proper safety measures. Always wear:
 - Use safety goggles.
 - Use safety gloves.
 - Use a face-mask
 - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
 - Be careful with long hair.
4. Turn on the main switch placed in the upper part of the unit. The motor will start up and the operation light (E) will lit up.
5. Place the model to be trimmed on the table (B). For a greater precision while trimming, the table has a set of marks to be used as reference to determinate the cutting angles.
6. When the work is over, turn the main switch off.
7. After operating the trimmer, it is strongly recommended to clean the table and the front casing with a damp cloth to remove the remains of plaster.

ABRASIVE DISC REPLACEMENT

When the abrasive element tears or wears out it must be replaced. **MESTRA** spare-parts should be always used, since they are the only ones that assure a perfect balance of the unit and top quality. To replace the disc, proceed as follows:


1. Unplug the unit. Open the lock (F) and the front cover. The safety micro-switch will prevent an eventual start-up of the motor.
2. Remove the worn-out disc and place a new one. Make sure it is properly centered.
3. Close the cover again, making sure the lock (F) is properly closed.


ALUMINIUM PLATE MAINTENANCE

If plaster accumulates in the back of the aluminum plate, then the machine could vibrate. Then proceed to clean as follows:

1. Unplug the unit. Open the lock (F) and the front cover. The safety micro-switch will prevent an eventual start-up of the motor.
2. Remove the abrasive disc.
3. Use the Allen spanner to unscrew the bolt that fixes the disc to the motor shaft.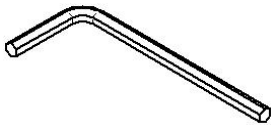

4. Remove the extraction bolt from the back of the unit and screw it up in the middle of the plate. Then pull the screw and extract the plate.

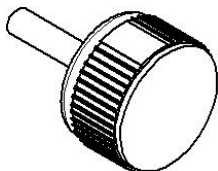5. Dip the plate in water, remove the plaster with a brush and dry.
6. When you place the plate again, do not forget the pin that couples it to the motor.

MAINTENANCE AND CLEANING

To ensure a correct, reliable and durable performance of the unit we strongly recommend:

- After using the unit, clean the dust and plaster remains from the table (B) and the front cover.
- Do not use alcohol or any other solvent to clean the unit.
- Never immerse the unit in a liquid.
- If you notice unusual noises or vibrations, send the unit to a technical assistance service.
- Replace the abrasive disc whenever you notice a significant fall in performance.
- After replacing the disc, check that there are not unusual vibrations.
- The unit has been designed to operate correctly leaning on the four rubber blocks in its base. These elements are strong enough to bear the movement of the trimming.

PRECAUTIONS

- Before connecting the unit to the power source, make sure the voltage is correct (grounded 230 V, 50/60 Hz)
- When trimming a model, protect yourself suitably. Always wear gloves, safety goggles, and a mask. Use proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine. Be careful with long hair.
- Always use the water-cleaning system to clean the discs.
- Do not allow children or unskilled personnel to operate the unit.
- Do not allow anybody to by-pass the safety micro-switch that detects that the cover/door is closed or to operate the unit when the cover/door is open.
- If the motor shaft jams during normal operation, disconnect the unit from the power source before trying to free the shaft by forcing its rotation manually.
- Do not polish, brush, or grind materials that could release particles that are toxic when inhaled.
- Before using the unit, make sure it is placed on a firm and solid base.
- Do not use the unit close to solvents or other inflammable materials.
- When replacing the disc, make sure that the sprinkler (I) is in good condition and free from plaster.

English

- When installing the unit, make sure that the disc-cleaning circuit is perfectly leak-proof.
- Do not trim materials for which the unit has not been designed, especially metal objects.
- Do not handle or adjust the table when the unit is working.

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