Hornos de precalentamiento

Pre-heating furnaces

Fours de chauffe

Ref. 080148

Ref. 080144

Ref. 080142

Ref. 080132

Ref. 080134

Ref. 080138

MESTRA®

HP-100 Ref. 080148
HP-25 Ref. 080142

HP-50 Ref. 080144

HORNOS DE PRECALENTAMIENTO PRE-HEATING FURNACES FOURS DE CHAUFFE

Inox

DIRECTO-50 D Ref. 080130 50 D DIRECT DIRECT-50 D

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Extremely high performance pre-heating furnace family, composed of three models with different muffle capacity (2.5, 5 and 10 litres). Made of stainless steel, the most convenient material for the extraordinary thermic requirements this kind of devices has to bear. The high quality of the muffle ensures an exceptionally long working life, even when continuously working at temperatures near 1100 °C. When the thermocouple fails, the electronic control will announce it. This element is very economical and easy to replace by the user himself. On the other hand, the electronic control allows the programming of cycles composed of multiple parameters; you can define ramps and plateaus with temperatures, maintenance times, heating in a determined time or in degrees per minute. The memory can store up to twenty different programmes. The furnace is factory supplied with a ceramic tray.

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It is perfect for those works that do not require a classical programming cycle of ramps and plateaus. Just select the set point temperature, and the furnace will start heating directly until this temperature is reached. Then, this temperature will be kept for a time programmed by the user. Its digital controls ensure an amazing temperature accuracy and prevent the effects of thermal inertia. Besides, it features a connection for an external catalytic converter, and this element will operate from the beginning of the cycle up to the stopping temperature programmed by the user. The size, operation and other features of this furnace are similar to those of the HP-50 model. The only different feature is its controls. Tank capacity: 4,8 litres (4 to 6 Ø 80 casting rings).

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Twin display screen: one for temperature, the other for time. Graphic showing temperature increase and holding, with indication of what process is ongoing.

EASY TO HANDLE

- ▶ **SEARCH KEY:** press to enter programming mode. The setting to be changed on the graph flashes.
- ▲ **UP KEY:** press to increase the time or temperature setting flashing on the graphic.
- ▼ **DOWN KEY:** press to decrease the time or temperature setting flashing on the graphic.
-  **ESCAPE KEY:** press to exit programming mode and memorise settings.
-  **STOP/START KEY:** hold down for 5 seconds to turn the oven on and off.
-  **PROGRAM KEY (20 Prog.)**

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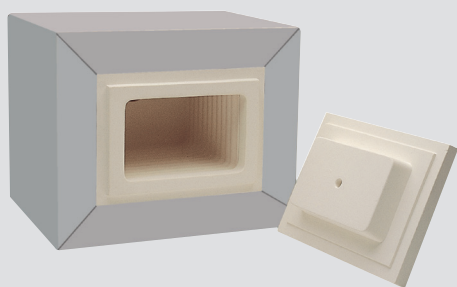
The muffle is made in ceramic material with heaters wrapped along the upper, lower and on both sides. This way, there is a uniform interior distribution of temperatures. The exterior sides of the muffle are thermally insulated by zirconium silicate, probably the best thermal insulator used in industrial processes.

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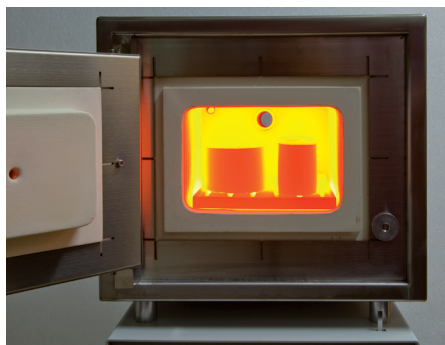
Magnetic closing system is easy and simple. A safety sensor detects door opening, and disconnects heaters while it is opened.

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Gas flow inside the muffle has been carefully studied. There is an air inlet at the door and a ventilation pipe at the back, both with calibrated sections. This way air inside the muffle is renovated without reducing thermal efficiency of the oven nor producing temperature differences.

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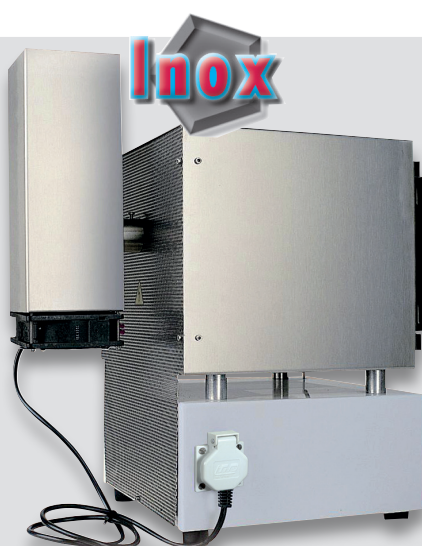
A practical socket allows the synchronised operation of the oven together with an extraction unit. Automatic disconnection can also be programmed between 75° C and 850° C. When temperature is over 850° C, the control automatically disconnects the power supply.

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The Smoke Extractor Ref. 080125 is an interesting attachment to the HP Preheating Furnaces. Installing this device, you succeed in getting rid of furnace gases generated in the muffle in a continuous and smooth way. It is completely made of stainless steel and its installation is easy. If it is connected to the MESTRA HP furnaces socket, the extractor ranges from 75° to 850°c as programmed by the user. If it is connected to another socket it runs continuously.

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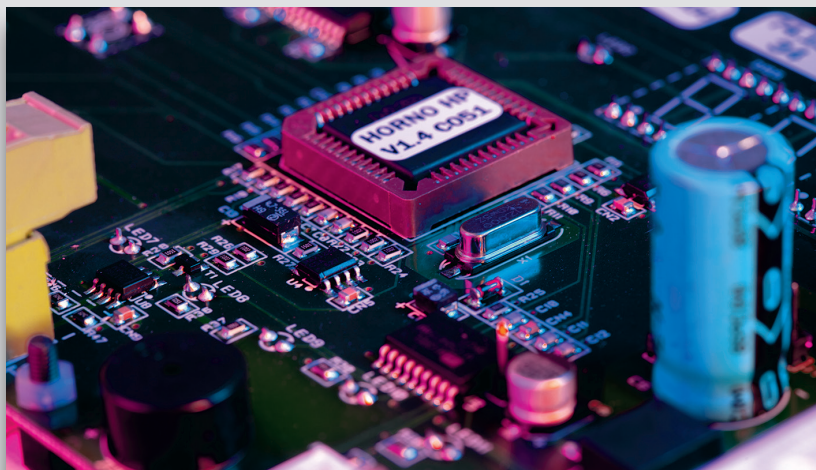
Simple device attached to the smoke vent of any pre-heat furnace to remove gases and combustion residues that are generated in the furnace as a result of the combustion of waxes and other materials.

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With this electronic control run by a microprocessor temperature can be defined accurately in the three heating ramps and in the three plateaux that can be programmed in a cycle, and also the required times in order to reach these temperatures. As the most relevant characteristics of the control it can be pointed out the following.

- Continuous monitorization of temperatures and times through its two displays.
- Precise control of temperatures, with grade by grade signal.
- Heating resistance control by triac with digital switch on when going over "0".
- PID algorithm Software for a best control of thermal inertia.
- Memory of up to 20 programmes.
- Synoptic panel indicating the stage in which we are every time.
- Thermocouple breakage indicator.
- Warning buzzing of the most significant steps of the cycle.
- It is possible to programme a time relay switch on. (99 h. 59')
- Overvoltage or voltage variation protection.

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The "Vulcano" controls family has been developed as an economical alternative to the **MESTRA**'s standard electronic controls. The only difference between a "Vulcano" product and a standard one lies in the controls; all the other components and functions are identical. The performance and features of the "Vulcano" controls meet perfectly the needs and expectations of most of the users, but at a very attractive price. For this reason, it is recommended for those users who do not need very high levels of performance. Apart from the price, its main advantages are its ease of operation and programming. The "Vulcano" line is available for all the **MESTRA** furnace range.

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