

Ref. 080218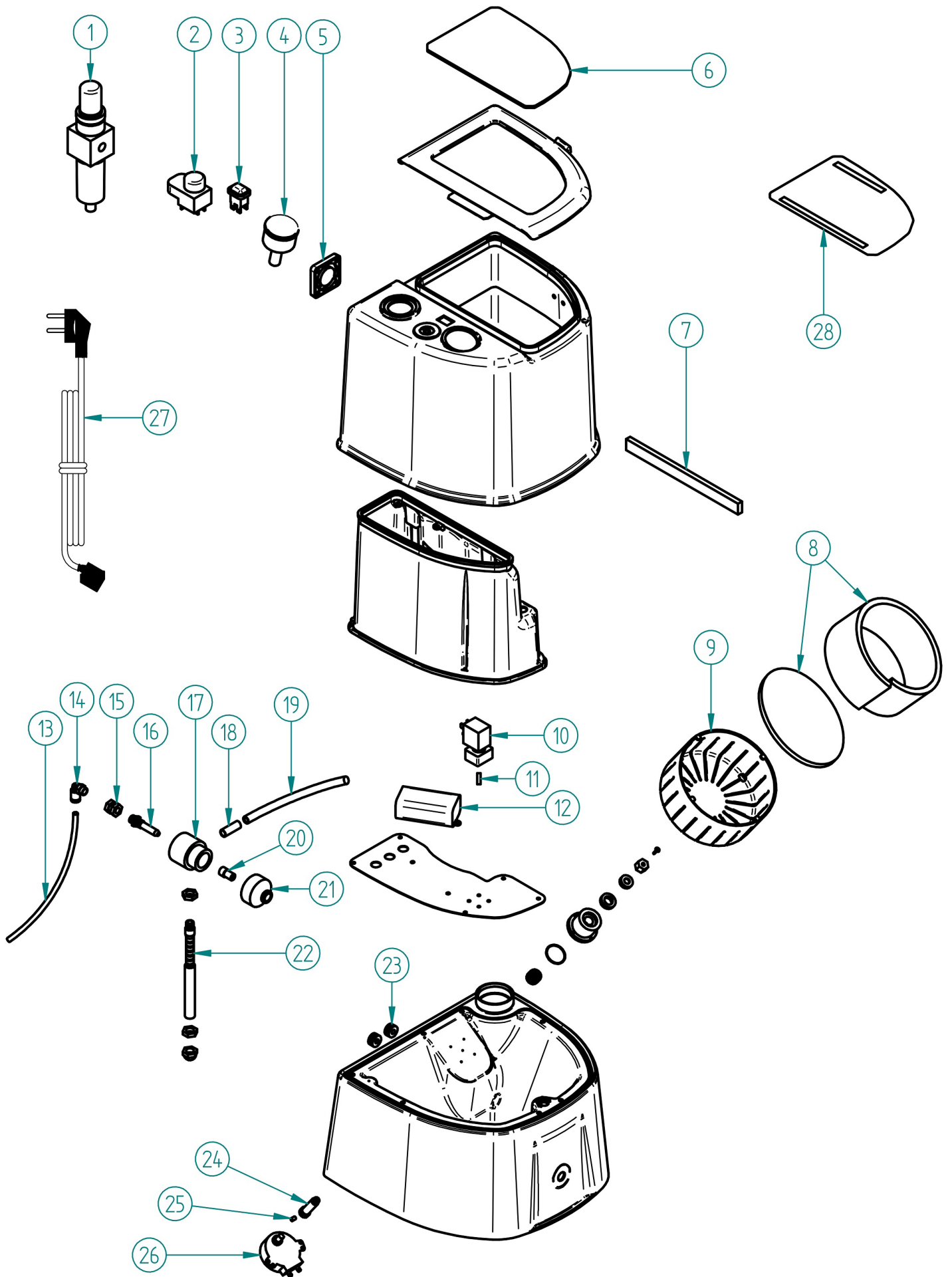

Nº	Ref.	Concepto	Description
1	080240-14	Manorregulador	Pressure control
2	080235-18	Temporizador	Timer
3	080422-17	Interruptor luz	Switch
4	080220-11	Manómetro	Pressure gauge
5	080218-07	Rejilla ventilación	Ventilation grill
6	080218-05	Cristal	Glass
7	080508-15	Barra de LED	LED bar
8	080218-11	Sufrideras cesta	Basket rubbers
9	080218-04	Cesta	Basket
10	080253-10	Electroválvula 24V	Electrovalve 24V
11	080242-24	Filtro electroválvula	Electrovalve filter
12	080614-20	Fuente alimentación	Power source
13	000101	Tubo Ø4x6	Tube Ø4x6
14	000312	Racor codo plástico 1/8 Ø6	1/8 elbow Ø6 plastic connector
15	080235-11	Racor prolongación inyector	Prolongation injector connector
16	080235-10	Inyector	Injector
17	080235-09	Cuerpo pistola	Gun body
18	080235-12	Casquillo pistola	Guns sleeve
19	080235-13	Manguera aspiración	Aspiration hose
20	080270-26	Boquilla tungsteno	Tungsten nozzle
21	080270-31	Porta boquillas	Nozzle holder
22	080235-08	Flexo pistola	Gun arm
23	080010-05	Pasa cables	Cable bushing
24	080235-27	Eje	Shaft
25	030350-03	Espárrago	Stud
26	080550-15	Motor	Motor
27	000200	Cable	Cable
28	080218-23	Protector cristal (10 u)	Glass protector (10 u)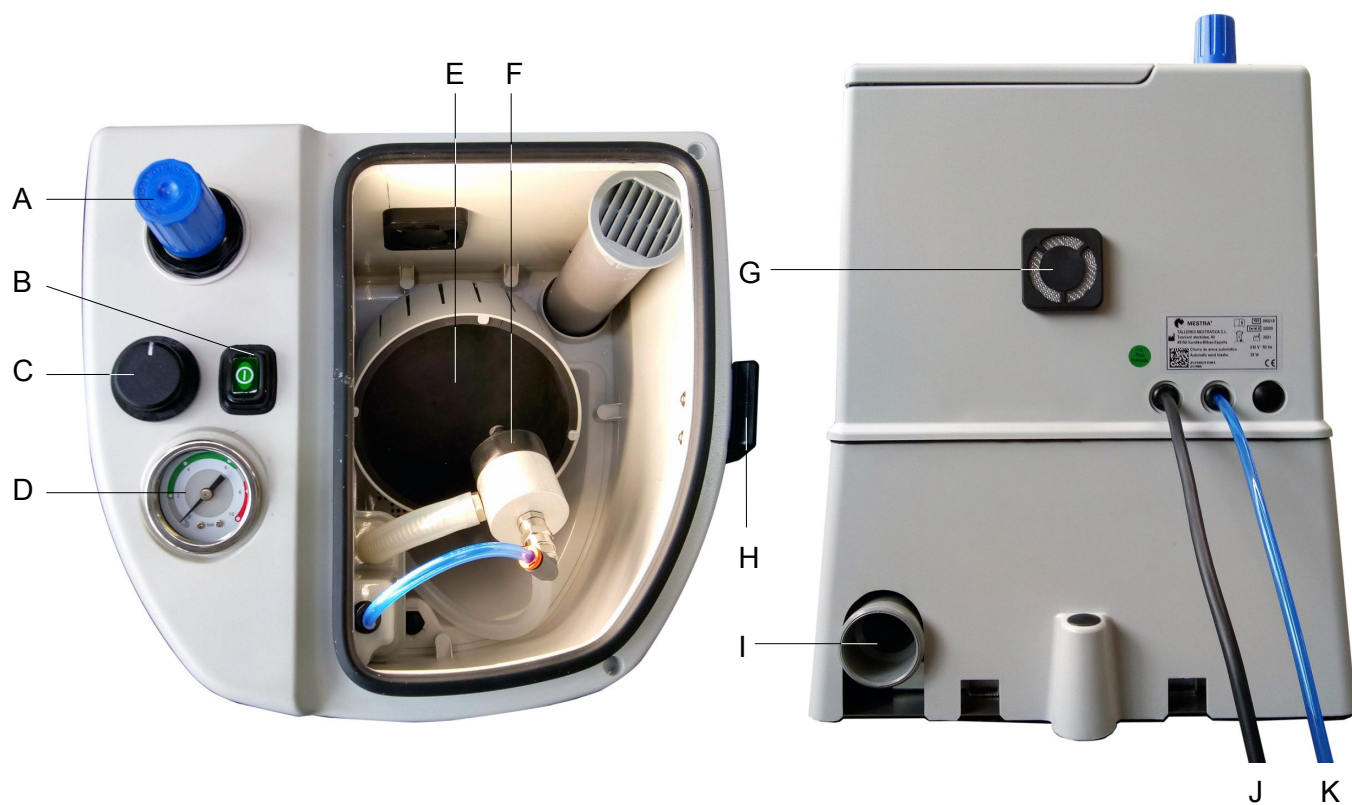

A	Manorregulador / Pressure regulation valve
B	Interruptor luz / Light switch
C	Temporizador / Timer
D	Manómetro / Pressure gauge
E	Cesta / Basket
F	Pistola sopladora / Blasting gun
G	Ventilación / Ventilation
H	Cierre tapa / Door lock
I	Conexión aspiración / Aspiration connection
J	Cable alimentación / Power conr
K	Conexión aire a presión / Pressure air connection

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NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

Ref. 080235 is an automatic sand blaster specially designed for sandblasting small pieces within the field of dental prosthesis on continuous automatic mode.

INSTALLATION

- Carefully unpack the unit.
- Choose a firm, flat surface to place the unit, far from heat or vibration sources. The room atmosphere should never be damp. An excess of humidity could compact the aluminium oxide particles inside the cabin and, therefore, obstruct the blasting nozzles. Finally, place the unit at a height that allows an easy manipulation of the objects inside the cabin.
- Fit the connecting pipe (K) into an intake of pressured air fed by a compressor with a minimum pressure of 7 bar (0.7 MPa) and a minimum flow of 100 litre/minute.
- Connect the power cord (J) to a grounded 230 V, 50/60 Hz power source.
- The unit is factory supplied with a 3 mm tungsten nozzle (Ref. 080270-26).
- Fill the hopper in the cabin with aluminium oxide of a maximum diameter of 300 micron.

NOTE: the unit is optimized for 200-micron particles and getting an stable jet. With smaller particle diameters (100 microns) the jet may be more irregular.

PRESSURE REGULATION

The blowing pressure of the blasting gun can be regulated at will between 2 and 5 bar (0.2 and 0.5 MPa), according to the nature and special features of the work to be done. Take into account that the highest pressure will be limited by the maximum pressure of the air supply we are using. In any case, the pressure should never be higher than 6 bar (0.6 MPa).

The unit features a pressure regulation valve (G) to select the working pressure easily. To adjust the pressure, pull the valve outwards and turn it until the intended pressure has been reached. The value of the pressure is shown in the built-in pressure gauge (H). After adjusting the pressure, block the valve knob by pressing it inwards.

The pressure gauge shows the gun's blowing pressure at any time, even when the gun is not being used.

CONNECTING TO THE ASPIRATION UNIT

The blasting cabin should always be used connected to a pressured aspiration unit. For this purpose, there is an outlet hose at the back side of the unit (I). The aspiration unit to be used has to create a light depression inside the cabin for the dust particles to circulate smoothly and continuously towards the outlet hose.

NOTE: make sure that the aspiration unit used can suck abrasive particles. We strongly recommend the MESTRA Eolo Plus Ref. 080532 aspiration unit.

OPERATION

Ref. 080218 automatic sandblaster can be operated on automatic mode.

1. Fit the basket (E) into the motor shaft.


Never force the motor shaft to rotate by hand, whether the basket is fitted or not. The motor's inner mechanism could be damaged.

2. Put the pieces you want to sandblast inside the basket.
3. Position the gun (F) facing the pieces, bearing in mind that the basket will rotate clockwise. At the beginning the motor may start to rotate anti-clockwise, but before completing its first turn it will start to rotate clockwise.
4. Direct the sandblast accurately to prevent the sand going straight to the cabin walls. The sandblast could damage the cabin.
5. Close the cabin lid making sure the glass fits tightly into its frame.
6. Turn on the pressured aspiration unit.
7. For a better vision of the piece, turn on the light inside the cabin by pressing the switch (B).
8. To select the automatic operation time, turn the timer (C) clockwise (maximum 30 minutes).
9. The basket will start to rotate and the gun will blast aluminium oxide automatically.
10. If you want to stop the automatic operation, turn the timer (C) anti-clockwise.

MAINTENANCE AND CLEANING

In order to achieve a long working life of the machine, we strongly recommend you:

- Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide.
- Preferably use dry-piston air compressors. The presence of oil particles in the air could make the sand compact and obstruct the gun's nozzle.
- To prevent the inner walls of the cabin wearing away due to abrasion, do not aim the aluminium oxide blast straight at the cabin walls.
- Periodically remove the aluminium oxide from the cabin. To do this, use the drain plug in the bottom of the hopper.
- For an overall cleaning of the unit, use a dry cloth or moistened with a proper product.
- In case of obstruction of the gun (F), dismount the nozzle holder. Clear the nozzle by blowing in the opposite direction to the air flow, or, if needed, by using a pin or a thin wire. Then set the gun again, making sure it is correctly centred.

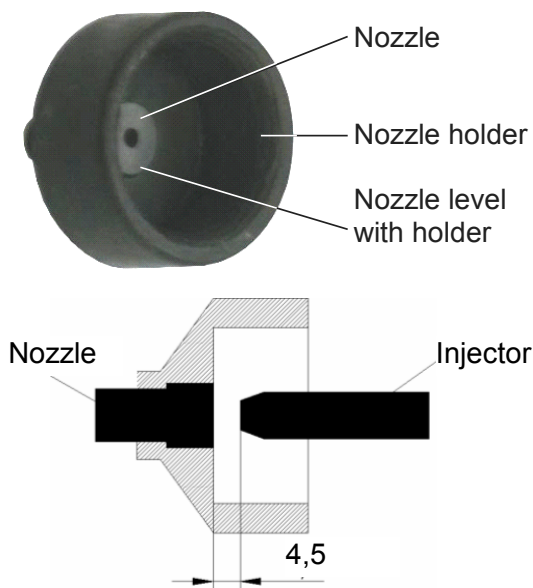- If the nozzle came loose, glue it to its holder with cianocrilate. The nozzle should be level with the inner surface of the nozzle holder, to make sure that the distance from the injector to the nozzle is 4.5 mm and the Venturi effect takes place.
- Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide. The gun (\varnothing 3 mm nozzle) can blast aluminium oxide up to particles of \varnothing 300 micron. If you use aluminium oxide particles bigger than \varnothing 300 micron, you may need to remove the filter (K) placed at the end of the gun's pipe, inside the hopper (Fig. 2). Without any filter obstructions in the nozzle will be more.

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- Never exceed a 6-bar pressure (0.6 MPa). The optimal working pressure lies between 2 and 5 bar (0.2 and 0.5 MPa).
- Never force the motor shaft to rotate by hand, whether the basket is fitted or not. The motor's inner mechanism could be damaged.
- Always work with the cabin lid closed and correctly protected. Use protection mask and goggles.
- Connect the cabin's outlet hose to an aspiration unit.
- If the protection sleeves (C) tear or wear, replace them immediately.
- Do not allow children or unskilled personnel to operate the machine.
- Always use a high-quality aluminium oxide (it is strongly recommended to use **MESTRA aluminium oxide**). A very low-quality aluminium oxide can produce obstructions in the nozzle.
- Make sure that the aspiration unit can suck abrasive particles. We strongly recommend the MESTRA Eolo Plus Ref. 080532 aspiration unit.
- Connect the power cord to a grounded 230 V, 50/60 Hz power source.
- In case of leaks in the pressured air circuit, consult the installation engineer immediately.

TECHNICAL CHARACTERISTICS

Height:	430 mm
Width:	320 mm
Depth:	310 mm
Weight:	5.5 kg
Working pressure:	2 - 5 bar 0.2 - 0.5 MPa
Maximum air consumption:	90 L/min (5 bar)
Voltage:	230 V, 50/60 Hz
Power consumption:	15 W