
## Ref. 080235

ES... ARENADORA AUTOMÁTICA GALAXY .....	6
EN ... GALAXY AUTOMATIC SANDBLASTER.....	19
FR... SABLEUSE AUTOMATIQUE GALAXY .....	12
DE ... GALAXY SANDSTRAHLAUTOMAT .....	15
PO ... AREADORA AUTOMÁTICA GALAXY .....	18
IT..... SABBIATRICE AUTOMATICA GALAXY .....	21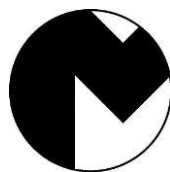

**MESTRA®**




TALLERES MESTRAITUA S.L.


Txori-Erri Etorbidea, 60



Tel. + 34 944530388 - Fax + 34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA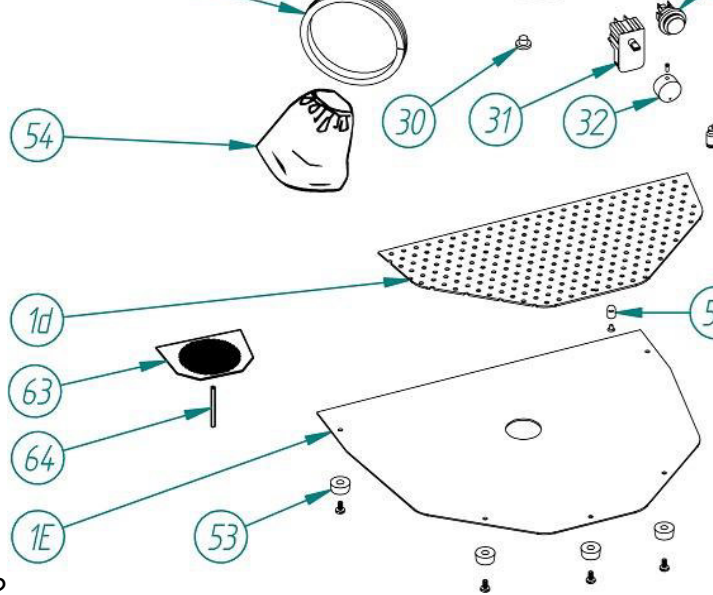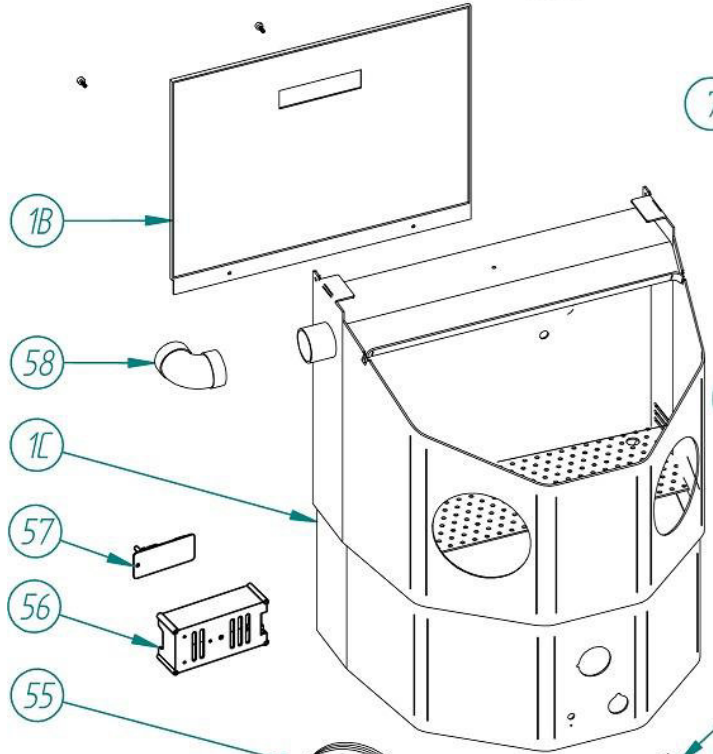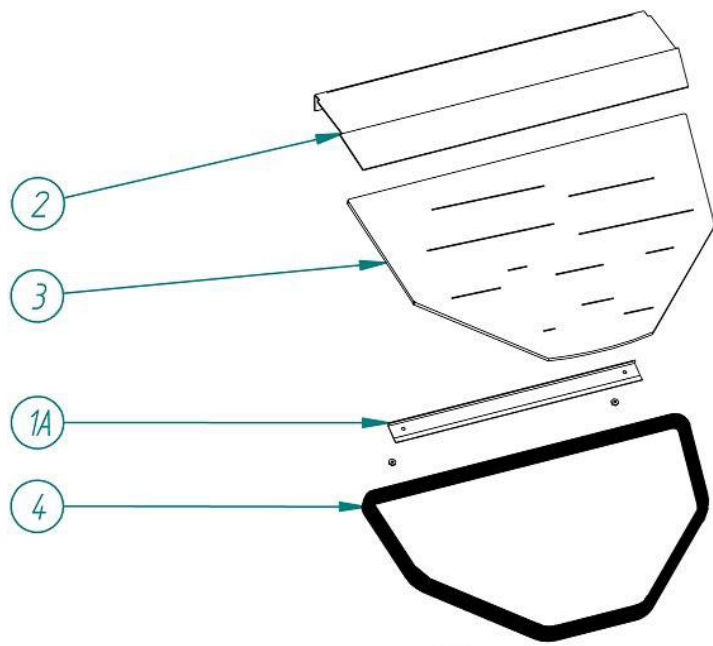

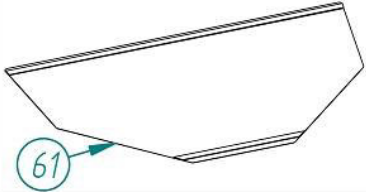	
61	62
R-080235-50 (10 u.)	R-080235-51

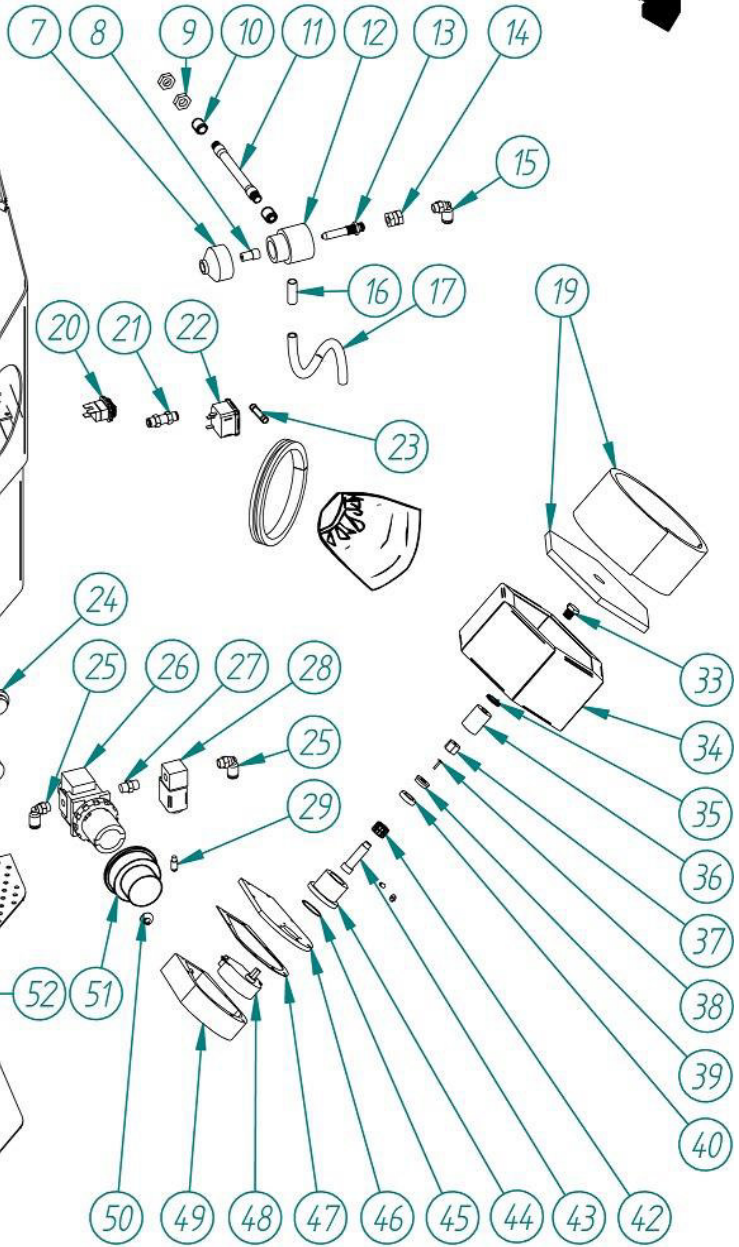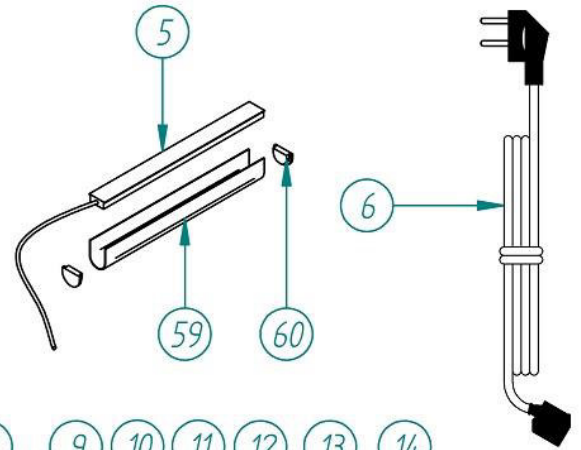Nº	Ref.	Concepto	Description
1A	080235-38	Bisagra cristal	Glass hinge
1B	080235-39	Tapa trasera carcasa	Housing endloader lid
1C	080235-40	Carcasa	Housing
1D	080235-67	Bandeja perforada con pomo	Slotted tray with knob
1E	080235-42	Base carcasa	Housing base
2	080235-02	Visera cristal	Glass visor
3	080235-03	Cristal	Glass
4	080262-03	Burlete cristal	Glass strip
5	080235-60	Barra de LED	LED bar
6	000250	Cable	Cable
7	080270-31	Porta boquillas	Nozzle holder
8	080235-25	Boquilla carburo de boro	Boron carbide nozzle
9	080235-07	Tuerca pistola	Gun nut
10	030350-03	Espárrago	Stud
11	080235-08	Flexo pistola	Gun arm
12	080235-09	Cuerpo pistola	Gun body
13	080235-10	Inyector	Injector
14	080235-11	Racor prolongación inyector	Prolongation injector connector
15	000312	Racor codo plástico 1/8 Ø6	1/8 elbow Ø6 plastic connector
16	080235-12	Casquillo pistola	Guns sleeve
17	080235-13	Manguera aspiración	Aspiration hose
19	080235-15	Sufrideras cesta	Basket rubbers
20	080422-17	Interruptor luz	Switch
21	080251-07	Pasamuros recto doble Ø6	Double connector Ø6
22	080490-06	Base con porta fusibles	Fuse base
23	000231	Fusible 4 A	Fuse 4 A
24	080235-16	Pulsador	Pusher
25	000310	Racor codo plástico Ø6 1/4	1/4 elbow Ø6 plastic connector
26	080235-17	Manorregulador	Pressure control
27	080246-07	Acople electroválvula	Couple electrovalve
28	080242-23	Electroválvula	Electrovalve
29	080242-24	Filtro electroválvula	Electrovalve filter
30	080270-33	Tapón vaciado tolva	Drain plug

Nº	Ref.	Concepto	Description
31	080235-18	Temporizador	Timer
32	080550-11	Pomo temporizador	Time pome
33	080235-19	Tomillo cesta	Basket screw
34	080235-20	Cesta	Basket
35	080235-21	Arandela estriada cesta	Basket washer
36	080235-22	Bocallave	Socket
37	080523-08	Tuerca de eje	Shaft nut
38	080523-07	Tomillo punta eje	Shaft tip screw
39	080235-24	V-ring	V-ring
40	080515-08	Casquillo con fieltro	Sleeve with filter
42	080235-37	Bloqueo motor	Blockage motor
43	080235-27	Eje	Shaft
44	080235-29	Casquillo motor	Motor sleeve
45	080235-30	Junta tórica casquillo	O-ring sleeve
46	080235-31	Tapa motor	Motor lid
47	080235-32	Junta motor	Motor O-ring
48	080550-15	Motor	Motor
49	080235-33	Carcasa motor	Housing motor
50	080010-05	Pasa cables	Cable bushing
51	080235-55	Protector manorregulador	Pressure control protector
52	080075-07	Pomo bandeja	Salver pome
53	080490-31	Pata	Leg
54	080242-28	Manguito	Wool sleeves
55	080270-30	Brida	Flanges
56	080093-44	Caja fuente alimentación	Power source box
57	080093-15	Fuente alimentación	Power source
58	080242-25	Codo salida aspiración	Exit aspiration elbow
59	080235-62	Protector LED	LED protector
60	080235-63	Tapa protector LED	LED protector lid
61	080235-50	Protector cristal (10 u)	Glass protector (10 u)
62	080235-51	Protector mandos	Command protector
63	080235-66	Rejilla fondo	Bottom grating
64	080235-61	Varilla rejilla	Grating column

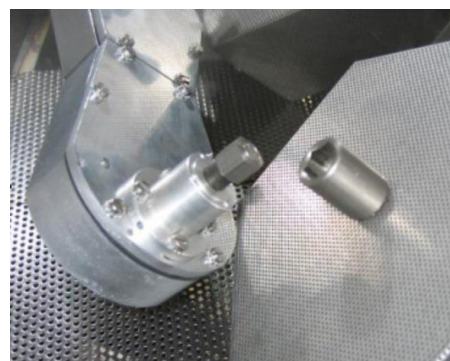
Fig. 1

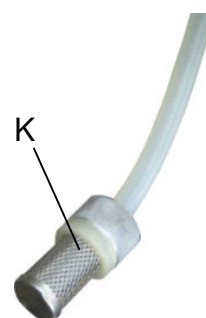
Fig. 2

A	Pistola sopladora / <b>Blasting gun</b> / <i>Buse et porte-buse</i> / Injektor / <b>Pistola soffiatrice</b> / <i>Pistola assopradora</i>
B	Cesta / <b>Basket</b> / <i>Panier</i> / Korb / <b>Cestello</b> / <i>Cesto</i>
C	Manguitos / <b>Sleeves</b> / <i>Manchons</i> / Durchgriffe mit Schläuchen / <b>Manicotti</b> / <i>Mangas</i>
D	Interruptor luz / <b>Light switch</b> / <i>Interrupteur lumière</i> / Lichtschalter / <b>Interruttore luce</b> / <i>Interruptor luz</i>
E	Entrada de aire / <b>Air intake</b> / <i>Entrée d'air</i> / Lufteintritt / <b>Ingresso aria</b> / <i>Entrada de ar</i>
F	Toma de corriente / <b>Power outlet</b> / <i>Prise de courant</i> / Stromanschluss / <b>Presa corrente</b> / <i>Tomada de corrente</i>
G	Manorregulador / <b>Pressure regulation valve</b> / <i>Mano régulateur</i> / Druckregler / <b>Regolatore pressione</b> / <i>Regulador de pressão</i>
H	Manómetro / <b>Pressure gauge</b> / <i>Manomètre</i> / Manometer / <b>Manometro</b> / <i>Manómetro</i>
I	Temporizador / <b>Timer</b> / <i>Temporisateur</i> / Zeitschalter / <b>Timer</b> / <i>Temporizador</i>
J	Botón funcionamiento manual / <b>Manual operation button</b> / <i>Bouton mode manuel</i> / Handbetriebstaste / <b>Pulsante funzionamento manuale</b> / <i>Botão funcionamento manual</i>
K	Filtro / <b>Filter</b> / <i>Filtre</i> / Filter / <b>Filtro</b> / <i>Filtro</i>

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The Galaxy Ref. 080235 is an automatic sand blaster specially designed for sandblasting small pieces within the field of dental prosthesis on continuous automatic mode or ion manual mode. In order to achieve an optimal operation and performance of the unit, we strongly recommend you to read carefully the following instructions:

## INSTALLATION

Carefully unpack the unit. You will find the following accessories inside the cabin:

- A Ø 4 / 6 mm pipe to connect the unit to the air supply.
- A 90° PVC elbow to be fitted into the outlet aspiration hose.
- Two plastic covers to protect the glass (Ref. 080235-50).
- A basket.

Choose a firm, flat surface to place the unit, far from heat or vibration sources. The room atmosphere should never be damp. An excess of humidity could compact the aluminium oxide particles inside the cabin and, therefore, obstruct the blasting nozzles. Finally, place the unit at a height that allows an easy manipulation of the objects inside the cabin.

Fit the connecting pipe supplied with the unit into the inlet adaptor (E) placed at the right side panel of the machine. The other end should be fitted into an intake of pressured air fed by a compressor with a minimum pressure of 7 bar (0.7 MPa) and a minimum flow of 100 litre/minute.

Connect the power cord to a grounded 230 V, 50/60 Hz power source.

Fill the hopper in the cabin with aluminium oxide of a maximum diameter of 300 micron. To obtain a maximum autonomy, load the hopper up to a maximum height of 2 cm below the tray.

**NOTE: the unit is optimized for 200 micron particles and getting an stable jet. With smaller particle diameters (100 microns) the jet may be more irregular.**

**The unit is factory supplied with a 3 mm boron carbide nozzle (Ref. 080235-25), material much more abrasion-resistant than the traditional tungsten.**

## PRESSURE REGULATION

The blowing pressure of the blasting gun can be regulated at will between 2 and 5 bar (0.2 and 0.5 MPa), according to the nature and special features

of the work to be done. Take into account that the highest pressure will be limited by the maximum pressure of the air supply we are using. In any case, the pressure should never be higher than 6 bar (0.6 MPa).

The unit features a pressure regulation valve (G) to select the working pressure easily. To adjust the pressure pull the valve outwards and turn it until the intended pressure has been reached. The value of the pressure is shown in the built-in pressure gauge (H). After adjusting the pressure, block the valve knob by pressing it inwards.

The pressure gauge shows the gun's blowing pressure at any time, even when the gun is not being used.

## CONNECTING TO THE ASPIRATION UNIT

The blasting cabin should always be used connected to a pressured aspiration unit. For this purpose there is an outlet hose on the upper left side panel of the unit. The aspiration unit to be used has to create a light depression inside the cabin for the dust particles to circulate smoothly and continuously towards the outlet hose.

**NOTE: make sure that the aspiration unit used can suck abrasive particles. We strongly recommend the MESTRA Eolo Plus Ref. 080531 aspiration unit.**

The factory-supplied PVC elbow can be adapted according to the needs and shape of the pressured aspiration unit to be used.

## OPERATION

The Galaxy Ref. 080235 automatic sandblaster can be operated on automatic or manual mode.

### Automatic mode:

1. Fit the basket (B) into the motor shaft (Fig. 1).

**Never force the motor shaft to rotate by hand, whether the basket is fitted or not. The motor's inner mechanism could be damaged.**

2. Put the pieces you want to sandblast inside the basket.
3. Position the gun (A) facing the pieces, bearing in mind that the basket will rotate clockwise. At the beginning the motor may start to rotate anti-clockwise, but before completing its first turn it will start to rotate clockwise.

- Direct the sandblast accurately to prevent the sand going straight to the cabin walls. The sandblast could damage the cabin.
- Close the cabin lid making sure the glass fits tightly into its frame.
- Turn on the pressured aspiration unit.
- For a better vision of the piece, turn on the light inside the cabin by pressing the switch (D).
- To select the automatic operation time, turn the timer (I) clockwise (maximum 30 minutes).
- The basket will start to rotate and the gun will blast aluminium oxide automatically.
- If you want to stop the automatic operation, turn the timer (I) anti-clockwise.

### Manual mode:

The blasting gun can be operated without rotating the basket or using the timer.

- The basket can be removed for an easier operation (on manual mode the motor does not rotate).
- Open the cabin lid and put the piece you want to manipulate inside it. The pieces can be introduced through the sleeve hole (C) as well. Make sure the level of the aluminium oxide in the hopper is correct.
- Close the cabin lid making sure the glass fits tightly into its frame.
- Protect yourself properly: use protection goggles, gloves and mask.
- Turn on the pressured aspiration unit.
- For a better vision of the piece, turn on the light inside the cabin by pressing the switch (D).
- Position the piece to face the blast by manipulating it through the built-in sleeves (C) inside the cabin.
- For the gun to start blasting aluminium oxide press the button (J). On manual mode the motor does not rotate.

### MAINTENANCE AND CLEANING

In order to achieve a long working life of the machine, we strongly recommend you:


Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide.

Preferably use dry-piston air compressors. The presence of oil particles in the air could make the sand compact and obstruct the gun's nozzle. To prevent the inner walls of the cabin wearing away due to abrasion, do not aim the aluminium oxide blast straight at the cabin walls.

Periodically remove the aluminium oxide from the cabin. To do this, use the drain plug in the bottom of the hopper.

For an overall cleaning of the unit, use a dry cloth or moistened with a proper product.

In case of obstruction of the gun, dismount the nozzle holder. Clear the nozzle by blowing in the opposite direction to the air flow, or, if needed, by using a pin or a thin wire. Then set the gun again, making sure it is correctly centred.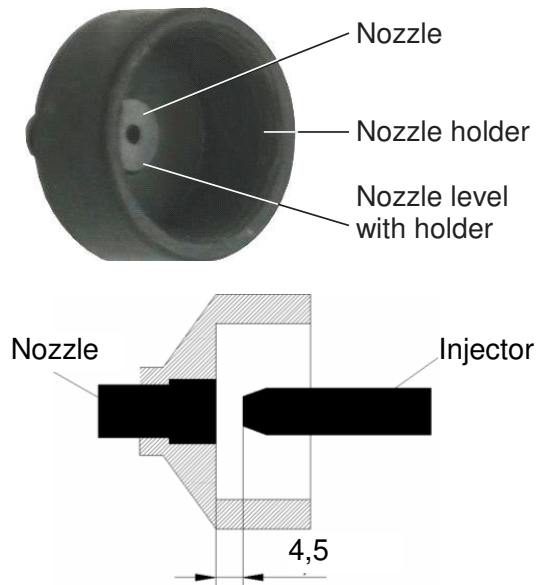

If the nozzle came loose, glue it to its holder with cianocrilate. The nozzle should be level with the inner surface of the nozzle holder, to make sure that the distance from the injector to the nozzle is 4.5 mm and the Venturi effect takes place.

Always use high quality aluminium oxide. **Talleres Mestraitua, S.L.** recommends the use of **MESTRA** aluminium oxide. The gun ( $\varnothing$  3 mm nozzle) can blast aluminium oxide up to particles of  $\varnothing$  300 micron. If you use aluminium oxide particles bigger than  $\varnothing$  300 micron, you may need to remove the filter (K) placed at the end of the gun's pipe, inside the hopper (Fig. 2). Without any filter obstructions in the nozzle will be more frequent.

### PRECAUTIONS

Never exceed a 6 bar pressure (0.6 MPa). The optimal working pressure lies between 2 and 5 bar (0.2 and 0.5 MPa).

Never force the motor shaft to rotate by hand, whether the basket is fitted or not. The motor's inner mechanism could be damaged.

Always work with the cabin lid closed and correctly protected. Use protection mask and goggles.

Connect the cabin's outlet hose to an aspiration unit.

If the protection sleeves (C) tear or wear, replace them immediately..

Do not allow children or unskilled personnel to operate the machine.

Always use a high quality aluminium oxide (it is strongly recommended to use **MESTRA aluminium oxide**). A very low quality aluminium oxide can produce obstructions in the nozzle.


Make sure that the aspiration unit can suck abrasive particles. We strongly recommend the MESTRA Eolo Plus Ref. 080531 aspiration unit. Connect the power cord to a grounded 230 V, 50/60 Hz power source.

In case of leaks in the pressured air circuit, consult the installation engineer immediately.

### MAINTENANCE OF THE PISTOL FLEX

Do not bend or twist excessively the flex that holds the pistol and allows positioning it. It could break and then the abrasive would not go up the pipe. When needed, replace it following these steps:

1. Loosen the two lower bolts in the back cover and remove the cover.
2. Undo the cap nut and the normal nut that hold the flex from the back cover. They may be tight due to the sealant.

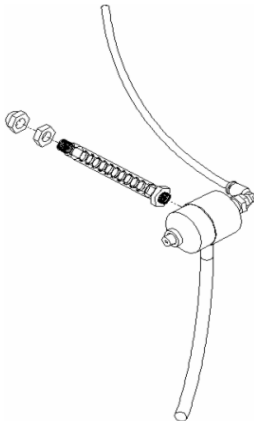3. Disengage the white pipe from the pistol by pulling it.

4. Disengage the blue pipe from the pistol by pulling it while you press the adaptor.
5. Unscrew the flex from the body of the pistol with a pair of pliers. It may be very tight due to the sealant. Hold the flex as near the pistol as possible.
6. Screw in the spare flex. Apply a sealant to prevent air leaks. Do not twist the flex. Hold the flex as near the pistol as possible.
7. Engage the two pipes again.
8. Fix the flex to the back cover with the two nuts. Apply a sealant to prevent air leaks.

### UNIT CONTROLS PLASTIC PROTECTOR

In order to protect the unit controls from abrasive powder, you can stick the supplied plastic protector on the housing (see the picture).

### TECHNICAL SPECIFICATIONS

Height:	505 mm
Width:	440 mm
Depth:	385 mm
Weight:	19 kg
Working pressure:	2 - 5 bar 0.2 - 0.5 MPa
Maximum air consumption:	90 litre/min at 5 bar
Voltage:	230 V, 50/60 Hz
Power consumption:	15 W

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*TALLERES MESTRAITUA S.L. is a company manufacturing tools and appliances for the dental prosthesis sector, from the business address shown at the bottom of the page.*

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**I HEREBY STATE:** That the **GALAXY AUTOMATIC SANDBLASTER Ref. 080235**, with the following characteristics:

Alto / Height:	505 mm
Ancho / Width:	440 mm
Fondo / Depth:	385 mm
Peso / Weight:	19 kg
Presión de trabajo / Working pressure:	2 - 5 bares 0.2 - 0.5 MPa
Consumo máximo de aire / Maximum air consumption:	90 l/min (5 bares)
Tensión / Voltage:	230 V, 50/60 Hz
Consumo / Power consumption:	15 W

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*Meets the low voltage requirements set out in Directive 2014/35/UE, in Directive 2014/30/UE concerning electromagnetic compatibility and in Directive 2006/42/EC concerning machinery safety.*

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*In witness whereof, the company issues this declaration of conformity in Sondika, on 03 December 2019.*

Firma / Signed:

Ignacio Mestraitua,  
Director Gerente / Managing Director

### MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. + 34 944530388 - Fax + 34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA