R-080422

E.....	BAÑO DE LIMPIEZA POR ULTRASONIDOS.....	3
GB.....	ULTRASONIC CLEANING BATH.....	7
F.....	BAIN DE NETTOYAGE PAR ULTRASONS.....	10
D.....	ULTRASCHALLREINIGUNGSGERÄT	13
I.....	BAGNO DI PULIZIA AD ULTRASUONI.....	17
P.....	BANHO DE LIMPEZA POR ULTRA SONS	21

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tfno. + 34 94 453 03 88 Fax + 34 94 471 17 25

E-mail: mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 09/03/05

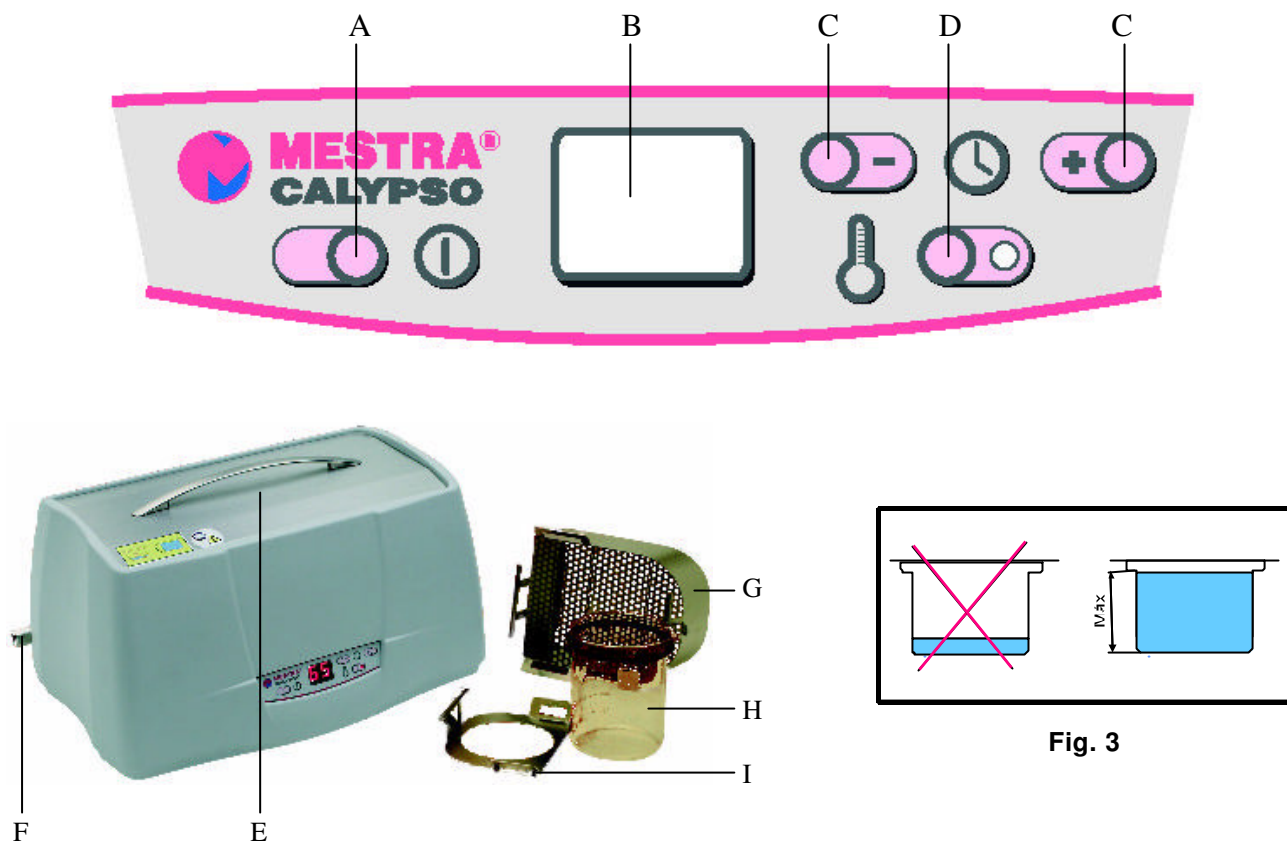
Fig. 3

A	Botón marcha-paro / Start-Stop button / <i>Bouton marche-arrêt</i> / Start-Stopp Taste / Pulsante Start-Stop / <i>Botão funcionamento-parada</i>
B	Display / Display / <i>Display</i> / Display / Display / <i>Display</i>
C	Teclas subir-bajar / Up-Down keys / <i>Touche monter-descendre</i> / Plus-Minus Taste / Tasti sollevare-abbassare / <i>Teclas subir-descer</i>
D	Tecla resistencia de calentamiento / Heater resistor key / <i>Touche résistance de chauffe</i> / Taste Heizwiderstand / Tecla resistênciã de aquecimento / <i>Tasto resistenza di riscaldamento</i>
E	Tapa / Lid / <i>Couvercle</i> / Deckel / Coperchio / <i>Tampa</i>
F	Desagüe / Waste pipe / <i>Écoulement</i> / Abfluss / Scarico / <i>Escoamento</i>
G	Cestillo de acero inoxidable / Stainless steel basket / <i>Panier en acier inoxydable</i> / Edelstahlkörbchen / Cestello in acciaio inox / <i>Cesto de aço inoxidável</i>
H	Vaso de cristal / Glass vessel / <i>Vase en verre</i> / Glasgefäß / Vasetto di vetro / <i>Vaso de vidro</i>
I	Soporte vaso de cristal / Glass vessel support / <i>Support du vase en verre</i> / Glasgefäßhalter / Supporto vasetto di vetro / <i>Suporte vaso de vidro</i>

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R-080422 is a practical Ultrasonic Bath specially designed for a wide range of applications. For a long working life and optimum performance, we recommend you read carefully the following instructions:

INSTALLATION

- Carefully unpack the Ultrasonic Bath. Check that in the packaging you find the following accessories:
 - 1 Stainless Steel Basket (G).
 - 1 Stainless Steel Glass vessel support (I).
 - 1 90 mm diameter Glass vessel and 600 ml capacity (H).
 - 1 Stainless Steel lid (E).
 - 1 set of Burr Cleaner Tweezers.
- Choose a flat and firm surface well away from heat sources or vibrations. It is advisable to place the machine near a sink, so as to keep the drainage pipe connected to the outlet.
- Connect the machine to 230 V, 50/60 Hz with earthing.
- Place the stainless steel basket (G) inside the tank. Fit the flanges on the sides of the basket into the slots on the unit.
- Fill the tank with water or a cleaning solution, taking into account that the minimum level should not be less than 10 cm from the bottom of the tank. Optimum level is on the press line on the upper section of the tank (Fig. 3).
- Depending on the cleaning operation, place the basket (G), the vessel support (I), the glass vessel (H) and the Burr Cleaner Tweezers at the appropriate locations inside the tank.
- The machine is ready for operation.

USING THE MACHINE

1. Place the items to be cleaned inside the basket (G).
2. In the case of small items or items exceeding tray dimensions, it is possible to use the glass vessel (H). It has been experimentally proven that the cleaning effect remains in objects inside a vessel filled with cleaning solution in contact with the tank bath. If you need to use this system, put the glass vessel (H) inside the tank, and make sure it is correctly positioned using the vessel support. When cleaning burrs or small items, we recommend you use the Burr Cleaner Tweezers supplied with the machine.
3. Check that the cleaning solution completely covers all the items in the basket or glass vessel.
4. The machine has a practical heating system which starts up when control key (D) is pressed. If this option is operational, a heating resistor will raise the temperature of the liquid in the tank to 50/55 °C. We recommend this heating system be used whenever the unit is started up with cold cleaning solution.
5. Regulate cleaning time using the Up/Down keys (C) on the electronic control unit. The selected time will be shown on the display (B). Machine operation time can be regulated between 1 and 99 minutes. The time required for cleaning objects depends on many aspects: type of cleaning solution used, the geometry of the item, materials, etc.
6. Press Start/Stop button (A). The machine will start working for the set time. The display (B) will blink to show the time remaining to end of cycle.
7. The machine can be stopped at any step of the cleaning cycle. It has two stop modes:

- **Momentary stop:** The machine is stopped merely for checking purposes, or to add or take away items cleaned. In this case you merely press the Start/Stop button (A) when the cycle is running. To continue the cycle press the Start/Stop button again.
 - **Cycle cancellation:** If you want to completely cancel a cycle, keep the Start/Stop button depressed for 3/5 seconds. The control will then emit an acoustic signal and return to Stand-by mode.
8. Once the programmed cycle finishes the control will emit an acoustic signal and return to Stand-by mode.
 9. Take the basket out of the tank. If required, place the items in the basket under a jet of water to rinse them.

When operating, the machine heats the cleaning solution in the tank. It must be taken into account that the electrical power supplied by the ultrasonic transducers is transmitted entirely to the cleaning solution, and this produces a warming effect. In any case, liquid temperature should never exceed 70/80 °C. When operating with lid (E), the warming effect is greater than during operations carried out without the lid.

CLEANING SOLUTIONS

Cleaning solutions can be classified into two types: those with water as the basic element and cleaning solution, and solvents.

The most common cleaning method is with water, as it has an acceptable capacity to dissolve inorganic substances. However, it is extremely limited in dissolving organic substances. For this type of compound, water is a good medium for the dispersion of waste particles, and a good support for specific detergents which dissolve organic substances.

In most cases a mixture of water and domestic washing up liquid can be used as a cleaning solution. Nevertheless, when the quantity or nature of dirt is resistant to this type of cleaning solutions, specific solvents for ultrasonic baths have to be used. **Under no circumstances should inflammable solvents or environmentally unfriendly products be used.**

MAINTENANCE AND CLEANING

In order to ensure optimum performance and reliability of the unit, we recommend the following:

- Do not overload the tank with objects to be cleaned. This makes the bath less efficient, and can damage the unit.
- Do not place very heavy or large objects in the bottom of the tank, if this can be avoided.
- Never allow the cleaning solution to reach levels less than 10 cm from the bottom of the tank.
- Use chemical cleaning products compatible with the objects to be treated. Avoid corrosive solvents.
- When items (especially valuable items) are being cleaned for the first time, carry out a trial run first with just one item. Ask the manufacturer which cleaning solution is recommended.
- During normal operation the bath and the base of the machine can heat up considerably. Values below 70/80 °C are completely normal, and can be reached after long continuous operation with the lid on. If you wish to reduce the temperature, take off the lid and leave the machine for a few minutes before starting a new cycle.
- After using the bath, clean the tank and the external body with a cloth moistened in soapy water. Always unplug the unit before cleaning it.
- The machine has a security sensor which disconnects it from the mains in the event of overheating, or operation without cleaning liquid. If the sensor activates, disconnect the unit from the mains and allow it to cool down for at least an hour before using it again.
- It is advisable to clean the waste pipe filter inside the tank on a regular basis.
- Do not start the unit when the tank is empty.

IMPORTANT- The Ultrasonic Bath transducers' supply voltage can be hundreds of volts, and so it is important the Safety Regulations be observed:

- 1) Do not open the lid when the cable is connected to the mains.
- 2) Avoid splashing water or other conducting liquids onto the unit's housing.
- 3) Always connect the unit to earthed power outlets.

PRECAUTIONS

- ◆ Connect the unit to a 230 V, 50/60 Hz earthed power outlet.
- ◆ Do not use inflammable solvents as cleaning solutions.
- ◆ Do not start up the unit with less than 10 cm of liquid at the bottom of the tank.
- ◆ Do not allow children or unskilled people to handle the unit.
- ◆ Do not overload the tank.
- ◆ Never submerge the unit in any liquid to clean it.
- ◆ Do not start up the unit when the tank is empty.
- ◆ Ultrasonic cleaning is not suitable for organic pearls.
- ◆ Do not open the unit while it is connected to the mains.
- ◆ The machine lid may be hot after a cycle. Remove it carefully.
- ◆ If you are not sure of the exact effects of the cleaning solution on a given item to be cleaned, first carry out a test with a single item.

TECHNICAL CHARACTERISTICS

Tank capacity:	4.5 L
Tank dimensions:	300 x 150 x 100 mm
Outer dimensions:	425 x 270 x 220 mm
Supply voltage:	230 V, 50/60 Hz
Construction material:	Stainless steel
Transducer power	100 W
Total power level	240 W
Frequency	40 KHz

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