R-080502


E.....	BOX CON ASPIRACIÓN PARA PULIDO	5
GB	BOX WITH SUCTION FACILITY FOR POLISHING .	7
F.....	BOX POUR POLISSAGE AVEC ASPIRATION	9
D	BOX MIT ABSAUGUNG ZUM POLIEREN	11
P.....	BOX COM ASPIRAÇÃO PARA POLIMENTO.....	13
I	BOX CON ASPIRAZIONE PER PULITURA.....	15

MESTRA®


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60
Tel. +34 944530388 - Fax +34 944711725
mestra@mestra.es - www.mestra.es
48150 SONDIKA - BILBAO - ESPAÑA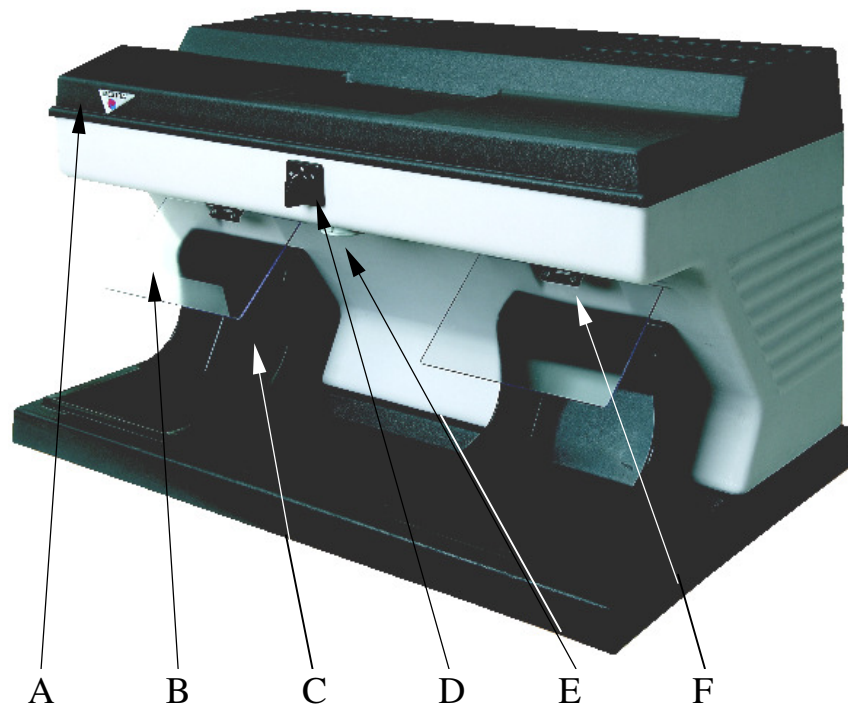

R-080502

- | | |
|---|--|
| A | Tapa / Cover / <i>Couvercle</i> / Deckel / Tampa / <i>Coperchio</i> |
| B | Visera protectora / Protection guard / <i>Visière de protection</i> / Schutzblende / Viseira protectora / <i>Visiera di protezione</i> |
| C | Pantalla / Screen / <i>Augette</i> / Gummischirm / Ecrã / <i>Schermo</i> |
| D | Interruptor de marcha / Operation switch / <i>Interrupteur</i> / Betriebsschalter / Interruptor de funcionamiento / <i>Interruttore di avvio</i> |
| E | Toma de corriente / Current pickup / <i>Prise de Courant</i> / Stromanschluss / Tomada de corrente / <i>Presa di corrente</i> |
| F | Entrada de aire / Air inlet / <i>Admission d'air</i> / Lufteintritt / Entrada de ar / <i>Entrata d'aria</i> |

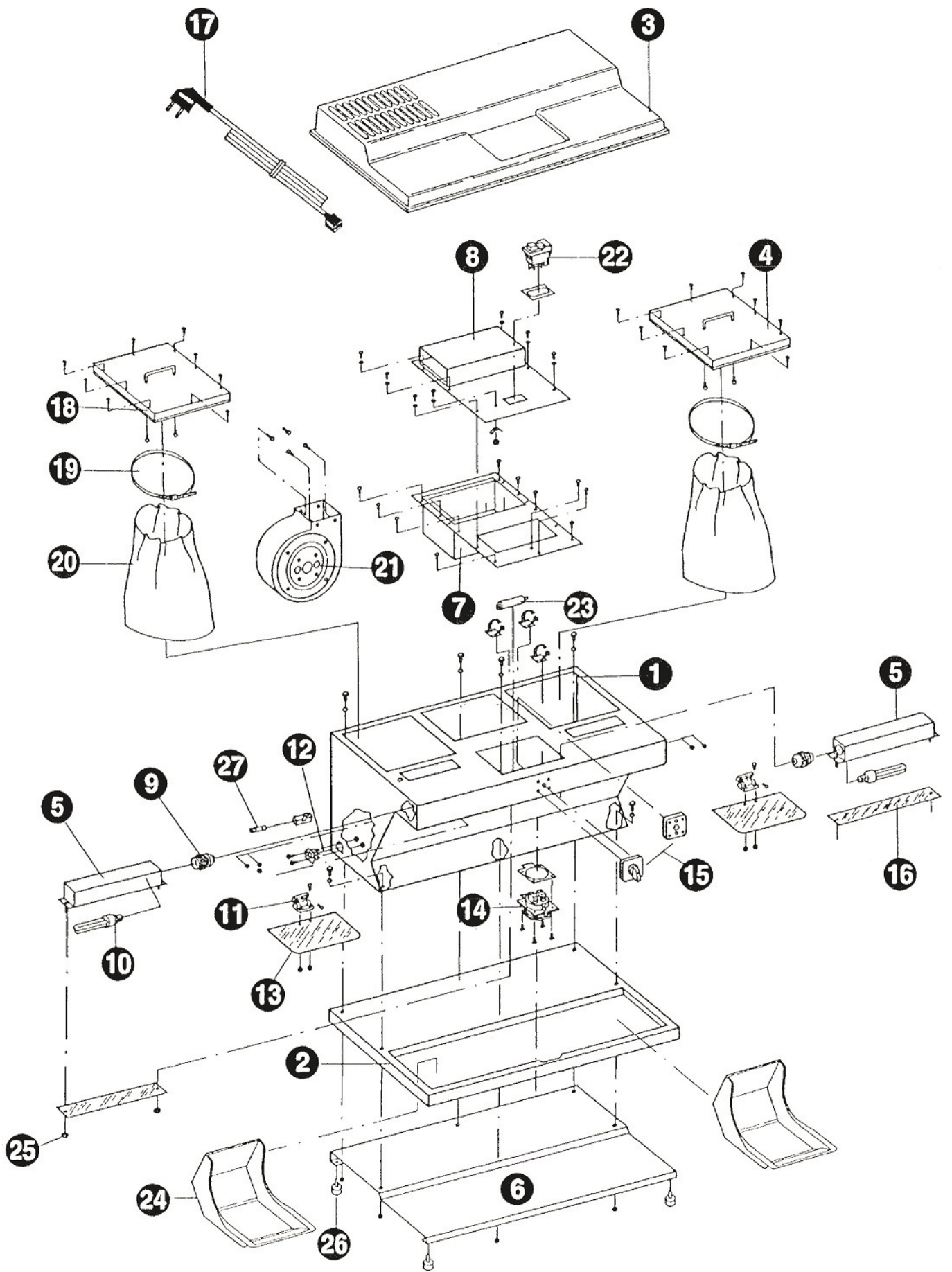Nº	Ref.	DESCRIPCION	DESCRIPTION
1	080502-01	Carcasa	Housing
2	080502-02	Base plástico	Plastic base
3	080500-02	Tapa plástico	Plastic lid
4	080502-04	Tapa portafiltros	Filter holder lid
5	080502-05	Porta lámparas	Lamp holder
6	080502-06	Base chapa	Metal base
7	080502-07	Porta ventilador	Fan holder
8	080502-08	Tapa superior	Upper lid
9	080502-09	Casquillo	Bushing
10	080502-10	Lámpara (230V)	Lamp (230V)
	080502-10B	Lámpara (115V)	Lamp (115V)
11	080502-11	Bisagra	Hinge
12	080502-12	Base de conexión	Connection base
13	080502-13	Visera	Visor
14	080084-01	Base enchufe Schuko	Schuko base
15	080500-16	Interruptor ventilador	Fan switch
16	080502-16	Pantalla lámpara	Lamp shield
17	000250	Cable (230V)	Cable (230V)
	000252	Cable (115V)	Cable (115V)
18	080500-03	Porta filtros	Filter holder
19	080500-04	Abrazadera	Clamp
20	080500-05	Filtro	Filter
21	080500-23	Extractor (230V)	Extractor (230V)
	080500-23B	Extractor (115V)	Extractor (115V)
22	080080-40	Interruptor de seguridad (230V)	Safe switch (230V)
	080080-40B	Interruptor de seguridad (115V)	Safe switch (115V)
23	080500-33	Condensador 6 μ F (230V)	6 μ F capacitor (230V)
	080500-33B	Condensador 20 μ F (115V)	20 μ F capacitor (115V)
24	080500-17	Pantalla goma	Rubber guard
25	080500-11	Tuerca lámparas	Lamp nuts
26	080500-42	Patas	Legs
27	000236	Fusible 7A	Fuse 7A
	000237	Fusible 15A	Fuse 15A

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The R-080502 is a practical Box with a suction unit for polishing. It has been specially designed for use in the dental prosthesis sector.

The Box has a suction and filtering system, a lighting system and two polycarbonate protection screens (C). The design has also taken account of adaptations for a laboratory polisher.

For the user's comfort, the appliance has a current pickup (E) to the rear for connection of the polisher.

INSTALLING THE POLISHER

As mentioned above, the box has been designed to incorporate a polishing unit. The various procedures for installation are as follows:

- If you purchase the Box with one of the MESTRA polishers (R-080150, R-080160, R-080162) the polisher is supplied with a bolt fastening to the base of the Box.
- If you intend to use the Box with a non-MESTRA polisher, place the polisher on the base of the Box and ensure that the unit is secure. Connect the polisher to the Box current pickup (E).
- If you already have a MESTRA polisher, this may be bolted to the base of the Box. For this operation you will have to use a hand drill to perforate the plastic base. This is easy to drill through, since underneath there is a metal chassis over which the reference holes have already been drilled. Once the holes have been drilled, you merely secure the polishing unit with four M6 bolts.
- The Box is supplied with four rubber studs for the base.

CONDITIONS FOR USE

We recommend the following instructions be observed for proper use of the machine:

- Carefully unpack the box and suction unit for polishing. Check that it has suffered no damage in transit.
- Connect the machine to an earthed supply of 230 V, 50/60 Hz.
- Lower the safety guard, and check for correct positioning of the rubber screens (C). Incorrect positioning of these items is not only dangerous, but will reduce suction efficiency.
- The machine has a security switch just behind the cover (A). Every you plug in the machine you need to switch on the security switch.
- Use switch (D) to select between the two operating modes for the box and its suction facility. Turning the switch to the right will activate the suction and lighting simultaneously. If

the switch is turned to the left, the lighting will be operational with no suction.

Operation without the suction facility is only to be used in extraordinary circumstances, in which the suction facility could impair operation of the unit. As a general rule, the switch should be turned to the right, thus obtaining simultaneous lighting and suction.

- The following protective equipment is recommended for polishing or grind-ing:
 - safety goggles;
 - safety gauntlets;
 - face-mask;
 - As a general rule do not use the polisher without the exhauster.
 - Proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine.
 - Be careful with long hair.
 - No objects similar to ropes or chains that could be hooked by the machine.
 - Place the protective screen properly.
 - Work with the visor in place.
- After the unit has been used, we recommend you clean the inside of the protection screen and the support tray.

The machine has a protection system to prevent sudden start-ups. If this system were not installed, the polisher motor would stop in the event of a power outage. However, when the power returned, the machine could then start up again more or less unexpectedly, and an accident could occur if someone were handling the shaft of the motor at that time.

To prevent such risks, TALLERES MESTRAITUA, S.L. has included a switch for the box and its suction facility for polishing, which automatically disconnects the unit in the event of a power outage, and this is located underneath the upper dust guard cover (A) on the unit. Also with this device it is possible to set up olden polisher to meet the security normative. If an outage should occur during a work session the protection device will activate, and the unit must then be manually reset by using the switch again.

CHANGING THE SUCTION FILTERS

The R-080502 box has a closed circuit for suction, which collects polishing dust in two filters inside the unit. You will know when the filters are dirty because the machine will be less efficient, and the noise from the suction motor will become louder. If these two

phenomena are observed, we recommend you replace the filter as soon as possible:

1. Disconnect the machine from the mains power supply and remove the dust guard cover.
2. Inside the machine you will find two square cast aluminium covers, with a grip for each. The covers are not physically attached to the polisher casing, and so they may be lifted up directly.
3. The filters are hanging from the bottom of each cover, and are attached to the covers by a clamp. Use a screwdriver to release the clamp.
4. Remove the dirty filter and insert a new filter. TALLERES MESTRAITÚA, S.L. recommends that **MESTRA** spares be used, since these are the only spare parts which can ensure best performance and reliability.
5. Check that the filter outlet is correctly positioned over the circular mouth of the cover, and that there are no wrinkles, uncovered spaces or points which air could leak through. Tighten the clamp back into position.
6. Reassemble the covers with the new filter as before. Two cover centring stubs show the correct position and housing. You must also check that the sponge seals around the perimeter of the cover fit properly over the support surface, and do not allow any air to leak out.
7. Refit the dust guard cover (A). The machine is now ready for use.

Since each operational side for the machine (left/right) may be used with varying frequency and has an independent filter, it is possible that occasionally it is necessary to change the filter on one side only. Check periodically whether the filters are full, and replace them as required.

We also recommend you always replace the filters with an original spare part, and do not attempt to clean them. Cleaning these items by simply emptying the filter bag is not recommended, since dirt particles are trapped inside the filter fabric. Operations carried out using a filter which has been cleaned using this procedure considerably reduces suction performance.

MAINTENANCE AND CLEANING

To ensure proper, reliable and longlasting operation, we recommend the following:

- After the machine has been used, use a brush to clean the dust off the inside of the protection screen (C), and clean the surface of the front tray.
- Clean the screens thoroughly on a regular basis, and also wash the polisher's outer stainless steel casing with a damp soapy cloth.
- A damp soapy cloth should also be used to clean the motor and the support. Never use alcohol, solvents or other inflammable liquids.
- Should abnormal noises or vibrations be detected, send the machine to the technical assistance unit.

- Replace the suction filter when it is dirty. You will know it is dirty because the performance of the machine will be impaired, and an abnormal noise will be heard when the suction unit is activated. To replace filters, proceed as explained above.

PRECAUTIONARY MEASURES

Before connecting the machine to the power supply, ensure that supply voltage is correct (230 V, 50/60 Hz).

- During operation of the unit, use sufficient protection – gloves, safety goggles and a face-mask.
- Do not allow children or non-specialist staff to handle the machine.
- Never polish, brush or grind materials which may release particles which are toxic when inhaled.
- Before using the machine, ensure that it is resting on a firm and solid base.
- Do not use the machine in close proximity to solvents, alcohol or other inflammable materials. Extreme care should be taken when grinding parts which may cause sparks.
- Do not use the machine without the suction unit. If the nature and characteristics of a specific task require this exceptional circumstance, disconnect the suction facility and try to minimise the duration of such operations.

TECHNICAL CHARACTERISTICS

Approximate consumption:	250 W
Suction flow rate:	500 m ³ /h
Voltage:	230 V, 50/60 Hz
Height:	500 mm
Width:	800 mm
Depth:	500 mm
Weight:	30 kg

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