R-080538

ES	UNIDAD DE ASPIRACIÓN GEMINI MINI	5
EN	GEMINI ASPIRATION UNIT MINI.....	7
DE	GEMINI ABSAUGGERÄT MINI.....	9
PO	UNIDADE DE ASPIRAÇÃO GEMINI MINI.....	11

MESTRA®

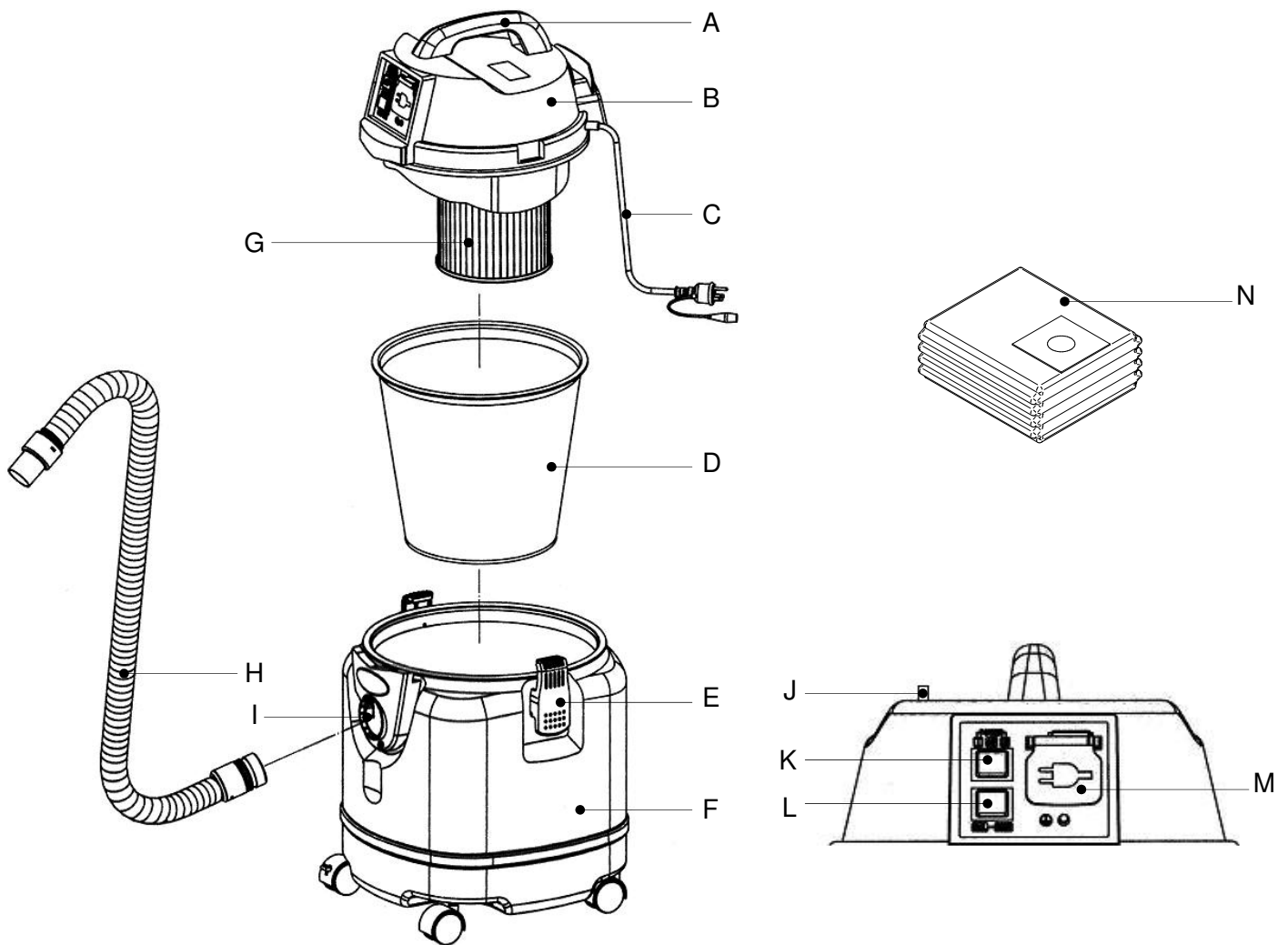
TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

- | | |
|---|--|
| A | Asa / Handle / Asa |
| B | Tapa / Lid / Tampa |
| C | Cable de alimentación / Power cord / Cabo de corrente |
| D | Bolsa filtro tela / Filter bag / Filtro bolsa |
| E | Cierre / Lock / Fecho |
| F | Tanque / Tank / Tanque |
| G | Filtro circular motor / Motor round filter / Filtro de papel circular |
| H | Manguera / Hose / Mangueira |
| I | Boca de aspiración / Aspiration nozzle / Boca de aspiração |
| J | Potenciómetro / Potentiometer / Potenciómetro |
| K | Interruptor On-Off / On-Off switch / Interruptor On-Off |
| L | Interruptor Automático-Manual / Auto-Man switch / Interruptor Automático-Manual |
| M | Toma de corriente / Power outlet / Toma corrente |
| N | Bolsa filtro papel / Paper filter bag / Saco de filtro de papel |

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Do not suck cigarette butt. Fire risk if they are not totally extinguished.

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This machine is only allowed to operate on smooth land.

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After each use, take the plug off to avoid leakage of electricity.

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If air flow was blocked when operating, stop operating and eliminate the blockage before start again.

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When disconnecting, hold the plug, do not pull the power cord.

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Use this machine according to the voltage rated in the label.

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It is prohibited aspirating chemical, foamy cleaners, sharp and point items.

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If cord is damaged call the technical service, do not repair by yourself.

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It is prohibited using near to fire sources.

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Before operating this machine, check if filter bag is installed in the right position or it will damage to motor.

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INSTALLATION

1. Carefully unpack your Gemini aspiration unit. You will find a flexible connecting hose inside the packing.
2. Choose a firm surface to place the unit, far from heat sources, damp or vibrations.
3. Open the lid (B) and make sure the filter-bag is correctly placed. Close the lid again.
4. Connect the unit's aspiration hose to the aspiration nozzle (I). Firmly turn it clockwise. The other end of the hose can be connected to a peg, sand blast, dry trimmer or any other similar device.
5. Connect the power cord to a grounded 230V, 50 Hz power source.
6. The unit is ready to be operated.

IMPORTANT

In case of liquid aspirating, remove first the paper filter (G).

OPERATION

1. Select the operational mode with the switch [TOOL/VACUUM].
 - If you select the mode *VACUUM*, aspiration motor will immediately start after pressing the switch [ON/OFF].
 - If you select the mode *TOOL*, the unit will work synchronously with a tool previously connected to the power outlet (M).

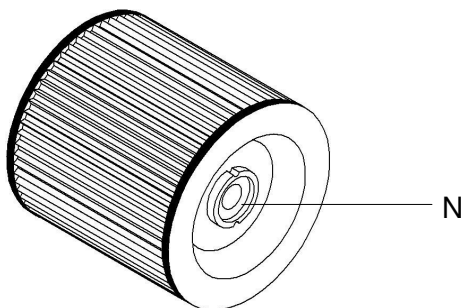
NOTE: Min. power need for the automatic detections is 100 W.
Max. power supported by the power outlet (M) is 750 W.

2. Press the switch [ON/OFF]. The unit will start if you selected *VACUUM*. It will keep in stand-by if you selected *TOOL*.
3. You can regulate the motor power (from 650 to 1200 W) by turning the knob (J).

UNIT EMPTYING - FILTER CLEANING

Periodically, check the tank filling level. Empty and clean it if necessary in the following way:

1. Disconnect the unit from the power supply.
2. Open the two lateral locks (E).
3. Pull the handle (A) upwards to remove the lid (B).
4. Remove the filter-bag (D).
5. Remove dust inside the tank (F).
6. Clean the round filter (G) or replace it with a new one and place again in its position. In order to remove the round filter, turn the knob (N) counterclockwise.

7. Place the two filters in the original position again. Do not forget locking the round filter by turning the knob (N) counterclockwise.
8. Place the upper lid again and close the two locks (E).

MAINTENANCE AND CLEANING

In order to achieve the machine's optimal performance and a long working life, we strongly recommend you:

- In case of liquid aspirating, remove first the paper filter (G).
- Clean the outer casing of the unit by always using a cloth. Do not use solvents or any other inflammable product.
- Periodically check the filling level of tank and filters.
- Replace the filters when damaged or if you can not clean them. Always use original spare parts.
- Make sure the aspiration hoses installed in your lab do not have losses due to cracks, breaks or faulty connections.

PRECAUTIONS

- Before connecting the machine, make sure that the power source is a grounded 230 V, 50 Hz.
- Do not connect micromotors or other devices with a power consumption of more than 750 W in the Schuko power outlet.
- Do not allow children or unskilled personnel to operate the machine.
- Choose a rigid and well-balanced surface to place the aspiration unit.
- Do not open the upper lid when the unit is in operation.
- Disconnect the machine from the power source during the cleaning and filter replacement operations.

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