## **R-080550 SUN BEAM**

<b>E.....</b>	<b>LÁMPARA POLIMERIZADORA.....</b>	<b>4</b>
<b>GB ...</b>	<b>POLYMERISATION LAMP .....</b>	<b>6</b>
<b>F.....</b>	<b>APPAREIL A PHOTOPOLYMERISER .....</b>	<b>8</b>
<b>D .....</b>	<b>LICHTHÄRTEGERÄT SUN BEAM .....</b>	<b>10</b>
<b>I.....</b>	<b>LAMPADA POLIMERIZZATRICE .....</b>	<b>12</b>
<b>P.....</b>	<b>LÂMPADA POLIMERIZADORA.....</b>	<b>14</b>

**MESTRA®**

**TALLERES MESTRAITUA S.L.**

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

**48150 SONDIKA - BILBAO - ESPAÑA**

1. Ventana / **Window** / *Fenêtre* / Fenster / **Finestra** / *Janela*
2. Puerta de Cámara Polimerización / **Polymerisation chamber door** / *Porte de la chambre à photopolymériser* / Polymerisationskammertür / **Portello Camera di Polimerizzazione** / *Porta da câmara de polimerização*
3. Cámara de Polimerización / **Polymerisation chamber** / *Chambre à photopolymériser* / Polymerisationskammer / **Camera di Polimerizzazione** / *Câmara de polimerização*
4. Interruptor General / **Main switch** / *Interrupteur général* / Hauptschalter / **Interruttore generale** / *Interruptor geral*
5. Display digital / **Digital display** / *Display digital* / Digitaldisplay / **Display digitale** / *Display digital*
6. Mando del Temporizador / **Timer control** / *Minuterie* / Bedienelement des Zeitschalters / **Comando del Temporizzatore** / *Comando do temporizador*

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The Sun Beam Polymerisation Lamp (R-080550) polymerises photopolymerisable materials swiftly and efficiently. It has a frequency range of 280 - 550 nanometers, and this allows it to polymerise the vast majority of materials used in the Dental Sector. We recommend you read the following Instructions for Use carefully for proper usage of the unit and best lamp performance:

### INSTALLING THE LAMP

- Carefully unpack your **Sun Beam** Polymerisation Lamp. Check that no damage has been sustained in transit, and read this “*User’s Manual*” in full.
- Choose a flat rigid surface for the facility, protected against sources of heat or vibrations. The unit should also be protected from direct sunlight.
- The lamps (6 x 9 W.) have been disassembled to prevent breakage during transit. Proceed with reassembly by inserting each lamp into its holder. You will observe that three lamps have been supplied with tone S/71, and a further three lamps with tone S/78. Each type of lamp must be alternated during assembly in order to obtain uniform lighting for the polymerisation chamber. Extreme care must be exercised during this operation to avoid any breakages.
- Connect the appliance to 230 V, 50/60 Hz, with earthing.
- Activate the main switch on the left-hand side of the unit. You will observe that the display shows the operating time set. The **Sun Beam** Lamp is now ready for use.

### OPERATION

- Uncover the polymerisation chamber, and place the items you wish to polymerise on the prosthesis plate. Up to 4 models may be polymerised, depending on the size of the items.
- Close the polymerisation chamber, and use the timer control to select the time required. The electronic display will show the time set in minutes. The polymerisation time range is 3 - 20 minutes.
- When the time has been selected, press the timer control knob. The facility will start up, the polymerisation chamber light comes on, and the prosthesis plate rotates.
- It is possible to cancel a polymerisation cycle during operation by pressing the timer control again. The machine then stops, and the display shows the last time selected.
- If the polymerisation chamber door opens during operation, the machine’s lights will go out and the prosthesis plate will cease to rotate for as long as the door is open. The timing function, however, continues to operate normally. This aspect is particularly useful when we wish to add or remove parts during a cycle which has already commenced.
- When the time set has elapsed, the electronic control unit will emit a warning signal, and the machine will stop. Remove the items from the polymerisation chamber.

### CHANGING THE LAMPS

The polymerisation lamps’ service life is limited. Optimum change is recommended every 500 hours, to a maximum of 1,000 hours of operation. The **Sun Beam** polymerisation lamp has an automatic lamp-change warning system: after 500 hours of operation, when the machine is hooked up to the mains **CL** (Change Lamps) will appear on the display. This will also appear as a reminder to the user every time a cycle commences. When the lamps have been changed, the hour meter must be reset. Proceed as follows:

- Disconnect the facility from the mains. Press the timer control knob and reconnect the facility to the mains, keeping the knob depressed.
- It will be observed that the appliance emits a signal for a few seconds, and “00” then appears on the display.

- Keep the timer control knob depressed until “00” disappears. When it has disappeared, the internal hour meter will have been reset to zero.

### MAINTENANCE AND CLEANING

- The polymerisation lamps’ service life is limited. Optimum change is recommended every 500 hours, to a maximum of 1,000 hours of operation (see point 3 in this Instructions Manual). Since the appliance uses two different types of lamps (S/71 and S/78), you must specify the type of lamp in requests for spare parts. Talleres Mestraitua, S.L. recommends that original **MESTRA** spare parts be used.
- The lamp types must be arranged alternately inside the polymerisation chamber.
- There is a fuse-holder on the left-hand side of the unit. The 2 amp fuse is to be changed as required.
- The unit should be cleaned on a regular basis, using a cloth soaked in soapy water. The unit should be disconnected from the power supply during this operation.
- The efficiency of the polymerisation lamp depends to a large extent on the clarity of the polymerisation chamber, and so it is advisable to carry out periodic cleaning.

### PRECAUTIONARY MEASURES

- ◆ Connect the unit to 230 V, 50/60 Hz, with an earthing facility.
- ◆ Do not submerge the unit in any liquids for cleaning purposes.
- ◆ Do not use the Polymerisation Lamp in locations which are particularly humid, or next to inflammable liquids (alcohol or solvents).
- ◆ The lamps must be changed with all due caution to prevent any breakage.
- ◆ Do not use the Polymerisation Lamp for any purpose other than polymerisation of Dental Materials.
- ◆ Do not allow children or unqualified staff to handle the unit.
- ◆ Operations must be carried out at all times with the polymerisation chamber door shut. Avoid all eye contact with the light emitted by the lamps, since ultra-violet radiation is being produced, and this can damage eyesight.

### TECHNICAL CHARACTERISTICS

Height:	315 mm
Width:	233 mm
Depth:	210 mm
Weight:	6.1 kg
Spectrum:	280/550 nm
Power supply:	230 V, 50/60 Hz
Illumination intensity:	8750 Lux

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