Ref. 080592
BOOMERANG PRECISION II

ES	DOSIFICADORA DE ESCAYOLA-AGUA.....	5
EN	PLASTER-WATER DISPENSER.....	10
FR	DOSEUR PLÂTRE-EAU	15

MESTRA®

TALLERES MESTRAITUA S.L.

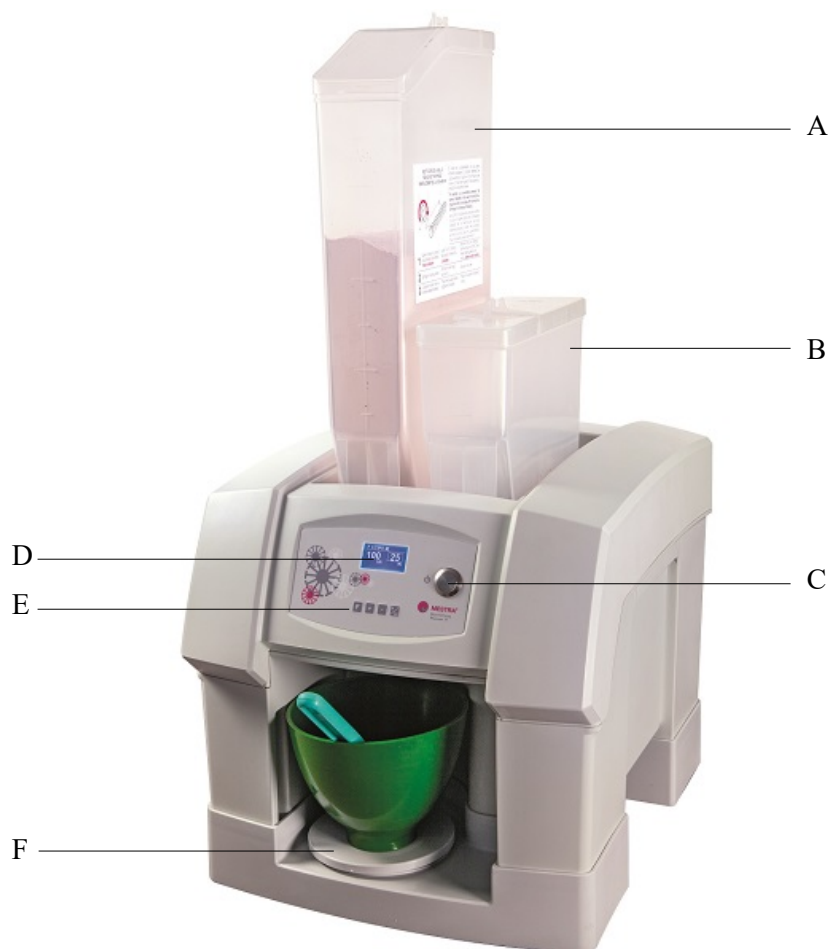
Txori-erri etorbidea, 60

48150 SONDIKA - BILBAO - ESPAÑA


Tel. (+34) 944530388 - Fax (+34) 944711725

mestra@mestra.es - www.mestra.es

Rev. 07/04/2021

- | | |
|---|---|
| A | Depósito grande para polvo / Big tank for powder / <i>Grand réservoir pour plâtre</i> |
| B | Depósito pequeño para líquido / Small tank for water / <i>Petit réservoir pour l'eau</i> |
| C | Botón start-stop / Start-stop key / <i>Bouton start-stop</i> |
| D | Display polvo / Powder display / <i>Display plâtre</i> |
| E | Display líquido / Water display / <i>Display eau</i> |
| F | Teclado / Keyboard / <i>Digits de fonction</i> |
| G | Balanza / Scale / <i>Balance</i> |

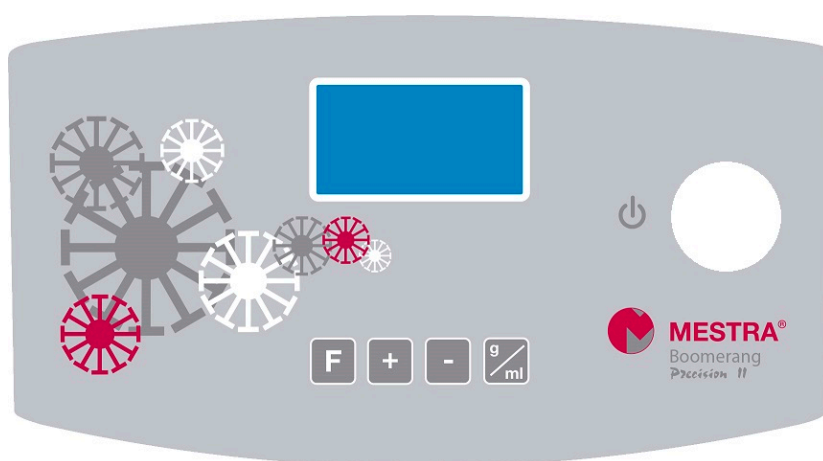F	Función Function <i>Function</i>
+	Subir Up <i>Monter</i>
-	Bajar Down <i>Descender</i>
g/ml	Cambio de display Display change <i>Changement de display</i>
	Marcha-paro Start-stop <i>Marche-arrêt</i>

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The Ref. 080592 dispenser has been specially designed for the field of dental prosthesis, and it measures the required amount of plaster and water thoroughly and automatically. Besides it allows you to switch easily from a type of plaster or other material to another type.

NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

MEANING OF THE SYMBOLS ON THE UNIT

Danger. Risk of injury.

Read completely these instructions before operating the unit.

Electric shock risk.

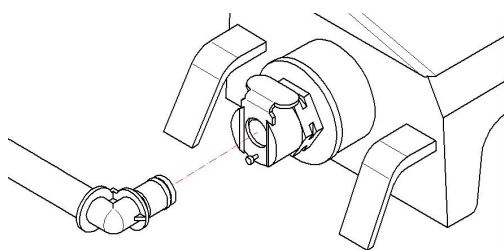
Fig. 1

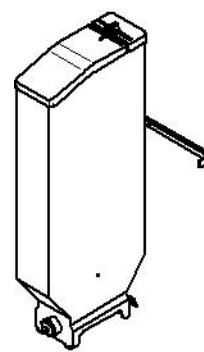
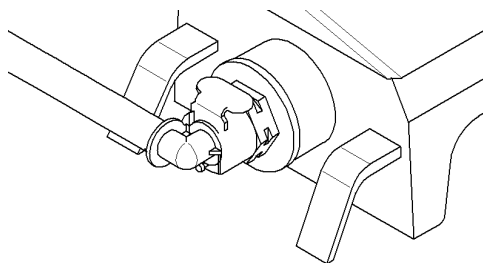
Fig. 2

INSTALLATION

Environmental conditions:

- Use indoors only.
- Environmental temperature: 5-40 °C [41-104 °F]
- Max. relative humidity: 0-95 %
- Degrees of protection: IP20

1. Carefully unpack the unit. Make sure it has not suffered any damage during its delivery. Together with the unit you will find: 2 plaster tanks, 1 water tank, 1 bowl, 1 spatula, 1 power cord, 1 supporting bar to hang the tanks on the wall.
2. Place the unit in a horizontal, flat and firm surface, away from heat or vibration sources.
3. Place the device so that it is easy to use the disconnecting device (main switch).
4. Mount the two tanks in place on the casing. To do this press the flexible pins against the stop and insert the forward rims into the holes on the casing.
5. **The engine pinion and the spindle in the powder tank must fit together. If the tank does not fit easily, insert your forefinger in the spindle opening and turn the spindle slightly.**

Do not forget to **connect the water pipe** by pressing it until you hear a “click” (Fig. 1).

6. You can screw the supporting bar to the wall to hang plaster tanks, and have them at hand filled with different types of plaster (Fig. 2).
7. Connect the unit to a grounded 230 V, 50/60 Hz power source.

USE

1. DISPLAY

Home screen:

Program number

Program name

Plaster grames

Water mililiters

2. BLEEDING THE LIQUID PUMP

When using the unit for the first time or when connecting the liquid tank, the liquid circuit usually takes some air and you need to bleed it.

- Mount only the small tank filled with liquid and make sure you have connected the water pipe by pressing until you hear the “click” (Fig. 1)
- Place a bowl to collect the liquid.
- Keep pressing the key **g/ml** for some seconds. The pump will start running. Keep pressing to make sure the pump is properly purged and the air comes out without bubbles.

3. DOSAGE

- Place a bowl to collect the mixture.
- Press the **START/STOP**  key once for the unit to dispense the dose selected.

4. USER SETTINGS

- From the home screen, keep pressing the key **F**.

CUSTOMER .CONFIG

1.0 User
2.0 Calibration

- Press again shortly the key **F** to enter option *1.0 User*.

2.0 User

TADVERT = 2 "
LANGUAGE = English

- Use the keys **+** and **–** to select the dosing end advising time.
- Press again shortly the key **F** and use the keys **+** and **–** to select the language.
- Press **N** to save.

5. SCALES CALIBRATION

Note: Follow this procedure only if necessary (the scale is weighing incorrectly).

- From the home screen, keep pressing the key **F**.

CONFIG.CLIENTE

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- Press again shortly the key **F** to enter option *2.0 Calibration*. Then, press the key **F**.

2.0 Calibration

2.1 Scales
2.2 Water pump

- Use the keys **+** and **–** to select option *2.1 Scales*. Then, press the key **F**.

2.1 Scales

CALIBR.0gr = NO
CALIBR.100gr= NO
RESULT = 1.2

- Remove any object from the scale.
- Use the keys **+** and **–** to select the option **CALIBR.0gr = YES**.
- Then, press the key **F**.
- Place an exact weight of 100 grams on the scale.
- Then press the **F** key.
- The electronic control will automatically calibrate the scale.

6. WATER PUMP CALLIBRATION

Note: Follow this procedure only if necessary (the pump is dosing the amount of water very wrongly).

- From the home screen, keep pressing the key **F**.

CUSTOMER .CONFIG

1.0 User
2.0 Calibration

- Press again shortly the key **F** to enter option 2.0 *Calibration*. Then, press the key **F**.

2.0 Calibration

2.1 Scales
2.2 Water pump

- Use the keys **+** and **-** to select option 2.2 *Water pump*. Then, press the key **F**.

2.2 Water pump

K.PUMP= 4.11 ml/s
POUR100ml= NO
RESULT = 100.0
0.0

- Place a bowl to collect the liquid.
- Use the key **g/ml** and the keys **+** and **-** to select option
POUR100ml= **YES**
- Then, press the key **F**.
- The pump will try to pour 100 ml of water.
- Next, the machine will weigh the amount of water actually poured and will automatically correct the pump constant.
- If the value obtained is very different from the expected one, a warning message will appear so that the operation is repeated or the constant is modified manually.

7. PLASTER TYPE SELLECTION

Use the keys **+** and **-** to select the plaster type:

P1 Free
P2 Type II
P3 Type III
P4 Type IV

8. CHANGING THE PLASTER/WATER PROPORTION

- Once a type of plaster is selected, to change the required proportion of water press the key **F**.
- Press the key **F** to select different parameters:

P 3 TYPE III

POWDER = 100gr — Plaster quantity
WATER = 25 ml — Water quantity
100/ 25 — Water %

- Press **START/STOP**  to save.
- If you have selected a value for water %, thereafter, only the amount of dust can be modified directly, and the water will automatically vary proportionally. This fact will be indicated with a rectangle around the amount of water on the initial screen:

P 3 TYPE III

100 GR 25 ML — Fixed proportion indicator

- If you want to be able to independently modify the plaster and water quantities, select the value “- - -” as % of water:

P 1 FREE

POWDER = 100gr —
WATER = 25 ml —
100/ - - - — Water % (free)

The Boomerang Precision dispenser is factory set according to the following proportions of plaster/water:

P1	Free	- - -
P2	Type II	100 g / 35 ml
P3	Type III	100 g / 25 ml
P4	Type IV	100 g / 20 ml

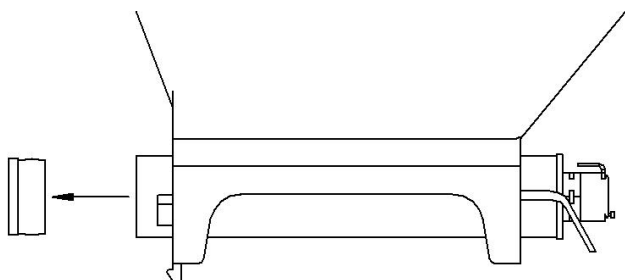
For greater accuracy follow the plaster manufacturer's instructions.

9. CHANGING THE DOSAGE

- From the home screen, press the key **g/ml** to select the plaster quantity.
- Use the keys **+** and **-** to modify the plaster quantity.
- Shortly press again the key **g/ml** to modify the water quantity (only in case you have set the proportion free).
- Press **START/STOP**  to save.

REPLACING THE FILTER

The water tank has a filter to prevent particles entering the tank. If it needs to be replaced, proceed as follows:


1. Remove the cap by pushing it with your finger from inside the tank.

2. Unscrew the filter with a 13 mm tube spanner.

REPLACING THE SPINDLE

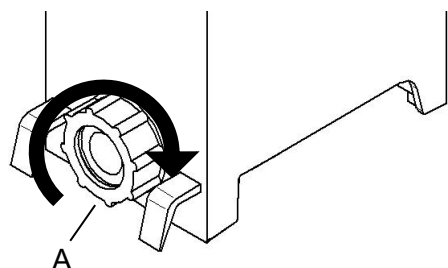The spindle is a perishable element. Its duration depends on the use of the machine. A spare spindle is included with the machine. To replace it, proceed as following:

1. Loosen nut (A) rotating in the direction of the arrow.
2. Remove the damaged spindle (B).
3. Place the new spindle and tighten the nut again.

ERROR MESSAGES

“RECIPIENTE”

There is no bowl on the plate.

“POWDER DEFECT”

There is no plaster in the tank. The spindle is damaged.

“DEFECTO AGUA”

There is no water in the tank. The pump is very uncalibrated.

MAINTENANCE AND CLEANING

To ensure an ideal reliability and a long working life of the unit, we strongly recommend:

- Keep the unit and the surroundings clean.
- Be careful when filling the powder tank; do not let particles enter the water tank, since the pump could be damaged.
- Keep the powder and water nozzles clean. It will prevent dirt gathering and the spindle from seizing up.
- Clean the unit thoroughly when changing the type of powder, to prevent the pollution from the dirt that can gather in the unit.
- The outer casing of the unit should be always cleaned with a damp cloth. Do not use solvents.
- Watch the filling level of the tanks. Do not let the unit work when the tanks are empty.
- Check the filter and the tanks periodically, as well as the possible wear of the interior pieces and the watertightness of the water tank.
- Use only original spare parts supplied by the manufacturer.

PRECAUTIONS

- Before connecting the unit to the power source, make sure the voltage is correct (grounded 230 V, 50/60 Hz).
- The cord may only be replaced by one with the same characteristics as supplied by the manufacturer.
- Read these instructions carefully before operating the unit. They contain very important information about safety during the installation, operation and maintenance of the unit.
- Keep these instructions for further consultation.
- Check out the state of the unit upon unpacking it.
- Before connecting the device, make sure the voltage in the identification plate matches the power source to be used. During the installation all the safety requirements must be observed and the instructions of the manufacturer followed.
- The manufacturer will not be liable for the damage caused by an inappropriate grounding system.
- Damage caused by a faulty installation or a wrong use of the unit is not covered by the warranty.
- When in doubt, do not start the unit. Contact the supplier for questions or detailed information about problems with the unit.
- Do not touch the unit or the power cord with wet hands.

- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.
- Do not allow children or non-qualified personnel handling the unit.
- Do not dismount, repair or modify the unit.
- Do not ever place or remove the tanks while the unit is in operation.
- Contact the supplier for questions or detailed information about problems with the unit.

SPECIFICATIONS

Height:	650 mm
Width:	350 mm
Depth:	430 mm
Weight:	7 kg
Big tank capacity:	5 L
Small tank capacity:	2 L
Voltage:	230 V, 50/60 Hz
Power:	70 W
Fuse:	Glass 2 A, 110-240 V
Sound power:	<60 dBA

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