
Ref. 080610

ES.....	PERFORADORA DE PINES.....	4
EN.....	PIN DRILL.....	6
FR.....	PERFOREUSE DE PINS.....	8
DE.....	PINBOHRGERÄT.....	10
PO.....	PERFURADORA DE PINS.....	12
IT.....	PERFORATRICE DI PIN.....	14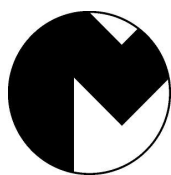

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

48150 SONDIKA - BILBAO - ESPAÑA

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

Ref. 080610

- | | |
|---|---|
| A | Láser de posicionamiento / Positioning laser / <i>Laser de positionnement</i> / Positionierungslaser / Laser de posicionamento / <i>Laser di posizionamento</i> |
| B | Mesa porta modelos / Model holder table / <i>Table porte-modèles</i> / Modellaufnahmetisch / Mesa porta modelos / <i>Tavola porta modelli</i> |
| C | Interruptor general / Main switch / <i>Interrupteur principal</i> / Hauptschalter / Interruptor geral / <i>Interruttore generale</i> |

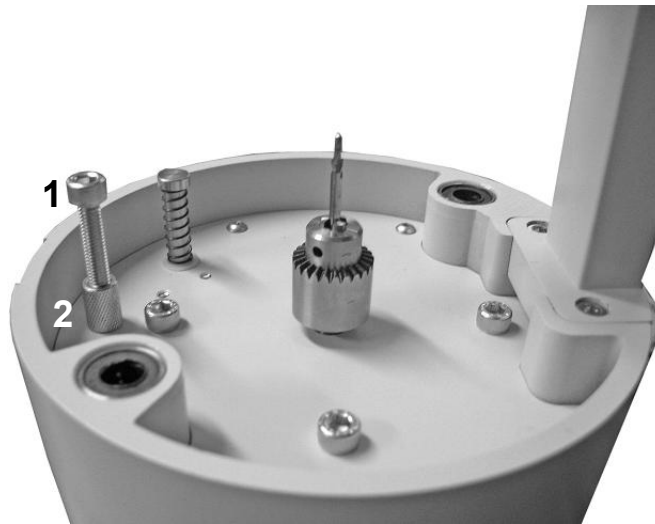
Fig. 1 / Abb. 1

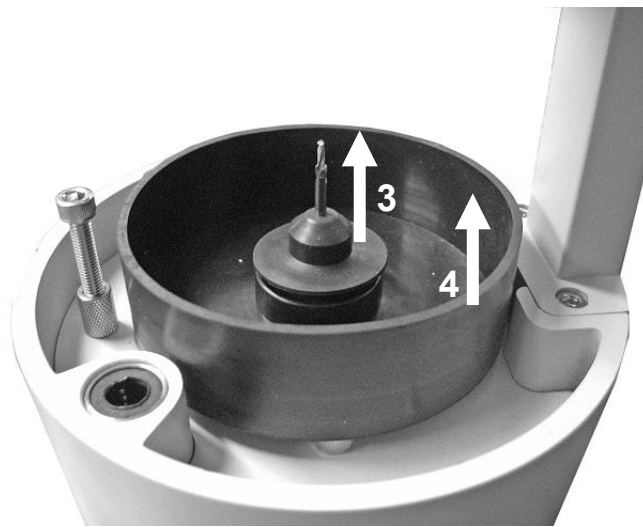
Fig. 2 / Abb. 2

Fig. 3 / Abb. 3

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- Carefully read this manual. It contains important information on safety during the installation, operation and maintenance of the unit.
- Keep this manual in a suitable place to consult it in the future.
- Check the overall state of the unit just upon unpacking it.
- Before connecting the unit, make sure the voltage shown in the identification plate matches that of the power source to be used. The installation must fulfil all the applicable safety standards and the instructions of the manufacturer.
- The manufacturer will not be responsible of the damage caused by an incorrect ground connection. The damage caused by a faulty installation or usage of the unit will not be covered by the warranty.
- In case of doubt, do not start the unit. Contact a qualified and authorised technician

- **Do not put your hand over the drilling hole.**
- **Disconnect the machine before handling the bit or its protections.**
- Check out the bit and the chuck before starting the drilling.
- If during the normal operation of the machine the engine shaft gets locked, disconnect the machine from the power source before attempting to unlock the shaft by forcing it to turn manually.

- Always wear protection goggles, gloves and mask while operating the machine.
- Do not drill materials that can give off particles toxic by inhalation.

- The machine has an in-built laser. Never look straight at the ray.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Remove any dirt or water remains from the cord connection.
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not insert plastic objects into the bowls or any other material that melts at low temperatures.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents,...).
- Be extremely cautious when drilling materials that can shoot out sparks.

- Do not allow children or non-qualified personnel handling the unit.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.

- Contact the supplier for questions or detailed information about problems with the unit.

The Ref. 080610 is a machine specially designed for the technique of individualizing pins. Easy to handle the machine allows to drill precision holes in the lower part of the model. Changing and adjusting the bit is an easy and intuitive operation. The machine works silently and without vibration. The laser in the upper part of the machine allows to locate exactly the place to drill.

INSTALLATION

For a correct operation of the machine, we strongly recommend to follow these directions:

- Unpack the pin drill carefully. Check it has not suffered any damage during its transport.
- The machine is factory supplied with the following accessories:
 - Chuck wrench
 - Ø 1.95 mm bit
- Connect the unit to a grounded 230 V, 50/60 Hz power source.

OPERATION

- Turn the main switch (C) on.
- Put the model over the model holder.
- With the help of the laser gauge, locate the model over the bit exactly in the place you want to drill.
- To start drilling press the model and the table downwards. The engine will start up automatically.
- While operating the drill, used a suitable protection:
 - Wear safety goggles.
 - Wear safety gloves.
 - Wear a mask.
- Once the work has been finished, turn the main switch (C) off.
- Periodically aspirate the dust accumulated in the rubber tray placed under the model holder table.

ADJUSTMENT OF THE DRILLING DEPTH

To adjust the drilling depth, remove the bolt (1) that acts as a stop and fix its position with the nut (2). (Fig. 1)

DISMOUNTING THE BIT

Always disconnect the unit before handling the bit or its protections.

The unit is factory supplied with a Ø 1.85 mm cutting bit. If you want to change it, proceed as follows:

1. Remove the model holder table (B).
2. Remove the rubber piece (3) that covers the chuck (Fig. 2).
3. Carefully remove the rubber dust tray (4) (Fig. 2) to prevent damaging the bit.
4. Loosen the chuck with the chuck wrench (5) and remove the bit (Fig. 3).
5. To mount the bit again, proceed as above, but in reverse order. Handle the rubber protection with care and make sure it is correctly placed.

MAINTENANCE AND CLEANING

To ensure an correct and reliable operation and a long life of the unit, we strongly recommend:

- To clean the dust and plaster that may have remained in the rubber tray, the surface where the drill has been installed or the body of the drill.
- To clean the unit with a cloth. Never use alcohol, solvents or any other flammable product.
- Do not ever immerse the drill in any liquid.
- If you notice unusual noises or vibrations, have the unit sent to the technical service.
- When changing the bit, make sure the drill works correctly, without unusual vibrations.
- If the machine fails to work, first of all check the fuse (6 A).

TECHNICAL SPECIFICATIONS

Voltage:	230 V, 50/60 Hz
Power:	100 W
Speed:	6000 rpm
Length:	165 mm
Width:	165 mm
Height:	300 mm
Weight:	5 kg

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