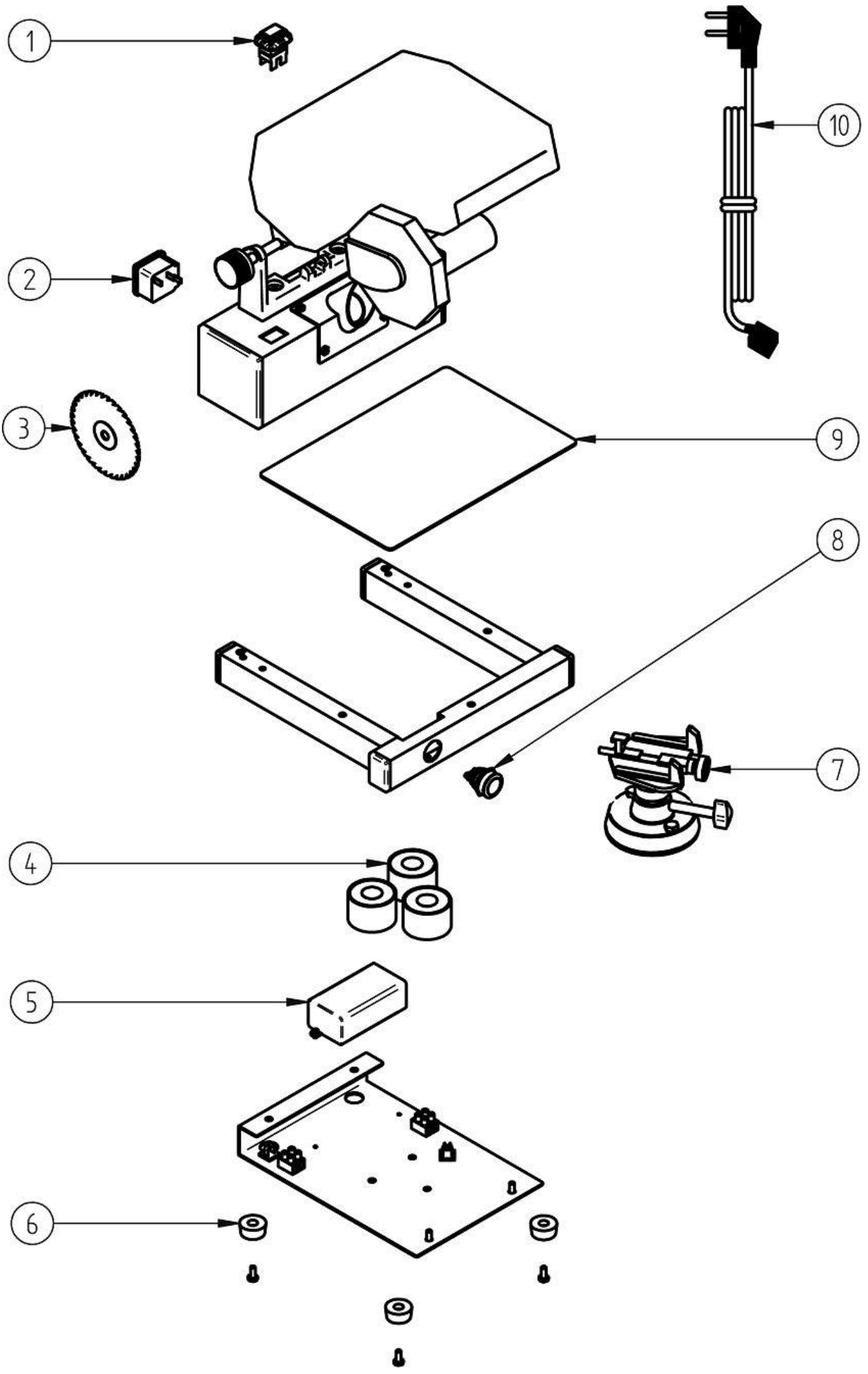
Ref. 080614

ES.....	5	Manual original
EN.....	7	
FR.....	9	

Nº	Ref.	ES	EN	FR
1	080422-17	Interruptor	Switch	Interrupteur
2	080490-06	Base de alimentación	Connection base	Base d'alimentation
3	080614-24	Sierra	Saw	Scie
4	080614-09	Electroimán	Electro magnet	Electro-aimant
5	080614-20	Fuente de alimentación	Power source	Source d'alimentation
6	080490-31	Pata	Leg	Patte
7	080614-22	Conjunto mordaza	Clamp assembly	Support modèle
8	080515-04	Pulsador de seguridad	Safe button	Bouton poussoir sécurité
9	080614-05	Mesa	Table	Table
10	000250	Cable	Cord	Câble

- | | |
|---|---|
| A | Cabezal / Head / <i>Tête</i> |
| B | Interruptor general / Main switch / <i>Interrupteur général</i> |
| C | Mordaza / Clamp / <i>Support modèle</i> |
| D | Pulsador imanes / Magnet button / <i>Poussoir aimant</i> |
| E | Palanca bloqueo cabezal / Head lock / <i>Clapet de blocuage de la tête</i> |
| F | Pulsador sierra / Saw button / <i>Poussoir scie</i> |
| G | Sierra / Saw / <i>Sierra</i> |
| H | Agujero aspiración / Suction hole / <i>Trou aspiration</i> |
| I | Mesa / Table / <i>Table</i> |

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Ignacio Mestraitua
Director Gerente
Sondika, 12.03.2020

NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

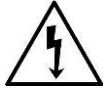
Meaning of the symbols on the unit

Danger. Risk of injury.

Read completely these instructions before operating the unit.

Electric shock risk.

Use safety goggles.

Wear safety gloves.

Wear a mask.

Precautions

- Keep this manual in a suitable place to consult it in the future.
- Check the overall state of the unit just upon unpacking it.
- The damage caused by a faulty installation or usage of the unit will not be covered by the warranty.
- the risk of electric shock, this equipment must only be connected to a properly grounded power source. Before connecting the appliance, check that the identification label indicates the correct voltage. The installation must comply with all applicable safety standards.
- Remove any dirt or water remains from the cord connection.
- Do not pull, twist or fold the power cord excessively.
- Do not touch the unit or the power cord with wet hands.
- Place the unit in a dry place, away from dust, water splashes and flammable materials (alcohol, solvents, etc.).
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not allow children or non-qualified personnel handling the unit.
- Do not use the unit if any doubts. Contact the supplier for questions or detailed information about problems with the unit.
- Do not dismount, repair or modify the unit.
- Unplug the unit in case of storm and when you are not going to use it for a long time.

- Unplug the unit before cleaning, do not use chemical products.
- Contact the supplier for questions or detailed information about problems with the unit.
- Before connecting the device, make sure it is placed on a firm and solid surface.
- The cord may only be replaced by one with the same characteristics as supplied by the manufacturer.
- During operation of the unit, use sufficient protection (gauntlets, safety goggles and a face-mask).
- Wear proper working cloth. Loose cloth, hanging or long hair could be caught by the machine.
- Always use a suction system.
- Do not bridge the safe switches.
- If the motor shaft locks during normal operation, disconnect the unit from the mains before attempting to release it by turning it manually.
- Never polish, brush or grind materials which may give off particles which are toxic if inhaled.
- Disconnect the unit from power supply before changing the saw.
- When you replace the saw, make sure that the protector is clean and in good conditions.
- Do not saw materials different from plaster models, specially metal objects.

Intended use

The R-080614 is a simple model saw designed be easily used in the field of dental mechanics.

Unit parts (page 2)

- A Head
- B Main switch
- C Clamp
- D Magnet button
- E Head lock
- F Saw button
- G Saw
- H Table

Installation

Environmental conditions:

- Use indoors only.
 - Environmental temperature: 5-40 °C [41-104 °F]
 - Max. relative humidity: 0-95 %
 - Degrees of protection: IP20
1. Carefully unpack the model trimmer. Make sure the unit has not suffered any damage during the transport.

2. Place the unit on a flat and firm surface, away from any heat or vibration source.
3. Connect the suction hole (H) to an aspiration unit, in order to remove the particles produced by sawing.
4. The unit has four rubber pads. When designing the unit, special care has been taken to give the unit enough weight and robustness to prevent any movement during its operation. No additional fixing element is needed.
5. Connect the unit to a proper and grounded power source.

Operation of the unit

1. Take proper safety measures:
 - Wear a mask, safety gloves and goggles.
 - Do not wear baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine. Be careful with long hair
 - Do not use the unit with long items that could be hooked.
2. Turn on the main switch (B) placed in the upper part of the unit (green). The operation pilot will light up.
3. Turn the aspiration on.
4. Depending on the model characteristics, fix it with the clamp and place on the table surface. With your left hand, press button (D) and electromagnets will lock the position.
5. Take the head handle with your right hand. To unlock the head movement, pull lever (E).
6. To start the motor, press button (F).

BECAUSE OF SAFENESS, BOTH BUTTONS MUST BE PRESSED SIMULTANEOUSLY TO START THE MOTOR.

7. Work finished, turn main switch (B) off.
8. After operating the trimmer, it is strongly recommended to clean the table and the front casing with a damp cloth to remove the remains of plaster.

Saw replacing

When the saw is damaged, you must replace it. Always use Mestra spare parts:

1. Turn the unit off.
2. Remove the saw protector with the Allen spanner (included).
3. Use a plane spanner at the right side of the saw to avoid it turn.
4. Release the saw central bolt with the Allen spanner (left side).
5. Remove the old saw and place a new one. Make sure it is in the correct position.

6. Place the protector again and make sure the assembly is correctly closed.

Maintenance and cleaning

To ensure a correct, reliable and durable performance of the unit we strongly recommend:

- After using the unit, clean the dust and plaster remains from the table and the cover.
- Do not use alcohol or any other solvent to clean the unit.
- Never immerse the unit in a liquid.
- If you notice unusual noises or vibrations, send the unit to a technical assistance service.
- Replace the saw whenever you notice a significant fall in performance.
- After replacing the saw, check that there are not unusual vibrations.
- The unit has been designed to operate correctly leaning on the four rubber blocks in its base. These elements are strong enough to bear the movement of the trimming.
- Use only original Mestra spare parts.

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Meets the requirements set out in the following directives:

2014/35/UE (low tension)
 2014/30/UE (electro magnetic compatibility)
 2006/42/CE (machinery safety)

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