
Ref. 100035

ES.....	LÁMPARA POLIMERIZADORA	2
EN.....	CURING LIGHT	3
FR.....	LAMPE DE POLYMÉRISATION	4
PO.....	LÂMPADA POLIMERIZADORA	5
IT.....	LAMPADA POLIMERIZZATRICE	6
DE.....	LICHTHÄRTEGERÄT FÜR LÖFFELMATERIAL	7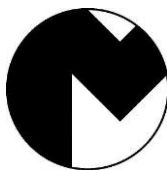

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

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The R-100035 Curing Light is used for fast effective curing of photo-polymerisable materials commonly used in the dental field. Its frequency range is 350 - 400 nanometers. Please read thoroughly the following instructions for correct use of this machine to obtain maximum output.

INSTALLATION

- Carefully unpack the Curing Light. Check that it has not been damaged in transit.
- Place the unit on a flat and firm surface well away from heat sources or vibrations.
- In order to avoid damage during transport, the lamps (4 x 9 W) are supplied disconnected. To install them, remove the tray and then place each lamp in its holder. Do this carefully to avoid possible breakage.
- Connect the machine to a 230 V, 50/60 Hz power supply with earth.
- Activate the main switch at the back of the machine. The machine is ready for operation.

HOW TO USE THE CURING LIGHT

- Remove the tray and place on it the parts to be cured (4 pieces maximum).
- Place the tray back on the machine and select the time with the timer. Curing time goes from 0 to 15 minutes.
- The lamps inside will light up. When the time selected ends, the timer will emit an acoustic signal.
- Lamps emit UV radiation which may be harmful to eyesight. If you have to open the tray while the machine is working, disconnect it first.
- When the cycle finishes, remove the cured parts.

MAINTENANCE AND CLEANING

- Curing Lights have a limited life span. We recommend changing the lamps between 500 (ideal) to 1,000 (maximum) working hours.
- There is a fuse holder on the power cable base. Change the fuse (2 A) when required.
- The machine should be cleaned periodically with a damp soapy cloth. Before cleaning the machine, disconnect from the power supply.

PRECAUTIONS

- ◆ Before connecting the machine, make sure it has 230 V, 50/60 Hz with earth.
- ◆ When cleaning the machine, do not immerse it in any kind of liquid.
- ◆ Do not use the Curing Light when it is too damp or next to any inflammable product (alcohol or solvents).
- ◆ Change the lamps carefully to avoid breakages.
- ◆ The Curing Light should only be used for curing Dental Materials.
- ◆ Do not allow children or unqualified individuals to handle the machine.
- ◆ Make sure that the tray is always closed during operation, and avoid eye contact with the light emitted, since UV radiation may be harmful to eyesight.

TECHNICAL DATA

Height:	115 mm
Width:	215 mm
Depth:	295 mm
Weight:	2 kg
Emission Range:	350/400 nm
Power supply:	230 V, 50/60 Hz
Power level:	4 x 9 W
Illumination intensity:	8250 Lux

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