
R-100096

E.....	LUPA DE AUMENTO BINOCULAR.....	4
GB	BINOCULAR MAGNIFYING GLASSES.....	6
F.....	LOUPE BINOCULAIRE	8
D.....	STEREO-LUPENBRILLE.....	10
P.....	LENTE DE AUMENTO BINOCULAR.....	12
I.....	LENTE DI INGRANDIMENTO BINOCULARE.....	14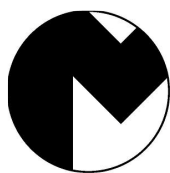

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

Rev. 09/05/16

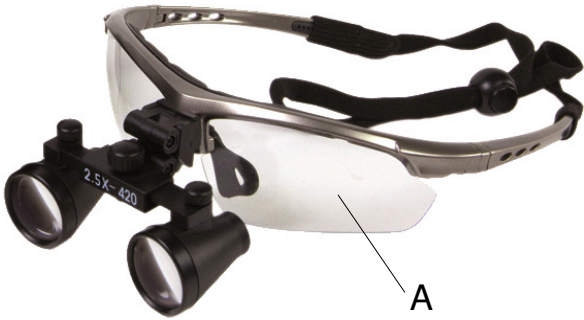
Fig. 1
Abb. 1

Fig. 2
Abb. 2

Fig. 3
Abb. 3

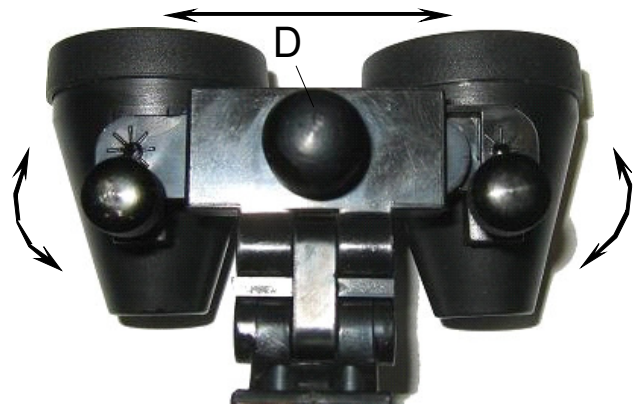
Fig. 4
Abb. 4

Fig. 5
Abb. 5

- A Cristales / **Glasses** / *Verres* / Gläser / **Vidros** / *Lenti occhiali*
- B Lentes / **Lenses** / *Verres* / Okulare / **Lentes** / *Lenti*
- C Protectores lentes / **Lens protectors** / *Protections verres* /
Okularschutzgläser / **Protectores lentes** / *Protettori lenti*
- D Tornillo distancia interpupilar / **Interpupillary distance adjustment screw** / *Vis* / Stellschraube Pupillendistanz / **Parafuso distancia interpupilar** / *Vite distanza interpupillare*
- E Articulación / **Joint** / *Articulation* / Gelenk / **Articulação** / *Snodo*

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Indispensable in the dental clinic and dental lab. In the clinic for exploration, diagnosis, minor surgery, etc. In the lab for precision works requiring greater visual sharpness.

Their main features are:

- High brightness optics, achieving outstanding clarity and sharpness.
- 2,5x magnification. Ideal for dental works.
- Working distance of 420 mm, together with an ideal depth of field.
- Side shields for protection against splashes and projections.
- Steel framed supporting goggles. Their glasses can be replaced for corrective glasses according to the user's needs.
- Comfortable and ergonomic nose-bridge.
- Interpupillary distance adjustment screw.
- Adjustable neck strap.
- Supplied with a practical black vinyl case and a screwdriver.

USE (see figures in previous pages)

1. If needed, the goggle's glasses (A) can be replaced for corrective glasses. Consult your optician.
2. The lenses (B) should never be touched.
3. The ideal working distance is 420 mm (maximum sharpness) (Fig. 3).
4. Adjust the interpupillary distance by turning the central screw (D) until achieving a single vision.
5. The alignment of the lenses can be adjusted when needed.
6. The lenses can be easily folded upwards when they are not needed for the work being performed (Fig. 5).
7. If you need to tighten or loosen the joint (E), use the screwdriver supplied.

TECHNICAL SPECIFICATIONS

Dimensions of the case:	250 x 135 x 75 mm
Total weight of the case:	310 g
Weight of the glasses:	100 g
Focus distance:	420 mm
Magnification:	2.5 X

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