R-100123

ES.... CALENTADOR DE CERA POR INMERSIÓN (3 RECIPIENTES).....	4
EN.... IMMERSION WAX HEATER (3 TANKS).....	6
PO ... RECHAUFFEUR DE CIRE PAR INMERSION (3 RECIPIENTS).....	8
IT..... RISCALDATORE DI CERA AD IMMERSIONE	10

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

48150 SONDIKA - BILBAO - ESPAÑA

Tfno. + 34 944530388 - Fax + 34 944711725

E-mail: mestra@mestra.es - www.mestra.es

1. Tapa / **Lid**
2. Selector de recipiente / **Tank selector**
3. Teclas subir-bajar / **Up-down keys**
4. Selector °C-°F / **°C-°F selector**
5. Recipientes / **Tanks**

ES

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WARNINGS – PREVENTIVE MEASURES

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Do not touch the wax tank during operation; it is very hot.
- To prevent burns, be cautious when handling the heater even when it is empty.
- Always keep the lid closed when you don't use the unit.
- After turning the main switch off, wait for 20 minutes before touching the bowls.
- Do not spill wax while removing it from the heater. The wax is very hot and it could cause small burns.

- The machine features a precision circuit. Prevent fortuitous impacts and stresses that could cause mechanical damage.

- Do not obstruct the ventilation grilles at the bottom of the unit.
- Remove any dirt or water remains from the cord connection.
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not insert plastic objects into the bowls or any other material that melts at low temperatures.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents,...).

- Do not allow children or non-qualified personnel handling the unit.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.

- Contact the supplier for questions or detailed information about problems with the unit.

INTRODUCTION

R-100123 is a practical wax heater specially designed for the professionals in the dental prosthesis sector. It can melt a wide range of waxes very quickly and to keep them in a liquid state without altering their properties. The selection of the set-point temperature is easy and quick.

- Read this manual carefully. It contains very important information about a safe installation, operation and maintenance of the machine.
- Store this manual in a suitable place for further consultation.
- Check the general state of the machine after removing the packing.
- Before connecting the unit, make sure the voltage in the identification plate matches the power source to be used. During the installation all the safety requirements must be observed and the instructions of the manufacturer followed.
- The manufacturer will not be liable for the damage caused by an inappropriate grounding system.
- Damage caused by incorrect installation or operation of the machine will not be covered by the warranty.
- If in doubt, do not start the machine. Contact a qualified and authorised technician.

INSTALLATION

- Choose a flat, horizontal and firm surface to place the unit, away from heat and vibration sources, and from inflammable materials.
- Connect the unit to a 230 V, 50/60 Hz power source.

OPERATION

- Place the wax particles you want to melt in the tank (4).
- Close the lid of the unit (1) and turn the main switch (10) on (green light).
- Press the [°C / °F] key to select Celsius or Fahrenheit degrees (the display will show °C / °F).

- Press the [S/C] key to programme the temperature. The pilot (S) will light up. Select the temperature by pressing the [▼▲] keys.
- Press the [S/C] key to see the actual temperature in the tank. The pilot (C) will light up.
- After about 10 minutes, the wax will be ready to be used. The pilot light (H) will turn off once the key-point temperature has been reached.
- Keep the lid (1) closed during the periods in which you do not need to remove any wax. You will prevent spilling the wax. Outside particles and impurities will not enter and contaminate the wax.
- After finishing the operation disconnect the heater by turning the main switch (10) off. Handle the heater cautiously for several minutes until it has cooled down completely.

MAINTENANCE AND CLEANING

In order to ensure a long working life and an optimal operation we strongly recommend:

- To clean the wax remains in the tank, lid and cover periodically. To do it, you can use a spatula. Make sure the wax heater has been previously disconnected from the power source.
- To prevent spilling melted wax over the cover of the machine. Be cautious when handling the spatula.

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