R-100144

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R-100144 is a practical Ultrasonic Bath specially designed for the dental sector. For a long working life and optimum performance, we recommend you to read carefully the following instructions:

INSTALLATION

- Carefully unpack the Ultrasonic Bath. Inside you will find the following accessories:
 - 1 plastic basket (E).
 - 1 detachable tank (G).
- Choose a flat and firm surface far from heat or vibration sources.
- Connect the machine to a grounded 230 V, 50/60 Hz power supply.
- Place the basket (E) inside the tank.
- Fill the tank (F) with water or a cleaning solution.
- The machine is ready for operation.

OPERATION

1. Put the detachable tank and move the lock button up to close it. If the detachable tank is not in the right position, the safety button will not close easily and the unit will not work.


2. Put the item (denture) in the basket to protect the tank from frictions.


3. Fill the stainless steel tank with water. Make sure that the water level does not exceed the *max.* mark when immersing the item. The unit will be damaged if operated without water.

4. Plug the machine and close the lid.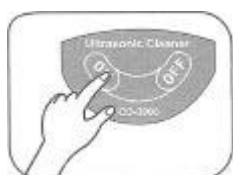


5. Press the "ON" key to start cleaning. The blue LED will turn on and the unit will produce a "buzz". The cleaning will finish automatically after 5 minutes. If more time is needed, just press the "ON" key again.

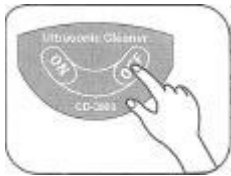6. If required, you can stop the cleaning process by pressing the “OFF” key. Meanwhile the LED will turn off.

7. When the cleaning is finished, unplug the unit and open the lid to remove the basket and the item.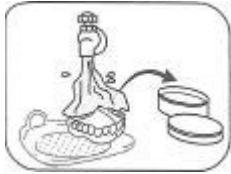

8. Wash the item and the basket with tap water. If a denture disinfectant is required, please use it following your dentist’s advice.


9. Open the lock, remove the tank and clean it with water. Then dry it with a clean cloth.


10. Finally, put the detachable tank back and move up the lock button. Dry it up for the next operation.


DIRT REMOVING PROCESS VIA ULTRASONIC TECHNOLOGY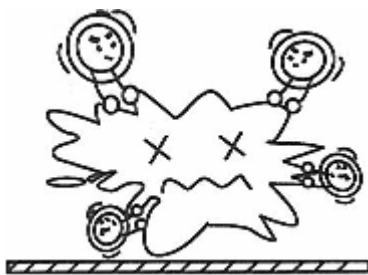

The ultrasounds generate myriads of small bubbles.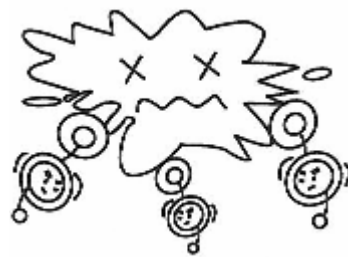

The ultrasounds make the bubbles implode constantly and impact against the object's surface

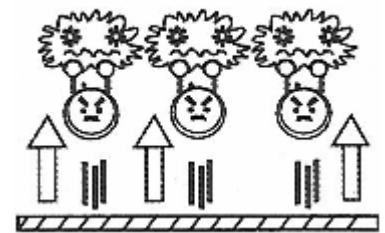The impact breaks up the dirt into many grains and separates them from the object's surface. The grains remain in the water.

CLEANING SOLUTIONS

Cleaning solutions can be classified into two types: those consisting of a cleaning solution mixed with water, and solvents.

The most common cleaning method is with water, as it has an acceptable capacity to dissolve inorganic substances. However, it is extremely limited in dissolving organic substances. For this type of compound, water is a good medium for the dispersion of dirt particles, and a good basis for specific detergents that dissolve organic substances.

In most cases a mixture of water and domestic washing up liquid can be used as a cleaning solution. Nevertheless, when due to the amount or nature of dirt to these cleaning solutions are not enough, specific solvents for ultrasonic baths should be used. **Under no circumstances should inflammable solvents or environmentally dangerous products be used.**

MAINTENANCE AND CLEANING

In order to ensure an optimum performance and reliability of the unit, we recommend the following:

- Unplug and empty the unit after using it. Do not keep water in the tank for a long time.
- In order to prevent overheating and prolong the machine's working life, it is strongly recommended to stop it for 5 minutes between operations.
- Dry the unit with a clean cloth. Do not use chemicals.
- Do not load the tank excessively; it could reduce the efficacy of the bath and damage the unit.
- Use chemical cleaning products compatible with the objects to be treated. Avoid corrosive solvents.
- When items (especially valuable items) are being cleaned for the first time, carry out a trial run first with just one item. Ask the manufacturer what cleaning solution is recommended.
- After using the unit, dean the tank and the external body with a cloth moistened in soapy water. Always unplug the unit before cleaning it.
- The unit should be stored in a cool and dry place.

IMPORTANT- *The input voltage of the Ultrasonic Bath transducers can reach hundreds of volts. Therefore, it is very important to observe the following Safety Regulations:*

- 1) Do not open the lid when the power cord is connected to the power supply.
- 2) Do not spill water or any other conducting fluid over the unit's housing.
- 3) Always connect the unit to grounded power outlets.

PRECAUTIONS

- ◆ Connect the unit to a grounded 230 V, 50/60 Hz power outlet.
- ◆ Do not use inflammable solvents as cleaning solutions.
- ◆ Keep the unit unplugged before filling it. Do not fill the tank above the *max.* mark. Do not start up the unit when the water level is below the *min.* mark.
- ◆ Do not allow children or unskilled people to handle the unit.
- ◆ Never submerge the unit in any liquid to clean it.
- ◆ Ultrasonic cleaning is not suitable for organic pearls.
- ◆ Do not open the unit while it is connected to the power supply.
- ◆ If you are not sure of the effects of the cleaning solution on a given type of item, carry out a previous test with a single item.
- ◆ Do not attempt to remove any screw or panel. The user does not need to manipulate anything inside.

- ◆ To prevent electric shock, never touch the power cord or socket with wet hands.
- ◆ Never operate the unit if the plug or the power cord is damaged. If the unit does not work properly, is damaged or has fallen into the water, refer it to a technical service for examination and repair.
- ◆ Do not touch the unit if it has fallen into the water. Unplug it immediately.
- ◆ Use the unit properly according to the user's guide. Do not use accessories not recommended by the manufacturer.
- ◆ The unit should never be left unattended when it is plugged.
- ◆ Do not operate the unit next to heat sources such as radiators.
- ◆ Install the unit on a dry and flat surface.

TECHNICAL SPECIFICATIONS

Voltage:	230 V, 50 Hz
Weight:	700 g
Power:	21 W
Dimensions:	178 x 120 x 120 mm
Capacity:	140 ml
Timer:	5 min

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