Ref. 100191



ES.....	MEZCLADORA AUTOMÁTICA DE ALGINATO	4
EN.....	AUTOMATIC ALGINATE MIXER	10
FR.....	AUTOMATIC ALGINATE MIXER	16

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TALLERES MESTRAITUA S.L.
Txori-Erri Etorbidea, 60
Tfno. + 34 944530388 - Fax + 34 944711725
E-mail: mestra@mestra.es - www.mestra.es
48150 SONDIKA - BILBAO - ESPAÑA

Rev. 01/09/11

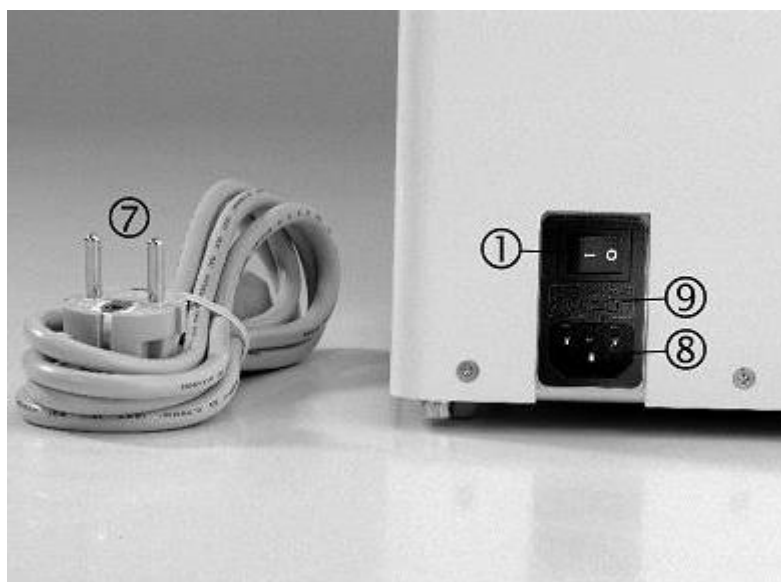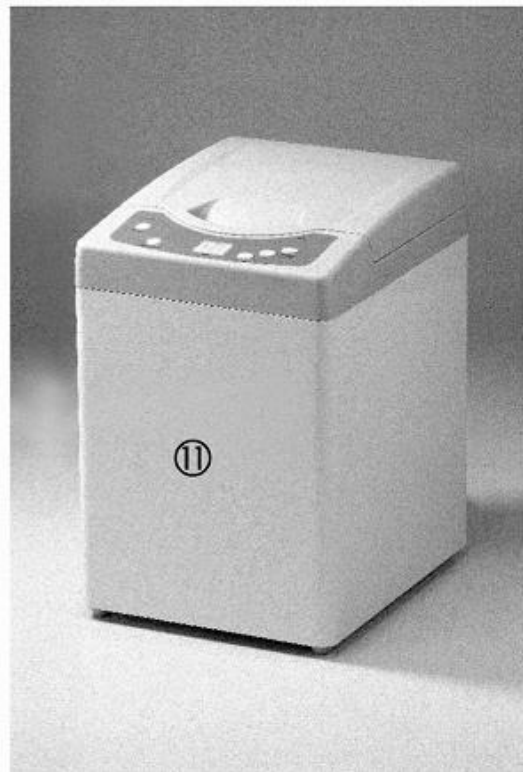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WARNINGS – PREVENTIVE MEASURES

- Attention: before using the machine, please completely read this operating manual.
- Keep this operating manual properly for future.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- The machine features a precision circuit. Prevent fortuitous impacts and stresses that could cause mechanical damage.
- Do not use the machine with a different product than alginate.

- Do not obstruct the ventilation grilles at the bottom of the unit.
- Remove any dirt or water remains from the cord connection.
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not install the unit near to heat sources.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents...).

- To prevent accidents, be cautious when using the unit.
- Always keep the lid closed when you don't use the unit.
- Always keep the device in idle mode (display off), except when mixing.
- Do not run the unit without impression material.
- Do not allow children or non-qualified personnel handling the unit.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.
- Do not dismount, repair or modify the machine.
- If you find a defective component or a problem do not use the machine.

- Contact the supplier for questions or detailed information about problems with the unit.

RESPONSIBILITY LIMITATION

1. Manufacturer shall not assume any responsibility if any malfunction, damage or accident if it is caused by the improper renewal, modification, maintenance, or repair as not being conducted by the dealer designated by us.
2. Manufacturer shall not assume any responsibility if any malfunction, damage or accident if it is caused by the use of the product not provided by us.
3. Manufacturer shall not assume any responsibility if any malfunction, damage or accident if it is caused by the improper removal, modification, maintenance or repair as not use the parts provided by us.
4. Manufacturer shall not assume any responsibility if any malfunction, damage or accident if it is caused by the improper operation as not in compliance with the manual.
5. Manufacturer shall not assume any responsibility if any malfunction, damage or accident if it is caused by the ambient environment as not in compliance with the power, installation environment and using conditions specified on this manual.
6. Manufacturer shall not assume any responsibility if any malfunction, damage or accident if it is caused by fire disaster, earthquake, flood or struck by lightning.

INSTALLATION CONDITIONS

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- Use inside the house.
- Use at the ambient temperature between 5 - 40° C.
- Use a 230 V, 50/60 Hz power source.

2. Installation:

- Install on the cabinet or table strong and firm with flat surface, the strength of which is sufficient to support.
- Never install near inflammable substances.
- Keep the device away from water or sources of moisture that can cause a shock or cause mechanical problems.
- Do not place any objects on top of the top cover or the control panel to prevent damage or spills.

3. Before / after transport:

- **Before installation, remove the 2 bolts fixed at the bottom of the machine (Fig. 1) and place them in the holes at the back of the unit for future use.**
- **If the mixer is out of order and needs to be sent for repair or delivered to some**

far place, please fasten by the fixing bolts attached in the delivery, otherwise it could possibly cause further malfunction or damage.

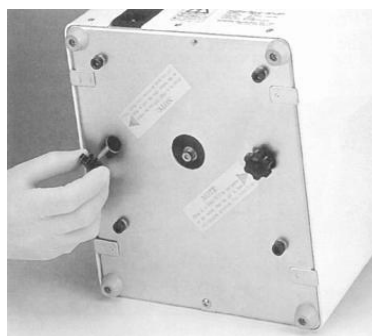
Fig. 1

! NOTE

There is a new design of rear cover where the bottom screws can fix.

4. Power

- Connect the power cable to the outlet on the reverse side of the machine.
- Connect to a 230 V, 50/60 Hz grounded power source.

Fig. 2

! NOTE

Spare fuse inside for replacement. Specifications: 10 A / 250 V.

OPERATION

Max. mix:		
Water + alginate	=	80 g

Alginate mixing:

1. Press down the main power switch on the reverse side of the machine (Fig. 3). A LED will light at the left side of the control panel.

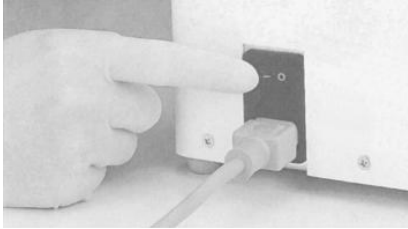
Fig. 3

2. Press down the power switch located at the control panel, and make sure that the indicator of digital display lights on (Fig. 4).

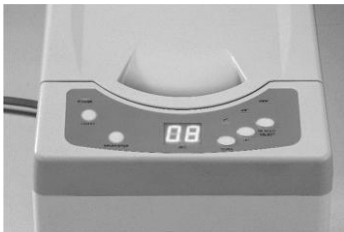
Fig. 4

3. Prepare the alginate powder following the instructions of the manufacturer. Stir the powder and do not compact.
4. Use the bottle to take the required quantity of water (10-20 °C). Pour in the mixing cup with the alginate powder (follow the instructions of the manufacturer for the proportion) (Fig. 5).

Fig. 5

! IMPORTANT
The hardening time can be controlled by water temperature. Particularly in the summer time, if not use cool water, the hardening time will be shortened

5. The amount of measure cup shall be calculated as per the correct scale designated by the supplier of alginate impression material, please also note

that the water amount added will affect the viscosity of alginate powder.

6. Fist mix powder and water with the stirrer for 2 seconds (Fig. 6). You can also just shake the cup some times. If not, you could obtain a bad result.

Fig. 6

Tip: before extracting the stirrer, clean it with the cup edge.

7. Close the mixing bowl cap turning to the right (Fig. 7).

Fig. 7

8. Make sure that the small hole at the center of cap is not obstructed. Follow the instructions and arrows printed on the lid.

Fig. 8

In the meanwhile, make sure that the unit is ready. Check also the spatula and impression tray

9. Insert the mixing glass into the mixing wheel. Make sure that it is perfectly clean and the cup correctly placed inside (Fig. 9).

Fig. 9

Note: Perform the above operations quickly to prevent the alginate mixture hardening too soon. Keep in mind that the longer the preparation lasts, the less time will be left to work before the mixture hardens.

10. Close the cover of mixer properly. Make sure whether the cover has been closed properly. If not, the mixer cannot be started on.

Fig. 10

11. Meet the mixing amount to set the required time by timer (please refer to the "Setting for Mixing Time"). When complete the setting, then press down the start button (Fig. 11). The digital display will count second by second till the time of completion. The cover of mixer is fully locked when running, so it is very safe as unable to open.

! NOTE

To protect the motor, the power will be gradually adjusted when starting to reach maximum speed in approximately 2 seconds.

Fig. 11

! IMPORTANT

The hardening time of the alginate depends on the mixing time that is programmed in the mixer. To have enough time to work, set the mixing time as short as possible.

Note:

- Work and hardening times increase the lower the water temperature. As the water temperature increases, these times are shortened.
- The hardness of the water causes the same effect: the higher the hardness of the water used for the mixture, the shorter the working and hardening time.

! IMPORTANT

If intends to stop running, press down the **STOP** button and the mixer will be stopped immediately. If finds any abnormal condition during running, the mixer shall be stopped immediately. Send it to repair.

12. After making sure that the mixer has been stopped, the cover can be opened to take the mixing bowl out. Use the spatula to have the mixture out (Fig. 12).

Fig. 12

SETTING THE MIXINT TIME

1. The timer has 3 programs established by the manufacturer that you can change.

Memory	Mixing time (seconds)
1	8
2	10
3	12

2. Press down the button of MEMORY SELECT to select the time required for mixing.
3. To alter the memory of mixing time established by the manufacturer, you can do adjustment by pressing the button **UP** or **DOWN** to key in the required mixing time. The maximum time is 16 seconds. You can make the adjustment second by second.

ATTENTION:
Increased mixing time accelerates hardening of impression material.

* The time setting for power of 50 Hz (the water temperature is between 10 y 20 °C) is listed as below:

Alginate cups	Mixing time (seconds)
1	7 - 10
2	8 - 11
3	9 - 12

* The time setting for power of 60 Hz (the water temperature is between 10 y 20 °C) is listed as below:

Alginate cups	Mixing time (seconds)
1	6 - 9
2	7 - 10
3	8 - 11

4. Cleaning:

If there is any dirt on the mixer, please use cloth to wipe. If it is serious, then please use a neutral cleaning agent.

Clean the unit at least once a week.

! ATTENTION:
Never use solvent or volatile oil to swipe for cleaning, that will cause surface faded.

! IMPORTANT

1. The mixing time shall adopt in average. If there is any difference between water temperature established by the supplier of alginate impression material, the setting time will be different.
2. Please set on the proper mixing time.

MANTENANCE

1. Before cleaning or maintenance, disconnect the unit.
2. Do not clean the unit while running or hot.
3. **Mixing bowl:**

After setting, the remains impression material inside the bowl can be cleaned and removed easily. The small hole at the center of cap shall be cleaned very often (Fig. 13).

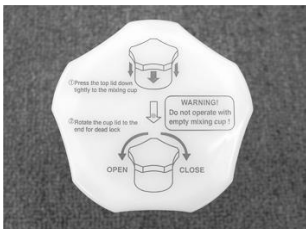
Fig. 13

! IMPORTANT

1. The small hole at the center of cap shall be cleaned very often. If jam, the cover will be very hard to open or close.
2. The water remained in the bowl after cleaning shall be removed completely. Otherwise it will cause error calculation to next use.

TROUBLE SHOOTING**! IMPORTANT**

If you find a problem different from the following, contact your distributor and send the unit to repair.

Problema	Causa	Solución
No power	Bad connection of power.	Inspect the connection of power.
	Fuse burn out.	Inspect fuse and change if needed.
Digital display has no image	Defective control board	Return to dealer for service.
	Defective driving motor or gear.	Return to dealer for service.
	Cover not fully closed.	Make sure cover of mixer is securely closed.
Slow start up to device	Defective driving motor or gear.	Return to dealer for service.
	Defective starting capacitor.	Return to dealer for service.
Abnormal noise or vibration during operation	Too much alginate and/or water.	Maximum amount of alginate is 80 grams (including water + alginate).
	Mixing bowl is empty.	Never use unit with an empty mixing bowl.
	Driving gear damage.	Return to dealer for service.
	Alginate powder spillage because of loose cap.	Secure bowl cap and tighten.

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