Ref. 100504

ES..... 3 Manual original  
EN..... 5

 **MESTRA®**

1. Bandejas / **Trays**
2. Lámpara UV / **UV lamp**
3. Interruptor / **Switch**
4. Piloto / **Pilot**
5. Sensor de puerta/ **Door sensor**
6. Fusible / **Fuse**

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## Warnings

### Preventive measures

- Attention: before using the machine, please completely read this operating manual.
- Keep this operating manual properly for future.
- Do not allow children or non-qualified personnel handling the unit.
- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- WARNING: to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Prevent fortuitous impacts and stresses that could cause mechanical damage or lamp break.

- Do not introduce potentially explosive substances.
- Remove any dirt or water from the cable connection.
- Unplug the appliance immediately if you detect noise, odor or smoke.
- Install the product away from heat sources.
- Do not use the appliance in particularly humid places, or near flammable materials (alcohol, solvents, etc.).

- Do not use the machine with products different than defined in the intended use.
- To prevent accidents, be cautious when using the unit.
- Always keep the device in idle mode, except when mixing.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.
- Do not dismount, repair or modify the machine.
- If you find a defective component or a problem do not use the machine.
- Contact us for questions or detailed troubleshooting information.

**NOTE:** Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

Thank you for purchasing our UV disinfection cabinet. Please read these instructions for use carefully before using the product.

### Operating principle

The germicidal properties of ultraviolet light are widely known. It is known that UV light with wavelength ranges between 185 and 254 nanometers is able of interacting with the DNA of germs, causing their death in a very short time.

For its part, light with a wavelength of 185 nanometers is able of ionizing the oxygen in the air, generating ozone (O<sub>3</sub>). This gas has widely proven germicidal properties. As a gas, it can be introduced through all the cracks of the object to be disinfected. On the other hand,—and due to the low concentration—the ozone gas generated is not harmful for health.

### Disinfection range

The device is suitable for the following microorganisms: bacteria, microbacteria, viruses (different types), spores, chlamydia, organisms of the fungus group, etc.

### Operation

1. Place the appliance on a stable surface or hanging vertically from the wall.
2. Put the objects to be sterilized into the chamber (clean and dry).
3. Make sure the door is properly closed.
4. Press the switch to start sterilization.
5. The lamps will start.
6. Press the switch again if you want to interrupt the cycle.
7. The chamber is equipped with a door open control and will turn off if the door is opened.

For reference, the mean sterilization time is usually 15 minutes.

## Precautions

- Do not look directly at the lamp when it is on.
- Be sure to plug the appliance into a 230V outlet. 50 Hz.
- Do not operate the appliance with wet hands.
- Do not introduce wet objects inside the disinfection chamber
- Do not open the door with the equipment running.
- Do not allow children to operate the appliance.

## Technical data

Power supply: ..... 230 V, 50 Hz  
Power: ..... 15 W  
Height: ..... 500 mm  
Width: ..... 350 mm  
Depth: ..... 260 mm  
Capacity: ..... 27 L  
Weight: ..... 8 kg

## Declaration of conformity UE

The firm **Talleres Mestraitua S.L.**

with social address: Txori-erri Etorbidea, 60  
48150 Sondika - Bilbao - Spain

Hereby declares that the product

UV Cabinet Ref. 100504

Meets the requirements set out in the following directives:

2014/35/UE (low tension)

2014/30/UE (electro magnetic compatibility)

Ignacio Mestraitua  
Managing Director  
Sondika, 2023-04-12