Ref. 080550 SUN BEAM

ES.....	LÁMPARA POLIMERIZADORA.....	3
EN.....	POLYMERISATION LAMP.....	5
FR.....	APPAREIL A PHOTOPOLYMERISER.....	7
DE.....	LICHTHÄRTEGERÄT SUN BEAM.....	9
IT.....	LAMPADA POLIMERIZZATRICE.....	11
PO.....	LÁMPADA POLIMERIZADORA.....	13

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

info@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

1. Ventana / **Window** / *Fenêtre* / Fenster / **Finestra** / *Janela*
2. Puerta de Cámara Polimerización / **Polymerisation chamber door** / *Porte de la chambre à photopolymériser* / Polymerisationskammertür / **Portello Camera di Polimerizzazione** / *Porta da câmara de polimerização*
3. Cámara de Polimerización / **Polymerisation chamber** / *Chambre à photopolymériser* / Polymerisationskammer / **Camera di Polimerizzazione** / *Câmara de polimerização*
4. Interruptor General / **Main switch** / *Interrupteur général* / Hauptschalter / **Interruttore generale** / *Interruptor geral*
5. Mando del temporizador / **Timer control** / *Minuterie* / Bedienelement des Zeitschalters / **Comando del temporizzatore** / *Comando do temporizador*

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The Sun Beam Polymerisation Lamp (Ref. 080550) polymerises photopolymerisable materials swiftly and efficiently. It has a frequency range of 280 - 550 nanometers, and this allows it to polymerise the vast majority of materials used in the Dental Sector. We recommend you read the following Instructions for Use carefully for proper usage of the unit and best lamp performance:

INSTALLING THE LAMP

- Carefully unpack your **Sun Beam** Polymerisation Lamp. Check that no damage has been sustained in transit, and read this “User’s Manual” in full.
- Choose a flat rigid surface for the facility, protected against sources of heat or vibrations. The unit should also be protected from direct sunlight.
- The lamps (6 x 9 W.) have been disassembled to prevent breakage during transit. Proceed with reassembly by inserting each lamp into its holder. You will observe that three lamps have been supplied with tone S/71, and a further three lamps with tone S/78. Each type of lamp must be alternated during assembly in order to obtain uniform lighting for the polymerisation chamber. Extreme care must be exercised during this operation to avoid any breakages.
- Connect the appliance to 230 V, 50/60 Hz, with earthing.
- Activate the main switch on the left-hand side of the unit. You will observe that the display shows the operating time set. The **Sun Beam** Lamp is now ready for use.

OPERATION

- Uncover the polymerisation chamber, and place the items you wish to polymerise on the prosthesis plate. Up to 4 models may be polymerised, depending on the size of the items.
- Close the polymerisation chamber, and use the timer control to select the time required.
- When the time has been selected, press the timer control knob. The facility will start up, the polymerisation chamber light comes on, and the prosthesis plate rotates.
- It is possible to cancel a polymerisation cycle during operation by pressing the timer control again. The machine then stops, and the display shows the last time selected.
- If the polymerisation chamber door opens during operation, the machine’s lights will go out and the prosthesis plate will cease to rotate for as long as the door is open. The timing function, however, continues to operate normally. This aspect is particularly useful when we wish to add or remove parts during a cycle which has already commenced.
- When the time set has elapsed the machine will stop. Remove the items from the polymerisation chamber.

CHANGING THE LAMPS

The polymerisation lamps’ service life is limited. Optimum change is recommended every 500 hours, to a maximum of 1000 hours of operation.

MAINTENANCE AND CLEANING

- The polymerisation lamps' service life is limited. Optimum change is recommended every 500 hours, to a maximum of 1000 hours of operation (see point 3 in this Instructions Manual). Since the appliance uses two different types of lamps (S/71 and S/78), you must specify the type of lamp in requests for spare parts. Talleres Mestraitua, S.L. recommends that original **MESTRA** spare parts be used.
- The lamp types must be arranged alternately inside the polymerisation chamber.
- There is a fuse-holder on the left-hand side of the unit. The 2 amp. fuse is to be changed as required.
- The unit should be cleaned on a regular basis, using a cloth soaked in soapy water. The unit should be disconnected from the power supply during this operation.
- The efficiency of the polymerisation lamp depends to a large extent on the clarity of the polymerisation chamber, and so it is advisable to carry out periodic cleaning.

PRECAUTIONARY MEASURES

- ◆ Connect the unit to 230 V, 50/60 Hz, with an earthing facility.
- ◆ Do not submerge the unit in any liquids for cleaning purposes.
- ◆ Do not use the Polymerisation Lamp in locations which are particularly humid, or next to inflammable liquids (alcohol or solvents).
- ◆ The lamps must be changed with all due caution to prevent any breakage.
- ◆ Do not use the Polymerisation Lamp for any purpose other than polymerisation of Dental Materials.
- ◆ Do not allow children or unqualified staff to handle the unit.
- ◆ Operations must be carried out at all times with the polymerisation chamber door shut. Avoid all eye contact with the light emitted by the lamps, since ultra-violet radiation is being produced, and this can damage eyesight.

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GUARANTEE

All **MESTRA®** units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA®** will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

MESTRA® will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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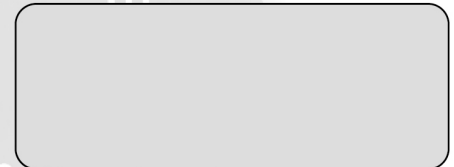
***TALLERES MESTRAITUA S.L.** is a company manufacturing tools and appliances for the dental prosthesis sector, from the business address shown at the bottom of the page.*

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***I HEREBY STATE:** That the “SUN BEAM” POLYMERISATION LAMP Ref. 080550, with the following characteristics:*

Alto / Height:	315 mm
Ancho / Width:	233 mm
Fondo / Depth:	210 mm
Ø plato giratorio / Ø plate:	135 mm
Peso / Weight:	6,1 kg
Espectro / Spectrum:	280/550 nm
Alimentación / Power supply:	230 V, 50/60 Hz
Luminosidad / Illumination intensity:	8750 Lux

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Meets the low voltage requirements set out in Directive 2014/35/UE, in Directive 2014/30/UE concerning electromagnetic compatibility and in Directive 2006/42/EC concerning machinery safety.

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In witness whereof, the company issues this declaration of conformity in Sondika, on 30 September 2024.

Firma / Signed:

Ignacio Mestraitua,
Director Gerente / *Managing Director*

MESTRA[®]

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

info@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA