




UV CABINET

Ref. 100500

- Install away from water sources. The plug of the product must be kept dry.
- Make sure there are no foreign bodies covering the air inlet. Make sure that a good air circulation is formed so as not to affect the working effect of the product or cause overheating.
- Do not dry items with low heat resistance, airtight, treated with rubber, fine materials (silk, feather) or with flammable oil.
- Do not use the product in a closet or in a closed environment, to avoid shutdown of the product due to malfunction or overheating protection caused by air ventilation.
- The product must not be exposed to direct sunlight for a long time, to avoid accelerated aging of the product material and affect the service life of the product.
- Stop using it if the power cord is damaged.
- If the unit does not work normally, turn off the power immediately and send it to the designated maintenance point or professional personnel for inspection and repair. Removing and replacing parts is not allowed.

Components

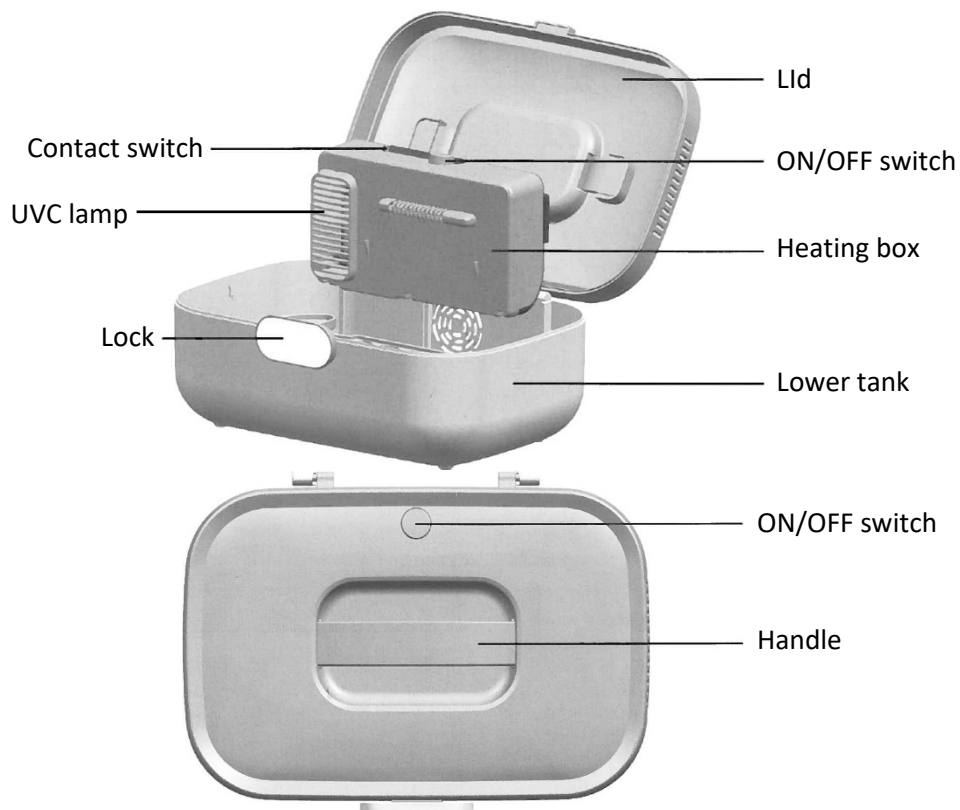

Use:

- After connecting the unit, the red light flashes.
- Press the key to start the drying and disinfection mode: the light will keep on, the machine starts working, the cooling fan starts working.
- Total drying time is 150 minutes
- Then the UV lamp for disinfection will be turned on for 10 minutes.
- Then the red light flashes. At this time, the heating and the UV lamp turn off. The cooling fan continues to run for 3 minutes and enters the standby state.

Separate disinfection mode:

- Keep pressing the key for 3 seconds. The blue light will turn on, indicating that the UV lamp is working.
 - It stops after 10 minutes, the red light flashes, and it enters the standby state.
-
- To stop the operation, press the key: the red light will flash and the machine stops working after 10 seconds.
 - Regardless of the mode, the UV lamp will turn on only when the machine cover is closed, and it will turn off automatically when the machine cover is opened.



Talleres Mestraitua S.L.

Txori-erri Etorbidea, 60
T: (+34)944530388 - F: (+34)944711725
mestra@mestra.es - www.mestra.es
48150 SONDIKA - BILBAO - ESPAÑA