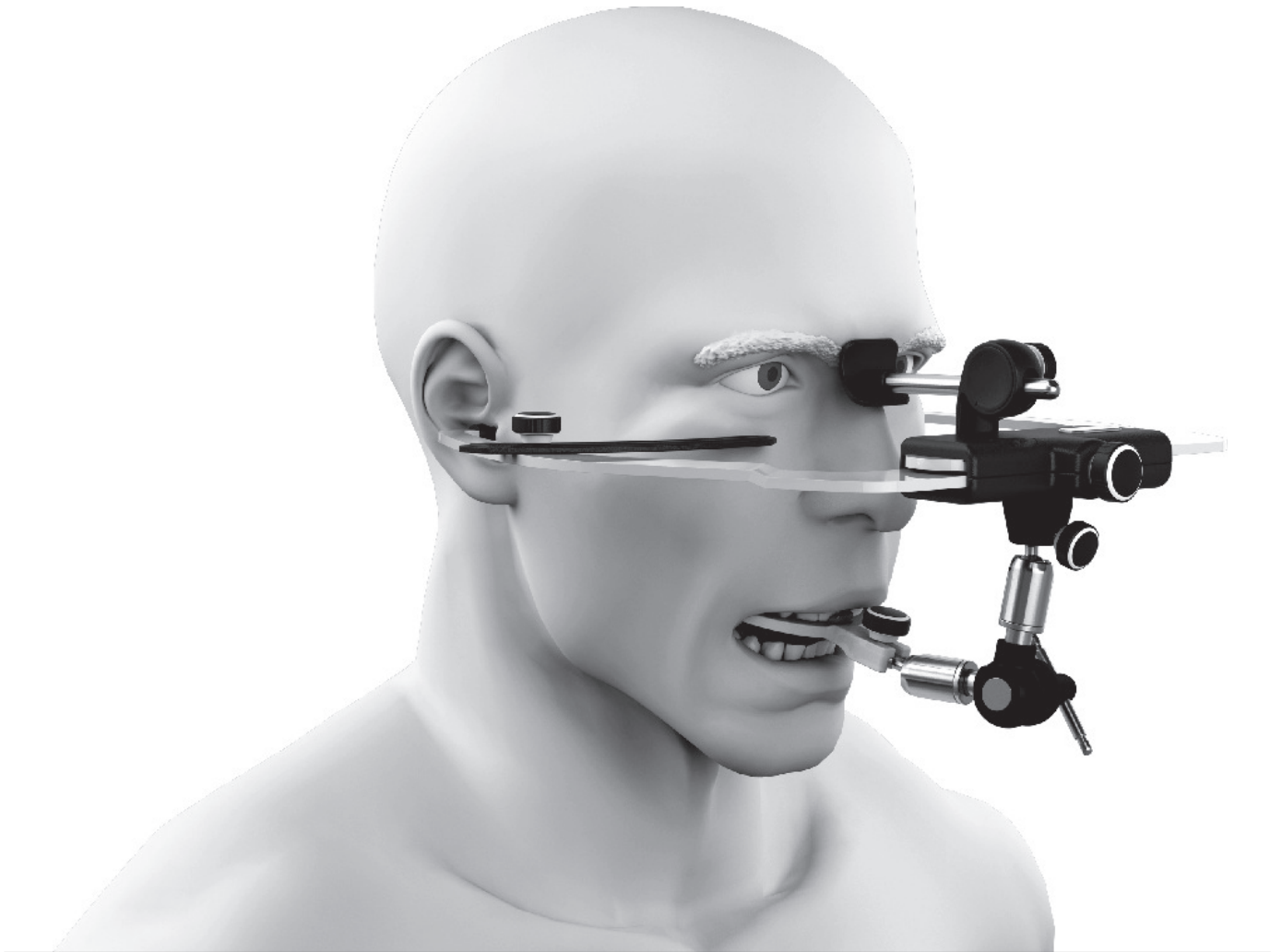
# Elite

## FaceBow

**Instruction manual**

---

# **Index**

1. Introduction . . . . .	01
2. General description . . . . .	02
3. Installation instructions . . . . .	03
4. Instructions for use . . . . .	03
5. Cares, precautions and safety . . . . .	07
6. Cleaning, disinfection and maintenance . . . . .	08
7. Storage conditions . . . . .	08
8. Problems and solutions . . . . .	09
9. Technical specifications . . . . .	09
10. Symbology . . . . .	09

## 1. Introduction

**Carefully read all information contained in this user manual before you start using your Elite Facebow.**

The Elite Facebow is an instrument especially developed to be used in the procedure of register and transfer of the upper dental arch spatial position of the patient in relation to the skull to the Mestra Articulators.

**For ensuring the total effectiveness in the final results of the work, the Elite Facebow should only be used with the Mestra Articulators.**

This user manual contains all the necessary technical information regarding the device features and functions. For the success in the final results of the work in which the equipment will be used, it is compulsory to have expertise through occlusion and/or prosthesis courses, which are offered in many educational institutions and also in the dentistry colleges.

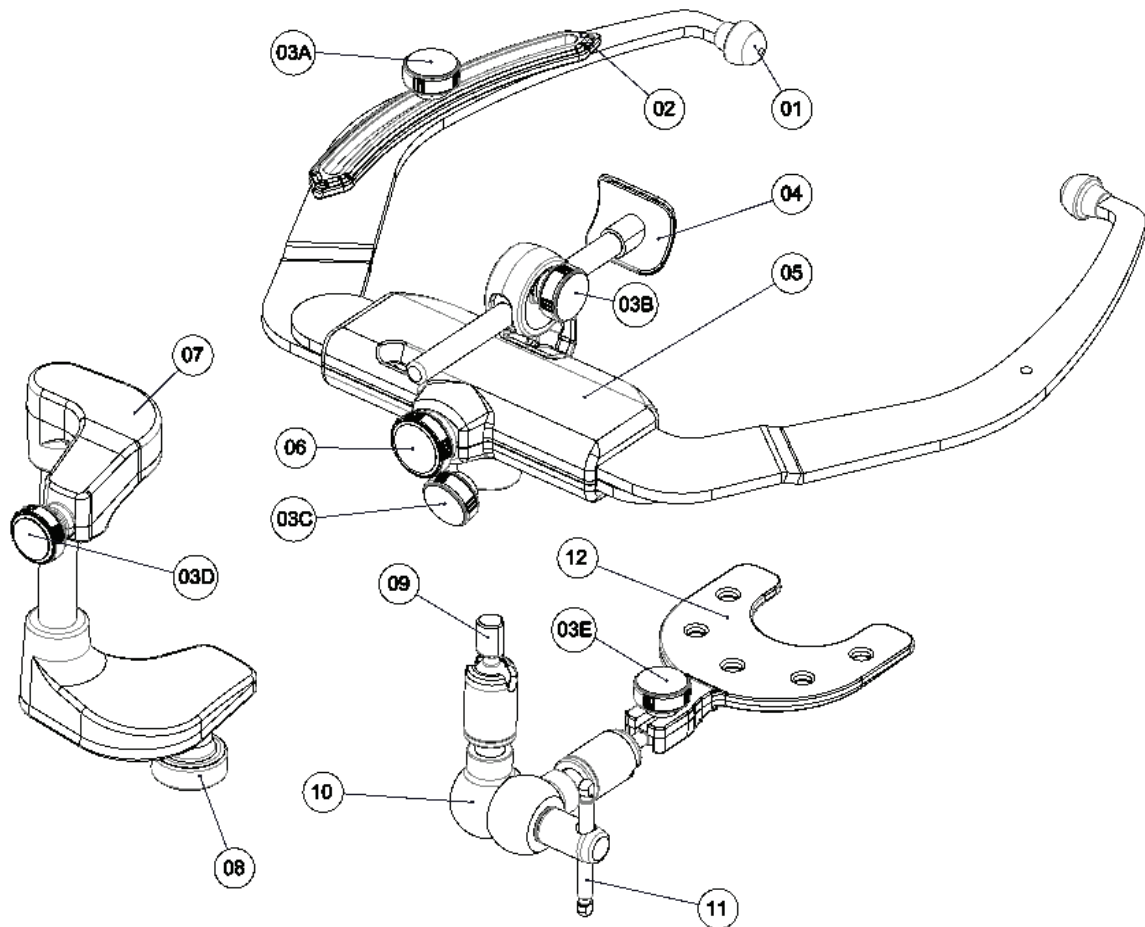
**The use of the Elite Facebow by non-trained professionals can cause damages to the product and/or the patient.**

It is important to emphasize that the use of the Elite Facebow jointly with the Mestra Semi-Adjustable Articulators offer a simple, fast and completely secure process for reproducing the patient mandibular movements, with a high level of precision and at a low cost. They present great advantages when compared to the techniques that imply a considerable time and the use of expensive and complex equipment, as well as the work carried out with simple "occluder" (hinge), where the models assembly in this case is arbitrary and the movements are limited.

It is concluded, therefore, that the use of the Elite Facebow jointly with the Mestra Semi-Adjustable Articulators are certainly recommended for most works with prostheses, occlusion and dental rehabilitation, among others, consisting of a simple, fast, and easy to apply technique, ensuring entirely satisfactory results both to the patient and the professional.

## 2. General Description

ITEM	QTY	DESCRIPTION
1	2	Ear pieces
2	1	Infra-orbital indicator
3	5	Fixation knobs
4	1	Naziun relator
5	1	Rack and gear housing
6	1	Brake knob
7	1	Transfer Set Support
8	1	Lower knob
9	1	Transfer Set Terminal (prism)
10	1	Transfer Set
11	1	Quick lock lever
12	1	Dentulous Bitefork

### 3. Installation instructions

Due to its technical characteristics, the Elite Facebow does not require specific instruction for its installation, and for its correct use, the professional should just follow carefully the information provided by the manufacturer, available in this user manual.

Thus, all the responsibility regarding misuse of the equipment is sole responsibility of the professional who uses it.

### 4. Instructions for use

The following instructions describe clear and objectively the procedures for the correct use of the Elite Facebow.

**Before using the Elite Facebow, always detach the ear pieces and dentulous bitefork and sterilize them in autoclave at 121°C of temperature. ATTENTION: it is not recommend sterilization in temperatures exceeding 121°C, because this process can damage these parts.**

#### 4.1 Facebow opening adjustment:

Use the brake knob to lock the facebow in the desired position, clockwise for locking and counterclockwise for unlocking (**Fig. 01A**), not exaggerating in the strength applied on the knob since it is not necessary and may damage the system.

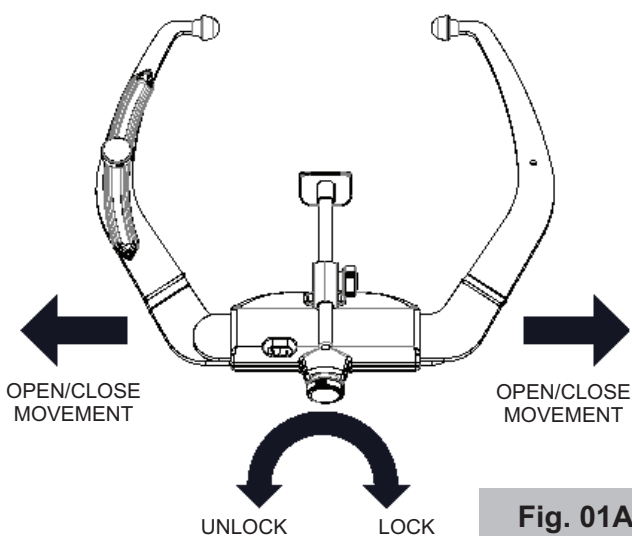
Fig. 01A

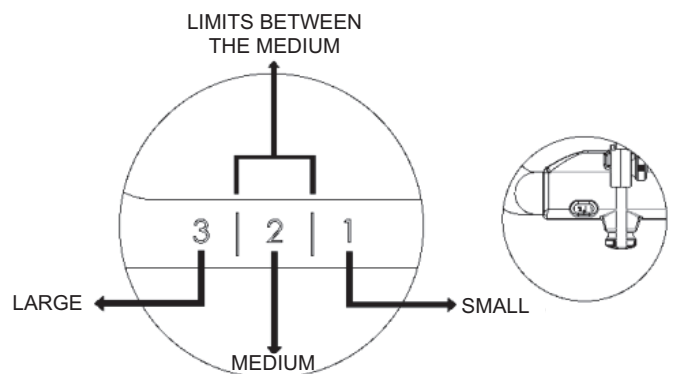
Fig. 01B

#### 4.2 Infra-orbital Indicator :

Use the knob for unlocking or locking the infra-orbital indicator (Fig.02)

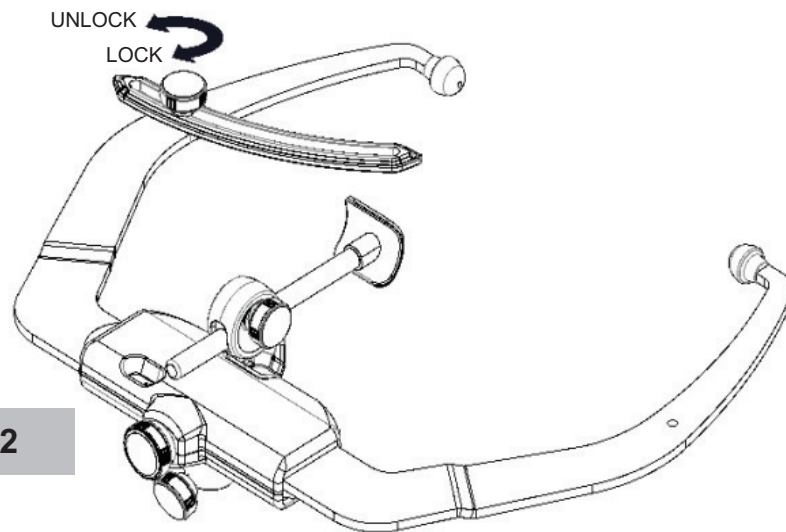
Fig. 02

#### 4.3 Naziun Relator Adjustment :

The Naziun Relator was especially developed to support the facebow as the third point in the patient horizontal plane. It should be regulated according to each patient's anatomy. For this, use it jointly with the infra-orbital indicator in order to reproduce the Frankfurt imaginary plan. For assembling the Naziun Relator in the facebow, fit it in the groove (rail) of the housing as illustration (Fig. 03A). For making the patient adjustment, unlock the knob that holds the naziun stick (counterclockwise), and after the correct positioning, lock it turning the knob clockwise (Fig. 03B).

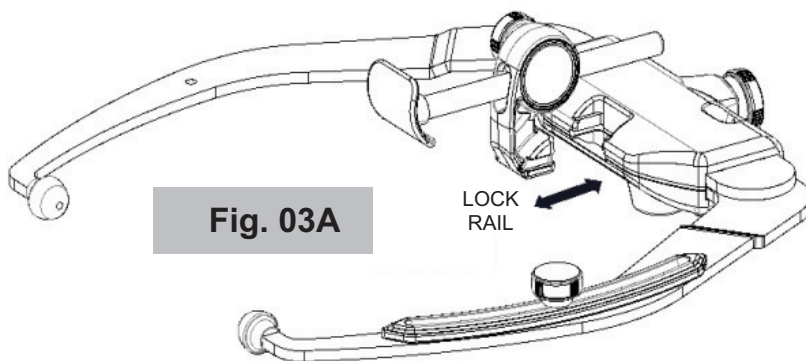
Fig. 03A

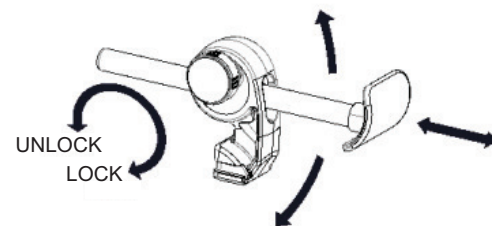
Fig. 03B

#### 4.4 Transfer Set:

For assembling in the facebow, fit the transfer set terminal until the end of the groove and fix it with the knob (Fig. 04A). The transfer set terminal has a prism fitting system that ensures repeatability in its positioning, since the quick lock lever is not released during the registration, and since it was docked up to the end of the groove. Always remember to leave the quick lock lever toward the right side during the patient reference making, avoiding possible interference with the transfer set support column during the transfer to the articulator.

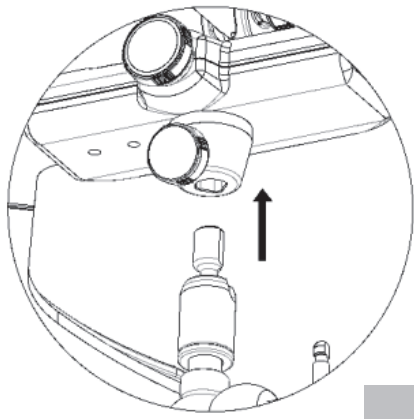
Fig. 04A

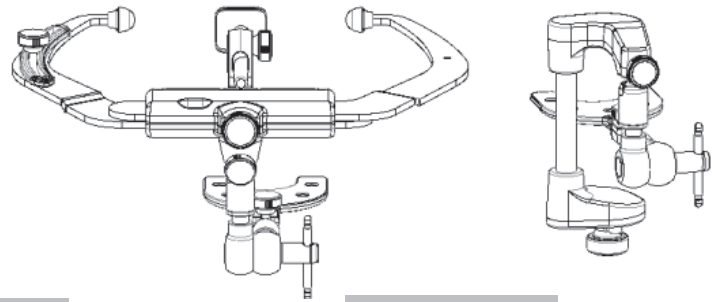
Fig. 04B 04C

**Make sure that the terminal of the Transfer Set is fully inserted inside the groove before making its fixation (Fig. 04B and 04C).**

**Always assemble the Transfer Set in the facebow with the quick lock lever towards the right side of the user.**

It is not necessary to unlock the lever totally for the patient positioning. Only release half of it for releasing the joints movement. After the patient's positioning turn the lever clockwise for locking the set (**Fig. 04D**).

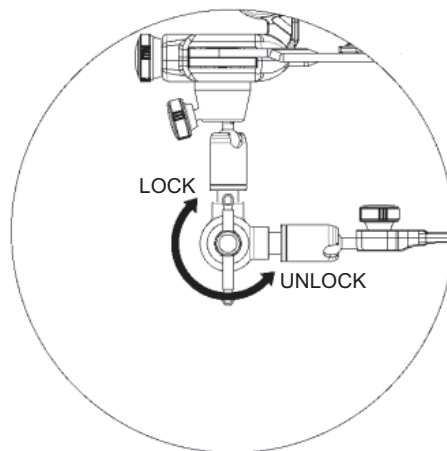
Fig. 04D

**The quick lever locking system was especially designed so that its locking occurs satisfactorily without the need of applying excessive strength.**

**4.5 Dentulous bitefork:** Position the bitefork in the transfer set until the final course of the transfer set and use the knobs for fixing it (**Fig. 05**).

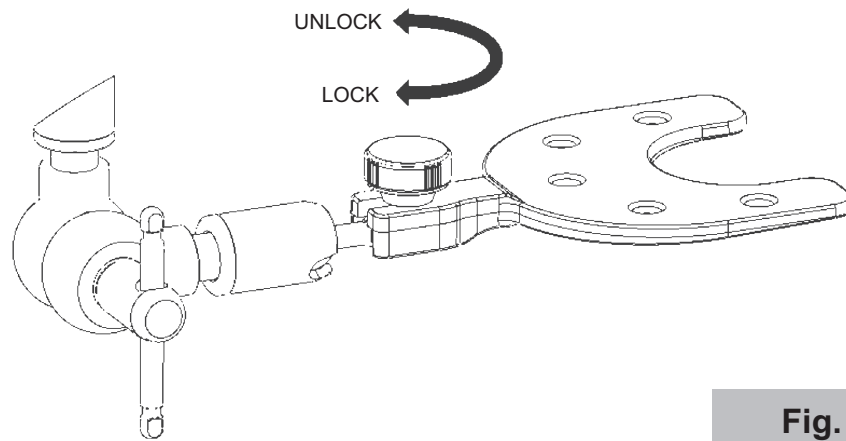
Fig. 05

**The Dentulous Bitefork should not be placed directly over the flame. If necessary, heat the bitefork using only hot water.**

**4.6 Transfer Set Support:** For assembling the Transfer Set in its Support, fit the terminal until the end of the groove and fix the terminal with the knob. The transfer set terminal has a prism fitting system that ensures repeatability in its positioning, since the quick lock lever is not released during the registration, and since it was docked up to the end of the groove (**Fig. 06**).


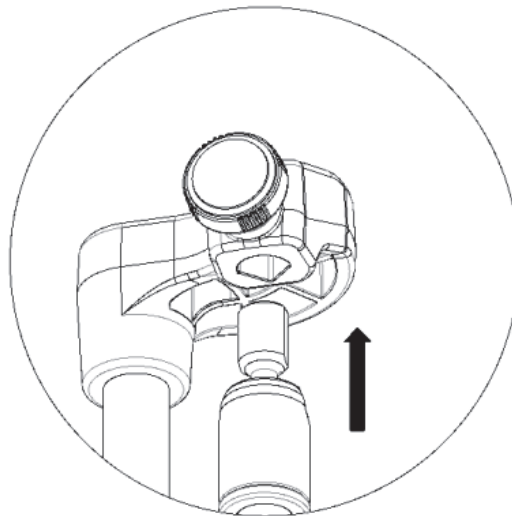
Fig. 06

**Make sure that the terminal of the Transfer Set is fully inserted in the Transfer Set Support groove before its fixing.**

**4.7 Articulator assembling:** When fitting transfer set support in the articulator, take it to the end of the groove for guaranteeing the register position and then lock the screw. (**Fig. 07**).

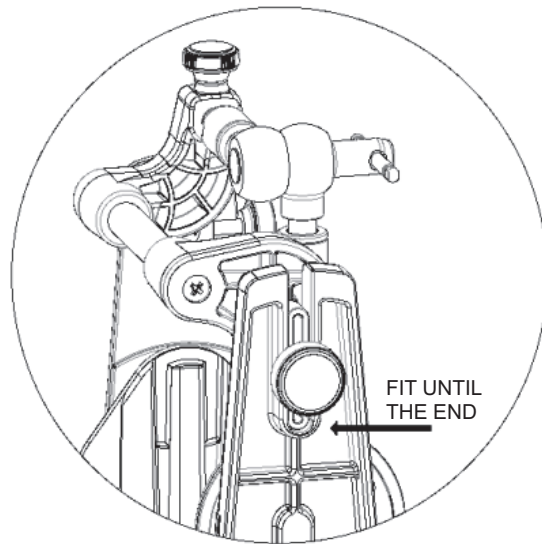**Fig. 07**

## **5. Cares, Caution and Safety**

This manual only contains the basic information related to the use of the device, which do not replace the proper course and user training, and the Elite Facebow is a product intended for use only by qualified professional in the area.

The Elite Facebow was designed for use only and exclusively with Mestra Articulators.

The facebow is equipped with rack and gear system, so do not force it for opening or closing beyond its lateral course, since this may damage the gear, compromising the equipment functionality.

The knobs and quick lock lever grips should be made without excess of strength for not damaging the equipment.

**Special care must be taken when using the bite registration material since excessive heating can cause burns in the patient. We strongly recommends that users follow the suppliers' instructions of "godiva" or any other material used for the registration process.**

**Before using the Elite Facebow, it should be cleaned with warm water, but the dentulous bifurk and ear pieces should be sterilized in autoclave at 121°C of temperature. The Naziun Relator and the infra-orbital indicator should be disinfected with a bactericidal tissue or with alcohol 70°.**

## 6. Cleaning, Disinfection and Maintenance

### 6.1 Cleaning and Disinfection

**Facebow and Transfer Set Support:** only use a soft sponge and water. Never use solvents, metal spatula or sharp objects for cleaning.

**Transfer Set:** should be cleaned under running water with a soft sponge and dry with absorbent towels or compressed air.

**Dentulous bitefork and ear pieces:** should be washed in an ultrasonic cleaner and then sterilized at 121°C of temperature.

**Do not use temperature above 121°C for the autoclaving process, because it can damage the parts.**

**Naziun Relator and Infra-orbital indicator:** should be washed in ultrasonic cleaner and then disinfected with a bactericidal tissue or with alcohol 70°.

**With the exception of the Dentulous Bitefork and Ear Pieces, no other parts of the Elite Facebow can be autoclaved.**

### 6.2 Maintenance

All maintenance or adjustments in the Elite Facebow, as well as any of its accessories, must be carried out by Authorized Technical Assistance.

**Never disassemble the Transfer Set, since it contains small parts that may be lost or incorrectly reassembled, impairing or preventing the equipment use.**

If you have questions, please contact Mestrat or any of its distributors for Authorized Technical Assistance .

## 7. Storage Conditions

The Elite Facebow, as well as its accessories, must be handled, transported and stored with care.

Even storing on the corresponding case, avoid falls, mechanical and physical shocks for maintaining the product quality and precision.

When not used for a long period of time, keep it in the corresponding case and in a dry place, away from heat sources and direct sunlight.

## 8. Problems and Solutions

Problem	Possible Cause	Solution
Facebow do not slide freely (open/close)	Brakes on.	Unlock the brake by turning the knob counterclockwise.
	Existence of any small object between the guides.	Remove the object with nippers.
The lever tightening is not generating the necessary fixation in the transfer set.	Spherical axis sleeve unscrewing.	Relieve the pressure on the lever and screw the spherical axis sleeve until it touches the sleeve cover.
	Wear of internal components of the transfer set.	Replace the parts at the Authorized Service Center.
Difficulty in the universal joint handling.	Locking of any mobile component of the joint.	Check if there is no accumulation of dirty or plaster between the mobile parts and clean them with warm water and soft brush if necessary.

## 9. Technical Specifications

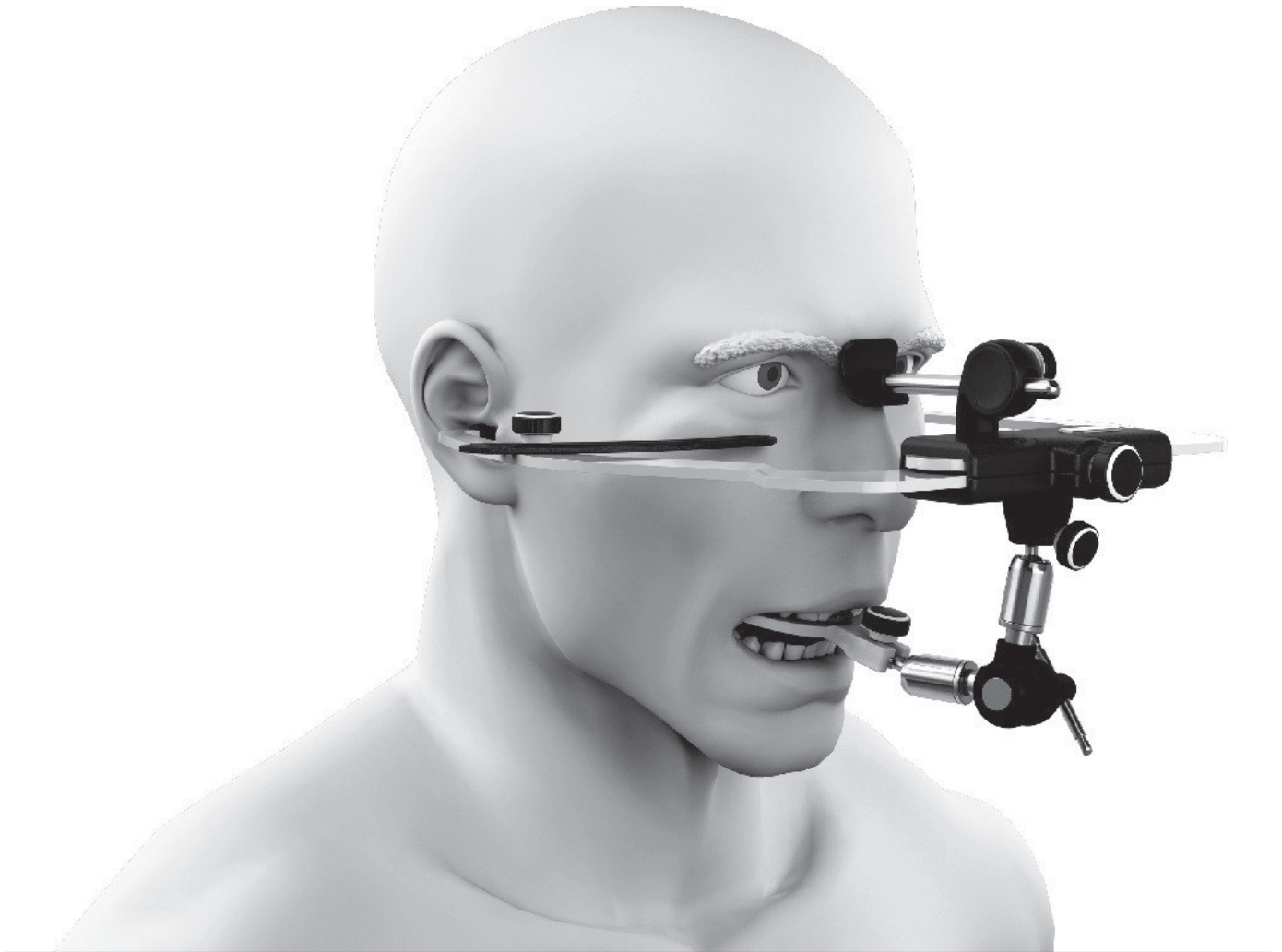
Dimensions :	220x215x120mm (WxDxH) (without package )
	250x250x60mm (WxDxH) (with package )
Weight:	350g. (without package)
	545g. (with package)
Opening:	96mm (minimum)
	177mm (maximum)

## 10. Symbology

					
This way up	Fragile handle with care	Keep dry	Maximum stacking 10 units	Consult instructions for use	Caution

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