


**Componentes**



- (1) Tapón
- (2) Front
- (3) Botón de encendido
- (4) Mango
- (5) Cubierta baterías
- (6) Base soporte
- (7) Dos baterías AA (no incluidas)
- (8) Punta grande (L 2.0 mm)
- (9) Punta mediana (M 1.2mm)
- (10) Punta pequeña (S 0.5 mm)

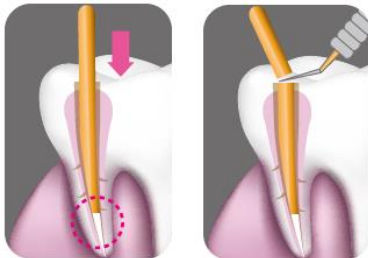

**Especificaciones**

Dimensiones (LxW)	20 x 1.85 cm (punta incluida)
Voltaje	2 baterías AA (3V cc)
Puntas	3 puntas (L: 2.0 mm; M: 1.2 mm; S: 0.5 mm)
Temperatura	>500 °C
Enfriamiento	Menos de 1 segundo
Soporte	3 cm x 2.5 cm

**Características**

1. Un solo botón, dos baterías, diseño de fácil uso.
2. Tres tamaños de punta disponibles
3. Tarda un segundo en alcanzar los 500 °C (932 °F).
4. Punta diseñada para un rápido enfriamiento, uso seguro.
5. Diseño de punta flexible, corte suave.

**Fácil de usar**

1. Instale dos baterías AA en la posición indicada cuando utilice por primera vez el aparato. Recomendamos pilas alcalinas.
2. Inserte la punta seleccionada en la parte frontal. La punta puede doblarse según sea necesario para un mejor acceso.
3. Haga contacto con la gutapercha y presione el botón de encendido. La punta se calentará en 2 segundos. Corte la gutapercha y suelte el botón
4. Por seguridad, espere 3 segundos antes de extraer el cortador de la cavidad oral del paciente.
5. El cuerpo del cortador de gutapercha puede desinfectarse o cubrirse con una barrera plástica (no incluida) antes del uso.
6. Las puntas intercambiables solo pueden desinfectarse en frío.
7. Sustituya las baterías cuando el cortador deje de funcionar adecuadamente.

**PRECAUCIÓN:**

- (1) No toque la punta durante el uso debido a su alta temperatura.
- (2) No mantenga el botón pulsado más de 5 segundos.
- (3) No salpique agua sobre el aparato.

El dispositivo ha sido probado para cumplir con los siguientes estándares: IEC 60601-1, IEC 60601-1-2, 93/42/EEC, and ISO 13485.

**Fabricante:**

**Motion Dental Equipment Corp.**

Dirección: 6F-2 & -3, No. 27, Lane 61, Sec. 1, Guangfu Rd., Sanchong District, New Taipei City, Taiwan (R.O.C)


TEL : +866-2-2278-2199 FAX: +886-2-8511-2492

Website: <http://www.motiodental.com> E-mail: [motion@twmotion.com](mailto:motion@twmotion.com)

## Components



- (1) Cap
- (2) Front
- (3) Power Button
- (4) Handle
- (5) Battery cover
- (6) Stand Base
- (7) Two AA batteries (not shown)
- (8) Large Tips (2.0 mm)
- (9) Medium Tips (1.2mm)
- (10) Small tips (0.5mm)

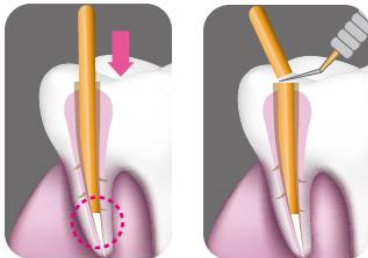

## Specification

Dimension (LxW)	20cm x 1.85 cm (tips include)
Voltage	2 AA batteries (DC 3V)
Tip Type	3 Tips (L: 2.0mm; M: 1.2mm; S: 0.5mm)
Temp. Reach	Over 500 degree
Cool down period	less than 1 second
Stand base size	3 cm x 2.5 cm

## GP CUT Feature

1. One button only, two batteries power supply, easy use design.
2. Three tip size available.
3. One second hip up to 500 degree high temperature.
4. Fast tip cool down design, safety use.
5. Flexible tip design, cutting smoothly.

## Simple to Use

1. Install two AA batteries according to the diagram on GP Cut body. For best results, use alkaline batteries.
2. Insert the selected tip on the front of GP Cut. Cutting wire may be bent as needed for access.
3. Contact gutta percha at desired cutting point and press power button for 2 seconds. Cut gutta percha and release power button.
4. for maximum patient safety, wait 3 seconds before withdrawing GP Cut from the oral cavity.
5. Body or GP Cut may be surface disinfected or covered with plastic barrier (sold separately) before use.
6. Interchangeable cutting tips may be disinfected only.
7. Replace batteries when unit stops functioning properly.

### CAUTION:

- (1) DO NOT touch tip during operation due to high temperature.
- (2) DO NOT press power button over 5 seconds.
- (3) NO water sprays on unit.

The device has been tested to meet the following standards: IEC 60601-1, IEC 60601-1-2, 93/42/EEC, and ISO 13485.

## Manufacturer:

### Motion Dental Equipment Corp.

Address: 6F-2 & -3, No. 27, Lane 61, Sec. 1, Guangfu Rd, Sanchong District, New Taipei City, Taiwan (R.O.C)

TEL : +866-2-2278-2199 FAX: +886-2-8511-2492

Website: <http://www.motiodental.com> E-mail: [motion@twmotion.com](mailto:motion@twmotion.com)