


Ref. 080221

ECLIPSE I

ES.....	5	Manual original
EN.....	7	
FR.....	9	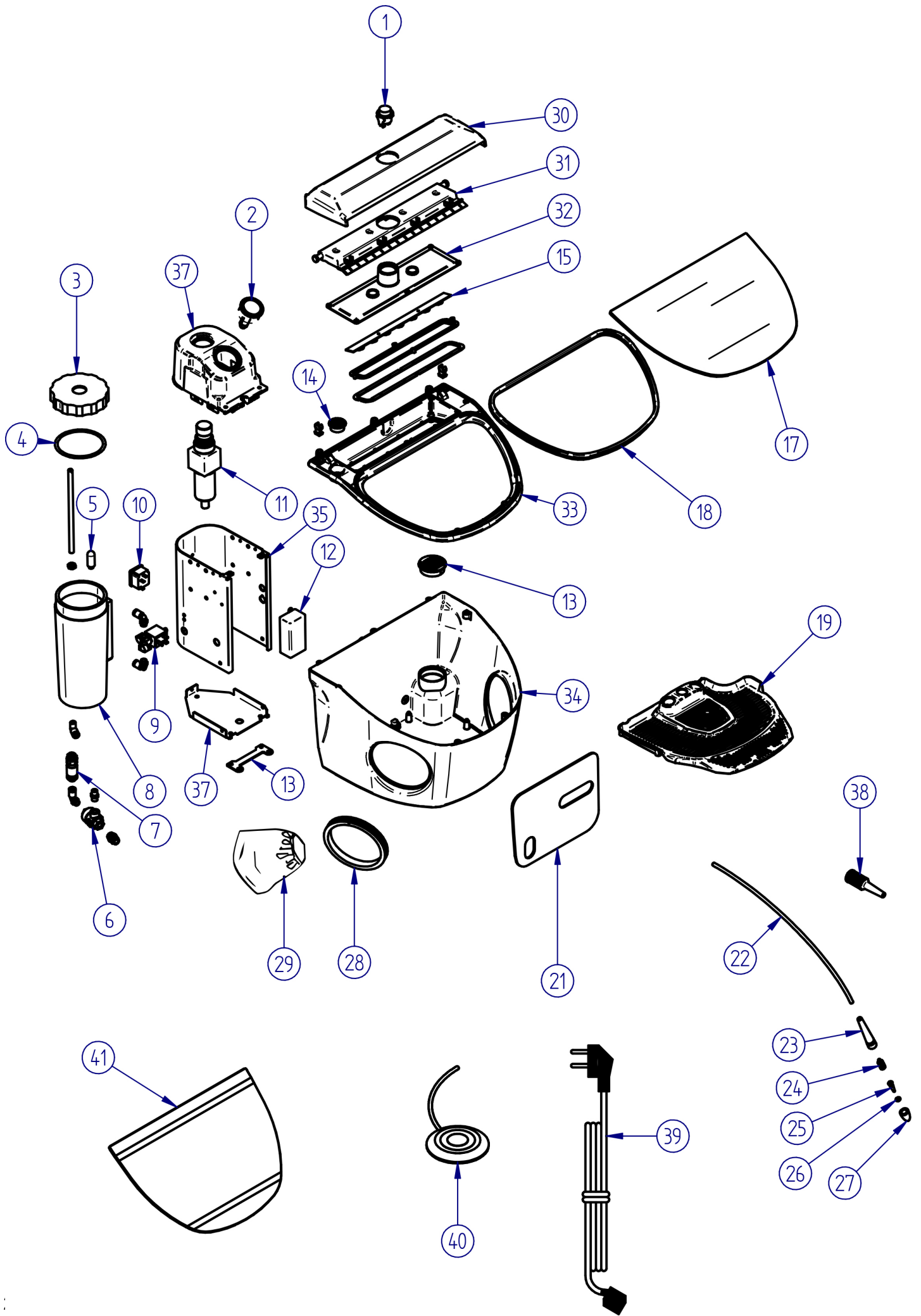

Nº	Ref.	CONCEPTO	DESCRIPTION
1	080233-02	Interruptor general	Main switch
2	080220-11	Manómetro	Pressure gauge
3	080262-24	Tapa tanque	Tank lid
4	080520-24	Junta tórica tapa	Lid O-ring
5	080262-13	Filtro de tanque	Tank filter
6	080242-35	Escape rápido	Quick escape
7	080259-12	Filtro en línea	Line filter
8	080262-14	Tanque	Tank
9	080253-10	Electroválvula 24V	Electrovalve 24 V
10	080490-06	Base de conexión	Conection base
11	080240-14	Manorreductor	Pressure control
12	080614-20	Fuente de alimentación 24V	Power source 24 V
13	080220-10	Rejilla ventilación grande	Large ventilation grill
14	080214-07	Rejilla ventilación pequeña	Small ventilation grill
15	080214-04	Barra de LED	LED bar
16	080214-05	Difusor	Difusser
17	080214-02	Cristal	Glass
18	080093-10	Burlete de goma	Rubber strip
19	080220-02	Bandeja	Tray
21	080220-13	Sufridera de goma	Rubber
22	000100	Tubo Ø4x6	Hose Ø4x6
23	080262-31	Cuerpo lapicero	Gun body
24	080262-32	Pieza intermedia lapicero	Gun intermediate part
25	080240-15	Boquilla 0,75 mm 50 micras	0,75 mm 50 micron nozzle
	080240-12	Boquilla 1,00 mm 150 micras	1,00 mm 150 micron nozzle
	080240-13	Boquilla 1,25 mm 250 micras	1,25 mm 250 micron nozzle
26	080262-28	Junta tórica	O-ring
27	080262-33	Punta de lapicero rojo	Gun point (red)
	080262-38	Punta de lapicero azul	Gun point (blue)
28	080270-30	Brida	Flange
29	080242-28	Manguito	Sleeve
30	080214-01B	Tapa cristal	Glass lid
31	080214-01C	Soporte cristal	Glass support
32	080214-01D	Soporte interruptor	Switch support
33	080214-01E	Marco cristal	Glass frame
34	080220-01	Cuerpo principal	Main body
35	080220-04	Chapa soporte superior	Upper support sheet
36	080220-05	Chapa soporte inferior	Lower support sheet
37	080220-03	Soporte manómetro	Pressure gauge support
38	050221	Soplador de aire	Air blower
39	000250	Cable 230 V	Cable 230 V
	000252	Cable 115 V	Cable 115 V
40	080242-33	Pedal	Pedal
41	080220-12	Protector cristal (10 U.)	Glass protector (10 units.)

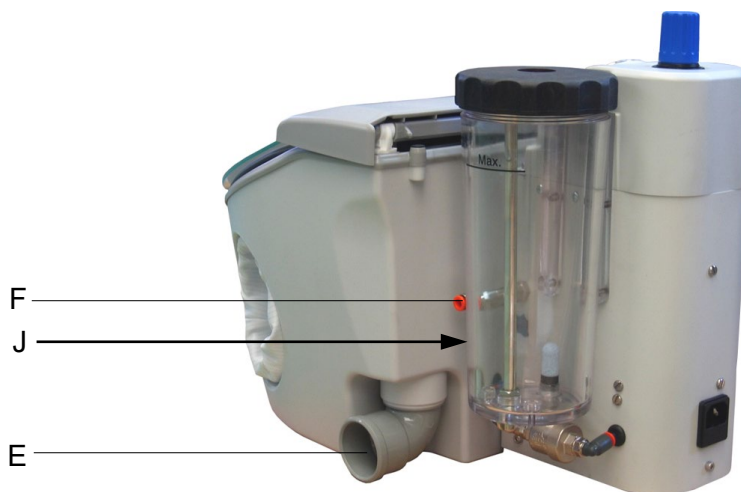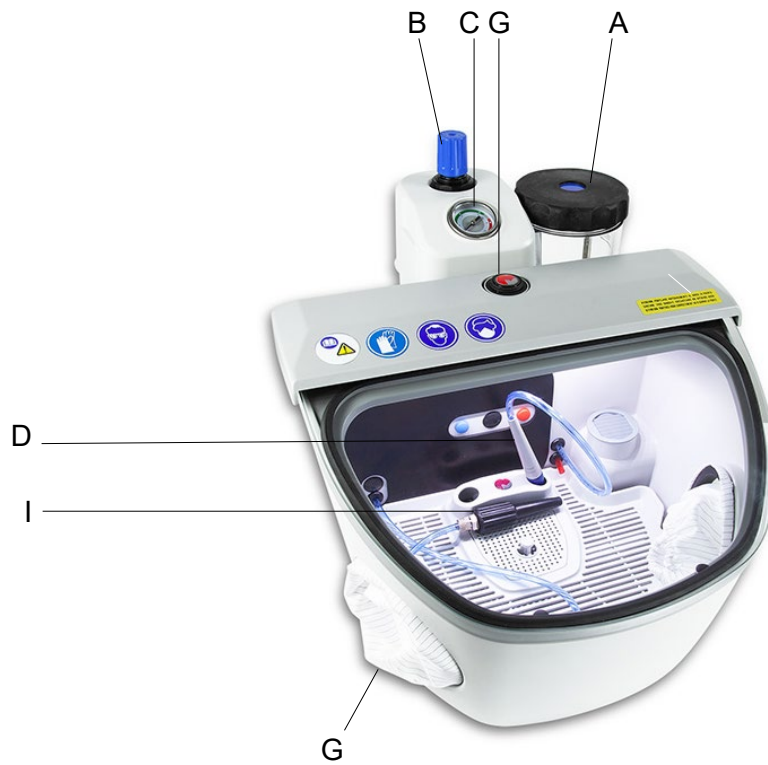A	Vaso depósito de esmeril / <b>Tank</b> / Réservoir de sables
B	Manorregulador vaso / <b>Pressure regulator</b> / Mano-régulateur
C	Manómetro de presión / <b>Pressure gauge</b> / Manomètre de pression
D	Lápiz de microarenado / <b>Microblasting gun</b> / Crayon de sablage
E	Tubo de entrada aspiración exterior / <b>External suction inlet tube</b> / Tube d'entrée de l'aspiration
F	Entrada aire a presión / <b>Pressure air inlet</b> / Entrée d'air sous pression
G	Interruptor luz / <b>Light switch</b> / Interrupteur de la lumière
H	Manguitos de tela / <b>Sleeves</b> / Manchons
I	Soplador de aire / <b>Air blower</b> / Soufflette
J	Conexión de pedal / <b>Pedal connection</b> / Connexion pédale

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Ignacio Mestraitua  
Director Gerente  
Sondika, 25.02.2025

The Sandblaster Eclipse is specially designed for precision-blasting small parts used in the dental prosthesis industry. For optimum output and performance of this unit, we advise you to read the following instructions with great care.

## Installing the unit

- Carefully unpack the unit.
- A rigid flat surface must be chosen for installation of the unit, at a safe distance from any sources of vibrations or heat, and the surrounding atmosphere must not contain excess moisture, since this could cause compacting of the emery particles inside the chamber, and lead to obstructions in the microblasting nozzles. The unit must also be positioned on a level which allows objects inside the chamber to be handled with ease.
- Hook up the air connection tube supplied with the unit to the connection at the rear of the unit. The other end must be fitted to a compressed air facility, with at least 6 bars of pressure and minimum flow rate 100 litres/minute.
- Hook up the electric plug to 115-230 V, 50/60 Hz, which must be earthed.
- Connect the pedal drive tube to the coupling nozzle on the right-hand side of the unit. Then position the pedal for comfortable foot operation.
- Unscrew the sandblasting module covers (A), and fill the tanks with emery. During this operation, **it is most important not to fill the tanks to more than ¾ of their capacity.** Put the covers back on, and ensure they have been screwed on tightly.
- The machine is supplied with one nozzle: Ø 1 mm (up to 150 microns).

## Regulating the pressure

The blowing pressure of the blasting guns can be regulated depending on the nature and characteristics of the operation to be carried out. Use the pressure-adjustment knob (B) on the top of the unit – pull the knob upwards, and rotate it to the required pressure. The unit has a pressure gauge (C) for easy selection of operating pressure. When the pressure has been adjusted, lock out the regulator knob by pushing it down to the initial position.

## Connection to the suction unit

The microblasting chamber must be permanently connected to forced suction via an outgoing tube on the top left of the unit (G). Suction must be able to produce a slight depression inside the chamber for continuous gentle circulation of the powder generated towards the outgoing tube.

Depending on the requirements and geometry of the external suction unit, it is possible to use the PVC elbow fitting supplied for specific needs.

## Using the unit

The following operations are recommended for proper precision-blasting:

1. Open the lid of the chamber, and put in the part you wish to blast. Parts may also be placed inside the chamber via sleeve (F). Check there is enough emery in the tanks.
2. Close the lid of the chamber, and ensure it is tightly shut (the glass should fit snug against the housing).
3. Suitable protection gear must be used: goggles, gauntlets and a mask.
4. Start up the forced suction facility. The light in the chamber may be activated for a better view of the part, if required, using switch (D).
5. Use the tank switch (E) to select the tank. The colour coding system used makes it easy to see which is the active tank at all times.
6. Pick up the mobile gun selected inside the chamber. The colour of the gun's nozzle-holder matches the colour on the tank covers (red, blue, yellow, green). When the gun has been selected, press the drive pedal to begin sandblasting.
7. Direct the blast carefully to prevent the stream hitting the sides of the chamber directly, since this could damage the chamber. If you wish to test the blasting pressure or strength, direct the stream at the rubber dolly inside the chamber.
8. After the blasting operation, release the pedal, stopping the sand flow and activating immediate air discharge to relieve pressure in the tanks. Then switch off the suction unit, open the lid of the chamber, and take out the part.

## Maintenance and cleaning

To ensure a lengthy service life, we recommend the following:

- Always use approved quality emery - **Talleres Mestraitua, S.L.** recommends **MESTRA** emery. The mobile blasting guns with nozzle diameters 1.25 mm and 1 mm can respectively blow emery of maximum diameter 250 and 150 microns. The mobile blasting gun with nozzle diameter 0.75 mm can blow a maximum diameter of 75 microns (see the table).
- It is preferable to use dry-piston air compressor units, since oil in the air could cause the sand to compact and obstruct the gun nozzles.
- Never fill the modules with emery deposited on the floor of the chamber, since particles with large diameters could obstruct the nozzles, and this emery is now contaminated with particles eroded off the parts blasted.
- The silencer filters inside the tanks must be replaced every 5000 or 10000 operations (around 1 or 2 years). The incoming filter fittings on each tank must be replaced every 500 or 1000 hours (much depends on the quality and metrage of the sand used). Original MESTRA spare parts must always be used.
- To prevent any abrasion wear on the walls inside the chamber, take care not to direct the stream of emery directly onto the walls.
- Clean out the emery inside the chamber on a periodic basis, using the practical emptying plug inside.
- Any water which has condensed inside the incoming air filter must be bled out regularly. The chamber has a window to view the amount of water in the filter.
- Use a dry cloth or a cloth moistened with soapy water for general cleaning of the unit.
- If the guns become obstructed, remove the nozzle-holder and take out the nozzle. Unblock the nozzle by blowing in the opposite direction to the flow of air, or use a pin or a thin wire to unblock it, and then refit the attachment.
- The blasting module guns on the 2-tank Eclipse model are supplied ex-works with nozzles 1 and 1.25 mm in diameter. 0.75 nozzles, however, may also be used with this model (consult your **MESTRA** distributor).
- The maximum diameter of emery particles in relation to nozzle diameter is shown in the table below:

MAXIMUM PARTICLE DIAMETER / NOZZLE	
Nozzle	Diameter
1.25 mm	Up to 250 microns
1 mm	Up to 150 microns
0.75 mm	Up to 75 microns

## Precautionary measures

- Never exceed 6 bars of pressure. Optimum working pressure is between 2 and 2 bars.
- Operations should always be carried out with the lid of the chamber closed, with suitable protection. Masks and goggles should be worn.
- Connect the chamber's outgoing tube to a suction facility.
- The protection sleeves (F) must be replaced immediately following any tearing or damage.
- Do not allow children or unqualified staff to operate the unit.
- Use only guaranteed-quality materials as emery (we recommend **MESTRA** emery). Poor quality emery may be toxic when inhaled, and may also cause obstructions in the blow nozzles.
- Hook up the power supply cable to 115-230 V, 50/60 Hz, earthed.
- Consult your fitter immediately if any leakages are observed in the compressed air circuit.
- Before carrying out any operation, first ensure that the micro blasting module covers are closed tightly.

## Technical characteristics

Height:	.....330 mm
Width:	.....360 mm
Length:	.....450 mm
Weight:	.....5 kg
Working pressure:	..... 2-6 bar
Voltage:	..... 115-230 V, 50/60 Hz
Power consumption:	..... 7 W
Air connection:	.....Ø6 mm

## Declaration of conformity UE

The firm **Talleres Mestraitua S.L.**  
with social address: Txori-erri Etorbidea, 60  
48150 Sondika - Bilbao - Spain

Hereby declares that the product

Sandblaster Eclipse I, Ref. 080221

Meets the requirements set out in the following directives:

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**GUARANTEE**

All **MESTRA®** units are guaranteed against every defect in manufacturing for a period of one year from date of purchase. Prior to honouring a guarantee, **MESTRA®** will require that the attached coupon be filled in with date of sale, duly stamped by the distributor.

The guarantee does not cover breakdowns caused by incorrect installation or use.

**MESTRA®** will not effect any repair outside its own workshops and will require transport costs.

Repairs carried out on the premises of the customer, be they with or without guarantee, are entrusted to the technical assistance of the distributor who has sold the unit or alternatively to the technicians he has contracted

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