


Ref. 080221

**ECLIPSE I**

|         |   |                 |
|---------|---|-----------------|
| ES..... | 5 | Manual original |
| EN..... | 7 |                 |
| FR..... | 9 |                 |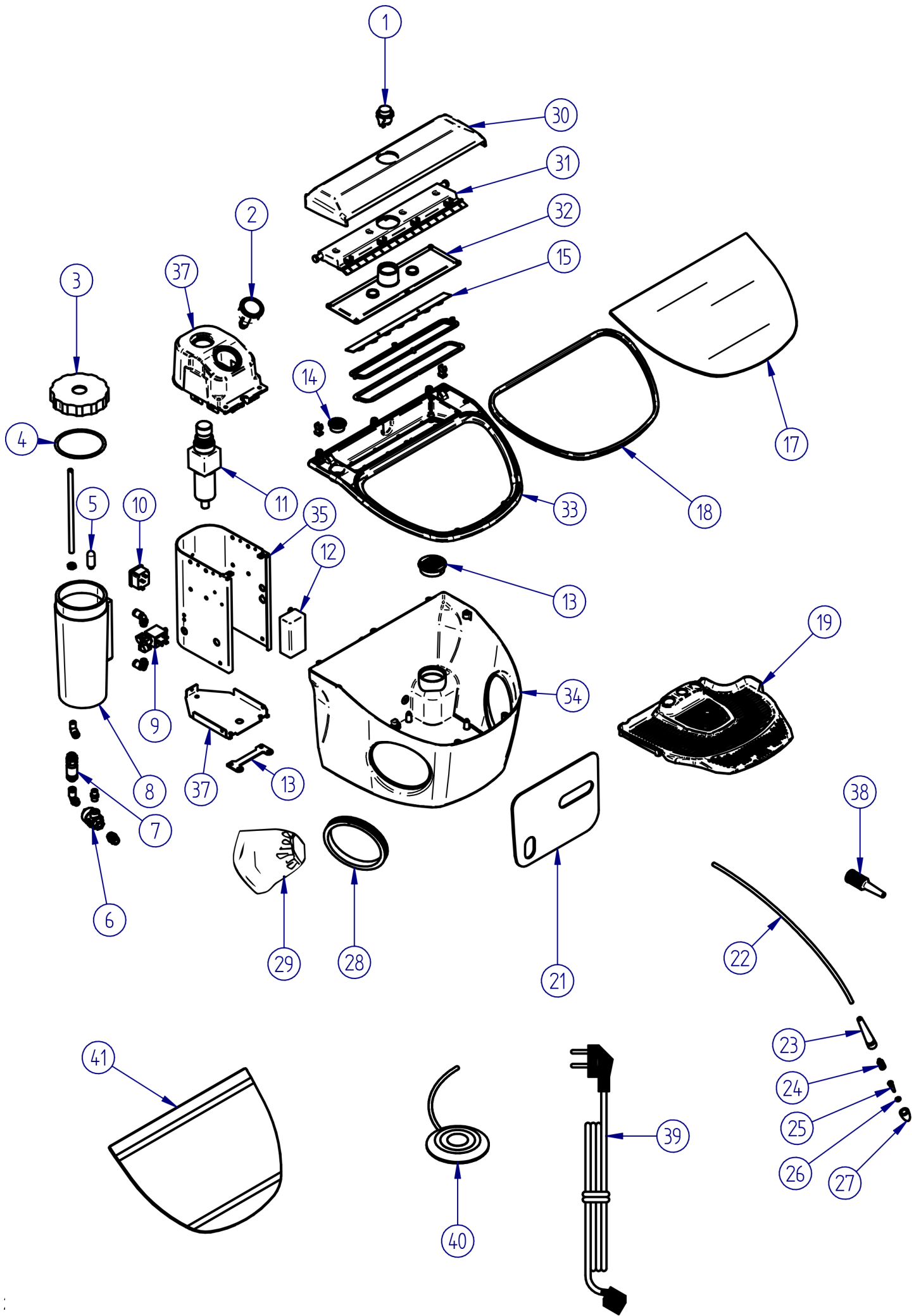

| Nº | Ref.       | CONCEPTO                    | DESCRIPTION                 |
|----|------------|-----------------------------|-----------------------------|
| 1  | 080233-02  | Interruptor general         | Main switch                 |
| 2  | 080220-11  | Manómetro                   | Pressure gauge              |
| 3  | 080262-24  | Tapa tanque                 | Tank lid                    |
| 4  | 080520-24  | Junta tórica tapa           | Lid O-ring                  |
| 5  | 080262-13  | Filtro de tanque            | Tank filter                 |
| 6  | 080242-35  | Escape rápido               | Quick escape                |
| 7  | 080259-12  | Filtro en línea             | Line filter                 |
| 8  | 080262-14  | Tanque                      | Tank                        |
| 9  | 080253-10  | Electroválvula 24V          | Electrovalve 24 V           |
| 10 | 080490-06  | Base de conexión            | Conection base              |
| 11 | 080240-14  | Manorreductor               | Pressure control            |
| 12 | 080614-20  | Fuente de alimentación 24V  | Power source 24 V           |
| 13 | 080220-10  | Rejilla ventilación grande  | Large ventilation grill     |
| 14 | 080214-07  | Rejilla ventilación pequeña | Small ventilation grill     |
| 15 | 080214-04  | Barra de LED                | LED bar                     |
| 16 | 080214-05  | Difusor                     | Difusser                    |
| 17 | 080214-02  | Cristal                     | Glass                       |
| 18 | 080093-10  | Burlete de goma             | Rubber strip                |
| 19 | 080220-02  | Bandeja                     | Tray                        |
| 21 | 080220-13  | Sufridera de goma           | Rubber                      |
| 22 | 000100     | Tubo Ø4x6                   | Hose Ø4x6                   |
| 23 | 080262-31  | Cuerpo lapicero             | Gun body                    |
| 24 | 080262-32  | Pieza intermedia lapicero   | Gun intermediate part       |
| 25 | 080240-15  | Boquilla 0,75 mm 50 micras  | 0,75 mm 50 micron nozzle    |
|    | 080240-12  | Boquilla 1,00 mm 150 micras | 1,00 mm 150 micron nozzle   |
|    | 080240-13  | Boquilla 1,25 mm 250 micras | 1,25 mm 250 micron nozzle   |
| 26 | 080262-28  | Junta tórica                | O-ring                      |
| 27 | 080262-33  | Punta de lapicero rojo      | Gun point (red)             |
|    | 080262-38  | Punta de lapicero azul      | Gun point (blue)            |
| 28 | 080270-30  | Brida                       | Flange                      |
| 29 | 080242-28  | Manguito                    | Sleeve                      |
| 30 | 080214-01B | Tapa cristal                | Glass lid                   |
| 31 | 080214-01C | Soporte cristal             | Glass support               |
| 32 | 080214-01D | Soporte interruptor         | Switch support              |
| 33 | 080214-01E | Marco cristal               | Glass frame                 |
| 34 | 080220-01  | Cuerpo principal            | Main body                   |
| 35 | 080220-04  | Chapa soporte superior      | Upper support sheet         |
| 36 | 080220-05  | Chapa soporte inferior      | Lower support sheet         |
| 37 | 080220-03  | Soporte manómetro           | Pressure gauge support      |
| 38 | 050221     | Soplador de aire            | Air blower                  |
| 39 | 000250     | Cable 230 V                 | Cable 230 V                 |
|    | 000252     | Cable 115 V                 | Cable 115 V                 |
| 40 | 080242-33  | Pedal                       | Pedal                       |
| 41 | 080220-12  | Protector cristal (10 U.)   | Glass protector (10 units.) |

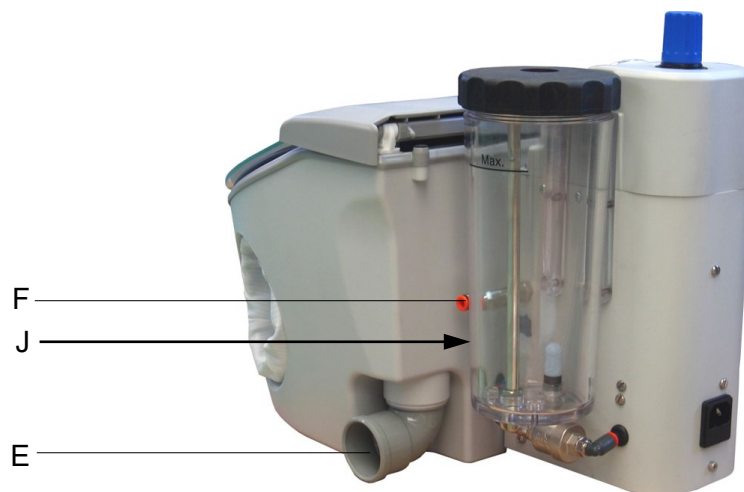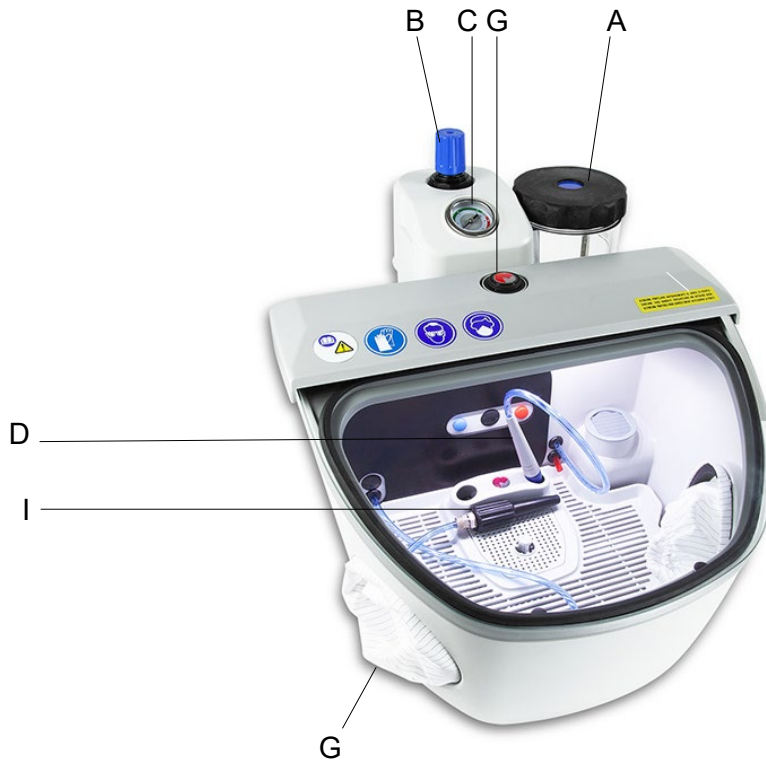|   |  |
|---|--|
| A | Vaso depósito de esmeril / <b>Tank</b> / Réservoir de sables   |
| B | Manorregulador vaso / <b>Pressure regulator</b> / Mano-régulateur  |
| C | Manómetro de presión / <b>Pressure gauge</b> / Manomètre de pression                                     |
| D | Lápiz de microarenado / <b>Microblasting gun</b> / Crayon de sablage                                     |
| E | Tubo de entrada aspiración exterior / <b>External suction inlet tube</b> / Tube d'entrée de l'aspiration |
| F | Entrada aire a presión / <b>Pressure air inlet</b> / Entrée d'air sous pression                          |
| G | Interruptor luz / <b>Light switch</b> / Interrupteur de la lumière                                       |
| H | Manguitos de tela / <b>Sleeves</b> / Manchons  |
| I | Soplador de aire / <b>Air blower</b> / Soufflette  |
| J | Conexión de pedal / <b>Pedal connection</b> / Connexion pédale   |

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The Sandblaster Eclipse is specially designed for precision-blasting small parts used in the dental prosthesis industry. For optimum output and performance of this unit, we advise you to read the following instructions with great care.

## Installing the unit

- Carefully unpack the unit.
- A rigid flat surface must be chosen for installation of the unit, at a safe distance from any sources of vibrations or heat, and the surrounding atmosphere must not contain excess moisture, since this could cause compacting of the emery particles inside the chamber, and lead to obstructions in the microblasting nozzles. The unit must also be positioned on a level which allows objects inside the chamber to be handled with ease.
- Hook up the air connection tube supplied with the unit to the connection at the rear of the unit. The other end must be fitted to a compressed air facility, with at least 6 bars of pressure and minimum flow rate 100 litres/minute.
- Hook up the electric plug to 115-230 V, 50/60 Hz, which must be earthed.
- Connect the pedal drive tube to the coupling nozzle on the right-hand side of the unit. Then position the pedal for comfortable foot operation.
- Unscrew the sandblasting module covers (A), and fill the tanks with emery. During this operation, **it is most important not to fill the tanks to more than ¾ of their capacity.** Put the covers back on, and ensure they have been screwed on tightly.
- The machine is supplied with one nozzle: Ø 1 mm (up to 150 microns).

## Regulating the pressure

The blowing pressure of the blasting guns can be regulated depending on the nature and characteristics of the operation to be carried out. Use the pressure-adjustment knob (B) on the top of the unit – pull the knob upwards, and rotate it to the required pressure. The unit has a pressure gauge (C) for easy selection of operating pressure. When the pressure has been adjusted, lock out the regulator knob by pushing it down to the initial position.

## Connection to the suction unit

The microblasting chamber must be permanently connected to forced suction via an outgoing tube on the top left of the unit (G). Suction must be able to produce a slight depression inside the chamber for continuous gentle circulation of the powder generated towards the outgoing tube.

Depending on the requirements and geometry of the external suction unit, it is possible to use the PVC elbow fitting supplied for specific needs.

## Using the unit

The following operations are recommended for proper precision-blasting:

1. Open the lid of the chamber, and put in the part you wish to blast. Parts may also be placed inside the chamber via sleeve (F). Check there is enough emery in the tanks.
2. Close the lid of the chamber, and ensure it is tightly shut (the glass should fit snug against the housing).
3. Suitable protection gear must be used: goggles, gauntlets and a mask.
4. Start up the forced suction facility. The light in the chamber may be activated for a better view of the part, if required, using switch (D).
5. Use the tank switch (E) to select the tank. The colour coding system used makes it easy to see which is the active tank at all times.
6. Pick up the mobile gun selected inside the chamber. The colour of the gun's nozzle-holder matches the colour on the tank covers (red, blue, yellow, green). When the gun has been selected, press the drive pedal to begin sandblasting.
7. Direct the blast carefully to prevent the stream hitting the sides of the chamber directly, since this could damage the chamber. If you wish to test the blasting pressure or strength, direct the stream at the rubber dolly inside the chamber.
8. After the blasting operation, release the pedal, stopping the sand flow and activating immediate air discharge to relieve pressure in the tanks. Then switch off the suction unit, open the lid of the chamber, and take out the part.

## Maintenance and cleaning

To ensure a lengthy service life, we recommend the following:

- Always use approved quality emery - **Talleres Mestraitua, S.L.** recommends **MESTRA** emery. The mobile blasting guns with nozzle diameters 1.25 mm and 1 mm can respectively blow emery of maximum diameter 250 and 150 microns. The mobile blasting gun with nozzle diameter 0.75 mm can blow a maximum diameter of 75 microns (see the table).
- It is preferable to use dry-piston air compressor units, since oil in the air could cause the sand to compact and obstruct the gun nozzles.
- Never fill the modules with emery deposited on the floor of the chamber, since particles with large diameters could obstruct the nozzles, and this emery is now contaminated with particles eroded off the parts blasted.
- The silencer filters inside the tanks must be replaced every 5000 or 10000 operations (around 1 or 2 years). The incoming filter fittings on each tank must be replaced every 500 or 1000 hours (much depends on the quality and metrage of the sand used). Original MESTRA spare parts must always be used.
- To prevent any abrasion wear on the walls inside the chamber, take care not to direct the stream of emery directly onto the walls.
- Clean out the emery inside the chamber on a periodic basis, using the practical emptying plug inside.
- Any water which has condensed inside the incoming air filter must be bled out regularly. The chamber has a window to view the amount of water in the filter.
- Use a dry cloth or a cloth moistened with soapy water for general cleaning of the unit.
- If the guns become obstructed, remove the nozzle-holder and take out the nozzle. Unblock the nozzle by blowing in the opposite direction to the flow of air, or use a pin or a thin wire to unblock it, and then refit the attachment.
- The blasting module guns on the 2-tank Eclipse model are supplied ex-works with nozzles 1 and 1.25 mm in diameter. 0.75 nozzles, however, may also be used with this model (consult your **MESTRA** distributor).
- The maximum diameter of emery particles in relation to nozzle diameter is shown in the table below:

| MAXIMUM PARTICLE DIAMETER / NOZZLE |                   |
|------------------------------------|-------------------|
| Nozzle                             | Diameter          |
| 1.25 mm                            | Up to 250 microns |
| 1 mm                               | Up to 150 microns |
| 0.75 mm                            | Up to 75 microns  |

## Precautionary measures

- Never exceed 6 bars of pressure. Optimum working pressure is between 2 and 2 bars.
- Operations should always be carried out with the lid of the chamber closed, with suitable protection. Masks and goggles should be worn.
- Connect the chamber's outgoing tube to a suction facility.
- The protection sleeves (F) must be replaced immediately following any tearing or damage.
- Do not allow children or unqualified staff to operate the unit.
- Use only guaranteed-quality materials as emery (we recommend **MESTRA** emery). Poor quality emery may be toxic when inhaled, and may also cause obstructions in the blow nozzles.
- Hook up the power supply cable to 115-230 V, 50/60 Hz, earthed.
- Consult your fitter immediately if any leakages are observed in the compressed air circuit.
- Before carrying out any operation, first ensure that the micro blasting module covers are closed tightly.

## Technical characteristics

|                          |                     |
|--------------------------|---------------------|
| Height: .....            | 330 mm              |
| Width: .....             | 360 mm              |
| Length:.....             | 450 mm              |
| Weight: .....            | 5 kg                |
| Working pressure: .....  | 2-6 bar             |
| Voltage: .....           | 115-230 V, 50/60 Hz |
| Power consumption: ..... | 7 W                 |
| Air connection: .....    | Ø6 mm               |

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