Ref. 100037

ES	LÁMPARA POLIMERIZADORA - LED	3
EN	CURING LIGHT - LED	5
FR	LAMPE A PHOTO-POLYMERISER - LED	7

MESTRA®


TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60


Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

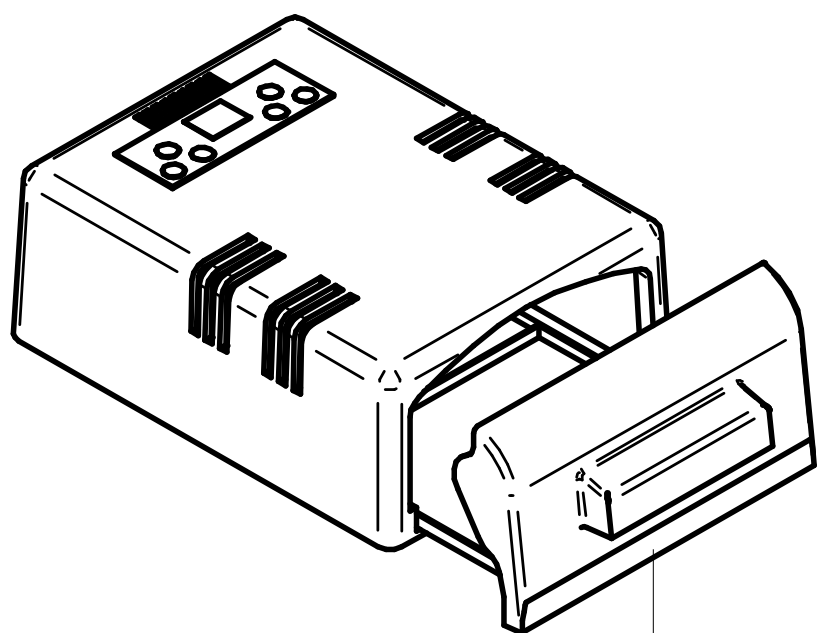Cajón
Drawer
Plateau tiroir

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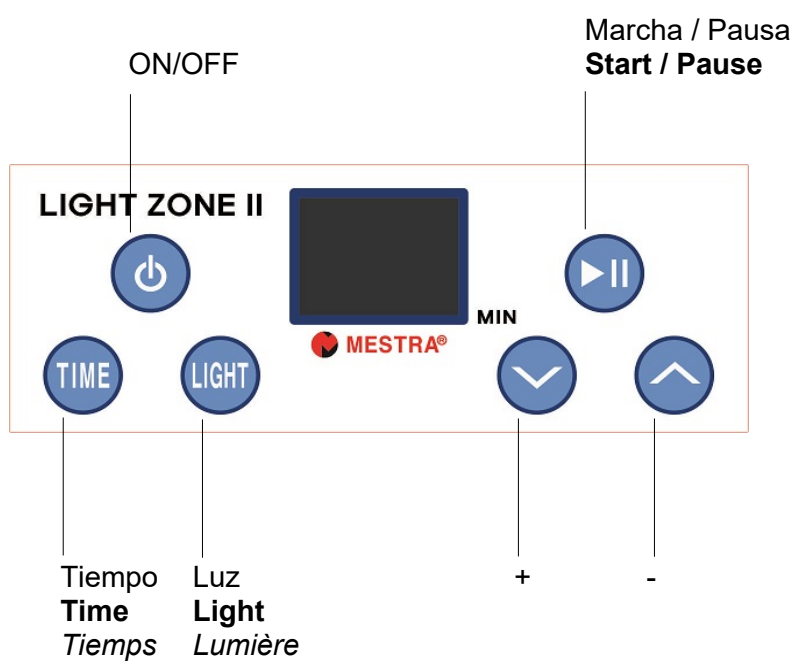9

Apague la máquina.

The Ref. 100035 Curing Light –LED is used for fast effective curing of photo-polymerisable materials commonly used in the dental field. Its frequency range is 320 - 480 nanometers. Please read thoroughly the following instructions for correct use of this machine to obtain maximum output.

INSTALLATION

- Carefully unpack the Curing Light. Check that it has not been damaged in transit.
- Place the unit on a flat and firm surface well away from heat sources or vibrations.
- Connect the machine to a 100-240 V, 50/60 Hz power supply with earth.
- Ambiental conditions:
 - Indoor use only.
 - Temperature: 5-40 °C [41-104 °F]
 - Humidity: 0-80 %
 - Supply voltage fluctuation: < 10 % of the nominal value.
 - Pollution degree: 2

MAINTENANCE AND CLEANING

The machine should be cleaned periodically with a soft cloth. Before cleaning the machine, disconnect from the power supply.

TECHNICAL DATA

Height:	107 mm
Width:	200 mm
Depth:	285 mm
Weight:	1,2 kg
Emission range:	400-405 nm
Power supply:	100-240 V, 50/60 Hz
Power:	65 W

PRECAUTIONS

-  - Read these instructions carefully before operating the unit. They contain very important information about safety during the installation, operation and maintenance of the unit.
- Keep these instructions for further consultation.
- Check out the state of the unit upon unpacking it.
- Before connecting the device, make sure the voltage in the identification plate matches the power source to be used. During the installation all the safety requirements must be observed and the

instructions of the manufacturer followed.

- The manufacturer will not be liable for the damage caused by an inappropriate grounding system.
- Damage caused by a faulty installation or a wrong use of the unit is not covered by the warranty.
- Do not use the polymerising lamp for uses other than the polymerization of dental materials.
- When in doubt, do not start the unit. Contact the supplier for questions or detailed information about problems with the unit.

- Always work with the polymerization tray closed and avoid that the light emitted by the lamp hits your eyes. The radiation is of the U.V. type, which can be harmful to the eyesight.
- Do not look directly into the product during the operation of the product.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 100-240 V, 50/60 Hz).

- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents...).
- Do not use a damaged or broken power cord or a loose plug.
- Do not block the ventilation grilles of the appliance.

- Do not allow children or non-qualified personnel handling the unit.
- Do not dismount, repair or modify the unit.

- Contact the supplier for questions or detailed information about problems with the unit.


OPERATION

1


Switch the unit on.

2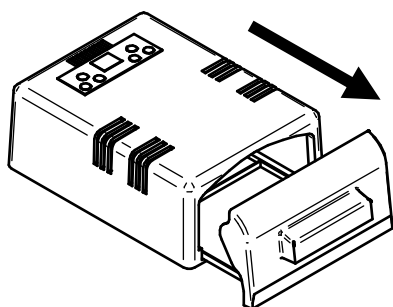


Open the drawer

3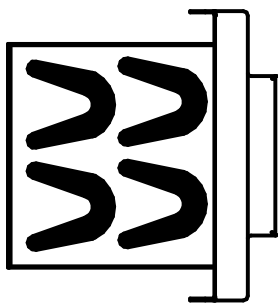

Place the objects in the tray.

4

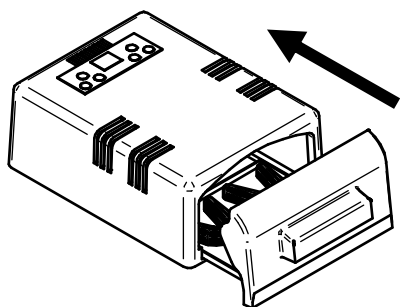Close the drawer.

5

Select the curing time (from 1 to 10 minutes).

6

Select the light intensity (from 1 to 10).


7

The lamp will light up inside.

When the time ends, you will hear an acoustic signal.

8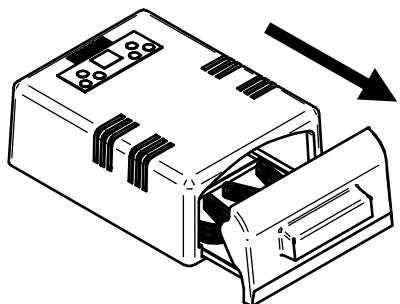

Open the drawer and remove the cured elements.

Lamp emits UV radiation which may be harmful to eyesight. If you have to open the tray while the machine is working, disconnect it before.

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