Ref. 100093

ES.....	2	Manual original
EN.....	4	
FR.....	6	

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Through-the-lens (TTL) loupes do not have a flip-up mechanism. They are rigidly attached to the eyeglass lens. Consequently, they are lighter, and the depth of field and field of view are greater and wider than with “flip-up” systems.

Thanks to our revolutionary new design, we can now offer these loupes with four different interpupillary distances: SS, S, R, and L. Choosing the correct interpupillary distance frame is easy. Simply insert the appropriate protective lenses into the frame and then screw on the loupes. With this simple operation, the Galilean loupes are ready for use.

- Flexible, padded, and anti-deformation temples.
- Adjustable nose pad.
- Brighter images.
- Wide field of view.
- Increased depth of field.
- No pupillary adjustment required.
- Ultra-lightweight.
- 3.0X magnification.

## Package Contents

- Galilean TTL Dental Loupe (ultra-light 3.0X).
- Protective case.
- SS/S/R/L protective lenses.
- Cloth bag.
- Cleaning cloth.
- Headband.
- Interpupillary distance ruler.

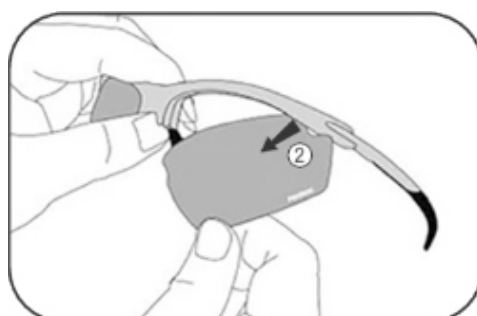
## Quick change

### TO REMOVE THE PROTECTIVE LENS


Hold the frame as shown in the image above (1). Then, grasp the center of the frame with your thumb and pull it down.

Finally, remove it laterally as shown in image (2).

### INSTALLING THE PROTECTIVE LENS

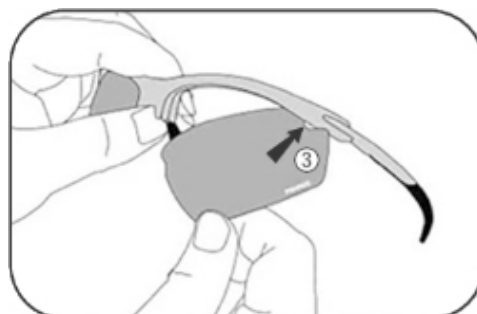Insert the edge portion of the lens as shown in image (3), ensuring it fits into the frame slot.

Then, use your thumb to hold the other corner of the lens and insert it into the slot on the opposite corner, making sure you hear a "click" sound to confirm it is in the correct position (4).

## How to measure interpupillary distance

1. Both the examiner and the person being examined should sit facing each other at a distance of 40 cm. They should also maintain eye level.
2. The examiner, using the thumb and index finger of their right hand, holds the interpupillary distance ruler. The other fingers rest on the face of the person being examined. Position the interpupillary distance ruler at the lowest point of the vertical line from the nose.
3. The examiner then closes their right eye and has the person being examined look with both eyes at the examiner's left eye.
4. The examiner, using their left eye, looks at the zero point of the DI meter to aim at the center of the pupil of the person being evaluated's right eye.
5. The examiner then opens their right eye and allows the person being evaluated to look into their left eye. The examiner then uses their right eye to read the measurement of the center of the pupil of the person being evaluated's left eye.
6. The process is completed by repeating steps (3) - (4) three times. An average of the three measurements should be taken to obtain the interpupillary distance (IPD) value.

## How to choose protective eyewear with the correct IPD

1. Once you know your IPD, subtract 2-3 mm to obtain the correct IPD value. Choose one of the following protective eyewear sizes: SS ( $56 \pm 1$  mm), S ( $58 \pm 1$  mm), R ( $60 \pm 1$  mm), L ( $64 \pm 1$  mm). These will be the appropriate IPDs for your IPD.
2. Next, insert the lenses (glasses) into the frame.
3. Screw the lens cylinder of the 3.0x magnifying glass onto the glasses. If needed, you can place one of the small rubber gaskets supplied on the neck of the cylinder for added stability and parallelism.

## Cleaning and caring for your magnifying glasses

Your magnifying glasses are precision instruments. You must take special care when handling and cleaning them.

- Moisten a soft cloth or cotton swab with isopropyl alcohol, soap, or mild disinfectant. Gently wipe the magnifying glasses to remove any loose particles.
- Dry the magnifying glasses with the included lens cloth.
- **Lens Cleaning:** Use an optical lens cleaner specifically formulated for cleaning coated lens surfaces.

### Important precautions

- Do not use harsh soaps or disinfectants.
- Do not immerse the magnifying glasses in water.

## Optional Wireless LED Front Light

Ref. 100094 Three different light intensities.

1. Snap-on front light + yellow filter.
2. Wireless front light controller (x2).
3. Power adapter.
4. Charging cable.

## Technical Specifications

Magnification:.....3X  
Working distance:.....  $420 \pm 20$  mm  
Depth of field:..... 75 mm  
Field of view:..... 70 mm  
Weight of goggles:..... 60 g  
Total weight:..... 415 g  
Packaging dimensions: ..... 220 x 155 x 95 mm

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