Ref. 100625

ES	2	Manual original
EN	5	
FR	8	

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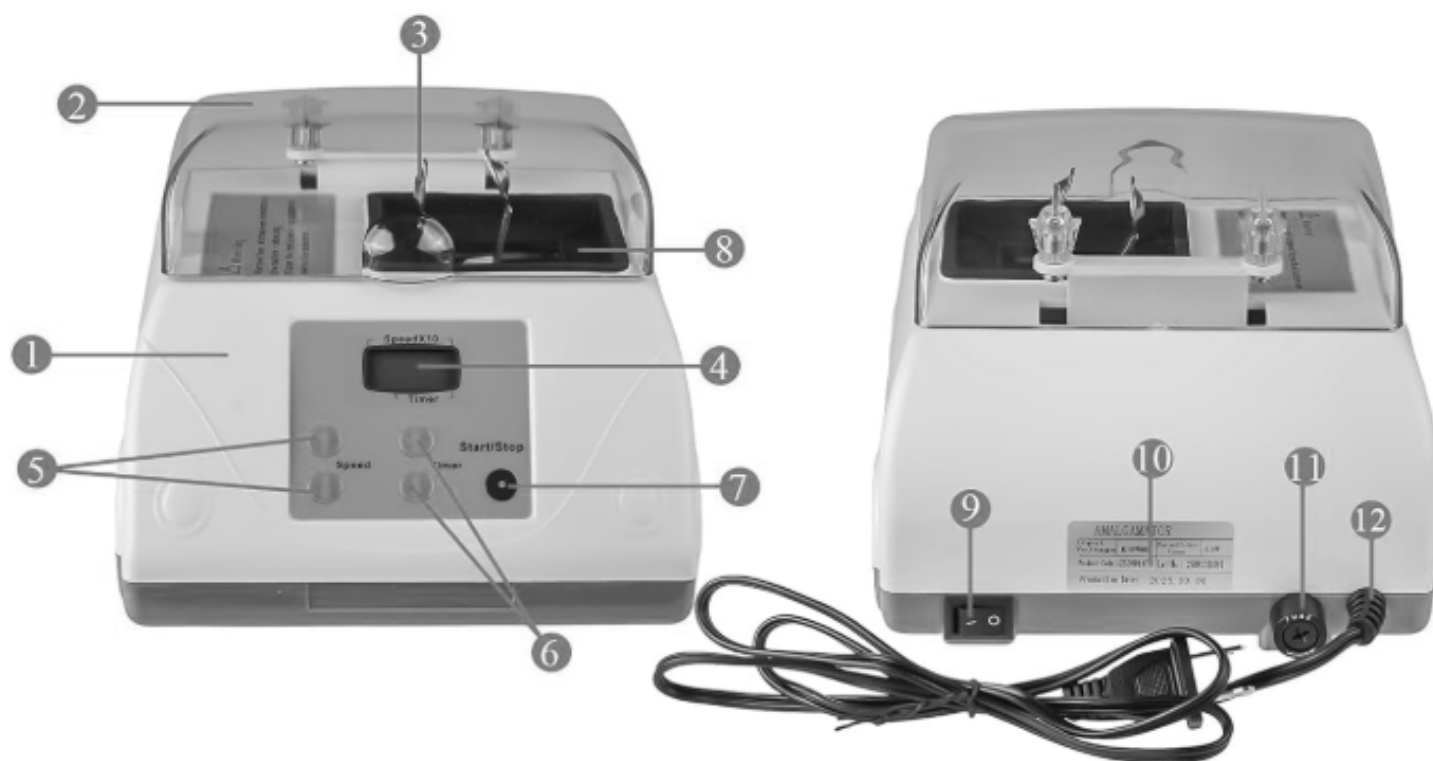
The capsule shaker Ref. 100625 is used in the dental sector to mix dental restoration materials, such as amalgam capsules, glass ionomer capsules, etc.

Please read this manual carefully. It contains important information regarding safe installation, use, and maintenance. Using the equipment in a manner not specified by the manufacturer may compromise its safety.

1. Housing
2. Lid
3. Capsule holder
4. Display
5. Speed buttons
6. Time buttons
7. Start/Stop button
8. Rubber tray

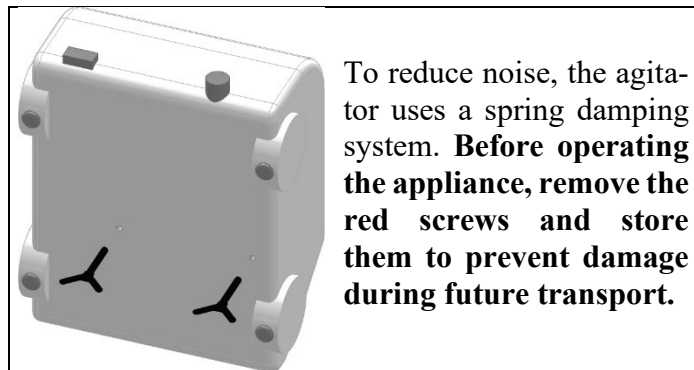
9. Main switch
10. Label
11. Fuse holder
12. Power cord

PRECAUTIONS

- Before connecting the device, make sure that it is a 220 V, 50 Hz outlet. It is also necessary that it be adequately protected with a thermal magnet. Consult your installer.
- Do not allow children or unqualified personnel to handle the appliance or get close to it when it is in operation.
- Do not leave the machine on unattended.
- Place the appliance on a rigid and flat surface, so that there is no risk of it tipping over.
- Disconnect the appliance from the electrical network before cleaning.
- Unplug the device from the power outlet when not in use.
- Do not operate the device if the power cord is damaged. Call your authorized dealer for replacement of any damaged parts.
- Handle the device with great care. It may be damaged due to a drop impact.
- Do not immerse the device in water or other liquids.
- Do not attempt to repair the appliance yourself. Do not disassemble it. For repair, take your device only to authorized dealers.
- Do not try to start the appliance with the lid open or open it during operation.
- Use the appliance exclusively for its intended use.

INSTALLATION

- Carefully unpack the appliance. You will find the following items:
 - 1 mixer;
 - 2 spare fuses;
 - 2 spare springs.
- Visually inspect the appliance, checking that it has not been damaged and that the lid opens and closes correctly.

To reduce noise, the agitator uses a spring damping system. **Before operating the appliance, remove the red screws and store them to prevent damage during future transport.**

- Choose a level, flat, stable, and rigid surface for the appliance, away from sources of heat or vibration.
- Before connecting the appliance, check that the nameplate indicates the correct voltage for the power supply.
- Connect the appliance to a 220 V, 50 Hz power outlet.

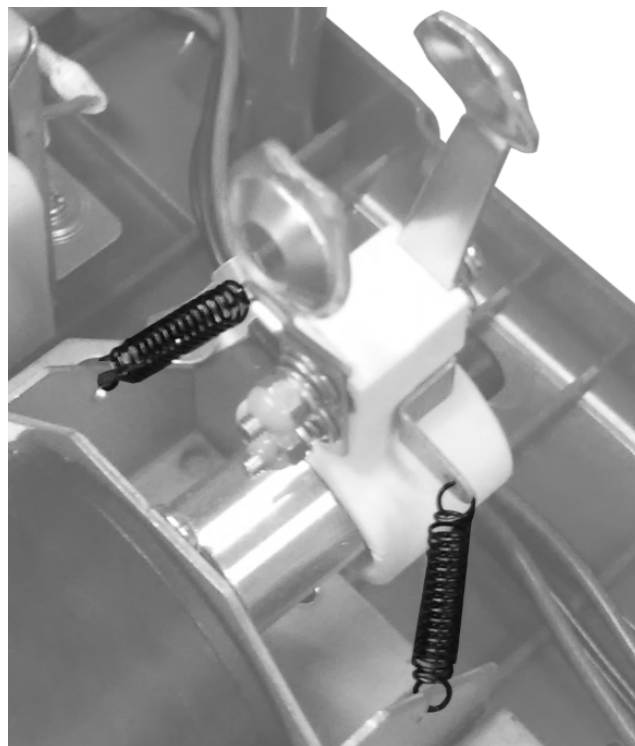
USAGE

1. Turn on the main switch located on the back of the machine. The display will light up.
2. Use the two Speed buttons to select the vibration speed.
3. Use the two Timer buttons to select the vibration time.
4. Place the capsule in the holder and close the door.
5. Press the Start/Stop button to begin vibration.
6. Turn off the device when you have finished using it.

MAINTENANCE

- Clean the appliance daily to maintain good hygiene.
- Never use solvents or chemicals that could damage the appliance.

- Periodically check the condition of the capsule holder and its springs. If they are broken or missing, replace them immediately.
- If the appliance does not work at all: Check if the fuse is blown. If so, replace it with a suitable F2A 5x20 fuse.
- If the blending result is not uniform:
 - Check if the speed and time are correct.
 - If the blend is so dry it resembles powder, the speed may be too slow or the time too short.
 - If the blend is too runny/dispersed, the speed may be too fast or the time too long.
 - Consider whether the capsules are expired or degraded. Try using new capsules.
- To reduce noise, the agitator uses a spring damping system. If these springs break or are lost, replace them.

TECHNICAL SPECIFICATIONS

Speed:	2800 - 5000 rpm
Timer:	3 - 180 "
Weight:	1.87 kg
Dimensions:	L210 x W175 x H150 mm
Fuse:	2 A
Power:	40 W
Supply:	220 V, 50 Hz
Noise:	≤ 65 dB(A)

WARRANTY CONDITIONS

Requirements for Warranty Validity

- All **MESTRA** machines are guaranteed against any manufacturing defects for one year from the date of delivery.
- Delivery is considered complete when the product is made available to the customer by the carrier.
- To process any warranty claim, it is essential to provide the purchase invoice.

Warranty Exclusions

The device has been designed and built for use by professionals in dental clinics, dental laboratories, or jewelry workshops. No other application is intended by the manufacturer, who consequently disclaims all liability for damages of any kind caused by improper use of the device.

Routine maintenance of each device, regardless of its frequency, is not covered by the warranty.

The following situations are excluded from the warranty:

- Parts subject to normal wear and tear.
- Damage caused by physical breakage, such as to casings, plastics, paint, enamel, glass, and similar materials.
- Damage caused by fire, flood, weather conditions, impact, crushing, inadequate ventilation, or application of incorrect voltage or power.
- Damage caused by misuse, specifically failure to follow the safety, installation, operating, handling, cleaning, maintenance, or setup instructions included with each appliance.
- Damage caused by the use of unsuitable or non-original accessories or spare parts.
- Appliances that do not have a factory serial number, or whose serial number has been altered or erased, or whose serial number does not match the one shown on the invoice.
- Appliances that have been modified, repaired, or tampered with by personnel not authorized by **MESTRA**.
- Normal wear and tear from use does not constitute a defect.
- **MESTRA** is not responsible for damage that an appliance may cause to external elements. **MESTRA** reserves the right to modify the design, construction, and appearance of its products without prior notice.
- All claims exceeding the scope of this warranty are excluded, particularly claims for compensation due to loss of profits, compensation for use, and indirect damages, unless such claims are mandatory under the law.

Damage caused by transport

- When transport is handled by **MESTRA**, the customer must inspect the product upon receipt. If any damage has occurred due to transport, the customer must immediately note it on the carrier's delivery note and notify **MESTRA**. Otherwise, **MESTRA** will not be able to make a claim against the carrier.

Repair conditions

MESTRA will repair or replace the machine as it deems appropriate, under the conditions stipulated below.

- Under this warranty, our company is only obligated to repair or replace the product or part recognized as defective and returned following the agreed procedure.
- During the repair or replacement period, no equivalent equipment will be supplied.
- The defective parts or components removed will become the property of **MESTRA**.

Repairs outside of Mestra's facilities

- **MESTRA** will not perform any repairs outside its workshop and is exempt from any transportation costs incurred.
- Repairs carried out at the customer's premises, whether or not covered by the warranty, are entrusted to the technical service department of the distributor who made the sale or to their contracted technicians.

Shipping conditions

- All product returns invoking the warranty must be notified and accepted in advance. For this purpose, the customer should contact our after-sales service.
- The defective product should preferably be returned in its original packaging or, failing that, securely packaged.
- The customer is responsible for all costs and risks associated with shipping the product, and shipping must always be done via courier to ensure the merchandise is insured against potential damage during transit.
- The equipment to be repaired must be accompanied by a note indicating the observed defect, the user's name, address, and telephone number. To expedite the process, a precise description of the defect is necessary.
- If the customer requests a replacement of a component under warranty, the warranty amount will be charged initially. The customer will then send the defective component to the factory for inspection and reimbursement.

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TALLERES MESTRAITUA S.L. is a company manufacturing tools and appliances for the dental prothesis sector, from the business address shown at the bottom of the page.

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I HEREBY STATE: That the **CAPSULE MIXER Ref. 100625**, with the following characteristics:

Velocidad / <i>Speed</i> :	2800 - 5000 rpm
Temporizador / <i>Timer</i> :	3 - 180 "
Peso / <i>Weight</i> :	1,87 kg
Dimensiones / <i>Dimensions</i> :	L210 x W175 x H150 mm
Fusible / <i>Fuse</i> :	2 A
Potencia / <i>Power</i> :	40 W
Alimentación / <i>Supply</i> :	220 V, 50 Hz
Ruido / <i>Noise</i> :	≤ 65 dB(A)

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Meets the low voltage requirements set out in Directive 2014/35/UE, in Directive 2014/30/UE concerning electromagnetic compatibility and in Directive 2006/42/EC concerning machinery safety.

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In witness whereof, the company issues this declaration of conformity in Sondika, on 10 July 2026.

Firma / *Signed*:

Ignacio Mestraitua,
Director Gerente / *Managing Director*

MESTRA®

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA

MESTRA®

Your trusted brand

Talleres Mestraitua S.L.

Txori-erri Etorbidea, 60

T: (+34)944530388 - F: (+34)944711725

mestra@mestra.es - www.mestra.es

48150 SONDIKA - BILBAO - ESPAÑA