Ref. 100505

ES.....	2	Manual original
EN.....	5	
FR.....	8	

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Continuous protection for clinical and laboratory instruments and work materials.

Important Note: This equipment is intended exclusively for the organized storage and environmental protection of work instruments. It is not a medical device and under no circumstances replaces regulatory sterilization methods (autoclaving or other systems approved by current regulations).

Please read this manual carefully. It contains important safety information regarding installation, use, and maintenance. If the equipment is used in a manner not specified by the manufacturer, the protection provided by the equipment may be impaired.

Features

- Removable trays/shelves.
- High-efficiency heating element.
- Door safety switch.
- Tempered glass door.
- Silicone door seal.

Installation

- Carefully unpack the device.
- Visually inspect the device, ensuring it has not been damaged by impacts and that the lid/door opens and closes correctly.
- Choose a horizontal, flat, stable, and rigid surface away from heat sources or vibrations for the location of the device.
- Leave a minimum clearance of 13 cm (5 inches) between the back of the device and the wall to ensure proper air circulation through the ventilation.
- Before connecting the device, verify that the rating plate indicates the correct voltage for the power supply.
- Place the two internal trays at the desired height.

Precautions

The unit is constructed with materials suitable for UV radiation and a safety mechanism to prevent exposure to such rays. The following instructions must be strictly followed to avoid hazards and health risks:

- Never look directly at the UV lamp during operation, whether directly or through a mirror. **Direct or excessive exposure to UV light can cause permanent eye damage.**
- The door panel must always remain closed during operation.
- Do not tamper with the automatic door shut-off safety mechanism or attempt to operate the unit with the panel door open.
- The device must be installed in a cool, dry place, away from any flammable or explosive materials. It must be operated maintaining a minimum distance of 13 cm (5 inches) from walls and other objects.
- Use an appropriate power outlet for the power cord. The use of a surge protector is also recommended.
- Do not come into contact with the surface of the heating area while the machine is in use. Also, keep all flammable and explosive materials away from the heating element. Do not place towels, footwear, plastics, or any other objects on the heating element.
- Do not allow children to touch or use the device.
- Do not operate the device if the plug, cord, or any other component is damaged or malfunctioning. In such cases, contact the manufacturer.
- Do not move, tilt, or disassemble the device while it is connected to the power supply.
- Open and close the device door carefully; if you find it difficult to open, hold the body of the machine firmly while doing so.
- Ensure that objects are dry before placing them in the machine for sterilization.
- Do not leave the door open for ventilation until 20 minutes have passed after the machine has completed its cycle.
- Keep the device away from moisture. If the machine gets wet, disconnect it from the power supply and allow it to dry completely.
- Always completely stop operation from the control panel before opening the door.
- Before use, ensure this product is connected to a dedicated, properly grounded power outlet suitable for use with this device.
- A short power cord is provided to reduce the risk of becoming entangled or tripping.
- Extension cords may be used if care is exercised in their use:
 - a) The cable gauge of the extension cord must be at least equal to that of the appliance's cord
 - b) The longer cord must be arranged so that it will not drape over the countertop or tabletop, where

it can be tripped over, snagged, or pulled unintentionally.

- Always unplug from the power outlet when not in use, before maintenance, cleaning, or replacing parts of the appliance. Always turn off the device before unplugging it.
- Do not operate this appliance if the cord or plug is damaged, or if it has come into contact with water.
- Do not let the cord hang over the edge of a table or counter, or touch hot surfaces.
- Never use the cord as a handle, nor pinch or pull it around sharp edges or corners. Do not run the equipment over the cord.
- To disconnect, grasp the plug and pull it from the power outlet. Never pull on the cord.
- Do not handle the plug or the device with wet hands.
- Always turn off all functions before unplugging the device.
- To protect against electric shock, do not immerse the cord, plugs, or the device in water or other liquids. Do not place or store the product where it can fall or be pulled into a tub or sink.
- Keep this device out of reach of children. Children should be supervised to ensure they do not play with the appliance.
- Do not use the appliance for other than its intended use.
- Never use accessories or parts from other manufacturers that are not specifically recommended for this device.
- For indoor use only.
- There are no user-serviceable parts inside the device. Do not attempt to open, repair, or service the device yourself. All maintenance must be performed by an authorized service representative. Failure to comply with this requirement will void any warranty, explicit or implied.
- Always place the device on a level, firm surface.
- Never move, tilt, or open the door of the device while in use. Always turn off and unplug the unit before moving it.
- Never place any object on top of the device.
- Do not use chlorine or harsh chemicals to clean the device, as this could cause damage to the unit. Always clean dry water spots inside the interior space after each use. Dry water spots can hinder the performance of the UV lights and cause damage to the stainless steel interior.

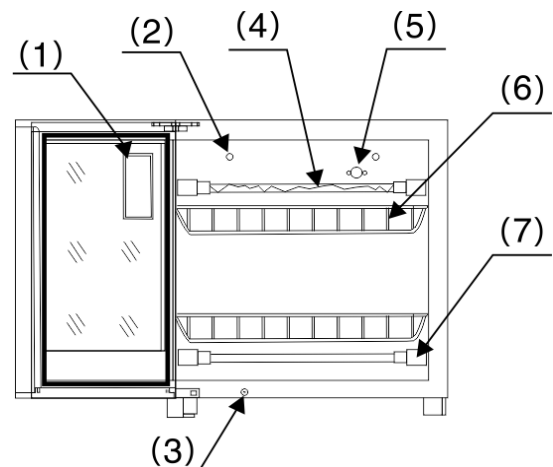
Daily usage tips

1. To eliminate any unusual odors or greasy residue, run the machine through 1 or 2 cycles before first use.
2. Do not place tableware/utensils in the cabinet unless completely dry.
3. Do not leave the device connected to the power supply for long periods. Disconnect the power when the machine is not in use. It is recommended to perform the sterilization process at least once every two days.
4. It is normal for minor noises and cracking sounds to occur as the machine heats up and cools down. If you notice loud noises, pungent odors, or smoke/steam coming from the machine, immediately disconnect the power and notify technical service.

Maintenance and cleaning


1. Disconnect the electrical power supply before cleaning the appliance.
2. You can use a soft cotton cloth or a sponge dampened with a neutral detergent to clean both the interior and exterior.
3. Do not use any products soaked in gasoline, polishes, or degreasing/disinfecting powders to clean the machine.
4. Do not spray water directly onto the appliance to prevent electrical damage.
5. Always clean dry water spots inside the interior space after each use. Dry water spots can hinder the performance of the UV lights and cause damage to the stainless steel interior.

Device diagram

1. Control panel
2. Exhaust vent / Air outlet
3. Door safety switch
4. UV lamp
5. Temperature sensor
6. Shelf / Tray
7. Heating element / Electrical resistor

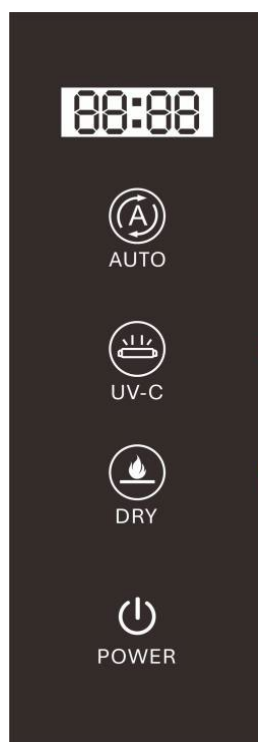Device operation – Control panel

- **AUTO:** When the machine is plugged in, pressing this button selects the AUTO (Automatic) function. The operating time is 60 minutes: the UV-C function operates for the first 15 minutes, followed by 45 minutes of the DRYING function (60~70 °C). The display will show the timer, and the machine will automatically turn off when the countdown finishes.
- **UV-C:** When the machine is plugged in, pressing this button selects the UV-C function. You can select an operating time of 15, 30, 90, or 120 minutes. The display will show the timer, and the machine will automatically turn off when the countdown finishes.
- **DRY:** When the machine is plugged in, pressing this button selects the **DRYING** function (60~70 °C). You can select an operating time of 30 or 45 minutes. The display will show the timer, and the machine will automatically turn off when the countdown finishes.
- **POWER:** Pressing this button will turn the machine on or off.

Technical specifications

Funciones:	UV + HEAT
Capacity:	38 L
Internal temperature:	60~75 °C
Safety features:	The UV lamp automatically turns off if the door is opened during sterilization.
Rated power:	460 W
Rated voltage:	220 V~
Material:	Stainless steel
Product dimensions:	440 × 365 × 405 mm
Net product weight:	6 kg
Packaging dimensions:	520 × 440 × 480 mm
Gross weight:	7 kg

Troubleshooting

Problem	Possible cause	Solution
No power	Power outage or the power cord is not plugged into the outlet.	Check the power supply and the plug. Verify the outlet is working by plugging in another functional device (e.g., a lamp).
	Circuit failure.	Contact the device distributor.
Strange odor	The inside of the cabinet is dirty.	Clean the interior of the cabinet.
	Evaporation of protective oils or grease.	Run the device for 1 to 2 cycles before first use.
UV light does not work	UV element (lamp) failure.	UV element (lamp) failure.
	Circuit failure.	Contact the device distributor.
Heat does not work	Heating element resistor failure.	Contact the device distributor.
Timer does not work	Component failure.	Contact the device distributor.
	Circuit failure.	Contact the device distributor.
Unknown faults	Unknown.	Unplug the unit immediately and contact the device distributor.

Warranty

- The machine is guaranteed against any manufacturing defects for **one year** from the date of purchase.
- To comply with any warranty, MESTRA will require the purchase invoice.
- Any breakdowns caused by improper installation or misuse of the machine are excluded from the warranty.
- Furthermore, MESTRA will not perform any repairs outside its workshop and is exempt from any shipping costs incurred.
- Repairs performed at the customer's premises, whether or not under warranty, are entrusted to the technical service of the distributor who made the sale or to its contracted services.

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